Owlet

Baby Care

Up all night
so you don't
have to be.

Tips for a Safe Sleep Environment

Some friendly tips to keep your baby healthy and safe.

- Always place your baby alone and on their back to sleep. Placing your baby to sleep on their side or on their stomach is not safe.
- Use a crib that meets current safety standards. Use a firm mattress that fits snugly in the crib and is covered with only a tight-fitting crib sheet.
- Do not put anything soft, loose, or fluffy in your baby's sleep space. This includes pillows, blankets, comforters, bumper pads, stuffed animals, toys, or any other soft items.
- Use a wearable blanket, or similar type sleeper, instead of traditional blankets to keep your baby safe and warm.
- Room-share instead of co-sleeping. Keep your baby's sleep space separate from, but close to, your bed. This will keep them safe, and make feeding easier. The Owlet Monitor does not function properly with co-sleeping. Your body will interfere with the Smart Sock's wireless signal.
- Avoid falling asleep with your baby in your bed, couch, or armchair.
- Never allow your baby to sleep on any soft surfaces such as adult or child-sized beds, sofas, chairs, waterbeds, pillows, cushions, comforters, sheepskins, or anything similar.
- Do not use pillows, wedges, or positioners to prop your baby up, keep them on their back, or prevent them from rolling.
- Make sure your baby doesn't get too warm. Use light sleepwear on your baby and keep the room temperature comfortable for a lightly-clothed adult.
- Consider offering a pacifier to your baby at naptime and bedtime. For breastfed babies, delay introducing a pacifier until 3-4 weeks of age, when nursing is well-established.
- Educate everyone who cares for your baby about these rules for safe sleep!

Box Contents

What you get

Base Station


Micro USB Smart Sock
Charging Cable

Newborn Fabric Sock

Size 1 Left


Size 1 Right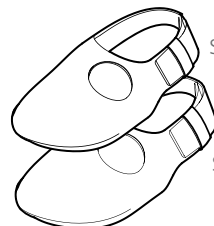


Size 2

Size 3

Fabric Sock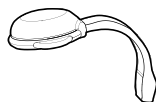

Sock Electronics

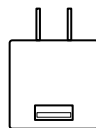Power USB Block


USB Power Cable

Charging and Connectivity


Get familiar with your monitor | Base Station

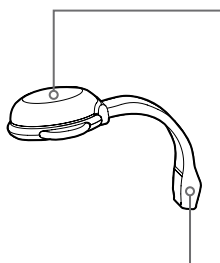
Sock Electronics dome light:

Indicates the Smart Sock is charging when it is blinking green. A solid green light indicates a full charge.

Sock Electronics charging port: Connects to the Base Station using the supplied two-inch white cord. Plug the Smart Sock in every day when not in use. Do not worry about removing the electronics from the Fabric Sock when recharging.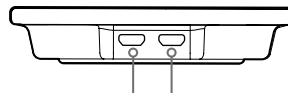

Power supply and Smart Sock charging ports: These ports can be used interchangeably for supplying power to the Base Station or recharging the Smart Sock.

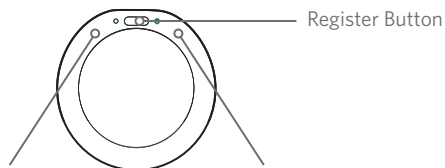Left light: Indicates Base Station connection to Wi-Fi

Right light: Indicates Smart Sock connection to the Base Station

Base Station Status

1: Slow pulsing green light: Heart rate and oxygen levels look good

2: Blinking green light: Getting first heart rate and oxygen readings

Base Station Alerts

1: Blue flashing light: The Smart Sock is disconnected from the Base Station

2: Yellow flashing light: The Smart Sock fell off or is poorly placed, or shifted from proper placement

3: Red flashing light: Heart rate or oxygen levels are outside specified limits


Dim Base Station Light

Press and hold to adjust brightness. There is a brighter light for daytime use and a dimmer light for nighttime use.

Power Off Sock Electronics

Place the bottom flat surface of the Sock Electronics against the center of the Owlet Base Station until red light inside the Sock Electronics stops flashing. Turn on Sock Electronics by plugging them into Base Station.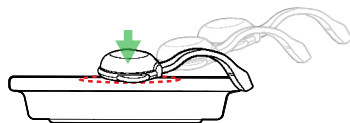

Charging

When the Smart Sock is charging, it will blink green. When it is fully charged, it will have a solid green light. The Base Station will glow cool-white when the Smart Sock is charging.

Turn Off Alerts

Press and release to turn off monitoring. The Base Station will beep, and its light will turn off.


Sock Sizing Guidelines

Size 1

0-1 MONTH

LEFT & RIGHT FOOT NEWBORN SOCKS

The Newborn Sock is always placed with the Sock Electronics on the bottom flat part of the foot. There is a RIGHT- and LEFT-footed Newborn Sock. Having an extra sock will allow you to switch the Socks between feet if needed. We recommend alternating between the right- and left-footed Newborn Size 1 Sock every 4 to 24 hours.

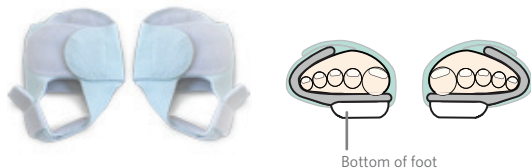Size 2

1-3 MONTHS

LEFT FOOT SOCK

The fabric sock sizes 2,3, and 4, are always placed with the sock electronics on the top of the LEFT foot with the sensor band running on the outside edge of the LEFT foot. Size 4 is available for order online at owletcare.com.


Size 3

3-10 MONTHS

LEFT FOOT SOCK


How to remove and insert the Smart Sock Electronics

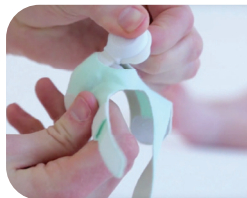1) Removing Sock Electronics-gently pull on the Electronics until it's completely removed

2) Inserting, gently pushing the electronics fully into the sock,

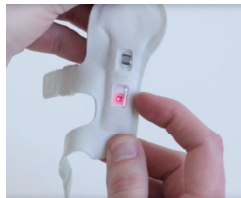3) Check if the sensors are fully visible inside the windows of the Fabric Sock

4) if not, Adjust until properly aligned within the windows.

ALWAYS remove the Smart Sock Electronics from the Fabric Sock before surface cleaning or washing the Fabric Sock.

Correctly position Sock and Sensors

Make sure that the sensor band falls about a centimeter, or a pinky width, behind where the small toe meets the foot. This is the zone for best readings. Pick a fabric sock size that keeps the electronics within this zone. If the electronics cover part of the toes, move to a bigger sock size.

Size 1 sock Left & Right zone for best readings.

Size 1 newborn sock is always placed with the Sock Electronics on the bottom flat part of the foot .

Size 2,3&4 socks zone for best readings.

Size 2,3 & 4 always place with the sock electronics on the top of the Left foot.

Guidelines to not over tighten

It is very important to not over tighten, If you notice red marks or pressure marks from the sock or sensor band, the sock is strapped too tight. Tips for a proper fit: After fully adjusting the sock, make sure the velcro hook falls towards the middle of the velcro pad. If any velcro is exposed on the bottom edge, indicating too small of a sock, go up a bigger sock size. If the back strap is not long enough to tuck into the strap loop go up a bigger sock size.

Zone where velcro pad fits best

Troubleshooting Guide

Initial Setup | Alerts | Connectivity

Initial Setup

Problem	Causes	Solutions
• Cannot see the Owlet Wi-Fi during setup	• The Base Station is not turned on • Already connected to the Owlet Wi-Fi • The Base Station is out of range	• Plug in the Base Station • Check if the Wi-Fi light on the bottom of the Base Station is on. If so, you can skip this step and continue setting up your account in the app • Move the Base Station closer to your Wi-Fi router
• Cannot see your home Wi-Fi in the Owlet App	• Your home router is 5.0 GHz (Owlet only supports 2.4 GHz) • You may have a hidden router	• Make sure your iOS device is connected to a 2.4 GHz network (most 5.0 GHz routers also support 2.4 GHz) • Visit owletcare.com/support for instructions on hidden routers
• The Base Station will not connect to your home Wi-Fi	• Password is incorrect • The Base Station is out of range of your Wi-Fi router • Your home Wi-Fi is not working	• Double check your internet password • Move the Base Station closer to your internet router for setup, then move the Base Station back to the nursery after set up • Try restarting the phone and reconnecting to your Wi-Fi • Reset your Wi-Fi router
• The Base Station remains white after the Smart Sock is unplugged	• Software update is in progress	• Plug the Smart Sock back into the Base Station until the update is complete
• The Base Station will not pair to your Owlet account	• The Base Station and your iOS device are not on the same Wi-Fi network • Holding the small Register Button	• Verify that your iOS device is connected to same Wi-Fi network as the Base Station • Press and release (do not hold) the small Register Button on the bottom of the Base Station and wait a few seconds before trying to pair your Owlet account again • Connect your iOS device to the same 2.4 GHz Wi-Fi network as your Base Station

*Need Technical Support? Email us at contact@owletcare.com or call us at +1 (844) 334-3717 from 7AM - 12AM (MST) Mon. - Sat.
For all general questions, email us at contact@owletcare.com or chat with us at owletcare.com/chat

Alerts

Problem	Causes	Solutions
• The Base Station is flashing blue and making noise	• The Base Station is out of range of the Smart Sock • The caretaker's body is blocking the Smart Sock to Base Station connection. This is common during feedings, diaper changes, snuggles, etc. • Low/dead battery	• Move the Base Station closer to the Smart Sock • Reposition your baby or consider turning off monitoring during feedings, diaper changes, snuggles, etc. • Recharge the Smart Sock
• The Base Station is flashing red and making noise	• Low/high heart rate • Low oxygen levels • The Smart Sock does not fit properly	• Make sure your baby's pinky toe goes all the way to the end of the Fabric Sock • Make sure the Smart Sock is snug (if not, it is difficult to get a reading, and more false alerts will occur) • Make sure the sensors are centered in the windows of the Fabric Sock • Consider using a different Fabric Sock size (To chat with a doctor go to www.owletcare.com/Dr-Chat)
• The Base Station is flashing yellow and making noise	• Difficulty getting a reading • The Smart Sock was placed incorrectly • The lighting in the room is too bright	• Make sure your baby's pinky toe goes all the way to the end of the Fabric Sock • Make sure the Smart Sock is snug (if not, it is difficult to get a reading, and more false alerts will occur) • Make sure the sensors are centered in the windows of the Fabric Sock • Dim the lighting in your baby's room

Connectivity

Problem	Causes	Solutions
• The Base Station is disconnected from the internet	• The Base Station is out of range of your Wi-Fi router • Your home Wi-Fi is not working	• Move the Base Station and the Wi-Fi router closer together (neither Wi-Fi range extenders nor second routers will help) • Reset your Wi-Fi router
• Unable to charge the Smart Sock	• The Smart Sock is not plugged in all the way • The Smart Sock requires a reset	• Plug the Smart Sock in to charge until you hear a click and it is blinking green • Turn the Sock Electronics off, then back on.
• Seeing dashes instead of heart rate and oxygen	• The Smart Sock is trying to get a reliable reading • Too much movement • Poor Smart Sock placement • Poor Fabric Sock fit	• Make sure the sensors are centered in the windows of the Fabric Sock • Do not use rockers or vibrating pads. They will interfere with monitoring • Check the Smart Sock placement • Check the Fabric Sock size

Limited Warranty and Disclaimer Information

Limited Warranty | Disclaimer and Venue

LIMITED WARRANTY

Owlet warrants that this Baby Monitor will be free from defects in materials and workmanship, under normal use and service, for a period of one (1) year from the date of purchase as established by the purchase receipt. This Limited Warranty is only available to the original purchaser of this Owlet Baby Monitor.

At its option, Owlet may repair or replace the defective Baby Monitor or defective component part(s) in accordance with the terms of this Limited Warranty Period. The replacement unit will be warranted only for the unexpired period of the original Monitor's Applicable Warranty Period. This Limited Warranty does not apply if the Owlet Baby Monitor has been damaged as a result of misuse, modifications or improper maintenance. This Limited Warranty is the only Warranty for this Owlet Baby Monitor. No one is authorized to make any other warranty on behalf of this Monitor. Some states or Countries do not allow limitations on how long an implied warranty lasts or for the exclusion of incidental or consequential damages, so these limitations or exclusions may not apply to you. This Limited Warranty gives you specific legal rights, and you may also have other rights depending on the laws of your State or Country.

DISCLAIMER AND VENUE

By using this Owlet Baby Monitor, the user agrees and affirms that Owlet Baby Care Inc. including its agents, employees, investors and members shall not be held liable for any loss, damage, or injury arising in connection with the use or operation of the Owlet Baby Monitor regardless of cause.

By using this Owlet Baby Monitor, the user agrees and affirms that any dispute arising out of its use or operation shall be brought in and heard by the Third Judicial District Court of Salt Lake County, State of Utah, USA.

The Owlet Monitor is meant for healthy babies; therefore, the alarm thresholds on the Owlet Monitor are not as stringent as a medical monitor.

Remember:

- Do not use Owlet as a diagnostic tool
- Do not use Owlet if your doctor recommends the use of a hospital pulse oximeter
- Do not use Owlet if your doctor recommends the use of a hospital apnea monitor
- Do not use Owlet as an excuse for unsafe sleep habits

READ THIS MANUAL

Inside you will find many helpful hints on how to use and maintain your Owlet Baby Monitor properly. The Troubleshooting section provides answers to some problems. If you review the Troubleshooting tips first, you may not need to contact us for assistance. Complete setup and use information is available online at www.owletcare.com/setup

READ THE SAFETY INFORMATION

Your safety and the safety of others are very important to us. There are many important safety messages in this manual and in the Owlet Baby Care App. Always read and obey all safety messages.

This symbol is used in this manual as an indication of Important Safety

Information! All safety messages will follow the safety alert symbol and either the word "DANGER", "WARNING", "CAUTION" OR "NOTICE". These words mean:

DANGER = An imminently hazardous situation that will result in death or serious injury.

WARNING = A potentially hazardous situation that could result in death or serious injury and/or damage to property.

CAUTION = A potentially hazardous situation that may result in minor or moderate injury.

NOTICE = Hazards that could result in things other than personal injury, like property damage.

WARNINGS AND IMPORTANT SAFETY INFORMATION

WARNING: This monitor is not a medical device. The Owlet Baby Monitor alerts you if your baby's heart rate or blood oxygen levels are outside of the provided range. The monitor is designed for newborns to eighteen (18) months not for after

eighteen (18) months. The Owlet Baby Monitor is not a medical device. It does not

- Diagnose, cure, lessen, monitor, treat, or prevent disease or injury;
- Affect the function or structure of the body;
- Achieve primary intended purposes through chemical action;
- Does not prevent Sudden Infant Death Syndrome (SIDS).

This manual serves two primary functions: To provide the user with the basic directions and recommendations for the proper installation and use of the Owlet Baby Monitor; and to explain features, operation, safety precautions, maintenance and troubleshooting of the Owlet Baby Monitor. It is essential that all persons who are expected to operate or adjust this Owlet Baby Monitor read the instructions carefully, that they understand how to properly operate the Owlet Baby Monitor and they follow these instructions.

If you do not understand these instructions, please contact us right away at contact@owletcare.com or 1-844-334-5330 from 7AM - 9PM (MST) Mon. - Sat.

FOR YOUR SAFETY AND THE SAFETY OF THE BABY

• ALWAYS KEEP THE BASE STATION AT LEAST THREE FEET AWAY FROM YOUR BABY'S CRIB BECAUSE THE CORD CAN POSE A STRANGULATION HAZARD.

- THE OWLET BABY MONITOR IS AN AID. IT IS NOT A SUBSTITUTE FOR ADULT SUPERVISION.
- FOR PREMATURE BABIES, BABIES ON SUPPLEMENTAL OXYGEN OR BABIES WITH HEALTH CONDITIONS, PLEASE CONSULT YOUR HEALTH PROFESSIONAL TO DETERMINE WHETHER OR NOT THE OWLET BABY MONITOR IS RIGHT FOR YOU.
- RESPOND IMMEDIATELY WHENEVER AN ALERT SOUNDS.

NORTH AMERICA COMPLIANCE STATEMENTS

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

This device complies with Part 15 of the FCC rules and with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

Your device is a radio transmitter and receiver. It is designed and manufactured not to exceed the emission limits for exposure to radio frequency (RF) energy set by the Federal Communications Commission of the U.S. Government. The FCC has established criteria for the amount of radio frequency energy various products may produce depending on their intended usage. This product has been evaluated and found to comply with the FCC's exposure criteria. For body worn operation, the FCC RF exposure guidelines were also met when used with the accessories supplied or designed for this product. Use of other accessories may not ensure compliance with FCC RF exposure guidelines and should be avoided.

Caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this device.

Please ONLY use the supplied power adapter. Failure to do so may void our limited warranty.

This device must not be co-located with other transmitters.

The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website: http://www.hc-sc.gc.ca/ewh-semt/pubs/radiation/radio_guide-lignes_direct/index-eng.php

IMPORTANT NOTE:

The antennas used for the Base Station transmitter must be installed to provide a separation distance of at least 20 cm from all persons and must not be located or operating in conjunction with any other antenna or transmitter.

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