P E R I F I T

support@perifit.co

- 1 English
- 5 Français
- 9 Deutsch
- 13 Español

FCC Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Any changes or modifications to this device not explicitly approved by manufacturer could void your authority to operate this equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

RF Exposure Information

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

ISED Statement

English: This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

The digital apparatus complies with Canadian CAN ICES-3 (B)/NMB-3(B).

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This equipment complies with Canada radiation exposure limits set forth for an uncontrolled environment.

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P E R I F I T

English user guide

Thank you for purchasing Perifit. We hope that Perifit will bring you quick and enduring results.

Here are the first steps to get started using your Perifit:

1. Download the Perifit app on your smartphone or tablet (look for Perifit in the App Store or Google Play).
2. Activate Bluetooth on your phone, but don't try to connect it to Perifit within your Bluetooth settings.
3. Launch the Perifit app and follow the detailed in-app instructions on how to use Perifit.

You can reach our customer support at any time by using the email button inside the app, we would be more than happy to help you. Detailed instructions are available in LEARN tab of the Perifit APP or on: <https://perifit.co/pages/using-perifit-for-the-first-time>

How to take care of your Perifit

Clean your Perifit before and after every use with warm, soapy water.

Remember to rinse thoroughly. It is important that your Perifit is completely dry before you store it. Store your Perifit in a clean and dry environment.

If you notice any change in device performance please turn off your Perifit and contact us using the Perifit app: "?" / "ask a question" button.

Perifit is not chargeable. The battery is intended for 1000 sessions of 15 min.

DO NOT:

- Wash in boiling water
- Sterilize
- Use cleaning agents
- Microwave
- Clean in a washing machine
- Leave it trapped under any object
- Use your Perifit during a bath
- Silicone based lubricant

AVOID:

- Extreme heat
- Extreme cold
- Sustained pressure

Safety Instructions

Do not use your Perifit if:

- You are pregnant.
- You have given birth within the last 6 weeks.
- You experience persistent pain when inserting or using Perifit, even with water-based lubricant.
- You are suffering from a severe condition affecting your pelvic floor (severe prolapse, severe vaginismus, etc.) and

have not already spoken to a pelvic floor specialist.

- You have a medical condition that cause pain around the vulva (known as vulvodynia) or vaginal tightness during penetration (vaginismus) that makes inserting the Perifit painful
- You have moderate to severe prolapse and you have not yet consulted a medical professional

Intended use: Perifit is a device intended to help you strengthen your Pelvic floor, and can be used to treat incontinence disorders due to a weak pelvic floor, reduce prolapse symptoms, improve intimate wellbeing. Perifit can be used actively or preemptively. It must not be used to diagnose a medical condition. If you are unsure about your condition please consult a women health specialist.

Please consult a women health specialist to ensure you use and understand correct pelvic floor contraction technique, or if you feel any pain during or after use of your Perifit. Using the Perifit during perimenopause or menopause: lower estrogen can cause the vaginal lining to become drier, thinner, and less elastic. Hormone replacement therapy (HRT) may reduce some of these symptoms, but you may also want to use a small amount of water-based lubricant to make inserting the Perifit easier and more comfortable. If you have any doubts about the health of your vaginal tissue, please check in with your health professional.

Perifit is a single patient device, intended to be operated by adult female patients, who do not need to have any formal education, but it is expected that they know how to read and use a smartphone. Perifit is not safe for children.

Do not leave your Perifit in for more than one hour at a time.

For further information, please refer to the LEARN tab in the Perifit app. For any question or suggestion please contact us using the Perifit app: "?" / "ask a question" button."

Guarantee

Perifit is made from medical-grade silicone and is backed by a limited manufacturer's guarantee against any defects in materials and/or workmanship for 2 years from the date of purchase.

Defects discovered within the guarantee period should be reported to X6 Innovations.

X6 Innovations reserves the right to exercise its discretion in determining whether or not to replace a defective Perifit free of charge.

This guarantee does not cover cosmetic deterioration or damage caused by general wear and tear, physical or natural destruction, accident, misuse, neglect or other external causes.

Any attempt to take the Perifit apart will invalidate this guarantee. Any guarantee claims must be supported with reasonable evidence, including proof of date of purchase, alongside these guarantee conditions. X6 Innovations must be notified of any defective Perifit.

We will then provide you with a Return Merchandise Authorisation number, which must be displayed on the outside packaging. In the event that X6 Innovations decides to replace your Perifit, any replacement Perifit will not extend the life of this guarantee.

DISCLAIMER:

Users of Perifit do so at their own risk. Neither X6 Innovations nor its retailers assume any responsibility or liability associated with or for the use of this product.

X6 Innovations reserves the right to revise and/or update this manual without being obligated to notify any users.

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Icons explanation

This packaging contains a printed user guide (as you can see).

This is a medical device certified CE class 1.

This medical device has no electrical contact with the patient.

This device should be kept between 15°C and 60°C (59°F and 140°F).

This device should be kept between 15% and 80% humidity.

Do not throw away.

Presence of non-ionizing electromagnetic radiation.

RoHS

This device is compliant with the RoHS directive about hazardous substances.

IP26

Perifit can be temporarily submerged in water.