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Widex SoundConnect

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US ENGLISH

USER INSTRUCTIONS
Widex SoundConnect

⚠ Read these instructions carefully before you start using the device.

Intended use

The device is intended to transmit audio signals from a Personal Computer directly to supported hearing aids.

NOTE:

The device uses a USB A connector. If your computer is without an USB A configuration, you may use a converter.

HOW TO USE

Connect the device to your computer. (Ilru. 1 and 2)

Restart your hearing aids. The hearing aids will then be in pairing mode for 3 minutes. (Ilru. 3)

Press and hold the button on the device until the light indicator is flashing blue, then

Symbol	Title/Description
	Manufacturer: The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.
	Catalogue number: The product's catalogue (item) number.
	Consult instructions for use The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.
	Warning: Text marked with a warning symbol must be read before using the product.

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LIGHT INDICATIONS

Flashing white: Device has not been paired to hearing aids.

Flashing blue: Device is in pairing mode.

Flashing green: Device is searching for paired hearing aids.

Constant green (low intensity): Device is connected to hearings aids, but no audio is streamed.

Constant green (high intensity): Device is connected and streaming audio.

Constant orange: Device is in service mode, see troubleshooting.

Flashing red: Device is in error mode, see troubleshooting.

MAINTENANCE

Clean your device using a soft, dry cloth. Do not use any liquids or chemicals and do not immerse in liquids.

When the device is not in use keep it in a dry location out of reach of children and pets. Never try to open or repair the device yourself. This may only be done by authorised technicians.

REGULATORY INFORMATION

FCC ANDISED STATEMENTS

FCC ID: 2XADT-WSC

IC: 26428-WSC

Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and

(2) This device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee

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Problem	Possible cause	Solution
The device does not work	1. Not paired 2. Out of range	1. Pair with hearing aids 2. Move into range
No sound	1. Not paired 2. Device is out of the hearing aids' range 3. Audio from PC is muted 4. Another audio output device is selected on the computer	1. Repeat the pairing process 2. Move into range 3. Unmute PC audio, and try to turn up the volume 4. Windows: Click the loudspeaker icon in the taskbar, and choose the Widex SoundConnect as output device
The device does not respond, and the light indicator is solid red	Device is in a service mode intended for the hearing care professional	Press the button to return to regular use
The device does not respond, and the light indicator is solid orange	Device error	Contact your hearing care professional
Audio from "Microsoft Teams" is coming out of another audio device, even while the SoundConnect is the main audio output device on the computer	On Windows operation systems "Microsoft Teams" has its own audio settings that overrules the ones for the operating system	Go into settings in "Microsoft Teams", choose "Devices" and set the "Speaker" option to the SoundConnect

that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by WSAUD A/S could void the user's authority to operate the equipment.

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