

Removing a Tag from an Infant

To avoid causing an alarm, de-enroll the Tag on the Perceptis Control and Display Computer to stop monitoring before removing the Tag (see the Perceptis Hugs System manual - 805U1601).

With a pair of bandage scissors, cut the strap in the middle and remove the Tag from the infant.

To remove the severed strap from the Tag, push each piece of the strap through the tag slots, moving from the bottom to the top of the Tag.

Cleaning a Tag

The Hugs Tag can be cleaned using a soft-bristle brush and a disinfectant soap or similar solution (for example, Butcher's Heptagon Disinfectant Cleaner). Dry the Tag thoroughly before use.

A small soft-bristle brush can be used to clean the tag slots. Pass the brush carefully through the tag slot moving from the bottom to top of the Tag.

Warnings:

- Do not use an autoclave to clean the Tag or serious damage may result.
- Do not soak the Tag in alcohol.

Reference Documents

Please refer to the Perceptis Hugs System Manual (p/n 805U1601), Perceptis Installation Manual (p/n 805I0301), and the Perceptis System Manual (p/n 805U0601) for more information.

Warnings

Follow Installation Instructions Precisely - instructions must be carefully followed throughout the installation of all Hugs Tags. Failure to follow the instructions may significantly affect performance.

Battery Handling Warning - this device contains a lithium battery. Risk of fire or explosion. Do not force open, heat to 100° C, or dispose of in fire.

Storage

Store Hugs Tags at room temperature.

Statements

United States - FCC

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Canada - ISC

This device complies with RSS-210 of Industry and Science Canada. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

Warranty

Instantel's products are warranted against defects in materials and workmanship and shall perform in accordance with published specifications for a period of one year. Instantel Inc. reserves the right to change specifications without notice.

Limitation of Liability

This Product has been designed for use to assist in the prevention of infant abduction.

The range, accuracy, function and performance of this Product may vary from the published specifications due to many factors, including, without limitation, site impairments from structural effects, metal objects in the vicinity, placement of the receiver and transmitter, interference from other electrical devices, atmospheric effects, installation, and maintenance. There may be other factors which also affect performance of this Product. Instantel does not guarantee that this Product will detect 100% of infant abductions, nor does Instantel guarantee that this Product will not return false reports of infant abductions or Product tampering.

Monthly testing and maintenance of this Product, and its proper use, as described in the Product documentation, are essential to verify that the system is operating correctly and to ensure that the probability of detecting an alarm and locating the tag are maximized. The failure to undertake regular testing and maintenance, or to use the Product properly, will increase the risk of system failure and failure to detect infant abduction or tampering with the Product, whether the tag or the strap.

Instantel hereby disclaims all warranties, express or implied, arising out of or in connection with any of its Products of the use or performance thereof, including but not limited to, where allowable by law, all other implied warranties or conditions of merchantable quality and fitness for a particular purpose and those arising by statute or otherwise in law or from a course of dealing or usage of trade.

Instantel's liability to you or anyone claiming through or on behalf of you with respect to any claim or loss arising out of the use or misuse of Instantel's Product, defective products or materials, improper installation or maintenance of Instantel's Product or products or the system in which they are incorporated, or alleged to have resulted from an act or omission of Instantel or any person, negligent or otherwise, shall be limited to:

The repair or replacement of defective Product or materials supplied by Instantel during the warranty period as set out in the Product documentation; or, at the option of Instantel, Refund of the purchase price of the Product supplied by Instantel.

In no event shall Instantel be liable for general, specific, indirect, consequential, incidental, exemplary or punitive damages or any losses or expenses suffered by you or anyone else, whether or not Instantel, or its employees, officers, agents, resellers or installers has been informed of the risk of such loss or expense and whether or not such losses or expenses were foreseeable.

© 1998 Instantel Inc. All Rights Reserved. Instantel and the Instantel logo are registered trademarks of Instantel Inc. in North America. Perceptis, the Perceptis logo, and Hugs are trademarks of Instantel Inc. Other brand and product names are trademarks or registered trademarks of their respective holders. As part of our continuing commitment to product excellence, Instantel reserves the right to change specifications without notice. Printed in Canada.

PerceptisTM HugsTM Tag Instructions

Instantel[®] Inc.
826 Proctor Avenue
Ogdensburg, NY
U.S.A., 13669

362 Terry Fox Drive
Kanata, Ontario
Canada, K2K 2P5

Telephone: 613-592-4642 or 800-267-9111

Facsimile: 613-592-4296

e-mail: info@instantel.com

Website: www.instantel.com

Model No.: Hugs

Part No.: 805A1701

Made in Canada

805U1701 Rev 02