







# ➤ Charge

included charging cable.

Red LED = charging

Blue LED = fully charged

Input: 5V == 500mA

## A Note

To avoid damaging the product, please defer from using any "quick chame" features.

#### ▶ Power ON/OFF

- Slide to "ON" position to turn the headphone on. The headphone will notify battery level based on the current state.
   Slide to "OFF" position to turn the headphone off.

#### > Pairing



- Initial use
   The headphone will auto-emter Bluetooth pairing state. Red and blue LED will blink attendately.
   Select "EDIFIER Enhance One" in your phone setting to connect. Blue LED will blink now
- and then when successfully connected.

  3. Follow the "Edifier Assist" app instructions to complete the hearing test; export the test results.
- to the headphone;

  4. Once completed, the headphone will auto-connect to this smartphone device whenever the device's Bluetooth is turned on.

#### A Note

- When smartphone device's Bluetooth is turned on, the headphone can auto-connect to the device, given that it has been paired before;
   If the afore mentioned device's Bluetooth feature is ever turned off, you'll need to manually
- 2. In the arter trimitation control is better the state of the state that the select the headphone under Blustooth devices in your smartphone device's settings:
  3. If you wish to connect to a new smartphone device: turn the headphone on; press and hold the " " button for around 5s to enter Blustooth pairing state, then select the headphone under Blustooth devices in new smartphone device's settings.

#### > Functional operation



- ─ Volume button Press to decrease volume
- Volume + button Press to increase volume
- Phone call button Press to play/pause
  Press to accept/end a call
  Press and hold for 5s to enter Bluetooth pairing mode

#### ➤ Reset

Turn the headphone ON, and press the " button five times in quick successions; The process is completed once the indicator light blinks rapidly in Nava.

- The 'C\_D' button is only used for acceptinglending phone calls; any calls via Brief party appor are not supported.

  2. For IGS device, if the call is accepted using the answer button on the phone, the audio will then be played via the phone speaker and not the heapthone.

  3. Any sound rearrain part calls in the heapthone will be cliented upon reset.

## For more information, please visit our website www.edifier.com $\bigcirc$ @Edifier\_Global @Edifier\_Global Edifier International Limited P.O. Box 6264 nternational Limited. All rights reserved. NOTICE: For the need of technical improvement and system upgrade, information contained herein may be subject to change from time to time without prior notice.

Products of EDIFIER will be customized for different applications. Pictures and illustrations shown on this manual may be slightly different from actual product. It any difference is found, the actual product prevails.

V1.0

#### > FAQs

#### > The indicator is off during charging.

Please ensure the headphone is correctly connected to a power source.

#### No sound

- Check if the headphone is operating.
   Check if the headphone volume is at an appropriate level.

#### > The hearing supplement quality of the headphone is not good.

- Please ensure the hearing test has been completed and the test results has been exported to
  the headphone successfully through the "Edifier Assist" app.
   The product is designed for people with moderate, to moderately severe hearing loss.
   If the product is repairments are beyond this range, the headphone may not achieve the desired

### >> Important safety information

- Electronic devices

   To suici districting with other electronic devices or causing any dangers, do not use the product in places where wireless devices are expressly disallowed.

  Medical devices

   Please abide by regulations therein and turn the product off in places for medical treatment or
- Please abide by regulations therein and turn the product off in places for medical treatment or health care, where winderes device are expressly disablened.
   Radio waves generated by the product may affect the normal use of implantable medical devices or personal medical devices (e.g., posemistes, coordier implants, etc.). If you use these devices, please consult their imanufactures about restinations on using the product, the expression implaces to the product, please consult your doctor about whether there are marginer, imentates contained in the product, please consulty your doctor about whether
- There are simplest instituted contained for product, please consult your doctor about whether pays all failed implicate modical devices.

   Do not use the product in the colority (close than 15cm) of the implicated medical devices (e.g., 15cm) and the product products and the colority of the colorit

- Surroundings

  Do not use the product in places with dust, high humidity, and smudges or near magnetic fields
- On all code in graduation in public with a count register better graduation (and in the public of the graduation in public in which complete code of the graduation is been deep and graduation of the gradua
- beside a window.

  Do not place the product near any heat source or open fire (e.g., heaters, microwave ovens, ovens, water heaters, stowes, candles, etc.), or any other places that may generate heat.

  Do not place sharp metal objects (e.g., pirs etc.) near the product's receiver or speaker to prevent metal objects from adhering to and causing injury to you.

#### Important safety information

- Use only accessories approved by the manufacture and attached in the package. Otherwise, it is may violate warms by them and relevant regulations of the county where the product is located, even lead to safely accident. Please contact the manufacture for the approved accessories. The power source case must be approved by relative departments and in accordance with electrical apporting that the production of evices as well as local regulatory requirements (e.g., UI, CSA, VCE, COC), otherwise them may be dangers of the, explosion, etc.
- electrical apportations or devices as well as occu registrary requirements (e.g., U.L.L.SA, V.U.CCC), otherwise from they be darges of the replaction, et.

  Batteries

   Do not demantife or modify batteries, inserf toreign objects into batteries, or immerse batteries into sater or other (quids to avoid battery leskage, overheating, the or explosion. Batteries hould only be registrated by manufactures.

   Do not from, squeece or purchase batteries. High external pressure may couse batteries to short-forcium and/or overheat.
- short-circuit and/or overheat.

  \*\*Do not use batheries after being immersed in water. Batteries subjected to extremely low air pressure may result in an explosion or leskinge etc. If the battery looks swollen, lesking, dissorbored, having odd or any abnormal appearance, stop using a. Immediately dispose of batteries carefully in accordance with local laws and regulations regarding battery disposal.
- Maintenance

  Lithium-ion batteries require to be charged at least once in every three months.

  To reduce the risk of fire and electric shock, do not expose the product and its accessories to
- moisture.
  deaning and maintaining the product, please cease to use it and disable all applications, sconnect from other devices or unplug all connecting cables.
- and disconnect own other devices or unplug all connecting cables.

  Never use strong chemicals, cleamer or detergents to cleam the product or its accessories. Use only clean and dry clott in wipe the product or its accessories.

  Do not keep magnetic stripe cards (e.g., bank cards, telephone cards, etc.) in contact with the product for a long time, otherwise the magnetic stripe card may be damaged by the magnetic
- product or a long time, comments on endigence comps can may be entempted by the management. The longed product enterprises fire endigence comps can be enterpresed to the product endinger of the product endinger of the product endinger in preparation. Only use this product where the instantion is reflicted.

  The product is reflicted.

  The product is not explained to proque having cache office enterins, maying it, denoir. The product end on explained to proque having cache office enterins, maying it, denoir. The product is not explained to proque having cache office enterins, maying it, denoir. The product is not explained to proque having cache office enterins, maying it, denoir.

  The product is not explained to proque having cache office enterins, maying it, denoir.

  The product is not explained to proque having cache office enterins on the product in the product enterins of the pr

#### Important safety information

- Keep product away from children under the age of 14. Please make sure product is inaccessible to young children and pets, to avoid them from swallowing parts of the product. Immediately seek professional medical assistance if the fifthum battery built-in the product is
- accidentally smallowed.

   To reduce the risk of fire and personal injury, do not open or remove the product housing for repair.

  In case of any product damages, it must be repaired by professionals or contact the manufacturer.

   Never throw batteries into fire or places of high temperature, the battery will explode in over

# FCC asternets. This device complies with Part 15 of the FCC Rules. Operation is subject to the following. This device complies with Part 15 of the FCC Rules. Operation is subject to the following the Part 15 of the Rules of th Input 5V - 500mA