## USER INSTRUCTIONS

### Widex SoundConnect

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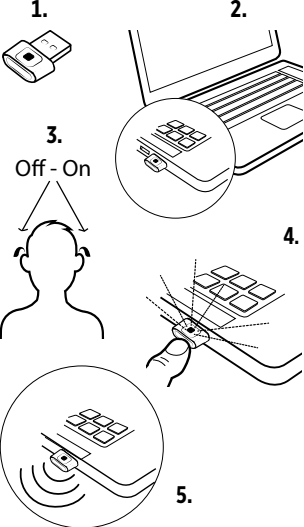
**EU Directives**  
Directive 2014/53/EU  
WSAUD A/S hereby declares that this SoundConnect model is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU. The device contains radio transmitters operating at: 2.4 GHz, 0.9 mW EIRP. A copy of the Declaration of Conformity can be found at: <https://global.widex.com/doc>

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**ENGLISH**  
**Read these instructions carefully before you start using the device.**  
**Intended use**  
The device is intended to transmit audio signals from a Personal Computer directly to supported hearing aids.  
**NOTE:**  
The device uses a USB A connector. If your computer is without an USB A configuration, you may use a converter.  
**HOW TO USE**  
Connect the device to your computer. (Illu. 1 and 2)  
Restart your hearing aids. The hearing aids will then be in pairing mode for 3 minutes. (Illu. 3)  
Press and hold the button on the device until the light indicator is flashing blue, then release. (Illu. 4) The SoundConnect is now in pairing mode.

**Information regarding disposal**  
Do not dispose of hearing aids, hearing aid accessories and batteries along with ordinary household waste. Hearing aids, batteries and hearing aid accessories should be disposed of at sites intended for electrical waste and electronic equipment, or given to your hearing care professional for safe disposal.

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Make sure the SoundConnect device and your hearing aids are within 1 m range of each other. The pairing sequence will start automatically. The light indicator will glow constant green, when the pairing is successfully completed and if sound is played from the computer the audio will be streamed to your hearing aids. (Illu. 5)

The SoundConnect can only be paired to one set of hearing aids. To pair a new set of hearing aids, you must perform the same steps and the new pairing will overwrite the existing one.

**NOTE**  
After pairing and you unplug the SoundConnect and reconnect the device, it will keep the pairing.

**RANGE**  
Line of site up to 8 meters  
Outside in open air up to 5 meters

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| Problem                                                                                                                                         | Possible cause                                                                                                                                          | Solution                                                                                                                                                                                                              |
|-------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The device does not work                                                                                                                        | 1. Not paired<br>2. Out of range                                                                                                                        | 1. Pair with hearing aids<br>2. Move into range                                                                                                                                                                       |
| No sound                                                                                                                                        | 1. Not paired<br>2. Device is out of the hearing aids' range<br>3. Audio from PC is muted<br>4. Another audio output device is selected on the computer | 1. Repeat the pairing process<br>2. Move into range<br>3. Unmute PC audio, and try to turn up the volume<br>4. Windows: Click the loudspeaker icon in the taskbar, and choose the Widex SoundConnect as output device |
| The device does not respond, and the light indicator is solid orange                                                                            | Device is in a service mode intended for the hearing care professional                                                                                  | Press the button to return to regular use                                                                                                                                                                             |
| Device does not respond, and the light indicator is flashing red                                                                                | Device error                                                                                                                                            | Contact your hearing care professional                                                                                                                                                                                |
| Audio from "Microsoft Teams" is coming out of another audio device, even while the SoundConnect is the main audio output device on the computer | On Windows operation systems "Microsoft Teams" has its own audio settings that overrules the ones for the operating system                              | Go into settings in "Microsoft Teams", choose "Devices" and set the "Speaker" option to the SoundConnect                                                                                                              |

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