User Manual

USB OTOSCOPE

Contents

English	02-17
Deutsch	18-33
中文-	34-49
日本峪	50-65

General Information

Manual:

rfYTI This operation manual allows you to work safely
^=4^ with camera. Please keep this manual for your
Information. Liability is void for any damage caused by
not following this manual.

Maintenance

To clean this device, use a damp cloth or mild, water
based cleaner.

Fully charge the battery before you leave this device idle for
long time to prevent damage to the battery.

Storage and transport:

Improper transport can harm the device. Always
transport this device in the provided box in order to prevent
damage.

Information on disposal:

Electronic equipment does not belong into domestic
waste, but must be disposed in accordance with the applicable
statutory provisions.


Description

This otoscope camera is a portable, handheld video camera with short focal length, it's compatible with iPhone, iPad, Android devices, PC and Tablet. The high resolution camera provides great video quality, there are 6 adjustable LED lights on the tip of the camera which makes the light path and increase the visual field of view. It can be used to view the tympanic membrane and external ear canal to diagnose outer and middle ear pathologies.


Safety

- Please use this device with an disinfected adapter
- Clean the surface of the device with 75% alcohol solution before contacting skin.
- Disinfect the adapters with saturated steam for 10 minutes at 134°C.
- This appliance is not intended for use by persons with limited physical, sensory or mental capabilities.
- Keep bystanders, children away while operating the tool.

6.12mm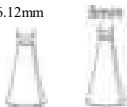

Select an adapter for your use, put it onto the head of the camera. Please notice that when you use it as an otoscope, it should only be used to examine ear when an adapter is fitted.

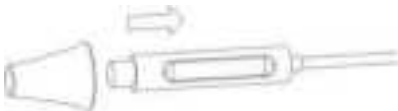
Clean the surface of the device with 75% alcohol solution before contacting skin.

Disinfect the adapters with saturated steam for 10 minutes at 134r.

Functions

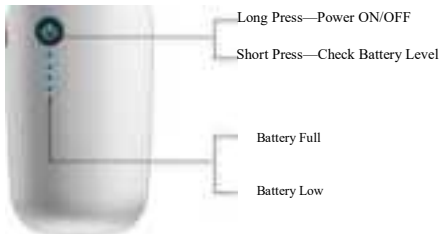Functions

Download the App

Search for "WiFi image" on App Store or Google Play Store, download and install the app.

WiFi image

Download on the
W AppStore

Start to use

Step 1:

Plug in the USB camera

Step 2:

Long Press the power button 0 to power on the device (Make sure the battery level is high)

Step 3:

Enter Wi-Fi settings on your phone and connect your phone to WiFi_Endoscope XXXXXX.

1 Notice: Please ALLOW the appto use data if you see the message as below

Run the application

Take
Photo

Record
Video Files

Captured Settings Full
Screen

- 1 Change the resolution of the camera

1. Enter Settings
2. Select the resolution
3. Click on "Set Resolution"

640X480
1280X720

Software Function

Use the USB otoscope alone

PC: ©Windows ② Mac

Please install suitable APP for your devices.

----- **Windows** -----

You can choose any of the following methods.

@ Only for Windows 10: Plug the USB otoscope to your computer.

Search for application named "Camera" in the start menu and run it, click on the switch icon if you have more than one camera on your PC.

Plug the USB otoscope to your computer. ② Download and install "Endoscope Camera". Plug the USB otoscope to your computer. Enter setting and select camera in the "Device list". Then you can see the video stream in the "Preview

Area".

!^You can only run one camera application when using the camera.

Video Record Button

Preview Area

Switch
Window

Size

Captured
Files List

R4CWd_0S

RwonLO?

总总总養. Ci. 2
McE* IUw4 II Rccrd_12

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Settings

Software for PC users:

Download:

<http://bit.ly/2BZwTYy>

A Please select “Trust this application” if you see warnings from your Antivirus software, your software is guaranteed to be safe.

③ Download and install “Amcap”

Download: <http://gto.so/amcap.zip>

Please make sure to select the right device name in the menu.
Plug the USB otoscope to your computer.

Step 1: Click “Devices” in the menu, choose^KTeslong Camera”.

Step 2: Click “Options” in the menu, choose “Preview”.

Mac

You can use Photo Booth or QuickTime Player to work with the camera, please make sure you select the right camera in the menu.

For Photo Booth, Please choose the device
"Teslong Camera" / "USB2.0 PC Camera" in the setting

"Camera", otherwise it will open the computer camera.

Q For
QuickTime

Player,

ording",you can also click the
setting button to change camera
and image quality.

click "File",
choose "New Movie Rec

Charge the battery

Only use chargers marked: Output 5V 500mA (or 1A, 2A). If you do not know the voltage and current of your charger, use the USB cable to charge the device with your computer. The camera charges to 100% in approximately 3 hours under 5V 500mA.

Technical Data

I Description

Data

Network Standard

IEEE 802.11 b/g/n

Antenna

Built-in IPEX antenna

Operating Frequency

2.4GHz

Data Rate

Up to 150Mbps

Camera

Interchangeable USB Type

Image Sensor

CMOS

Camera Light Source

adjustable high-intensity LEDs

Operating Temperature

32° to 113° F (0° to 45° C)

Battery Type

900mAh Li-polymer Battery

Battery life

1 hour

Power Input

DC 5V 500mA

Certifications

CE FCC RoHS

Safety Information

When operating your equipment:

I WARNING: This product is not intended for use in patient health care environments unless specifically designated. **This product is not designed for use in flammable or explosive environments.**

! _ WARNING: Do not operate this equipment with any cover* (including bezels, panels, housings, etc) removed.

^^ WARNING: Do not use this equipment in a wet environment. Protect the equipment **from** liquid **intrusion**.

- Do not use damaged equipment, including exposed, frayed, or damaged power cords.
- Disconnect your device and all peripherals from any wall connection during an electrical (lightning) storm or when you **will** be away for extended periods.
- Do not attempt to service the equipment yourself.
- Contact us if your equipment does not operate normally.

Power Safety

When connecting your equipment to a power source:

- Check the voltage rating before you connect this equipment to an electrical outlet.
- Use only branded AC adapter approved for use with this device.
- Place the AC adapter and the this equipment in a ventilated area, such as desk top or on the floor, when you use it to charge the battery.
- The AC adapter and this equipment may become hot during charge operation.

Troubleshootin

- * **No image or defects, noise in the image**

- * Check the Wi-Fi settings on your smartphone, make sure it has connected to the device.

- * Check the USB camera, make sure it's connected to the device correctly and tightly.

- * Disconnect and reconnect the camera.

- * Close the app and open it again.

- * Check the battery level, recharge the battery if the power indicator is red

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FCC Warning Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Radiation Exposure Statement

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.