Sleep Tracking Pad  
P300

User Manual

Sleepace

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
09

01. Product Introduction

P300 is used to track your sleep data including heart rate, respiratory rate, turning over, sleep cycle etc and provides a comprehensive sleep analysis and sleep advice to help improve your sleep quality.

02. Device Description

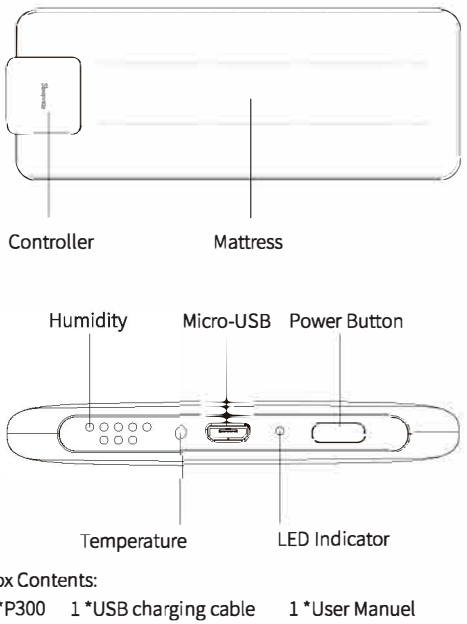
03. Operation Guide

Step1: Download the "Comfort Sleep" APP and register an account, Connect the P300 to APP.  
There are two ways to download the app.  
1. Scan the QR code.



2. Search "Comfort Sleep" in the App Store or Google Play for download. You will need a smart phone with Bluetooth 4.0 that supports iOS 8.0 and later or Android 4.3 and later.

Step 2: Connect P300 to APP.  
Make sure the controller and your smart phone are switched on and Bluetooth is activated on your smart phone. Follow the instructions given by the app to connect the device.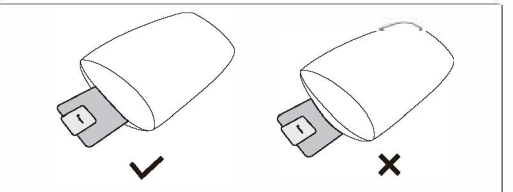


It is requested that the product should be put inside the pillow case. Please do not put it under or outside the pillow case.

11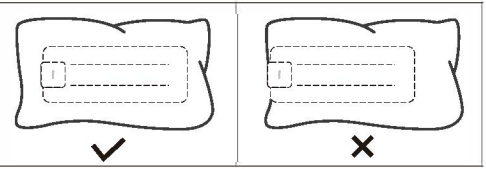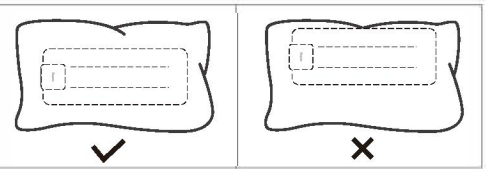

Please put the product exactly in the middle of the pillow.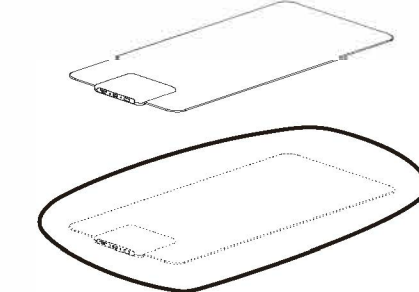

1. Fully charge the controller till the green LED indicator is ON.
2. Disconnect the charging cable and the controller.
3. Power on the controller by pressing the power button for 3 seconds. Connect P300 with APP and start monitoring.
4. Put P300 into the pillow.

Step 3: Sleep Monitoring  
Click "Start Sleep" on the homepage of APP to start sleep monitoring. The device will keep recording if leave bed shortly (e.g. going to the bathroom). If leave bed for more than 60 minutes, the device will stop monitoring automatically, you may have to click "Start Sleep" again.

Step 4: Sleep Report  
Click "Get Up" to stop sleep monitoring when you get up, the recorded data will be transferred to the App automatically with a comprehensive sleep report as well as your sleep quality analysis and personal sleep guidance. If forget to click "Get Up" via App, the device will stop the device will stop sleep monitoring. Please connect the device with App again to sync sleep data for sleep report.

04. LED Indicator

Status	Display
Power On (Press the Power button for 3 seconds)	Green LED indicator is ON for 3 seconds
Power Off (Press the Power button for 3 seconds)	Red LED indicator is ON for 3 seconds
Low Battery	Red LED indicator flashes every 2 seconds, please charge the device immediately
Charging Device	Red LED indicator is on constantly
Charging Completed	Green LED indicator stays ON

13

05. Technical Specifications

Controller Size	90mm*85*12mm
Product Size	535mm*200mm*12mm
Weight	170g
Communication Mode	Bluetooth 4.0
Bluetooth Transmission Range	max. 10 meters
Battery Life	Up to 2 weeks
Compatibility	Bluetooth 4.0 and up, iOS8.0 or later, Android 4.3 or later.
Power Supply	Re-chargeable Lithium-ion-battery 3.7V, 900mAH. Charging current 5.0V 1A


06. Precautions

Please note, this device is not a medical device and cannot be used to prevent sudden death. Users shall not be entirely dependent on this device and neglect the care of those persons under their care or being monitored.

14

07. Safety Notice

- Do not excessively bend this product. The build-in sensor may be damaged.
- Please keep the product away from high-temperature equipment, and do not use the product when using an electrical blanket.
- Do not touch the metal end of power plug when plugging into power socket.
- The sensor belt is nonconductive, safe to users.
- Please power the device off if for a long term non-using.
- For children and the elders, please use the product with the involvement of a guardian.
- Do not use the product during charging process
- The device must not be used close to any source of vibration (e.g. washing machines, loud music etc) as the vibrations could affect the functions of the device.
- NOTE: DO NOT disassemble or replace battery.

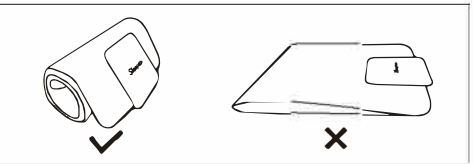When you pack and store the product, please roll it into a cylinder. Please do not fold it as it will damage the sensor.

08. Warranty

Please contact your dealers for warranty service. For more information about the product, warranty or service, please go to www.sleepace.com.

15

FCC Statements:

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications or changes to this equipment. Such modifications or changes could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:  
-Reorient or relocate the receiving antenna.  
-Increase the separation between the equipment and receiver.  
-Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.  
-Consult the dealer or an experienced radio/TV technician for help.  
FCC ID: 2ADIOP300

16

RF Exposure Information and Statement  
This equipment complies with CE and FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance of 20 cm between the radiator and your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Notice:  
The suitable temperature for the product and accessories is 0°C-40°C.  
Observe the national local regulations in the location where the device is to be used.

Frequency Band	Max output Power
BLE (2402-2480MHz)	0 dbm

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction. Federal Communication Commission (FCC Radiation Exposure Statement Power is so low that no RF exposure calculation is needed.

Hereby, Shenzhen Medica Technology Development Co., Ltd declares that the radio equipment type sleep tracker is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: <http://www.sleepace.com>

Shenzhen Medica Technology Development Co., Ltd.  
2F Building A, Tongfang Information Harbor, No. 11, East Langshan Road, Nanshan District, Shenzhen, China

17

Sleepace

Shenzhen Medica Technology Development Co., Ltd.

Web: www.sleepace.com  
Add: 2F Building A, Tongfang Information Harbor, No. 11, East Langshan Road, Nanshan District, Shenzhen, China

V1.1