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EN

Book an Appointment

You can book a 1-on-1 setup appointment with one of our support agents at avantree.com/book-a-call

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

Table of Contents

Product Introduction (Earphones)	3
Product Overview	3
Button Functions	4
LED Indicators	4
Product Introduction (Table Mic & Charging Case 2-in-1)	5
Product Overview	5
Button Functions	6
LED Indicators	6
Charging the Earphones & Case	7
Use as Bluetooth TWS Earphones	8
Use as Sound Amplifiers with the Table Mic	9
Use a Mono Earphone	10
Specifications	11
FAQ	12
Contact Us	57

Product Introduction (Earphones)

Product Overview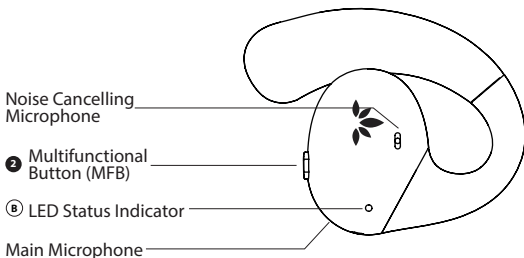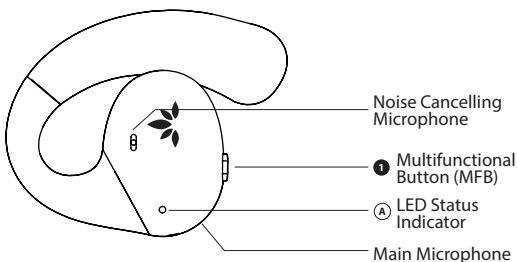

Button Functions



No.	Button	Functions
1 or 2	MFB on L / R earphone	Double-click to play / pause
		Click once to accept call
		Double-click to end call
		Press & hold for 2 seconds to reject call
		Press & hold for 5 seconds to turn on / off
		When using a single earphone, click once to cycle through 8 levels of volume
1	MFB on L earphone	When using both earphones, click once to decrease volume
2	MFB on R earphone	When using both earphones, click once to increase volume

LED Indicators

No.	Indicators & Voice Prompt	Status
A	Solid blue for 1 second	Power on
	Solid red for 1 second then off	Power off
	Solid blue for 5 seconds then off, "CONNECTED" voice indicator	Connected
	Solid red	Charging
	LED off	Fully charged
	Alternate red & blue flashes, "PAIRING" voice indicator	Pairing

Product Introduction (Table Mic & Charging Case 2-in-1)

Product Overview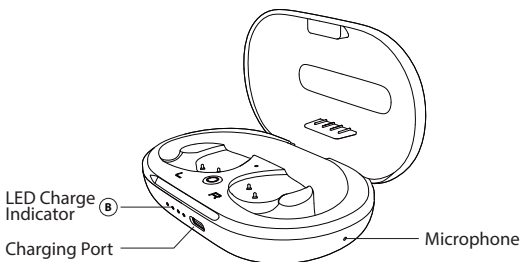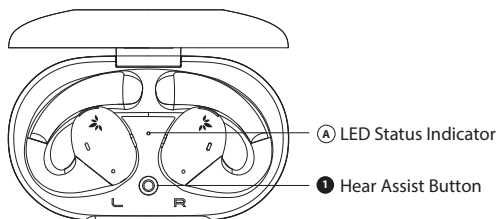

Button Functions

No.	Button	Functions
1		Press & hold for 1 second to turn Hear Assist mode on / off


LED Indicators

No.	Indicators & Voice Prompt	Status
A	Solid blue	Hear Assist on
	Solid red for 1 second then off	Hear Assist off
	Alternate red & blue flashes	Pairing mode
B	White flash every second	Charging
	Solid white on four LEDs	Fully charged

Charging the Earphones & Case


To charge the earphones:

Place the earphones into the case and then close the case. Solid red lights on the earphones indicate that charging is in progress; once the lights turn off, your earphones will be at full battery.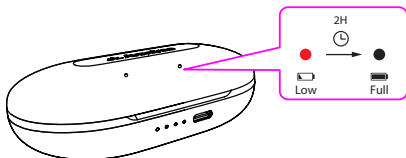

To charge the case:

If your case runs out of battery, when you open and then close it, the 4 LEDs on the case will flash white, and you'll need to recharge it in order to charge your earphones. Plug the provided USB C charging cable into the charging port, then connect the other end to a 5V/500mA-2A rated USB adapter (your phone charger or similar should work) or a PC USB port.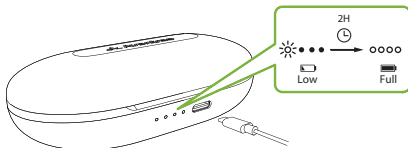


The case's LED charging indicators will flash white to indicate its charging status. At full charge, the four LEDs will turn solid white.

Use as Bluetooth TWS Earphones

STEP 1

When removed from the charging case, the earphones will enter PAIRING mode automatically, indicated by the LED flashing BLUE & RED.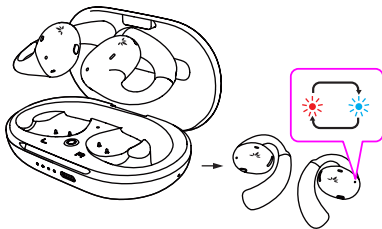

STEP 2

Turn on your device's Bluetooth and select "Avantree AcuEar Pebble" in the Bluetooth menu.


STEP 3

Keep the earphones close to your device and wait for 5-10 seconds or until they connect. Once they're successfully connected, both earphone's LED indicator will turn solid blue for 5 seconds then off.

For subsequent use, click the MFB button once to reconnect to the last-paired device or select "Avantree AcuEar Pebble" in your device's Bluetooth menu.


Use as Sound Amplifiers with the Table Mic

The table mic picks up the meeting participants' voices and sends them directly into your ears.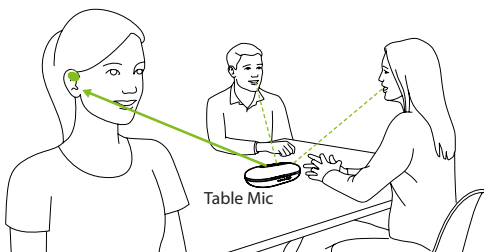


STEP 1

Long press the "Hear Assist" button for 1 second on the table mic; the table mic will enter hearing mode, indicated by the LED flashing BLUE.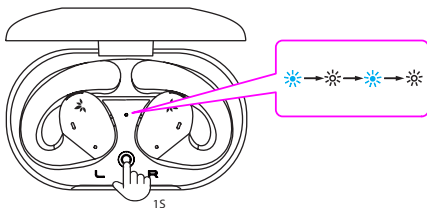

STEP 2

Wait for a few seconds. The table mic and the earphones will successfully connect, indicated by the case's LED turning solid BLUE.

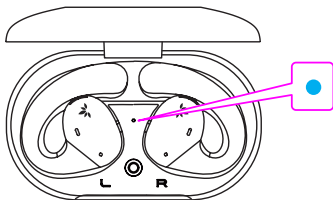NOTE: Your microphone typically works within a range of up to 10 meters / 33 feet to your sound amplifier. Note that human bodies and walls may reduce this distance. To maximize range, make sure you can see the AucEar Pebble microphone (i.e. line of sight).

Switch to Bluetooth Mode

Press and hold the “Hear Assist” button on the table mic for 1 second to turn it off. The earphones will automatically switch to Bluetooth mode. Then you can select “Avantree AcuEar Pebble” in your device’s Bluetooth menu.

Use a Mono Earphone

The R / L earphone can be used separately. When you use a mono (single) earphone, you can tap the button once to cycle through 8 levels of volume.

EN Specifications

Earphones

BT version	v5.3
Audio codec support	LC3, aptX-adaptive, aptX, SBC
Play time	Up to 10 hrs
Playtime w/ charging case	Up to 35 hrs
Charging time	2hrs
Drivers	16.5mm
Impedance	22Ω
Weight	26g

Table Mic & Charging Case

Bluetooth version	v5.3
Audio codec support	LC3
Play time	Up to 56 hrs
Charging time	2hrs
Weight	78g

FAQ

Below are some of the most common questions people have during setup.

The earphones can't connect with the table mic.

- Enter the earphones into pairing mode - Put the earphones into the case and turn off Bluetooth on your mobile phone. The earphones will enter pairing mode automatically, indicated by the LED flashing BLUE & RED.
- Enter the case into pairing mode - While the case is off, press and hold the button on the case for 5 seconds or until the LED flashes BLUE & RED.
- Keep the earphones and case together and they will connect with each other.

Only hearing out of one earphone / one side sound

How to adjust volume of the earphones?

You can find a full FAQ list at avantree.com/support/acuear-pebble, or contact us directly for additional assistance.

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Contact Us

Support Tickets avantree.com/submit-a-ticket

Support Email support@avantree.com

Support Numbers

US	+1 800 232 2078	9am-6pm PST	Mon-Fri
CA	+1 800 232 2078	9am-6pm PST	Mon-Fri
UK	+44 208 068 2023	9am-6pm GMT	Mon-Sat
DE	+49 322 2109 7297	11am-9pm CET	Mon-Sat
FR	+33 17 634 0312	11am-9pm CET	Mon-Sat
IT	+39 069 4803 330	9am-6pm CET	Mon-Sat
ES	+34 93 1786 261	9am-6pm CET	Mon-Sat
AU	+61 2 8310 9897	11am-7pm AEST	Mon-Fri

FAQ avantree.com/support/acuear-pebble

Product Registration See QR code on the product.

Manufacturer:
Avantronics Limited
Luohu District, Shenzhen 518000 China
+ 86 755 8228 5022 support@avantree.com

US Importer:
Avantree Corporation
San Jose, California 95119 USA
+1 800 232 2078 support@avantree.com

UK Importer:
Avantree Limited
Watford, WD180 0HQ UK
+44 20 8068 2023 support@avantree.com

Servicer in Europe:
Gizmex GmbH,
Sternstr. 67, 40479 Dusseldorf, Germany
+49 322 2109 7297 support@avantree.com

Model No.: PHA-126M
Dispose of the packaging and this product in
accordance with the latest provisions.
Z-PKQG-126M-V1

FCC Requirement

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.