



## PURELL® Point of Care Compliance Dispenser

---

### How does the POC Compliance Bracket work?

Newly designed SmartLink™ enabled bottle holder with a push paddle registers a hand hygiene event within the GOJO Activity Monitoring System. The blue LED light provides visual confirmation that the hand hygiene event was recorded. The LED light is designed to only illuminate when the user has depressed the push paddle far enough to receive a full efficacious dose.

Robust Mount Solutions provide reliable installation wherever needed within the patient zone to provide quick and easy access to PURELL Hand Sanitizer, improving Hand Hygiene compliance. Main areas for placement include: Foot of Bed, Horizontal and Vertical Pole/Mobile Equipment, Table/Equipment/Cart Edge, Vertical Wall/Surface Mount and Tabletop.

This dispenser/holder accommodates the 12oz bottle of PURELL(R) Advanced Instant Hand Sanitizer Gel.

### Is the Compliance Rate calculated?

Yes:

At the room level, if the Point of Care dispenser stays within the room, all the events captured are associated to the in/out opportunities of the room.

If the Point of Care dispenser moves from room to room, but stays within the unit, all events captured are associated to the in/out opportunities of the unit.

### What if the POC Compliance Bracket leaves the unit?

When the dispenser leaves the range of the AMS network/gateway, the electronics within the Point of Care dispenser will still continue to collect and store up to 200 events locally. Then, when the dispenser returns to the gateway range, the events will be pushed all at once to the gateway. Based on this, we recommend developing a process to ensure the POC dispensers do not leave the unit.


# PURELL® Point of Care Compliance Dispenser User Instructions

## PURELL® Bottle Installation

1. Remove battery tab



2. Unlock the pump by rotating the nozzle counter clockwise as shown.

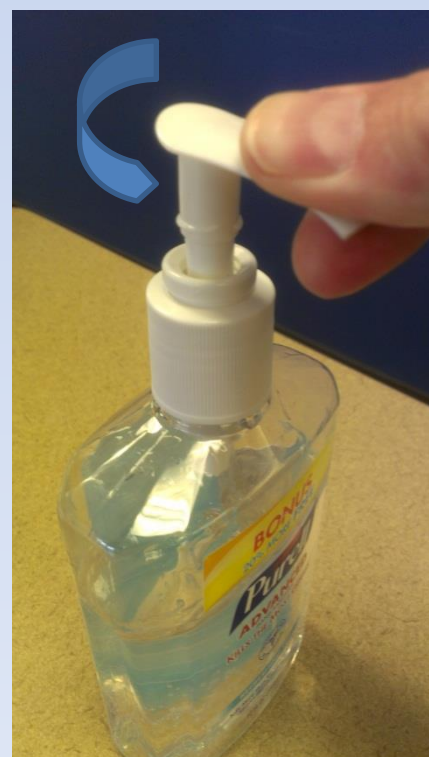

# PURELL® Point of Care Compliance Dispenser User Instructions

## PURELL® Bottle Installation – Un-Locked Position

3. Slide Locking Tab into un-lock position as shown



4. Squeeze the Collar and pull to open as shown



# PURELL® Point of Care Compliance Dispenser User Instructions

---

5. Insert bottle sliding bottom of bottle into tray first, then insert bottle into collar.



6. Push collar closed as shown.



# PURELL® Point of Care Compliance Dispenser User Instructions

## PURELL® Bottle Installation – Locked Position

1. With Collar open, slide Locking Tab into locked position as shown



2. Insert bottle sliding bottom of bottle into tray first, then insert bottle into Collar.





# PURELL® Point of Care Compliance Dispenser User Instructions

---

3. Push Collar closed as shown.



4. Insert pen tip into lock hole to release Collar, then squeeze and pull to open.



# PURELL® Point of Care Compliance Dispenser User Instructions

## SANITIZE INSTRUCTION

1..Depress Paddle until  
blue light appears.

2. Blue light indicates that  
a compliance event has  
occurred.



# PURELL® Point of Care Compliance Dispenser

## User Instructions

---

### Cover Plate Removal / Installation

1. Depress Cover Plate release in bottom of dispenser



2. Slide the Cover Plate down as shown.



3. To install the Cover Plate, slide the plate up until it clicks and locks into place.



5830-10-868

## PURELL® Point of Care Compliance Dispenser Installation Instructions

---

Determine the location and orientation for mounting the dispenser


### Foot of Bed Mounting Location



## PURELL® Point of Care Compliance Dispenser

---

1. Measure the width of the Bed Footboard.

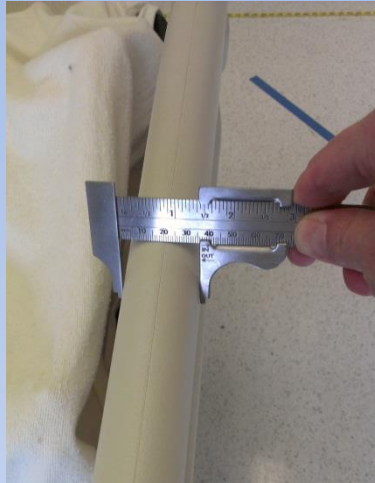

*For widths from  
½" to 1-1/4"  
(12mm to 31mm)  
use part number **5835-500***




*For widths from  
1-1/4" to 2"  
(31mm to 50 mm)  
use part number **5840-500***



## PURELL® Point of Care Compliance Dispenser

---

2. Slide the appropriate Bracket down onto the Bed Footboard.

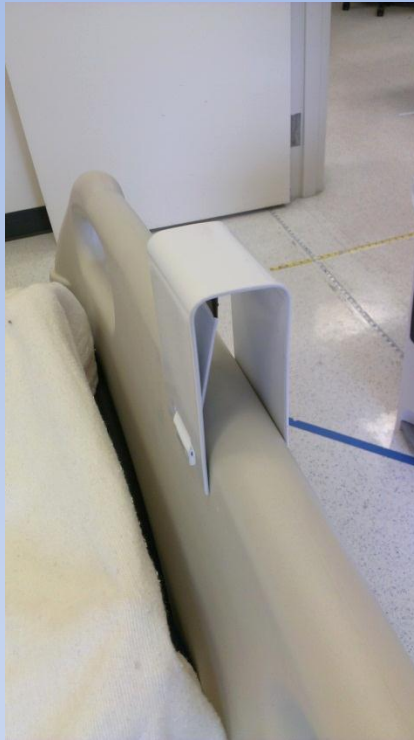

3. Slide the Dispenser with bottle onto the bracket. Pull up on the dispenser to ensure it's locked.



### Table /Cart Mounting Location

1. Use C-Clamp for table thicknesses up to 1-1/2".



2. Mount to table as shown.  
Hand tighten



## PURELL® Point of Care Compliance Dispenser

---

3. Slide the Dispenser onto the clamp as shown until it clicks. Pull on the dispenser to ensure it's locked in place.





### IV Pole Mounting Location

1. Install onto IV pole as shown



## Cleaning /Disinfecting Instructions

### Recommended Disinfecting Solutions or Wipes <sup>1</sup>

- K, Ethyl or isopropyl alcohol (70-90%)
- L, Sodium hypochlorite (5.25-6.15% household bleach diluted 1:500 provides >100 ppm available chlorine)
- M, Phenolic germicidal detergent solution (follow product label for use-dilution)
- N, Iodophor germicidal detergent solution (follow product label for use-dilution)
- O, Quaternary ammonium germicidal detergent solution (follow product label for use-dilution)

Follow manufacturer's usage guidelines for contact time.

1. Center for Disease Control (CDC)

“Guideline for Disinfection and Sterilization in Healthcare Facilities, 2008”

Table 1, Smooth Hard Surfaces, Non-Critical Items

# FCC Statement

- THIS DEVICE COMPLIES WITH PART 15 OF THE FCC RULES. OPERATION
- IS SUBJECT TO THE FOLLOWING TWO CONDITIONS: (1) THIS DEVICE MAY NOT CAUSE HARMFUL INTERFERENCE, AND (2) THIS DEVICE MUST
- ACCEPT ANY INTERFERENCE RECEIVED, INCLUDING INTERFERENCE THAT MAY CAUSE UNDESIRE OPERATION.
- NOTE: THE GRANTEE IS NOT RESPONSIBLE FOR ANY CHANGES OR MODIFICATIONS NOT EXPRESSLY APPROVED BY THE PARTY RESPONSIBLE FOR COMPLIANCE. SUCH MODIFICATIONS COULD VOID THE USER'S AUTHORITY TO OPERATE THE EQUIPMENT.