




Press: **1** **2** **3**

2. Unlock the screen by pressing each front button from left to right to unlock the remote screen.



Once unlocked, screen will stay active for 2 minutes before going into sleep mode.

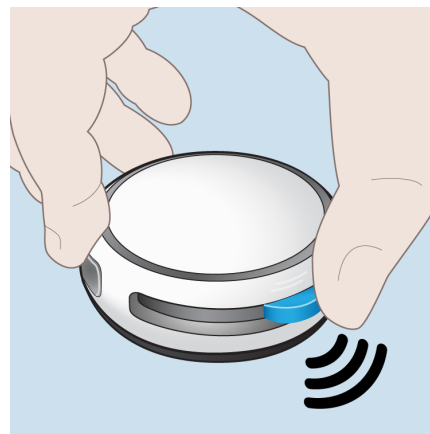

3. Place the pump and remote within 2 in (50 mm) of each other during the pairing process.
4. If your pump's serial number does not show up on screen after a minute, **press and hold** the pump's **blue button** for **10 seconds** to activate pairing mode. Then, check the screen again for your pump serial number.



5. On the remote, use   to highlight the pump's serial number and press .



6. After selecting your pump serial number you will be prompted to confirm the selected pump. Press  to confirm.



A Pairing To... message displays.

If pairing fails, [see "Pairing Failed Attention Alarm" on page 166](#)



7. Upon successful pairing the *Paired Pump* message displays. Press .

Walkaway Attention Alarm

To help you avoid forgetting your remote, setting this feature will let you know that you may have left your remote behind. The *Walkaway Attention Alarm* feature can be set to go off between 1 and 120 minutes.

Turn ON Walkaway Attention Alarm:

1. **Press** the **side button** to wake up the remote.
2. Unlock   the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .
5. Press the down arrow  to **Walkaway Alarm:OFF**.
6. Press .



7. If needed, press the  to edit the *Walkaway Attention Alarm* minutes.
8. Press the up and down arrows   to select between 1 and 120 minutes.
9. Press  to accept your selection.



10. Press the front button under **ON** to turn *ON* the *Walkaway Attention Alarm* feature.

11. Press  to accept your selection.

A Sending request to the pump message displays.

Turn OFF Walkaway Attention Alarm:

1. **Press** the **side button** to wake up the remote.
2. Unlock   the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .
5. Press the down arrow  to **Walkaway Alarm:ON**.
6. Press .



7. Press the front button under **OFF** to turn *OFF* the *Walkaway Attention Alarm* feature.
8. Press  to accept your selection.

A *Sending request to the pump* message displays.

Screen Timeout

The *Screen Timeout* feature allows you to set the time before the remote's screen will sleep.

1. **Press** the **side button** to wake up the remote.
2. Unlock   the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .
5. Press the down arrow  to **Screen Timeout**.
6. Press .

7. Press the up and down arrows   to select one of four times:

- One minute
- Two minutes
- Five minutes
- Ten minutes



8. Press  to accept your selection.

Any time the remote has not been used and the programmed *screen timeout* minutes have passed, you will need to **press the side button** to wake up the remote.

Remote Audio

The *Remote Audio* feature controls the feedback and volume options for the remote.

Feedback Options

1. **Press the side button** to wake up the remote.
2. Unlock   the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .
5. Press the down arrow  to **Remote Audio**.
6. Press .





7. The **Feedback** option is already highlighted.
8. Press .
9. Press the up and down arrows   to select one of three feedback options:
 - Audio and Vibration
 - Audio Only
 - Vibration Only
10. Press  to accept your selection.

Volume Options

1. **Press the side button** to wake up the remote.
2. Unlock   the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .
5. Press the down arrow  to **Remote Audio**.
6. Press .





7. Press the down arrow  to the **Volume** option.
8. Press .



9. Press the up and down arrows   to select one of five volume levels.
10. Press  to accept your selection.

Change Rate Passcode

The *Change Rate Passcode* is *Off* by default. When turned *On* a passcode will be required to change the rate.

Turn ON Change Rate Passcode

1. **Press** the **side button** to wake up the remote.
2. Unlock  the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .



5. Press the down arrow  to **Passcode: OFF**.
6. Press .



7. Press the front button under **ON**.



8. Use the up and down arrows   to scroll through the first digit selection.



9. When you have the desired number press the right arrow .
10. Repeat [Step 8](#) and [Step 9](#) for the remaining three digits.
11. Confirm the passcode by repeating [Step 8](#) and [Step 9](#) to enter your new four digit passcode.



- A message displays that the *Passcode is Enabled*.
12. Press  to return to the *Setup Menu*.

Turn OFF Change Rate Passcode

1. Press the **side button** to wake up the remote.
2. Unlock  >> the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .
5. Press the down arrow  to **Passcode: ON**.
6. Press .
7. Press the front button under **OFF**.



8. Use the up and down arrows   to enter the first digit of your current passcode.
9. When finished entering the number press the right arrow .
10. Repeat [Step 8](#) and [Step 9](#) for the remaining three digits.



A message displays that the *Passcode is Disabled*.

11. Press  to return to the *Setup Menu*.

Forgot My Change Rate Passcode

If you have forgotten your change rate passcode, you will need to follow the steps to factory reset your remote and pair to spare remote. [See "Factory Reset" on page 140.](#)

Airplane Mode

If you are asked to turn off all electronic devices during a commercial flight, follow the steps to put the device into airplane mode.

1. **Press** the **side button** to wake up the remote.
2. Unlock   the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .
5. Press the down arrow  to **Airplane Mode**.
6. Press .





7. Press  to enable airplane mode.


A Sending request to the pump message displays.



A Press and hold pump button to exit message displays.



The pump will continue to deliver medication even when the remote is in airplane mode.

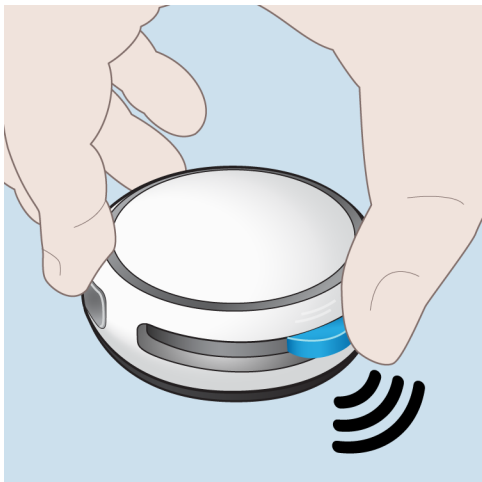

When the use of electronics is cleared during a commercial flight you can exit airplane mode.

8. You will need to **press and hold the blue pump button** until after you hear the *four-tone* delivering sound to exit airplane mode.



9. Verify the remote displays *Delivering* on the *Home* screen.

Factory Reset

A *Factory Reset* should not be performed unless directed by your specialty pharmacist.

1. **Press the side button** to wake up the remote.
2. Unlock  the screen by pressing each front button from left to right.
3. Use the down arrow  to scroll to **Setup**.
4. Press .
5. Press the down arrow  to scroll to **Factory Reset**.
6. Press .

You will need to select whether you want to reset the remote or the pump.

See "Remote Factory Reset" below.

See "Pump Factory Reset" on the facing page.

Remote Factory Reset



1. Press  to factory reset the remote.



*A **Reset the Remote to defaults? Pump will be disconnected!** message displays.*

2. Press  to confirm or  to cancel.

*A **Remote will restart when complete message displays.***

The remote will display *Searching* until the pump is paired to the remote. *See "Pair to a Spare Remote" on page 123.*

Pump Factory Reset



1. Press the down arrow  to highlight **Pump**.
2. Press  to factory reset the pump.