

Let's Get Started



Download the Owlet Pregnancy app



For more information, visit: [owletcare.com/band-manual](https://www.owletcare.com/band-manual)

What's in the box



Band



2 Size Panels



Medallion



Charging Station



Power Adapter

Warnings / Important Safety Information



WARNING: The Owlet Band is not a medical device, It is not intended for use as a medical device or to replace a medical device. It does not and is not intended to diagnose, cure, treat, alleviate or prevent any disease or health condition or investigate, replace or modify anatomy or any physiological process.

The Owlet Band is meant for healthy, pregnant women who are at least 20 weeks pregnant. It is not intended for use with high risk pregnancies.

- Do not use the Owlet Band as a diagnostic tool.
- Do not use the Owlet Band if your medical professional recommends the use of a medical device
- Do not use the Owlet Band as an excuse for unsafe pregnancy habits.
- Do not use the Owlet Band as a replacement for or in lieu of medical care.

THE OWLET BAND IS AN AID. IT IS NOT A SUBSTITUTE FOR MEDICAL SUPERVISION.

Remember: *This product is only intended to assist you in tracking your wellbeing during your pregnancy and is not intended to replace you as a caregiver. You are responsible for your own and your baby's health and wellbeing and following safe pregnancy practices, health and care guidelines.*

Warranty

Owlet proudly stands behind its products. Our warranties are found at: <https://owletcare.com/pages/terms>.

Terms and Conditions

The use of Owlet's products is subject to the terms and conditions found at: <https://owletcare.com/pages/terms>. By using an Owlet product, you agree to those terms.

For contact information specific to your region, please visit <https://support.owletcare.com>

Safety

Your safety and the safety of others are very important to us. There are many important safety messages in this manual and in the Owlet Baby Care App. Always read and obey all safety messages.



Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment. Please ONLY use the supplied power adapter. Failure to do so may void our limited warranty. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio

frequency energy and, if not installed and used in accordance with the manufacturer's instructions, may cause interference harmful to radio communications.

IC : 21386-SS1B6-OBSS1B, 21386-OB51B This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interferences, and (2) this device must accept interferences, including interferences that may cause undesired operation of the device.



Owlet Baby Care declares that the Owlet Band is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: <https://owletbabycare.co.uk/pages/ce-compliance>

2402.0 - 2480.0 (0,016 Output Watts)

2412.0 - 2462.0 (0,204 Output Watts)-Base Station

2402.0 - 2480.0 (0,018 Output Watts)-Sensor



Owlet is committed to meeting the requirements of directive 2012/19/EU on waste electrical and electronic equipment (WEEE). Please visit our website at owletcare.com to find out more about the proper disposal of Owlet products in the EU and UK.

Need Help?

If you have questions about the Owlet Band, visit our Help Center at support.owletcare.com