

Pet health scale

USER Manual



Model : ZBZ_PHS-001

Product Specifications

Size : 260*260*25mm

Power : 1.5V AAA X 4

Max Weight : 180Kg

Min Weight : 3Kg

Resolution : 0.1 Kg

Total Weight : 1.2Kg

User Guide

1. Open the bottom battery cover, tear open the insulating transparent plastic mold of the battery, and put in 4 AA batteries correctly pool.

2. Please put the product on a flat ground, after the display is turned off, stand with your feet in the middle, read Take the measured value.

3. After the measurement, make sure to stand firmly, slowly leave the scale body, and store the scale body after use.

There is a dry and cool place, if it is not used for a long time, please remove the battery to prevent the battery for a long time. If it is not used, the battery fluid will flow out and damage the scale body.

Note

1. The minimum weight to trigger the startup is 6Kg, the minimum 3Kg after startup, and the resolution is 100g.
2. Pregnant women, children, the elderly. It needs to be used in a safe situation.
3. Please do not use it in a humid environment. Please make sure the surface of the scale is dry when using money. No water.
4. When using it for the first time, please wait until the LED is completely off before using it. Otherwise, the measurement data may be inaccurate.
5. After the last measurement, wait for the LED to turn off for about 3 seconds before performing a measurement.
6. This product uses APP. Multiple user data can be obtained under the same account.
7. This product is used in precision instruments. There are weight sensors at the four corners. It is easy to damage. Please handle it with care.
8. When not in use for a long time, please take out the battery to prevent damage to the product.
9. This product is not waterproof. There is water on the scale surface and it can be easily wiped off. The bottom battery box and connection are not waterproof. Please be careful not to soak the product in water.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Federal Communication Commission Interference Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Federal Communication Commission (FCC) Radiation Exposure Statement
This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. But user must follow the specific operating instructions for satisfying RF exposure compliance.

FCC Caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.
This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 20cm between the radiator & your body.