

2

UP

spinning to tighten the fixing ring

Spinning to tighten the ear spoon

When use silicone ear spoon, connect the ring first, and then connect the silicone ear spoon

When the battery is low, the red light flashes, Please charge in time

3

4

5

6

7

Close the cover , Turn off

DOWN

Power off, light off

2

Product Details

Product model and accessories

WS1

Type - c cable

Alcohol swab

Metal earwax spoon

The silicone earwax spoon

Silica gel ear spoon adaptor

Feather brush

2

Product Specifications

Common problems and solutions

Frequently Asked Questions	Solution
No response after startup:	Please use the supplied charging cable to charge for 1 hour Turn off and restart the otoscope
Application Crash:	Uninstall the existing APP. Then scan the QR code install again
Power indicator flashes in red:	The battery is low, please use the supplied charging cable to charge, the charging time is 1 hour
After connect to the APP, the camera cannot display real-time image correctly:	Disconnect the phone from WiFi, restart the WiFi, and then reconnect to WiFi
WiFi connection is unstable and sometimes disconnect in seconds:	Restart the otoscope and reconnect the APP Some phones require authorization
If the above operation fails to solve the problem, please contact customer service, we will further provide you with practical solutions	

2

3

Take the following measures:

CE

FC

RoHS

Made in China

