User's Manual

Blood Pressure Monitor

Model: BP2, BP2A, BP2B, BP2C, BP2V, BP2W



2. The Basics

This manual contains the instructions necessary to operate the product safely and in accordance with its function and intended use. Observance of this manual is a prerequisite for proper product performance and correct operation and ensures patient and operator safety.

1.1 Safety

Warnings and Cautionary Advice

- Before using the product, please ensure that you have read this manual thoroughly and fully understand corresponding precautions and risks.
- This product has been designed for practical use, but is not a substitute for a visit to the doctor.
- This product is not designed or intended for complete diagnosis of cardiac conditions. This product should never be used as a basis for starting or modifying treatment without independent confirmation by medical examination.
- The data and results displayed on the product are for reference only and cannot be directly used for diagnostic interpretation or treatment.
- Do not attempt self-diagnosis or self-treatment based on the recording results and analysis. Self-diagnosis or self-treatment may lead to deterioration of your health.
- Users should always consult their physician if they notice changes in their health.
- We recommend not to use this product if you have a pacemaker or other implanted products. Follow the advice given by your doctor, if applicable.
- Do not use this product with a defibrillator.
- Never submerge the product in water or other liquids. Do not clean the product with acetone or other volatile solutions.
- Do not drop this product or subject it to strong impact.
- Do not place this product in pressure vessels or gas sterilization product.
- Do not disassemble and modify the product, as this could cause damage, malfunction or impede the operation of the product.
- Do not interconnect the product with other product not described in the Instruction for Use, as this could cause damage or malfunction.
- This product is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the product. Children should be supervised around the product to ensure they do not play with it.
- Do not allow the electrodes of the product to come into contact with other conductive parts (including earth).
- Do not use the product with persons with sensitive skin or allergies.
- DO NOT use this product on infants, toddlers, children or persons who cannot express

themselves.

- Do not store the product in the following locations: locations in which the product is exposed to direct sunlight, high temperatures or levels of moisture, or heavy contamination; locations near to sources of water or fire; or locations that are subject to strong electromagnetic influences.
- This product displays changes in the heart rhythm and blood pressure etc. which may have various different causes. These may be harmless, but may also be triggered by illnesses or diseases of differing degree of severity. Please consult a medical specialist if you believe you may have an illness or disease.
- Vital signs measurements, such as those taken with this product, cannot identify all diseases.
 Regardless of the measurement taken using this product, you should consult your doctor immediately if you experience symptoms that could indicate acute disease.
- Do not self-diagnose or self-medicate on the basis of this product without consulting your doctor.
 In particular, do not start taking any new medication or change the type and/or dosage of any existing medication without prior approval.
- This product is not a substitute for a medical examination or your heart or other organ function, or for medical electrocardiogram recordings, which require more complex measurements.
- We recommend that you record the ECG curves and other measurements and provide them to your doctor if required.
- Clean the product and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the product or cuff.
- Avoid tightly folding the cuff or storing the hose tightly twisted for long periods, as such treatment may shorten the life of the components.
- The product and cuff are not water-resistant. Prevent rain, sweat and water from soiling the product and cuff.
- To measure blood pressure, the arm must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the arm. This condition will appear especially when measurement is repeated successively. Any pain, numbness, or red marks will disappear with time.
- Too frequent measurements can cause injury to the patient due to blood flow interference.
- Consult your physician before using this product on an arm with an arterio-venous (A-V) shunt.
- Consult your physician before using this monitor if you have had a mastectomy or lymph node clearance.
- The pressurization of the CUFF can temporarily cause loss of function of simultaneously used monitoring product on the same limb.
- Consult your physician before using the product if you have severe blood flow problems or blood disorders as cuff inflation can cause bruising.
- Please prevent that operation of the product results in prolonged impairment of the circulation

of the blood of the patient.

- Do not apply the cuff on an arm with another medical electrical equipment attached. The equipment may not function properly.
- People who have a severe circulatory deficit in the arm must consult a doctor before using the product, to avoid medical problems.
- Do not self-diagnose the measurement results and start treatment by yourself. Always consult your doctor for evaluation of the results and treatment.
- Do not apply the cuff on an arm with an unhealed wound, as this can cause further injury.
- Do not apply the cuff on an arm receiving an intravenous drip or blood transfusion. It may cause injury or accidents.
- Remove tight-fitting or thick clothing from your arm while taking a measurement.
- If the patients' arm is outside the specified circumference range that may result in incorrect measurement results.
- The product is not intended for use with neonatal, pregnant, including pre-eclamptic, patients.
- Do not use the product where flammable gases such as anesthetic gases are present. It may cause an explosion.
- Do not use the product in the area of HF surgical equipment, MRI, or CT scanner, or in an oxygen rich environment.
- The battery intended to be changed only by service personnel with the use of a tool, and replacement by inadequately trained personnel may result in damage or burn.
- The patient is an intended operator.
- Do not carry out the servicing and maintenance while the product is in use.
- The patient can safely use all the functions of the product, and the patient can maintain the product by carefully reading Chapter 7.
- This product emits radio frequencies (RF) in the 2.4 GHz band. DO NOT use this product in locations where RF is restricted, such as on an aircraft. Turn off the Bluetooth feature in this product and remove batteries when in RF restricted areas. For further information on potential restrictions refer to documentation on the Bluetooth usage by the FCC.
- DO NOT use this product with other medical electrical (ME) equipment simultaneously. This may result in incorrect operation of the product and/ or cause an inaccurate blood pressure readings and/ or EKG recordings.
- Sources of electromagnetic disturbance may affect this product (e.g. mobile telephones, microwave cookers, diathermy, lithotripsy, electrocautery, RFID, electromagnetic anti-theft systems, and metal detectors), please try to stay away from them when making measurements.
- The use of accessories and cables other than those specified or provided by manufacture could result in increased electromagnetic emission or decreased electromagnetic immunity of the product and result in improper operation.
- Interpretations made by this product are potential findings, not a complete diagnosis of cardiac

conditions. All interpretations should be reviewed by a medical professional for clinical decision-making.

- DO NOT use this product in the presence of flammable anesthetics or drugs.
- DO NOT use this product while charging.
- Remain still while recording an ECG.
- The detectors of ECG have been developed and tested on Lead I and II recordings only.

2. Introduction

2.1 Device Description

The blood pressure monitor includes four models, BP2, BP2A, BP2B, BP2C, BP2V, BP2W, the difference of six models are shown as the table.

	BP2	BP2A	BP2B	BP2C	BP2V	BP2W
ECG	•	x	•	х	•	•
Heart Rate	•	x	•	x	•	•
Wifi	x	x	х	x	•	•
Colour	White	White	Dark green	Dark green	White	Black

Note: • Indicates that this function is available, × indicates that this function is not available.

2.2 Intended Use

The product is intended to measure, review and store adults' ECG data and blood pressure at home or in healthcare environment, does not contain auto-analysis function.

2.3 Contraindications

This product is contraindicated for use in ambulatory environments.

This product is contraindicated for use on aircraft.

The device is not intended for use with neonatal, pregnant, including pre-eclamptic, patients.

2.4 About the product

Product name: Blood Pressure Monitor

Product models: BP2, BP2A, BP2B, BP2C, BP2V, BP2W



- 1. LED screen
 - Display date, time and power status, etc.
 - Display ECG and blood pressure measurement process and results.
- 2. Start/ Stop button
 - Power On/ Off
 - > Power On:Press the button to power on.

- > Power off:Press and hold the button to power off.
- Press to power on the product and press again to start measuring blood pressure.
- Press to power on the product and touch the electrodes to start measuring ECG.
- 3. Memory button
 - Press to review historical data.
- 4. LED indicator
 - Blue light is on: the battery is being charged.
 - Blue light is off: the battery is full charged not charging.
- 5. ECG electrode
 - Touch them to start measuring ECG with different methods.
- 6. USB connector
 - It connects with charging cable.

2.5 Symbols

Symbol	Meaning
SYS	Systolic blood pressure
DIA	Diastolic blood pressure
•	Pulse beat
HR	Heart rate
	Battery indicator
	Charging
★	Application part type BF
	Manufacturer
IP22	Against ingress of solid foreign objects ≥12.5mm diameter, Against dripping(15° tilted)
8	Follow operating instructions
(MRI unsafe. Presents hazards in all MR environments as product contains strongly ferromagnetic materials.
((-))	Non-ionizing radiation
X	Symbol for "ENVIRONMENT PROTECTION – Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local authority or retailer for recycling advice".
CE1097	In conformity with medical device regulation (EU) 2017/745
EC REP	European Representative

3. Using the Product

3.1 Charge the Battery

Use the USB cable to charge the product. Connect the USB cable to a USB charger or to the PC. A fully charge will need 2 hours. When the battery charged fully the indicator will be blue.

The product works in a very low power consumption and one charge usually works for months. On-screen battery symbols which indicate the battery status can be seen on the screen.

Note: The product cannot be used during charging, and if choosing a third party charging adaptor, select one that complies with IEC60950 or IEC60601-1.

3.2 Measure Blood Pressure

3.2.1 Applying the arm cuff

1. Wrap the cuff around the upper arm, about 1 to 2 cm above the inside of the elbow, as shown.

Do not roll up shirtsleeve tightly



- 2. Place the cuff directly against the skin, as clothing may cause a faint pulse and result in a measurement error.
- 3. Constriction of the upper arm, caused by rolling up a shirtsleeve, may prevent accurate readings.
- 4. Confirm that the artery position mark is line up with the artery.

3.2.2 How to sit correctly

To take a measurement, you need to be relaxed and comfortably seated. Sit in a chair with your legs uncrossed and your feet flat on the floor. Place your left arm on a table so the cuff is level with your heart.



Note:

- The blood pressure can differ between the right arm and the left arm, and the measured blood pressure readings can be different. The Viatom recommends to always use the same arm for measurement. If the blood pressure readings between both arms differ substantially, check with your physician to determine as to which arm to use for your measurements.
- The time is about 5s required for the product to warm from the minimum storage temperature between uses until the product is ready for its intended use when the ambient temperature is 20°C, and the time is about 5s required for the product to cool from the maximum storage temperature between uses until the product is ready for its intended use when the ambient temperature is 20°C.

3.2.3 Measurement process

- 1. Press to power on the product and press again to start measuring blood pressure.
- 2. The product will automatically deflate the cuff slowly during the measurement, a typical measurement takes about 30s.
- 3. The blood pressure readings will scrolling appear in the product when the measurement finished.
- 4. The product will automatically release the cuff gas after the measurement is over.
- 5. Press the button to turn off the power after the measurement, then remove the cuff.
- 6. Press the memory button to review historical data. The blood pressure readings will appear in the product

Note:

- The product has an automatic power shut-off function, which turns off the power automatically in one minute after measurement.
- During the measurement, you should keep still and don't squeeze the cuff. Stop measuring when the pressure result appear in the product. Otherwise the measurement may be effected and the blood pressure readings may be inaccurate.
- The device can store maximum 100 readings for Blood Pressure data. The oldest record will be overwritten when the 101th readings is coming in. Please uploading data in time.

NIBP Measurement Principle

The NIBP measurement way is oscillation method. Oscillation measurement is using automatic inflator pump. When the pressure is high enough to block arterial blood flow, then it would deflate slowly, and record all the change of cuff pressure in the deflation process to calculate blood pressure based on certain algorithm. The computer will judge whether the quality of signal is accurate enough. If the signal is not accurate enough (Such as sudden movement or touch of cuff while measurement), the machine will stop deflating or re-inflate, or abandon this measurement and calculation.

The operating steps needed to obtain accurate routine resting blood pressure measurements for the condition hypertension including:

-- Patient position in normal use, including comfortably seated, legs uncrossed, feet flat on the floor, back and arm supported, middle of the cuff at the level of the right atrium of the heart.

-- The patient should be relaxes much as possible and should not talk during the measurement procedure.

- -- 5 minutes should elapse before the first reading is taken.
- -- Operator position in normal use.

3.2.4 Display Screen



- 1. Systolic blood pressure
- 2. Diastolic blood pressure
- 3. Pulse display

3.3 Measure ECG

3.3.1 Before using ECG

- Before using ECG function, pay attention to the following points in order to obtain precise measurements.
- The ECG electrode must be positioned directly against the skin.
- If your skin or hands are dry, moisten them using a damp cloth before taking the measurement.
- If the ECG electrodes are dirty, remove the dirt using a soft cloth or cotton bud dampened with disinfectant alcohol.
- During the measurement, do not touch your body with the hand with which you are taking the measurement.
- Please note that there must be no skin contact between your right and left hand. Otherwise, the measurement cannot be taken correctly.
- Stay still during the measurement, do not speak and hold the product still. Movements of any kind will falsify the measurements.
- If possible, take the measurement when sitting and not when standing.

3.3.2 Measurement process

1. Press to power on the product and touch the electrodes to start measuring ECG.

There are four methods for recording ECG by cablefree operation.

- Method A: Lead I, right hand to left hand
- Method B: Lead II, Right hand to left Leg
- Method C: Lead II, right hand to left abdomen
- Method D: Right hand to chest



2.Keep touching electrodes gently for 30 seconds.

3.When the bar if fully filled, the product will show the measurement result.

4. Press the memory button to review historical data.

Note:

- Do not press the product too firmly against your skin, which may result in EMG (electromyograph) interference.
- The device can store maximum 10 records for ECG data. The oldest record will be overwritten when the 11th record is coming in. Please uploading data in time.

3.3.3 Turn on/off Heartbeat Sound

The buzzer beep when heartbeat is detected during ECG recording. You can turn on/off the heartbeat sound on the App.

ECG Measurement Principle

The product collects the ECG data through the potential difference of the body surface through the ECG electrode, and obtains accurate ECG data after being amplified and filtered, then displays through the screen.

3.3.4 Display Screen



Regular ECG4 HR /min 568

- 1. Real time ECG waveform
- 2. Recording progress bar
- 3. Real time Heart Rate
- 4. ECG result
- 5. Heart Rate result

3.3.5 Criteria of ECG data description

Display content	Conditions	
High HR	More than 100/ min	
Low HR	Less than 60/ min	
	No R-waves detected for 2 seconds and more	
Irrogular boot	R-R variation rate is more than 10% of the normal intervals	
irregular beat	R-R interval is less than 80% of the normal intervals	
	When PVC detected	
Poor signal, please try again	The measurement is not completed.	
Regular beat None of the above		

3.4 Bluetooth

The product Bluetooth will be enabled automatically only when the screen lights up.

- 1) Ensure the product screen is on to keep the product Bluetooth enabled.
- 2) Make sure the phone Bluetooth is enabled.
- 3) Select the product ID from the phone, then the product will be paired successfully with your phone.
- 4) You can export the measured data including SYS, DIS, ECG data to your phone.

Note:

- The Bluetooth technology is based on a radio link that offers fast and reliable data transmissions.
 The Bluetooth uses a license-free, globally available frequency range in the ISM band-intended to ensure communication compatibility worldwide.
- The pairing and transmitting distance of wireless function is 1.5 meters in the normal. If the wireless communication is delay or failure between the phone and the product, you will try to narrow the distance between the phone and the product.

• The product can pair and transmit with the phone under the wireless coexistence environment (e.g. microwaves, cell phones, routers, radios, electromagnetic anti-theft systems, and metal detectors), but other wireless product may still interface with pairing and transmission between the phone and the product under uncertain environment. If the phone and the product display inconsistent, you may need to change the environment.

3.5 Wifi (Only for BP1V, BP1W)

3.5.1 Download the App

App name: ViHealth

iOS: App Store

Android: Google Play

3.5.2 Install the App

Install the ViHealth app on your smart device.

3.5.3 Connect the monitor with Wi-Fi network

- 1) Enable Bluetooth on your smart device settings.
- 2) Power on the monitor and run the ViHealth app.
- 3) Select your the monitor and Add it to ViHealth.



4) Tap "sign up" to register an account. If you already had a ViHealth account, please sign in directly.



5) Press both buttons on the device at the same time to pair with the monitor.



Press both buttons on the device at the same time to connect.

6) Tap "Start Wi-Fi Setup Connect" to connect the monitor with Wi-Fi network.



7) Choose a Wi-Fi network and enter the password.



8) When "Your device is ready " pops up, tap "Done" to compete Wi-Fi setup.



3.5.4 Data transmission

After the device is connected to the Wi-Fi network, the measurement results will be automatically uploaded to the cloud. You can download and view historical data in the App at anytime, regardless of whether the device is connected to the App.

4. Trouble Shooting

Problem	Possible Cause	Recommended Action	
The product cannot be	The product is out of Bluetooth range.	Verify product is in Bluetooth range while being paired (approximately 1.5 meters).	
connected to the phone	The radio frequency interference (ex. microwave)	Conduct the pairing process away from possible interference source.	
	The battery may be low.	Charge battery and try again.	
The product doesn't response to the button	The product is running in an unexpected status.	Reset the product by press and hold the button for 8s.	
press.	The device might be damaged.	Please contact your local distributor.	
Cannot get blood pressure measured results.	The measurement is interrupted by arm movement or unexpected bulb squeeze.	Keep arm still and don't squeeze the bulb during deflating-measure phase.	
	There is an over-leakage of press.	Check if the hose connection is loose.	
Cannot get ECG measured results.	Hold the electrode is too loose.	Please hold the electrode properly, and ensure no skin contact between the right and left hand.	
	The skin of hand is too dry.	Please keep the skin of hand a little moist.	

5. Accessories

Model	Description
540-00240-00	MICRO USB charge cable

6. Specifications

Classifications				
	MDR, (EU) 2017/745			
EC Directive	RED, 2014/53/EU			
Degree of protection against electrical shock	Type BF			
Protection against electric shock	Internally powered equi	pment		
Electro-magnetic Compatibility	Group I, Class B	Group I, Class B		
Apply part	Cuff and ECG electrode			
Environmental				
Item	Operating	Storage		
Temperature	5°C to 45°C	-25°C to 70°C		
Relative humidity (non-condensing)	10% to 95%	10% to 95%		
Barometric	700hPa to 1060 hPa	700hPa to 1060 hPa		
Degree of dust & water resistance	IP22			
Drop test	1.0 m			
Physical				
Size	135 mm×46mm×21mm	(main unit)		
Weight	about 220 g (main unit)			
Cuff size	22-42 cm			
Wireless connectivity	Built-in Bluetooth 4.0 BLE			
Power Supply				
Charge input	Micro USB, DC5V			
Battery type	Rechargeable lithium-polymer battery			
Battery run time	Approximately 1000 measurements			
Charge time	2 hours			
Blood Pressure				
Technology	Oscillometric method			

Pressure measurement range	0 – 300 mmHg		
Pressure measurement accuracy	±3mmHg or 2%, whichever is greater.		
Pulse rate range	40 to 200 /min		
Pulse rate accuracy	±2 bpm		
Clinical accuracy	Meet IEC80601-2-30		
ECG			
Lead type	Integrated ECG electrodes		
ECG channel	Single Channel		
Measure mode	Continuous		
Input impendence	≥10MΩ, 10Hz		
CMRR	>60 dB		
Center Frequency	50/60 Hz		
Linearity and dynamic range	10mV (Peak-to-valley)		
Frequency response	0.67 to 40 Hz		
Gain error (Max.)	±10%		
HR measurement range	30 to 250 bpm		
Accuracy	±2 bpm or ±2%, whichever is greater.		
Memory			
Capacity	up to 100 readings for Blood Pressure data + 10 records for ECG data.		
Bluetooth RF			
Frequency range	2.402-2.480 GHz GFSK Modulation		
	Adaptive Frequency Hopping (AFH)		
Wireless Quality of Service (QoS)	Transmission Distance: 1.5m Transmission Time: ≤0.5s for one Blood Pressure reading ≤10s for one ECG record Data integrity: 100%		
Network topology	Point-to-Point		
Band width	1Mbps		
WIFI			
Frequency range	2412-2462MHz		
Receiving Sensitivity	CCK, 1 Mbps : -90dBm CCK, 11 Mbps: -85dBm 6 Mbps (1/2 BPSK): -88dBm 54 Mbps (3/4 64-QAMI): -70dBm HT20,MCS7 (65 Mbps,72.2 Mbps):-67dBm		

Network topology	Point-to-Point	
Band width	1Mbps	
Durable period		
Expected service life	3 years	

7. Maintenance and Cleaning

7.1 Maintenance

To protect your product from damage, please observe the following:

- Store the product and the components in a clean, safe location.
- Do not wash the product and any components or immerse them in water.
- Do not disassemble or attempt to repair the product or components.
- Do not expose the product to extreme temperatures, humidity, dust or direct sunlight.
- The cuff contains a sensitive air-tight bubble. Handle this carefully and avoid all types of straining through twisting or buckling.
- Clean the product with a soft, dry cloth. Do not use petrol, thinners or similar solvent. Spots on the
- cuff can be removed carefully with a damp cloth and soapsuds. The cuff must not be washed!
- Do not drop the instrument or treat it roughly in any way. Avoid strong vibrations.
- Never open the product! Otherwise the manufacturer calibration becomes invalid!

7.2 Cleaning

The product can be repeatedly used. Please clean before reuse as follow:

- Clean the product with a soft, dry cloth with 70% alcohol.
- Do not use petrol, thinners or similar solvent.
- Clean the cuff carefully with cloth soaked 70% alcohol.
- The cuff must not be washed.
- Clean on the product and the arm cuff, and then let it air dry.

7.3 Disposal

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestics waste.

8. FCC Statement

BP2/BP2A/BP2B/BP2C	FCC ID: 2ADXK-8621
BP2V/BP2W	FCC ID: 2ADXK-8622

FCC Warning

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) this device may not cause harmful interference, and

(2) this device must accept any interference received, including interference that may cause undesired operation.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

9. Electromagnetic Compatibility

The product meets the requirements of EN 60601-1-2.

M Warnings and Cautionary Advices

- Using accessories other than those specified in this manual may result in increased electromagnetic emission or decreased electromagnetic immunity of the equipment.
- The product or its components should not be used adjacent to or stacked with other equipment.
- The product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided below.
- Other products may interfere with this product even though they meet the requirements of CISPR.
- When the inputted signal is below the minimum amplitude provided in technical specifications, erroneous measurements could result.

- Portable and mobile communication equipment may affect the performance of this product.
- Other products that have RF transmitter or source may affect this product (e.g. cell phones, PDAs, and PCs with wireless function).

Guidance and Declaration - Electromagnetic Emissions

The Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the Blood Pressure Monitor should assure that it is used in such an environment.

Emission tests	Compliance	Electromagnetic environment - guidance	
RF emissions CISPR 11	Group 1	The Blood Pressure Monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
RF emissions CISPR 11	Class B	The Blood Pressure Monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public	
Harmonic emissions IEC61000-3-2	N/A		
Voltage Fluctuations / Flicker Emissions IEC 61000-3-3	N/A	low-voltage power supply network that supplies buildings used for domestic purposes.	

Guidance and Declaration - Electromagnetic Immunity
The Blood Pressure Monitor is intended for use in the electromagnetic environment specified
below. The customer or the user of the Blood Pressure Monitor should assure that it is used in
such an environment.

Immunity test	IEC60601 test level	Compliance	Electromagnetic
initiality test		level	environment - guidance
Electrostatic	± 8kV contact	± 8kV contact	Floors should be wood, concrete or ceramic tile. If
discharge (ESD) IEC 61000-4-2	±2kV, ±4kV, ±8kV, ±15kV air	±2kV, ±4kV, ±8kV, ±15kV air	floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/ burst IEC 61000-4-4	± 2kV for power supply lines 100kHz repetition frequency ± 1kV for input/output lines	N/A	
Surge IEC 61000-4-5	± 0.5kV, ±1kV differential mode line-line	N/A	
Voltage dips, short Interruptions and Voltage variations on power supply input lines IEC 61000-4-11	$0\% U_T$ (100% dip in U _T) for 0.5 cycle at 0°, 45°, 90°, 135°,180°, 225°, 270°, and 315° 0% U _T (100% dip in U _T) for 1 cycle at 0° 70% U _T (30% dip in U _T) for 25/30 cycles at 0° 0% U _T (100% dip in U _T) for 250/300 cycles at 0°	N/A	
Power frequency	30 A/m, 50/60Hz		Power frequency magnetic

(50/60 HZ)		30 A/m,	fields should be at levels		
magnetic field IEC		50/60Hz	characteristic of a typical		
61000-4-8			location in a typical		
			commercial or hospital		
			environment.		
Note: U_T is the AC mains voltage prior to application of the test level.					

Guidance and Declaration - Electromagnetic Immunity							
The Blood Pressure Monitor is intended for use in the specified electromagnetic environment. The customer or the user of the Blood Pressure Monitor should assure that it is used in such an environment as described below.							
Immunity test	IEC60601 test level	Compliance level	Electromagnetic environment - guidance				
Conduced RF IEC61000-4-6	3Vrms 150kHz to 80 MHz 6Vrms 150 kHz to 80 MHz outside ISM bands	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the system,				
Radiated RF IEC61000-4-3	10V/m 80MHz to 2.7 GHz	10 V/m	including cables, than the recommended separation distance calculated from the equation appropriate for the frequency of the transmitter. Recommended separation distances: $d = \left[\frac{3.5}{V_1}\right]\sqrt{P}$ $d = \left[\frac{3.5}{E_1}\right]\sqrt{P}$ 80MHz to 800MHz $d = \left[\frac{7}{E_1}\right]\sqrt{P}$ 800MHz to 2.7GHz Where, <i>P</i> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <i>d</i> is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol:				

Note 1: At 80 MHz to 800 MHz, the separation distance for the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a The ISM (industrial, scientific and medical) bands between 0,15 MHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.

b The compliance levels in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2,7 GHz are intended to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas. For this reason, an additional factor of 10/3 has been incorporated into the formulae used in calculating the recommended separation distance for transmitters in these frequency ranges.

c Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Blood Pressure Monitor is used exceeds the applicable RF compliance level above, the Blood Pressure Monitor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Blood Pressure Monitor.

d Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications

equipment and the Blood Pressure Monitor

The Blood Pressure Monitor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Blood Pressure Monitor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Blood Pressure Monitor as recommended below, according to the maximum output power of the communications equipment.

	Separation distance according to frequency of the transmitter (m)				
Rated max. output	150kHz to 80MHz	80MHz to 800MHz	800MHz to 2.7GHz		
power of transmitter (W)	$d = [\frac{3.5}{V_1}]\sqrt{P}$	$d = \left[\frac{3.5}{E_1}\right]\sqrt{P}$	$d = \left[\frac{7}{E_1}\right]\sqrt{P}$		
0.01	0.12	0.04	0.07		
0.1	0.37	0.12	0.23		
1	1.17	0.35	0.70		
10	3.70	1.11	2.22		
100	11.70	3.50	7.00		

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.



Shenzhen Viatom Technology Co., Ltd.

4E,Building 3, Tingwei Industrial Park, No.6 Liufang Road, Block 67, Xin'an Street, Baoan District, Shenzhen 518101 Guangdong China

www.viatomtech.com info@viatomtech.com

Well Kang Limited



EC REP

The Black Church, St. Mary's Place,Dublin 7, D07 P4AX, Ireland Tel 1: +353(1)2542900 Tel 2: +353 (1)4433560 Fax: +353(1)6864856 Email:AR@Well-Kang.com

PN: 255-01761-00

Version: A

Oct, 2020

Annex Bland-Altman Plot of Blood Pressure

Note:

The Bland-Altman Plots originated form clinical test, which data was gathered from pediatric /adult healthy volunteers in a controlled study.

The clinical test applies for the device with the cuff size (22-42 cm) comparing with Mercury sphygmomanometer.

The result met the criteria specified in ISO 81060-2:2013; In addition, there were no reported adverse effects during these investigations.

Please refer to following for detail.



Figure 1. Plot of SBP difference between the paired blood pressure determinations against the reference systolic measurement



Figure 2. Plot of DBP difference between the paired blood pressure determinations against the reference diastolic measurement