

BrailleSense 6

User Manual

ENGLISH MANUAL
for Software Version 1.0

SELVAS Healthcare, Inc.

Safety Precautions

For your safety and protection of the BrailleSense 6, please read and abide by the following safety precautions.

1. The input voltage of the AC adapter is 100V - 240V, and the outputs are DC 5V 2A and 9V 2A
2. When the BrailleSense 6 is shipped, the battery may be packaged separate from the unit to avoid damage. When you use the BrailleSense 6 for the first time, you may need to open the package, remove the battery, and insert it into the battery slot at the underside of the unit. (See section 1, introduction, for more details on inserting the battery.) It is possible that the dealer or distributor from whom you purchased the BrailleSense 6 has already done this for you. The battery may not be fully charged when it is shipped. Before you use the BrailleSense 6, you should ensure the battery is fully charged.
3. When you use the BrailleSense 6 with the battery for the first time, the battery status may not be displayed accurately. To avoid this, leave the BrailleSense 6 connected to the AC adapter for about five hours with the unit turned on. If you want to use the BrailleSense 6 immediately, you may use it while it is charging.
4. If you want or need to remove the battery from the unit, power the unit off, and then remove the battery. If the unit is connected to AC power, before re-inserting the battery into the unit, make certain that the power is off.
5. There is a risk of explosion if the battery is replaced by an incorrect type. Please make certain to use only battery packs produced specifically for the BrailleSense 6 by SELVAS Healthcare. Please dispose of used batteries according to the instructions.

6. If the usage time is less than 50% as compared to the stated battery run time in the product specifications, even after a full charge, it is recommended that you replace the battery.
7. It is recommended to remove and store the battery separate from the product during long periods of disuse.
8. If you drill into or disassemble the product or battery using a sharp point, it may cause electric shock, explosion or fire.
9. When you are using the BrailleSense 6 on battery power, the status of the remaining battery power is announced when the battery charge is low. When the battery's charge falls below 15 percent, connect the AC adapter to the unit for recharging. If the remaining battery charge drops to five percent, and the unit is not connected to the AC adapter, the BrailleSense 6 goes in to Sleep Mode one minute after the announcement of the battery's status. If the battery level falls to 2%, the unit shuts down to preserve settings, etc. The amount of battery life per charge varies depending on the options you have set and the number and type of tasks you are running.
10. Your BrailleSense 6 needs to be handled with care. The BrailleSense 6 is a very delicate machine. Please handle the unit in a proper, careful manner. The BrailleSense 6 is very sensitive to dust. Make sure to keep it away from dusty environments.
11. The Braille cells are very sensitive, intricate parts. If the Braille cells are not working properly due to the presence of dust or for any other reason, please contact our repair center, or the dealer from whom you purchased the BrailleSense 6 for support.
12. You should register your BrailleSense 6 in order to ensure further maintenance, service, and upgrade information. In the US, please visit www.hims-inc.com, to register your information. If you have any trouble when registering your information, please contact the dealer from whom

you purchased your BrailleSense 6 for help. International customers should contact their regional dealer for product registration information.

13. Do not disassemble the BrailleSense 6 yourself. Do not have anyone else who is not authorized by HIMS take apart the BrailleSense 6. If an unqualified person disassembles the unit, serious damage may occur. If an unauthorized person disassembles the unit, it is excluded from any free maintenance, and the warranty becomes void. If any liquid or external force damages the unit, it may also be excluded from free maintenance, even if the damage occurs during the warranty period. The BrailleSense 6 contains a Liquid Contact Indicator sticker that will change color if the unit comes in contact with water or other liquids. This indicator will not activate under humid or other conditions that fall within the product's normal environmental requirements.
14. Do not leave your BrailleSense 6 in closed, high temperature environments such as inside a car on a hot summer day, as the battery may be damaged or catch fire. Please do not let your BrailleSense 6 remain in such environments for long periods of time.
15. To prevent possible damage to your hearing, do not listen to audio at high volume levels for long periods.

Operation is subject to the following two conditions.

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

This equipment can generate, use and radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

I. Your BrailleSense 6 packing box should contain the following items:

- 1) BrailleSense 6
- 2) Battery
- 3) Carrying Case
- 4) AC Adapter
- 5) USB C to C Cable
- 6) USB C to A Cable
- 7) Earbuds
- 8) Braille Commands Summary

II. How to use this manual

In this manual you will see references to hot keys and shortcut keys. These keys refer to ways to access menus and functions quickly by using a combination of keystrokes. Please note that not all hot keys and shortcut keys will work from every location on the BrailleSense 6. Some hot keys and shortcut keys are program specific, thus, they require that you be within a specific program for them to work.

Shortcut keys refer to keystrokes used to quickly activate a function from within a menu. Hot keys refer to keys that will activate a function regardless of where you are within the program. When functions are described in this manual, shortcut keys are shown in parentheses, and hot keys are shown in quotes.

This user manual notes how to press hot keys and shortcut keys in the following manner: Keys that are to be pressed at the same time are separated by a - (dash). When you see "Space-b (dots 1-2)," this means that you should press "Space" and dots 1 and 2 simultaneously.

Table of Contents

Safety Precautions	2
I. Your BrailleSense 6 packing box should contain the following items:	6
II. How to use this manual	6
Table of Contents	7
1. Introduction	20
1.1 WHAT IS THE BRAILLESense 6?	20
1.2 PHYSICAL TOUR	21
<i>1.2.1 The Top Panel</i>	<i>21</i>
<i>1.2.2 The Right Panel</i>	<i>22</i>
<i>1.2.3 The Left Panel</i>	<i>23</i>
<i>1.2.4 The Rear Panel</i>	<i>23</i>
<i>1.2.5 The Front Panel</i>	<i>23</i>
<i>1.2.6 The Bottom Face</i>	<i>24</i>
1.3 HARDWARE SPECIFICATIONS	25
1.4 MEMORY ON THE BRAILLESense 6	26
2. Basic Functions of the BrailleSense 6.....	26
2.1 HOW TO ENTER COMMANDS	26
<i>2.1.1 Basic Alerts and Messages</i>	<i>27</i>
2.2 MENU CONCEPTS	27
<i>2.2.1 Control Symbols</i>	<i>31</i>
2.3 USING THE FUNCTION KEYS.....	35
2.4 THE AC POWER SUPPLY AND THE BATTERY PACK	36

2.5 CONTROLLING THE VOLUME, SPEED, AND PITCH OF THE TTS	38
2.6 USING SPEECH AND BRAILLE.....	38
2.7 ENTERING TEXT	39
2.8 MULTI-TASKING.....	39
2.9 SWITCHING MEDIA MODES.....	41
2.10 SWITCHING KEY LOCK.....	41
2.11 ONE-HANDED MODE.....	42
2.12 USING TYPING MODE.....	43
2.13 CREATING SYSTEM LOGS AND SCREEN RECORDINGS.....	43
3. Customizing your Notetaker.....	44
3.1 SET TIME AND DATE	45
3.2 SETUP INTERNET	47
<i>3.2.1 Setup Wireless LAN</i>	<i>47</i>
<i>3.2.2 LAN Setup</i>	<i>50</i>
3.3 BLUETOOTH MANAGER.....	51
<i>3.3.1 Bluetooth Device List.....</i>	<i>52</i>
<i>3.3.2 Bluetooth Service List.....</i>	<i>53</i>
<i>3.3.2.1 Connecting a Service</i>	<i>55</i>
3.4 MENU MANAGER	56
3.5 BACKUP/RESTORE BRAILLESENSE SETTINGS.....	57
3.6 CHANGE DEVICE NAME	58
3.7 QUICK START GUIDE	59
3.8 PASSWORD PROTECTION.....	59
<i>3.8.1 Setting and Modifying Passwords</i>	<i>60</i>
<i>3.8.2 Password Configuration</i>	<i>61</i>

3.8.3 <i>Reset Password to Factory Defaults</i>	61
3.9 INITIALIZE BRAILLESENSE OPTIONS	62
3.10 VOICE OPTIONS	62
3.10.1 <i>BrailleSense Vocalizer</i>	62
3.10.2 <i>Android TTS</i>	63
3.11 BRAILLESENSE GLOBAL OPTIONS	64
3.11.1 <i>Global Options in Detail</i>	65
3.12 ANDROID BACKUP & RESET	70
3.13 ANDROID SYSTEM SETTINGS	71
4. File Manager	71
4.1 OVERVIEW	71
4.1.1 <i>Navigating the File List</i>	72
4.1.2 <i>Selecting Folders and Files</i>	73
4.2 WORKING WITH FILES AND FOLDERS	74
4.2.1 <i>Entering and Exiting Folders</i>	74
4.2.2 <i>Ejecting USB Drives and SD Cards</i>	74
4.3 THE FILE MENU	75
4.4 THE EDIT MENU	77
4.5 THE VIEW MENU	77
4.6 GOOGLE DRIVE	78
4.7 USING THE ADDRESS WINDOW	79
5. Word Processor	80
5.1 FILE MENU	81
5.2 EDIT MENU	82
5.3 INSERT MENU	83

5.4 GO TO MENU	84
5.5 READ MENU	85
5.6 LAYOUT MENU.....	86
5.7 MISCELLANEOUS FUNCTIONS	86
6. Notepad	87
6.1 FILE MENU	88
6.2 EDIT MENU.....	89
6.3 INSERT MENU	90
6.4 GO TO MENU	91
6.5 READ MENU	92
6.6 LAYOUT MENU.....	93
7. E-mail.....	94
7.1 EXECUTING E-MAIL.....	94
7.2 SETTING UP AN ACCOUNT	95
7.3 MANAGING ACCOUNTS	99
7.4 RECEIVING E-MAIL.....	101
7.5 READING MESSAGES	101
7.6 WRITING E-MAIL.....	104
7.6.1 <i>Attaching a File</i>	105
7.6.2 <i>Saving E-mail to the Outbox</i>	106
7.7 NAVIGATING ACCOUNTS AND MAILBOXES.....	107
7.7.1 <i>Managing Mailboxes</i>	107
7.7.2 <i>Copying and Moving to Mailboxes</i>	108
7.7.3 <i>Importing Email</i>	109
7.8 ADVANCED MESSAGE FEATURES	109

7.8.1 Reply and Reply All	109
7.8.2 Forwarding a Message.....	110
7.8.3 Saving an E-mail	111
7.8.4 Finding Messages	111
7.8.5 Move to Next Unread Message.....	112
7.8.6 Activating Links	112
7.9 USING THE TOOLS MENU	113
7.9.1 Set Path.....	113
7.9.2 Set Options.....	113
7.9.3 Spam Settings	115
8. Exchange E-mail.....	117
8.1 EXECUTING EXCHANGE E-MAIL.....	118
8.2 SETTING UP AN EXCHANGE E-MAIL ACCOUNT	118
8.3 SAVING EXCHANGE MESSAGES OFFLINE.....	119
8.4 NAVIGATING MAILBOXES	120
8.5 EXCHANGE OPTIONS	120
9. Media	121
9.1 MEDIA PLAYER	121
9.1.1 Using the Media Buttons	121
9.1.2 Media Player Layout.....	123
9.1.3 The File Menu	124
9.1.4 The Play Menu	125
9.1.5 Making Recordings.....	126
9.1.6 The Position Menu	126
9.1.7 The Mark Menu	126

9.1.8 <i>The Settings Menu</i>	127
9.2 FM RADIO	127
9.2.1 <i>Listening to FM Radio via the Internal Speakers</i>	128
9.2.2 <i>Frequency Control</i>	128
9.2.3 <i>Adding and Navigating Presets</i>	129
9.2.4 <i>Recording from the FM Radio</i>	130
9.2.4.1 <i>Changing record folder</i>	130
9.2.5 <i>Additional Features</i>	131
9.3 DAISY PLAYER	131
9.3.1 <i>Arrangement of the DAISY Player</i>	132
9.3.2 <i>The File Menu</i>	134
9.3.3 <i>The Document Menu</i>	134
9.3.4 <i>The Mark Menu</i>	135
9.3.5 <i>The Headings Menu</i>	135
9.3.6 <i>The Memo Menu</i>	135
10. Organizer	136
10.1 ADDRESS MANAGER	136
10.1.1 <i>Adding an Address</i>	137
10.1.2 <i>Setting Add Address Fields</i>	138
10.1.3 <i>Searching For an Address</i>	138
10.1.4 <i>Setting Searching Address Fields</i>	139
10.1.5 <i>Backing up and Restoring the Address List</i>	139
10.1.6 <i>Importing from CSV</i>	140
10.1.7 <i>Exporting to CSV</i>	141
10.1.8 <i>Editing, Saving, and Deleting Addresses</i>	142

10.1.9 Syncing Contacts	142
10.2 SCHEDULE MANAGER.....	143
10.2.1 Adding an Appointment	144
10.2.2 Setting a Recurring Appointment	145
10.2.3 Search Appointments	147
10.2.4 Editing, Saving, and Deleting Appointments	149
10.2.5 Backing up and Restoring Your Appointments	150
10.2.6 Multiple Accounts and Syncing	152
10.2.7 Setting Alarm Options	153
11. Web Tools	154
11.1 WEB BROWSER	154
11.1.1 The File Menu.....	155
11.1.2 The Edit Menu	155
11.1.3 The Go To Menu.....	156
11.1.4 The Read Menu.....	156
11.1.5 The Favorites Menu.....	157
11.1.6 Options	157
11.1.7 Navigating Page Elements	158
11.1.8 Table Navigation	160
11.1.9 Playback of Streaming Audio Files	161
11.2 GOOGLE SEARCH	162
11.2.1 The Search Dialog.....	162
11.2.2 The Results Dialog	163
11.2.3 Choosing your Browser	164
12. EXTRAS	164

12.1 EXCEL VIEWER.....	164
12.1.1 <i>General Navigation.....</i>	165
12.1.2 <i>The File Menu</i>	166
12.1.3 <i>The Edit Menu</i>	167
12.1.4 <i>The Go To Menu</i>	167
12.1.5 <i>The Read Menu.....</i>	167
12.1.6 <i>The View Menu</i>	168
12.2 ONLINE DAISY	169
12.3 SENSE DICTIONARY	170
13. Utilities	171
13.1 CALCULATOR	171
13.1.1 <i>General Functions.....</i>	172
13.1.2 <i>Fractional Arithmetic.....</i>	172
13.1.3 <i>Memory Functions.....</i>	173
13.1.4 <i>Sine Functions.....</i>	175
13.1.5 <i>Cosine Functions.....</i>	175
13.1.6 <i>Tangent Functions.....</i>	175
13.1.7 <i>Logarithm Functions.....</i>	175
13.1.8 <i>Convert Unit</i>	176
13.1.9 <i>Copy to Clipboard.....</i>	177
13.1.10 <i>Return Formula</i>	178
13.1.11 <i>Option Settings</i>	178
13.2 BRAILLESense MATH.....	179
13.2.1 <i>The File Menu</i>	180
13.2.2 <i>The Edit Menu</i>	180

13.2.3 <i>The Graph Menu</i>	181
13.2.4 <i>The Output Menu</i>	181
13.3 DISPLAY COMPASS HEADING	182
13.3.1 <i>Options of Gyrocompass</i>	182
13.4 DISPLAY TIME AND DATE	183
13.5 WAKE UP ALARM	183
13.6 STOPWATCH	185
13.7 TERMINAL FOR SCREEN READER	186
13.7.1 <i>Terminal Clipboard</i>	187
13.8 DISPLAY NETWORK STATUS	188
13.9 DISPLAY POWER STATUS	189
13.10 FORMAT	189
13.11 SET SLEEP TIMER	190
13.12 USING MACROS	191
13.12.1 <i>Recording a macro</i>	191
13.12.2 <i>Executing a Macro</i>	192
13.12.3 <i>The Macro Manager</i>	192
13.13 UPGRADE BRAILLESENSE 6 FIRMWARE	193
13.13.1 <i>Upgrading the BrailleSense Firmware Using the Internet</i>	195
13.13.2 <i>Upgrading the BrailleSense Firmware from a Disk</i>	196
14. Installing and Using Native Android Apps	196
15. Using Help	200
16. Troubleshooting Guide	201
16.1 THE BRAILLESENSE 6 WILL NOT TURN ON	201

16.2 THE VOICE CANNOT BE HEARD	202
16.3 THE VOICE IS TOO FAST TO UNDERSTAND.....	202
16.4 THE VOICE PITCH IS TOO LOW OR TOO HIGH TO UNDERSTAND.	202
16.5 YOU DON'T KNOW WHERE YOU ARE	202
16.6 NO INTERNET CONNECTION	203
16.7 E-MAIL	203
16.8 THE BRAILLE DISPLAY DOES NOT DISPLAY BRAILLE.....	203
16.9 THE BRAILLESense 6 DOES NOT SPEAK WHEN USING PLAY STORE OR ANDROID APPS	203
16.10 THE BRAILLESense MENUS ARE NOT DISPLAYED OR SPOKEN.....	204
16.11 THE BRAILLESense 6 DOES NOT RESPOND.....	204
16.12 THE LCD DOES NOT COME ON	205
16.13 USB OR SD FILES SEEM TO CONTAIN 0 BYTES	205
17. Command Summary.....	205
17.1 COMMON COMBINATION KEYS	206
17.2 QUICK LAUNCH COMMANDS	208
17.3 FILE MANAGER	210
<i>17.3.1 Navigation Keys for the File and Menu List.....</i>	<i>210</i>
<i>17.3.2 Navigation Keys for the File List.....</i>	<i>211</i>
<i>17.3.3 Selection Commands</i>	<i>211</i>
<i>17.3.4 File Menu Commands</i>	<i>211</i>
<i>17.3.5 Edit Menu Commands.....</i>	<i>212</i>
<i>17.3.6 View Menu Commands</i>	<i>212</i>
17.4 WORD PROCESSOR.....	213

17.4.1 File Menu Commands	213
17.4.2 Edit Menu Commands.....	213
17.4.3 Insert Menu Commands.....	214
17.4.4 Go To Commands	214
17.4.5 Read Commands	215
17.4.6 Layout Commands	215
17.4.7 Miscellaneous Hot Keys.....	216
17.5 NOTEPAD	217
17.5.1 File Menu Commands	217
17.5.2 Edit Menu Commands.....	217
17.5.3 Insert Menu Commands.....	218
17.5.4 Go To Commands	218
17.5.5 Read Commands	219
17.5.6 Layout Commands	219
17.5.7 Miscellaneous Hot Keys.....	219
17.6 E-MAIL/EXCHANGE E-MAIL	220
17.6.1 Hot Keys for the Inbox.....	220
17.6.2 Hot keys for Sending E-mail.....	221
17.6.3 Hot Keys for Reading/Writing an E-mail Message	222
17.7 MEDIA.....	222
17.7.1 Media Player	222
17.7.1.1 File Commands	222
17.7.1.2 Playback Commands	223
17.7.1.3 Recording Commands.....	224
17.7.1.4 Hot Keys in the Play Information Tab.....	225
17.7.1.5 Hot Keys in the Playlist Tab.....	225

17.7.2 FM Radio	225
17.7.3 DAISY Player	226
17.8 ORGANIZER.....	228
17.8.1 Address Manager	228
17.8.1.1 Navigation When Adding an Address	229
17.8.1.2 Navigation in Search Results List	229
17.8.1.3 General Commands	229
17.8.2 Schedule Manager	230
17.8.2.1 General Commands	230
17.8.2.2 Navigating Search Results	231
17.9 WEB TOOLS	231
17.9.1 Web Browser	231
17.9.1.1 General Commands	231
17.9.1.2 Hot Keys When Playing Audio Files in Web Browser	233
17.9.1.3 Hot Keys for Reading Web Pages	233
17.10 EXTRAS	234
17.10.1 Excel Viewer	234
17.10.1.1 Navigation Commands	234
17.10.1.2 File Commands	235
17.10.1.3 Go To Commands	235
17.10.1.4 Read Commands	236
17.10.1.5 View Commands	237
17.10.1.6 Edit Commands	237
17.10.2 Online DAISY	237
17.11 UTILITIES.....	237
17.11.1 Calculator	237
17.11.2 BrailleSense Math	239

17.11.2.1 File Menu Commands	239
17.11.2.2 The Edit Menu	239
17.11.2.3 The Graph Menu	240
17.11.2.4 The Output Menu.....	240
17.11.3 Display Compass Heading	240
17.11.4 Alarm	240
17.11.5 Stopwatch.....	241
17.11.6 Terminal for Screen Reader	241
17.11.7 Macro Commands	241
17.11.8 Other Utility Commands	242
17.12 SETTINGS.....	242
17.12.1 Setting Time and Date.....	242
17.12.2 Bluetooth Manager.....	243
17.12.3 Backup/Restore Personalized Settings.....	243
17.12.4 Password Protection.	243
17.13 SCREEN READER COMMANDS.....	244
17.13.1 Moving Focus.....	244
17.13.2 When Editing Text	245
17.13.3 Media Key Functions when in "App Mode"	246

1. Introduction

1.1 What is the BrailleSense 6?

The BrailleSense 6 is a HIMS branded Braille note taker developed and manufactured by SELVAS Healthcare, for people who are blind and visually impaired. The BrailleSense 6 has a variety of powerful capabilities, including the following.

1. Read TXT, BRF, RTF, PDF, EPUB, and Microsoft Word, Excel and PowerPoint files. Take notes or create formatted documents and e-mails.
2. Play media, DAISY books, and FM Radio and YouTube videos, and stream content from 3rd party services such as Apple music and Netflix.
3. Organize your Appointments and contacts, and synchronize all information with your Google account.
4. Perform scientific and graphic calculations, set alarms and explore the Lunar and solar calendars.
5. Use the BrailleSense 6 as a USB or Bluetooth Braille display with a compatible screen reader, such as JAWS, NVDA, SuperNova, System Access, VoiceOver for Mac and iOS and TalkBack for Android.
6. Use the built-in GPS sensor and digital compass to navigate independently without the need for external devices.
7. Download and install 3rd party applications from the Google Play Store.
8. Utilize Google Services such as Drive, Docs, Sheets, Slides, and Classroom via our included screen reader, specially customized to enhance access to these applications.
9. Attend or present at virtual meetings or conferences via Zoom, Google Meet, Microsoft Teams or other accessible platforms.

10. Obtain information and perform functions using your voice with the included Google Assistant, or third party voice assistants like Amazon Alexa.
11. Connect to your favorite smart watch for portable access to BrailleSense notifications and services.
12. Use the built-in 13 megapixel camera, or an external USB camera to scan printed materials, join a video conference, or get sighted assistance from friends and family or via apps such as Envision or AIRA.
13. Use the 3 available USB host ports to connect and use compatible devices such as USB portable monitors, keyboards and mice, USB audio devices, printers, and other Android compatible USB peripherals.

1.2 Physical Tour

To get the most from your BrailleSense 6, it is important to become oriented to the buttons and ports as well as learning their functions. This section will orient you to your BrailleSense.

Place the BrailleSense on a desk or table top with the Braille display nearest you.

1.2.1 THE TOP PANEL

On the top face of the BrailleSense 6 is located a 9 key Perkins-style keyboard, consisting of Braille dots 1:6, "SPACE", "BACKSPACE", and "ENTER".

On the same row as the Space bar are Control and Alt, as well as 4 marked function keys. Moving from left to right, the row contains F1, F2, Control, Space, Alt, F3 and F4.

Below the keyboard is a 32 cell Braille display. Above the Braille display, there are cursor routing buttons corresponding to each of the 32 Braille cells. When

you are editing documents, you can use these keys to route your cursor to the character of the corresponding Braille cell. The cursor routing keys also serve as shortcut keys for navigating lists and groups of fields in certain places on the unit. A dot is placed on each 5th cursor routing button for easier orientation.

On each side of the Braille display are two capsule shaped keys. These are the "UP" and "DOWN" scroll keys. The scroll keys are used to scroll the Braille display through menus, fields, lists or text. You can use the scroll keys to scroll by display length, paragraph, line, sentence, or character. You can set the left and right sets of scroll keys independently in the "Global Options". (See chapter 3.)

Near the top center of the unit's top panel, is a liquid crystal display (LCD). The LCD displays the output of the unit visually so that a sighted person can see what is shown on the BrailleSense 6. There are also 2 LED indicators to the right of the LCD. The indicator nearest the LCD shows On/Off status, and the LED to the right indicates charging status.

Stereo speakers are located on the top left and right corners for audio output.

1.2.2 THE RIGHT PANEL

The Right edge of the BrailleSense 6 contains 3 USB ports, 2 USB C ports with a USB A port between them. The USB C port nearest the rear of the unit is for powering and charging the unit, as well as connecting to a PC for use as a Braille display, to transfer data, or operate the BrailleSense using Android companion software for Windows. The USB C port nearest the front of the unit will host other compatible USB C devices as well. To eliminate confusion over which port offers which functions, Braille letter indicators are placed on the top of the casing near each port: the letter p for the power and data port, and the letter v for the host port.

Between the 2 USB C ports is an additional USB A host port for connecting drives, printers, audio devices, cameras, etc.

1.2.3 THE LEFT PANEL

Near the front of the unit on the left panel are 2 small buttons for raising and lowering the master volume of the unit. Use the button nearer the front to decrease the volume, and the button behind it to increase it.

Behind the volume buttons are located 2 small, round jacks. The one nearest the front is the stereo headphone jack, and the one nearer the rear of the unit is the stereo microphone jack for connecting a stereo microphone or line level recording source.

Nearest the back of the left panel is an SD card slot for auxiliary storage.

1.2.4 THE REAR PANEL

The rear panel of the BrailleSense 6 contains another USB A host port, for connecting drives, keyboards and mice, cameras, printers, audio gear and other compatible USB peripherals.

1.2.5 THE FRONT PANEL

On the far left of the front panel is located a 3-position slide switch. This is the key "Lock" switch. This switch is explained in more detail in section 2.11 of this manual.

To the right of it is another 3-position slide switch. This is the "Media Mode" switch, and determines which type of content the front panel buttons control. This switch is explained in more detail in the next chapter.

To the right of the "Media Mode" switch, are five buttons, with a dot on the center one. These buttons control media and DAISY playback as well as providing shortcut keys for other areas of the unit when they are set to Application Mode. As explained above, the type of content controlled by these buttons depends on the position of the "Media Mode" switch. These five buttons are explained further in subsequent chapters of this manual.

To the right of the media buttons is the Power button. To turn on the unit, press and hold this button until you hear a beep and feel a vibration. The first time you power on the unit, it will take some time to boot up.

Once the unit boots, you can press the Power button quickly to put the unit in standby "screen off" mode. This is similar to locking the screen on a phone or tablet. Press it quickly again to unlock it and return to the position at which you entered Screen off Mode. Press and hold the Power button to completely power down the system.

1.2.6 THE BOTTOM FACE

The underside of the unit contains the battery compartment and the camera. The battery contains a rounded edge along one of the sides of its length. This rounded edge should be placed alongside the back edge of the unit's casing. The battery has 2 small protrusions that should be pointed to the right. Place the battery in the unit right side first, and then press down on the left side to click it into place. To remove the battery, Use a finger to pull the battery release lock toward the left edge of the unit, and pull the battery out.

The 13 megapixel camera is located near the upper right corner of the underside of the unit. Applications for OCR and photo capture can be downloaded from the Google Play Store. For best results, hold the BrailleSense

9 to 12 inches above the paper, with the unit off set to the left to account for the camera position.

Congratulations! You are now familiar with the basic functions of the buttons and ports of the BrailleSense 6. The rest of the manual discusses each function in greater detail to help you maximize the potential of your BrailleSense 6.

1.3 Hardware Specifications

1. Operating system: Android 10
2. Internal storage: 128GB UFS 2.1 (around 100GB available to user).
3. RAM: 6GB LPDDR4X.
4. CPU: Octa-core (4 x 2.0 GHz A76, 4 x 2.0 GHz A55)
5. Battery: Lithium Polymer (detachable), 4590mAh.
6. Battery running time: About 18 hours (with Braille and TTS, internal speaker at mid volume)
7. Keyboard: 9 key Perkins-Style Braille keyboard, Control and Alt, 4 function keys, 32 cursor routing keys, 4 scroll buttons.
8. Additional buttons and switches: key lock switch, Media Mode switch, 5 media buttons.
9. Braille display: 32 refreshable Braille cells.
10. Wireless: Wi-Fi: 802.11ac, Dual-band, 2T2R MU-MIMO (IEEE a/b/g/n compatible); Bluetooth: 5.1 dual mode for LE and Classic
11. Sensors: GPS, RTC, Accelerometer, Compass, Gyroscope
12. Camera: 13 MP Sony internal Camera, USB camera supported.
13. Interface: USB C x 2, USB A x 2, 1 SD card slot (FAT32 and exFAT supported)

14. Sound: Internal stereo speakers, stereo headphone jack.
15. Voice recording: Internal stereo microphones, external stereo microphone jack, headset microphone, USB audio supported.
16. Vibrate motor: installed.

1.4 Memory on the BrailleSense 6

The BrailleSense 6 has 6GB of RAM (random access memory) and 128GB of flash memory. The RAM is memory reserved for running programs. RAM is a temporary storage area for any program files that are running or any currently open documents. The data in the RAM area disappears if your battery depletes, and the AC adapter is not connected. The RAM data also disappears when BrailleSense 6 reboots for any reason. This means, if you have an open document which you have not saved, the unsaved data disappears.

2. Basic Functions of the BrailleSense 6

2.1 How to Enter Commands

The commands used to operate the BrailleSense are comprised of various combinations of the Braille and function keys. In this user manual, the "-" (dash) is used to indicate that keys should be pressed simultaneously. For example, "Space-Z" indicates that the user should press "space" and Z at the same time. This manual also contains expressions like "Space-2-4-6". This means that the spacebar with dots 2, 4, and 6 should be pressed and released simultaneously. In a menu or dialog, when your cursor is placed on your desired command, you can execute most commands by pressing "Enter". To cancel a function or close a program or menu, and return to your previous location, press "Space-Z".

- Note: In Android applications, “Space-Z” serves as the “Back” button. You can also use “Space-E” or “F4” as the Android “Back” button, used to move back one screen, or exit your current function or application.

2.1.1 BASIC ALERTS AND MESSAGES

There are instances where the BrailleSense alerts you to events for which you have asked it to notify you. For example, by default, the BrailleSense checks the Schedule Manager when you power on, and notifies you of any appointments scheduled for that day. When you are notified of an appointment, you can interact with the Schedule Manager as you normally would. To exit the Schedule Manager, press "Space-Z" or "Space-E" and you are returned to your previous location.

Other BrailleSense system messages are displayed and spoken, and automatically disappear when any key is pressed, or after a set number of seconds. For example, when you connect the AC adapter to the BrailleSense, the BrailleSense speaks the message "AC adapter connected." When you disconnect it, the BrailleSense says, "AC adaptor disconnected."

You will also hear alerts from the Android system and applications. You can review Android notifications by pressing “Space-1-2-3-4-5-6”, and can delete most notifications using “Space-D”. Press Enter on the notification to open the associated program. You can control how various notifications and alerts are handled in the Global options. (See Chapter 3)

2.2 Menu Concepts

Menus on the BrailleSense provide you with a list of programs, actions, or options from which you can select. If a menu item has a sub menu, selecting it brings up another list of choices.

When using a program on the BrailleSense, press "Space-M" or F2 to bring up its menu. Press the "Up" scroll key or "Space-1" to move to the previous item, and press the "Down" scroll key or "Space-4" to move to the next item.

To move to the first item in the menu, press "Space-1-2-3." To move to the last item in the menu, press "Space-4-5-6." When placed on the first item of a sub menu, pressing "Backspace" returns you to the upper Menu. To repeat the current item, press "Space-R". To open a sub menu or execute the current item, press "Enter".

You can also quickly jump to and execute a menu item by pressing the menu item's shortcut key, usually the first letter of the item name. Often menu items can also be executed instantly, directly from your current position in the program, with a hotkey.

Shortcut keys are displayed in parentheses following the menu names. Hotkeys generally consist of a modifier such as "Enter," "Space," or "Backspace", pressed simultaneously with a letter or dot combination. These are displayed following the shortcut keys.

For example, if you are in the Word Processor, and press "Space-M" or "F2," then press "Enter" on "File," you hear "New (n) pull down Enter-n". The word "New" is the menu option. You would press the letter N to jump to "New" from within the "File" menu. Press "Enter-n" to access the "New" document command from anywhere in the Word Processor. The n is the shortcut key and "Enter-n" is the hotkey. It is important to note that some hotkey commands are global, thus will work from anywhere on the unit, and some are program specific. For example, if you want to activate the "new document" option in the Word Processor. You cannot press "Enter-n" from the "Program" menu to activate this option. You must be in the Word Processor for "Enter-n" to activate the "new document" command. However, if you wish to activate the "display time and

Date" option in the "Utilities" menu, this can be done from anywhere on the unit using "Space-T" as this is a global hot key.

Let's begin going through the various menus. First, turn the unit on as explained in chapter 1. Now, bring up the "Program" menu by pressing "F1". The "Program" menu is the main menu of the BrailleSense. You can access all of the programming, option settings, utilities, and help from this menu, as well as the Play Store and installed Android apps.

The "Program" menu consists of 5 programs: File Manager, Word Processor, Notepad, E-mail and Exchange E-mail; then 7 sub menus: Media, Organizer, Web Tools, Extras, Utilities, Settings, and Help. Finally, at the bottom of the menu are the Play Store, All Apps, and information.

The first item in the "Program" menu is "File Manager". If "Space-4" is pressed, the next menu item is displayed and spoken. Pressing "Space-4" repeatedly, continues your navigational progress down the list of items in the "Program" menu. When you reach the bottom of the list, the "information" menu item is displayed as it is the last item in the "Program" menu.

You could also jump quickly to the "Information" menu item by pressing "Space-4-5-6" when "File Manager" is displayed. Press "Space-1-2-3," and the BrailleSense jumps back to the top. Press "Enter" on an item you want to execute or open. If you are in a submenu, and wish to return to the main "Program" menu, press "Space-E" or F4.

The "Media" menu contains the "Media Player", "FM Radio" and "DAISY Player".

The "Organizer" menu contains the "Address Manager" and "Schedule Manager".

The "Web Tools" menu contains the "Web Browser" and "Google Search".

The "Extras" menu contains the optional "Sense Dictionary", "Excel Viewer" and "Online DAISY".

The "Utilities" menu contains the "Calculator", "BrailleSense Math", "Display time and Date", "Display Compass Heading", "Wake Up Alarm", "Stopwatch", "Terminal for Screen reader", "Display Network Status", "Display Power status", "Format", "Sleep Timer" and "Upgrade BrailleSense Firmware".

The "Settings" menu contains "Set Time and Date", "Setup Internet", "Bluetooth Manager", "Menu Manager", "Backup/Restore Settings", "Change device Name", "Quick Start Guide", "Password Protection", "Initialize BrailleSense Options", "Voice options", "BrailleSense Global Options", "Android Backup and reset" and "Android System Settings".

The "Help" item opens a navigable version of the BrailleSense User Manual.

The "Play Store" item opens the Google Play Store where you can search for and download native Android apps.

The "All Apps" menu displays a list of all native Android apps installed on the unit. When the unit ships, this list contains stock Android apps and other Google basics like Docs, Sheets and Slides.

The "Information" dialog provides the copyright and version of the BrailleSense 6, as well as the operating system version, the version of the BrailleSense 6 firmware, and other useful items such as Mac address and Key number. When the "information" dialog is open, you can save the information about the BrailleSense 6 by pressing "tab (Space-4-5 or F3)", to navigate to the "Save" button and press "Enter". The unit displays "Successfully saved", and returns you to the "Save" button. The saved file is named "bs-information.txt", and is stored in "flashdisk" for easy access.

For more information on a specific program, please see the section of this user manual related to that program. For information on using Android apps, see chapter 12.

The "F1" command for accessing the "Program" menu is an example of a global hotkey. This means that pressing the "F1" key takes you to the "Program" menu immediately from anywhere on the BrailleSense.

There are also global quick launch commands for most BrailleSense programs, usually comprised of either F1 or F3 and a letter or dot combination. These quick launch commands immediately launch their associated application from anywhere on the BrailleSense. See section 15.2 for a complete list of the quick launch commands.

You can also search for apps and uninstall non-BrailleSense apps from the Main Menu/All Apps list. To uninstall any 3rd party app, press "Space-D" when you are focused on it. You are prompted if you are sure you wish to uninstall the app, press "Enter" to confirm.

Searching for apps may be useful if you have many Android apps installed, or if you want to see which apps you have in a certain category, such as mail or music. Press "Space-F" from the Main Menu/All Apps list, and you are prompted to enter your search term. Type it, and press Enter, and results matching your search are displayed.

Press "Enter" on an app in the list to execute it. Press "Space-D" on any 3rd party app to uninstall it.

2.2.1 CONTROL SYMBOLS

The BrailleSense 6 uses several control symbols to note the type of information being displayed: examples are list items, menus, and edit boxes. These symbols can appear "Before" or "After" the text, or the appearance of this information

may be set to "off", determined by the "Control information" setting in the "Global Options". For example, if "Control information" is set to "before", and you bring up the Word Processor's program menu, The BrailleSense displays, "MN File (f) pull down".

- Note: In the Web Browser, the control symbols are displayed even if the "Control information" is set to "off".

The following table shows the types of controls and the symbols that are used to indicate them.

Name

Symbol

Comments

Menu

MN

A menu that has submenus.

Menu item

MI

A menu that does not have a sub menu.

List item

LI

An item that is part of a list.

Edit box

EB

After this symbol, the edit box name is displayed. You can find the edit box after the edit box name. In the web browser, the symbol "EB" may be followed by an edit box without an edit box name. Edit boxes that begin with the symbol "EB" are one line.

(Ex: "EB Last name: (cursor)" in the address manager program or "EB (cursor)"

Multi edit box

MEB

This indicates a multi-line edit box.

(Ex: "MEB Subject: (cursor)" in the e-mail program)

Computer edit box

CE

An edit box requiring ASCII Braille input.

(Ex: "CE To:" in the Email program)

Combo box

CB

Indicates a combo box in which you can choose an item by pressing the up scroll button or the down scroll button.

(Ex: "CB Type:" in the "Open dialog box" of the Word Processor)

Edit combo box

ECB

choose the desired item by pressing the up or down scroll button, or by directly typing in the item name.

(Ex: "ECB File name: (cursor) was in the "Open dialog box" of the word processor program)

Prompt button

PB

PB follows any question in which you have to choose "Yes" or "No." You can toggle between "Yes" and "No" by pressing "Space" or "Backspace" button.

(Ex: When you exit from the word processor, "PB Save: Yes")

Static box

ST

ST indicates the current status.

(Ex: "ST noname.hbl/insert mode/write" in the word processor program)

Link

LN

Indicates a link on a web page.

Anchor

ANC

Indicates an anchor on a web page.

Radio button

RB

Indicates a radio button, cycle among items using up scroll button, down scroll button, space, or Backspace.

(Ex: "RB Attribute: Write" in the "Information dialog box" of the file manager)

Radio button select

SRB

Indicates the currently selected radio button in the web browser

Radio button unselect

URB

It indicates the radio button is not selected.

Check box check

SCHB

Indicates the check box is selected.

Check box uncheck

UCHB

Indicates the check box is not selected.

- Note: Links, anchors, and radio buttons are controls that are only used in the Web Browser.

The symbols enable you to understand what type of input control or menu you are working with. For example, if "MN," "MI," or "LI" are displayed, you know you can move from one item to the previous and next items by pressing the "Up" and "Down" scroll keys. If "ECB," "EB," "CB," or "MEB" are displayed, you know you are in an input area where you must type the. If "BT" is displayed, you can press "Enter" to activate the "button".

2.3 Using the Function Keys

There are four function keys on the BrailleSense. You can use these function keys to open the menus, cancel tasks, and move to other items.

The "F1" key opens the BrailleSense "Program" menu from anywhere on the unit, and is used in combination with other keys to open various BrailleSense programs.

From within BrailleSense programs, the "F2" key opens the program specific menu, similar to pressing "Alt" in Windows.

- Note: you may also use "Space-M" to perform this function.

The "F3" key functions as "Tab ("Space-4-5")". "Space-F3" serves as "Shift-Tab ("Space-1-2")". F3 is also used together with other keys to open BrailleSense programs.

The "F4" key functions like the "Escape" key on a PC. You can also use "Space-E". Please note that "Escape" does not exit a program, as does "Space-Z" but rather, exits menus, dialogs, or prompt boxes. It also serves as the Back button in native Android apps.

You can use the function keys in combination to perform various other tasks. Pressing "F1" and "F2" at the same time functions as "Page Up". Pressing the "F3" and "F4" keys together serves as "Page Down". "F1" and "F4" together opens the task list. Pressing the "F2" and "F3" keys together cycles through the running programs; similar to pressing "alt-tab" on your PC. The Task List and task switching are explained in greater detail in section 2.9.

The keys to the left and right of the spacebar are referred to as Control and Alt. This is because, in native Android applications, these keys function as Control and Alt on a QWERTY keyboard so that you may take advantage of native Android keyboard shortcuts when they are present. In BrailleSense specific applications, these keys operate similarly to "Space-1" and "Space-4" and provide an alternative method for scrolling through menus and lists.

2.4 The AC Power Supply and the Battery Pack

To check the battery power status, press "Space-1-6." The unit displays the following 2 items:

1. Battery power level: expressed as a percentage of the remaining battery power compared to a full charge.

2. Power source (battery, USB or AC power)

If the battery power level drops under 15 percent, the BrailleSense displays the battery status on the Braille display, and it announces that the battery has less than 15 percent of its power remaining. If the battery power level falls below five percent, the BrailleSense gives you a warning indicating that your battery power level is under five percent, and the BrailleSense will be automatically put to sleep in one minute, unless you connect it to a power supply. When the level reaches 2 percent, the unit automatically shuts down.

When the BrailleSense is shipped, the battery pack is not installed. This is done to prevent damage during shipping. When you insert the battery in to the BrailleSense for the first time, the unit may display an inaccurate battery power level. Before using the BrailleSense for the first time, please charge the battery for at least 5 hours. You may use the unit during the initial charge, however, it is recommended if you do, that the unit remain connected to AC power for at least 5 hours to fully charge the battery.

When you remove the battery from the BrailleSense, make sure your unit is turned off; especially when the AC power adapter is not connected. If you remove the battery while the unit is on and AC power is not connected, you will lose all data stored in RAM, including open documents, emails, and contacts or appointments that have not been backed up.

BrailleSense can also be charged via a USB 3.0 or Micro USB cable. It is compatible with Qualcomm quick chargers for more rapid charging. You may also charge BrailleSense's battery pack via Qi compliant wireless charging decks.

2.5 Controlling the Volume, Speed, and Pitch of the TTS

Press "Backspace-F4" to increase the voice volume. Press "Backspace-F1" to decrease the voice volume. Press "Space-F4" to increase the speech rate. Press "Space-F1" to decrease the speech rate. Press "Enter-F4" to raise the pitch and "Enter-F1" to lower it.

When you press any of the above key combinations, the BrailleSense announces the status of its volume, rate, or pitch. The changes in the characteristics of the voice remain in effect until you change them again, or a hard reset is performed either manually or due to lack of power to the unit for an extended period of time.

- Note: the volume of the voice is controlled independent of the Main Volume on the unit, thus, you can adjust it above or below the volume of the system sounds, media playback, etc. using the Voice volume control described above. You can control the "Main Volume" of the unit using "Backspace-Space-F1" to lower it and "Backspace-Space-F4" to raise it. You can also use the Volume Up and Volume Down buttons on the left side of the BrailleSense to control master volume.

2.6 Using Speech and Braille

When using the BrailleSense, you can choose to read using Braille only or speech only. You can also use both Braille and speech simultaneously. However, you cannot turn off both Braille and speech as you would have no method of receiving information from the BrailleSense. Use "Backspace-F2" to toggle the speech on and off. Press "Backspace-F3" to toggle the Braille on and off. You can also control these settings in the Global options.

2.7 Entering Text

In a document, "edit box" or "multi edit box" on the BrailleSense, you can enter either grade 1 or grade 2 Braille. By default, the unit uses UEB Braille, and you are free to use UEB, even when writing e-mail or Web addresses. However, if using the traditional US Braille code, some areas of the unit, such as the "to" field of the E-mail program, contain a "computer edit box" requiring you to enter text in computer Braille.

To enter a capital letter in computer Braille, use the upper case indicator (Space-U" before the letter you want capitalized. For example, if you want to enter H in a computer edit box, press "Space-U" then H. If you press "Space-U" twice, caps lock is turned on. Press "Space-U" again to turn off caps lock.

You can also press "Space-Backspace" simultaneously with the letter to capitalize it. To input the "at sign," you may either use "Space-U" followed by dot 4, or "Space-Backspace-dot 4".

To enter numbers in contracted or uncontracted Braille, use the "number sign followed by the number. However, when using computer Braille, you use no number sign and must enter numbers as "dropped" numbers (Nemeth numbers).

2.8 Multi-Tasking

BrailleSense can run many tasks at once. For example, you can work with the Word Processor while listening to music, and place calculations into a document at the same time. To do this, open the Media Player and play a music file. While the audio is playing, press the "F1" key to bring up the "Program" menu and launch the Word Processor.

If you try to run a program that is currently running, The BrailleSense does not open a new instance of the program, but rather, returns you to the instance of the program that is already running. For example, while working with a document called "Notes" you decide to execute the "File Manager." Next, you navigate to a document called "journal" in the file list and press "Enter." The Word Processor does not load a second time, but rather, "journal" is opened in the same instance of the Word Processor in which "notes" is already loaded. You can switch among multiple open documents in a single instance of the Word Processor with F3 and Space-F3. Switching documents is explained in more detail in section 5.

To switch between currently running programs press "F1-F4" to open the Task List. The Task List shows all the currently running programs in list form. To return to any application, simply navigate to it and press "Enter". To force closing of an application in this list, press "Space-D". To close all applications, press "F3" to tab to the "Close All" button, and press "Enter", or press "Enter-Space-D".

If you have pressed "F1" to bring up the "Program" menu, or "F1-F4" to open the Task List and you decide not to access a different application, you can press "escape" ("Space-e" or "F4") to return to your previous location on the unit.

You can use "F2-F3" to immediately cycle among all running programs without opening the Task List. This is similar to using "alt-tab" on your PC. If you are running three programs and are in the second program, pressing "F2" and "F3" moves you to the third program. If you press "F2-F3" again, you are placed in the first program.

In the above case, if you were to press "Space-Z" to close the currently open program, you are then placed in the previous program. However, if there is only one program running, pressing "Space-Z" closes the program and places you in the "Program" menu.

2.9 Switching Media Modes

The media buttons on the front panel perform different functions according to the location of the "Media Mode" switch.

If you slide the "Media Mode" switch to the left-most position, the BrailleSense announces, "application mode" and the media buttons control operation of the current application. If you place the "Media Mode" switch in the center position, the BrailleSense says, "DAISY mode" and the media buttons control playback of DAISY content. If you slide the "Media Mode" switch to the right-most position, the BrailleSense announces, "media mode" and the media buttons control media playback.

It is very important to pay attention to the position of the "Media Mode" switch to achieve desired operation of the media buttons. For example, if you are using the Media Player, if the "Media Mode" switch is placed in the center position, the media buttons on the front panel control operation of the DAISY Player, even though your focus is in the Media Player. More detailed explanations of the functions of the Media buttons is given in the Mobile Screen Reader, DAISY Player, and Media Player sections in this manual.

2.10 Switching Key Lock

You can disable the keys on the BrailleSense to prevent the accidental pressing of keys using the key "Lock" switch on the front panel.

When the key "Lock" switch is placed in the left-most position, BrailleSense announces, "all keys locked". This means all keys on the unit are disabled including the on/off switch. If the key "Lock" switch is placed in the center position, BrailleSense says, "top panel locked". This means you can continue to use the buttons and switches on the front panel (media mode switch, media buttons and on/off switch), however, you cannot use any of the keys on the top

face of the unit. If the key "Lock" switch is placed in the right-most position, BrailleSense announces, "unlocked". This indicates all keys are operational.

To lock and unlock only the keys on the front media panel, press the "Back" and "Forward" media buttons simultaneously. The unit announces "Media keys locked". If you press the keys again, the unit announces "Media keys unlocked", and the media buttons are again operational.

2.11 One-Handed Mode

BrailleSense provides a One-handed mode for users restricted to the use of one hand for Braille input. To toggle One-handed mode on/off, press "Backspace-1-2-3". You can also turn One-handed mode on or off in the "Global Options" dialog.

When One-handed mode is on, use the following methods to input text:

1. To type a character, press the dots in any combination or order. When you have pressed all dots, press "Space" to complete the entry of the character. For example, if you want to type "h," you press "dot 1" and "dot 2" separately or together and then press "dot 5." Finally, press "Space".
2. To press a command that includes "Space," press "Space" both before and after the dot combination. For example, to press "Space-o (dots 1-3-5)," press "Space" then press "dot 1", "dot 3" and "dot 5" separately or together in any order. Then press "Space" again.
3. To press a command that includes "Backspace" or "Enter," press "Backspace" or "Enter" before the dot combination. Then press keys that make up the dot combination. Finally press "Space". For example, to press "Enter-l (dots 1-2-3)", press "Enter", and then press "dots 1-2-3." then, press "Space".

4. To press "Space," "Backspace" and "Enter", press the key twice. For example, to press "Space" to create a space between words, press "Space" twice.

The operation of the scroll keys, function keys, and media buttons remains the same as in normal mode.

If One-handed mode is on, the BrailleSense announces this each time it is powered on, as well as how to return to normal operation.

2.12 Using Typing Mode

While you are typing in a document, you may wish to turn off the navigation keystrokes, especially if you type very fast as you may accidentally navigate when you intend to enter text. To turn on "Typing Mode", press "Backspace-Space-Enter-M" from within a document. "Start typing mode" is announced and displayed.

When using "Typing Mode", if you press "Space-dot 4", instead of moving to the next line, the "Space" and "Dot 4" are entered into the document.

To cancel "Typing Mode", Press "Backspace-Space-Enter-M" once again. "End typing mode" is announced and displayed. After "Typing Mode" is canceled, navigation commands return to normal operation.

2.13 Creating System Logs and Screen Recordings

There may be times when it is necessary to report an issue in detail to support staff in order to clearly demonstrate a problem or process. BrailleSense offers several options for providing a picture of any given situation you may need to tell us about.

- Take a screen shot. Hot-key: F4-S.
 - i. It is saved in the “Music” folder, as that’s where general media is stored.
 - ii. A sub folder called “ScreenShots” is created, and each image is saved according to the date and time it was taken.
- Create a screen recording. Start/stop Key: F4-R.
 - i. It is saved in the “Music” folder.
 - ii. A sub folder called “ScreenRecording” is created,
- Create a system log and screen recording simultaneously. Start/stop key: F4-L.
 - i. A zip file is automatically created containing both the log data and the screen recording.
 - ii. The zip file is saved to Flashdisk/logData.
- Create a system log only. Start/stop key: Backspace-Space-Enter-2-3-5-6 from the Main Menu.
 - i. A log file is created and saved to Flashdisk/LogData, named according to the time and date it was created

3. Customizing your Notetaker

When you receive your notetaker, one of the first things you will want to do is to configure it to act the way you want it to. The "Settings" menu allows you to configure and customize your notetaker to meet your specific needs and preferences.

To open the "Settings" menu, press "F1" to bring up the main menu. Press "Space-4" until you reach "Settings" and press "Enter". Or, just press S from the main menu to quickly jump to and open the "Settings" menu.

The "Settings" menu contains the following items: "Set Time and Date", "Setup Internet", "Bluetooth Manager", "Backup/Restore BrailleSense Settings", "Change Device Name", "Quick Start Guide", "Password Protection", "Initialize BrailleSense Options", "Voice options", "BrailleSense Global Options", "Android

Backup & Reset” and “Android System Settings”. The following sections discuss each of these items in detail.

3.1 Set Time and Date

To set the BrailleSense's internal clock, press "Enter" on "Set Time and Date" in the "Settings" menu. Or press T to open the "set time and date" dialog from the menu. You can launch the "Set Time and Date" dialog from anywhere on the unit by pressing "F3-T".

The set time and date is a dialog, consisting of the following: "Time zone", "Time format", "Time", "Date Format", "Date", "Synchronize with time server", a "Confirm" button, and a "Cancel" button. You can move among these items by pressing “F3” or “Space-F3”.

When you activate "Set Time and Date", "Time zone?" appears followed by the default time zone. You can choose your desired time zone by navigating to it using "Space-1" or "Space-4" or by typing the first letter of the time zone. Once done, move to the next item using "F3".

The next item is "Time format", the options for which are 12 hour and 24 hour time. You can toggle between the two using "Space-1" or "Space-4".

The next item is "Time". When typing the time, separate the hour and minute using a space or a colon.

If you select 12 hour clock in the "Time format", A.M. or P.M. is displayed after the hour and minute. In the 12 hour clock, you can toggle A.M. it P.M. by pressing "Space-X".

Tab to "Date format". You can choose to display the date in either the "short" or "long" format. The short format displays the date numerically, while the long

format displays the Month spelled out and shows the date as it would appear in literary form. Toggle between the two using "Space-1" and "Space-4".

The next item is "Date". Type the "Date" month, date, and year as mm/dd/yyyy. Use 2 digits for the month, 2 digits for the day, and 4 digits for the year, separating them by a space or a slash.

The next item is "Synchronize with time server", used to synchronize time and date with a time server on the Internet. This function is extremely useful when the date and time of BrailleSense is faster or slower than the current time. To execute this function, your BrailleSense must be connected to the Internet. When connected, press "Enter" on the "Synchronize with time server" button. After synchronization, it displays the synchronized time in the "Time" edit combo box. The synchronized date is also displayed in the "Date" edit combo box.

If you do not wish to enter date and time information directly, you can navigate using the following commands:

When setting time:

Move forward 1 hour: Space-4

Move back 1 hour: Space-1

Move forward 10 minutes: Space-5

Move back 10 minutes: Space-2

Move forward 1 minute: Space-6

Move back 1 minute: Space-3

When setting date:

Move to the next day: Space-6

Move to the previous day: Space-3

Move to the next week: Space-5

Move to the previous week: Space-2

Move to the next month: Space-4

Move to the previous month: Space-1

Move to the next year: Space-5-6

Move to the previous year: Space-2-3

When you have completed entering all of the information, press "Enter" to save your settings, or "Tab" to "Confirm" and press "Enter", and you are returned to the "Settings" menu.

3.2 Setup Internet

The "Setup internet" function is used to configure your BrailleSense to use a LAN or Wireless connection to access the Internet.

- Note: to use a LAN connection, you will need a USB to Ethernet adapter.

To activate "Setup internet", press "Enter" on "Setup internet" in the "Settings" menu, or press "I". You can open the "Setup Internet" menu from anywhere on the unit by pressing "F3-I".

3.2.1 SETUP WIRELESS LAN

Before setting up a Wireless LAN connection, be sure wireless is turned on in the "Global Options" dialog. You can toggle wireless on and off from anywhere on the BrailleSense using "Backspace-1-4-5-6". In addition, when you choose this option, if Wireless is not turned on, BrailleSense automatically activates Wireless LAN. To connect to a wireless network, navigate to the "Wireless LAN" option in the connection types list under "Setup Internet" and press "Enter".

The "Wireless LAN" dialog consists of the "access points" list, "Advanced" button, "Options" button and "Close" button. You can move among these controls by pressing "F3" or "Space-F3". If wireless is not turned on, the "access points" list displays "no items".

When wireless is on, this list shows the wireless "access points" (networks) in range of your notetaker. The "Access points" list displays the "access point name", "encryption", and "sensitivity" of each item. (Ex: linksys, encryption: WEP, sensitivity: good.) If there is more than one access point in the list, you can move among them by pressing "Space-1" or "Space-4". Move to the "access point" to which you want to connect, and press "Enter". If the access point has no encryption, the BrailleSense announces, "Connecting to (access point name)". If connection is successful, the connecting sound is heard. If the network is encrypted and requires a network key, when you press "Enter" on the access point, the "Network key" edit box is displayed. Type the network key in computer Braille in the edit box and press "Enter". If your settings are correct, the unit says, "Completed setting profile" and the connecting sound is heard.

The "Advanced" dialog is used to modify an existing profile, manually add a new profile, or check the settings of an existing profile. Pressing "Enter" on the "advanced" button, while wireless LAN is turned on, displays a list of the wireless profiles saved on the BrailleSense.

The "Advanced" dialog has 5 controls: the "profile list", "Connect" button, "Add" common dialog button, "Remove" button, "Properties" common dialog button, and "Close" button. You can navigate among these controls by pressing "F3" or "Space-F3". Navigate the items in the Profile list using "Space-1" and "space-4". Press "Enter" to activate one of the buttons.

Use the "Add" button: to manually add a profile, specifically where an SSID is hidden, and thus does not appear in the "access points" list. The "Add" dialog" contains 10 controls.

- 1) "Setup IP: Automatic" combo box: You can switch between automatic and manual by pressing "Space-4" or "Space-1". If you select "Manual", you must enter your IP address, subnet mask, gateway IP address, and DNS IP address. You can move among these computer edit boxes by pressing "Space-4" or "Space-1". Press "tab (Space-4-5)" to move to the next control.
- 2) "Network name (SSID)" computer edit box: Type the network ID in the computer edit box.
- 3) "Network mode" combo box: You can select among "Infrastructure", and "802.11 ad-hoc" mode by pressing "Space-1" or "Space-4".
- 4) "Network authentication mode" combo box: Select among "Open", "Shared", "WPA-PSK", and "WPA2-PSK" by pressing "Space-1" or "Space-4".
- 5) "Data encryption" combo box: Select the data encryption method by pressing "Space-1" or "Space-4". If you select "Open" or "Shared" in network authentication mode, you can choose among "Disable" and "WEP". And if you select "WPA-PSK" or "WPA2-PSK" in network authentication mode, either "AES" or "TKIP" can be selected in the "Data encryption" combo box.
- 6) "The key is provided automatically" check box: If the "access point" does not provide a key for users automatically, uncheck this check box by pressing "Space". The default is checked.
- 7) "Network key" computer edit box: This is a computer edit box to type the network key if the key is not provided automatically.
- 8) "Key index" edit box: This is an edit box to type the key number of the wireless access point that matches the network key input.
- 9) "Confirm" button: Pressing this button applies the changes.
- 10) "Cancel" button: You can select this button to cancel the setup.

- Things to note:

1) If "Data encryption" has been set as "disabled", press "F3" and you are taken directly to the "Confirm" button.

2) "Network key" and "Key index" is displayed only if you uncheck "The key is provided automatically".

To delete a profile item in the profile list, select the profile item you wish to delete, and press R or "Tab" to the "Remove" button and press "Enter".

To get information about a selected profile item in the profile list, "Tab" to the "Properties" button and press "Enter". In the "Properties" dialog, you can reconfirm or modify the information for each profile.

To exit the "Advance" dialog, press "Space-E" or "Space-Z".

The "Options" dialog allows you to set how the BrailleSense scans for and detects wireless networks. The "Scanning Always Available" option allows you to choose whether BrailleSense auto scans for available networks, even when you don't open the Setup Internet dialog. The "Wi-Fi Frequency Band" determines what Wi-Fi networks are detected and used. Choose from "Auto", "5GHz" and "2.1GHz". Use "Space" to adjust the options in the dialog, and press "Enter" to save your settings.

3.2.2 LAN SETUP

Before attempting to set up a LAN connection, connect one end of a LAN cable to the BrailleSense via a USB to Ethernet adapter, and the other to a router or direct LAN connection to the Internet. ?. From the list of connection options in "Set up Internet", navigate to "LAN" and press "Enter".

"Setup IP: Automatic" is displayed. If your service provider does not require you to use a static IP address, simply press "Enter" on this option for automatic setup. For the automatic setup, your host server should provide DHCP service (Dynamic Host Configuration Protocol).

To set up your IP configuration manually, move to "Manual" by pressing "Space-1", and press "Enter". The following settings are displayed: "IP", "Subnet mask", "Gateway", and "Primary DNS". These settings require specific numeric addresses. Please contact your Internet service provider for the correct addresses. Use computer Braille to type the information for each setting, using "Space-1" and "Space-4" to move among the settings. When you have entered the information for each item, press "Enter" to save the settings. The unit announces, "Setting completed". And you are returned to "Setup internet". Your Internet connection should be active.

3.3 Bluetooth Manager

The BrailleSense is Bluetooth capable, or rather, you can connect the BrailleSense to various devices such as keyboards, printers, audio devices, and other computers without the need for cables. You can also use Bluetooth to transfer files, or to connect the BrailleSense as a Braille display with a compatible screen reader. Bluetooth is the latest in wireless technology and enables wireless connectivity via a Personal Area Network (PAN). Unlike a traditional Wireless Local Area Network (WLAN), these networks are limited to approximately 30 feet or less. Bluetooth enables users to exchange various types of information over this wireless connection.

You can toggle Bluetooth on/off from anywhere on the unit using "Backspace-dots-3-4-5-6".

To access the Bluetooth Manager from the program menu, navigate to the "Settings" menu, and press "Enter". Then, navigate to "Bluetooth Manager", and press "Enter". You can also press "F3-B" to launch the Bluetooth Manager from anywhere on the unit.

3.3.1 BLUETOOTH DEVICE LIST

When the Bluetooth Manager is activated, you are placed in a dialog with 2 items: a paired devices list, and a "Search" button to scan for available devices. For example, if you have a computer with Bluetooth enabled nearby, the BrailleSense attempts to find it. It may take 30 to 40 seconds to discover all of the Bluetooth devices in range. The BrailleSense announces and displays, "Scanning for devices. Please wait..." Full cells are filled in on the Braille display to indicate scanning progress.

When a Bluetooth device is detected, the BrailleSense announces "found (Bluetooth device name)".

- Note: while the BrailleSense is scanning for Bluetooth devices, it cannot respond to key presses. In addition, you may not cancel the scan once it has begun, but must wait for its completion before the BrailleSense returns to normal operation.

When the scanning process completes, you are placed in the Bluetooth Device List. If the BrailleSense has found other Bluetooth devices in range, they are displayed as "device name (untrusted (or trusted)) xx/yy" where xx is the number at which the device appears in the list, and yy is the total number of devices. "Trusted" indicates that the service has been authenticated, "untrusted" indicates the service has not been authenticated.

If there are no other Bluetooth devices in range, the BrailleSense announces, "no items". You can press "Space-1" or "Space-4" to move through the list. To select a device from the list, press "Enter" on the device name.

If the Bluetooth device is trusted, settings for connecting the device are automatically saved in the BrailleSense. If you want to delete this information, press "Space-D" on the Bluetooth device name in the list. The BrailleSense

announces, "Device settings removed". Removal of the settings applies only to the current device.

From the Bluetooth Device List, you can access the menu by pressing "F2". The "Bluetooth Device List menu" contains "Rescan for Devices", "Delete Pair Information", "Options" and "Exit". You can navigate among the menu items using "Space-1" or "Space-4".

The first menu item is "Rescan for Device". This option initiates a new scan for other Bluetooth devices. To activate this item, press "Enter" on "Rescan for Devices" in the menu. Or you can activate it directly from the list by pressing "Enter-R".

The next menu item is "Delete Pair Information". This item allows you to remove the connection settings for the current device. To activate this item, press "Enter" on "Delete Pair Information" from the menu, or press "Space-D" to activate it from the Device List.

The "Options" item lets you determine if the BrailleSense is discoverable to other Bluetooth devices. Press "Space" to check or uncheck the option.

The final item in the Bluetooth Device List menu is "Exit". This option exits the Bluetooth Manager program. To activate this item, press "Enter" on "Exit" from within the menu, or exit the program any time by pressing "Space-Z".

3.3.2 BLUETOOTH SERVICE LIST

When you locate a Bluetooth device to which you'd like to connect, press "Space-1" or "Space-4" to navigate to the device in the Bluetooth Device List, and press "Enter" on the device name. The unit announces, "Scanning for services. Please wait..." Please note The BrailleSense does not respond to key presses during the scanning process.

The BrailleSense supports the following Bluetooth services: FTP, Serial port, USB port, and headset. To access any of these functions, be sure that the remote Bluetooth device is in Discoverable mode, and that the service you wish to access is enabled on that device. It is possible to have certain services enabled on your remote Bluetooth device, while disabling others. The BrailleSense only sees and displays services which are enabled on your remote device, even if others are available.

When the BrailleSense has found all of the available services, the services are displayed in a list in the following format: "service name xx/yy" where xx is the number at which the service appears in the list, and yy is the total number of available services. You can move through the available services by pressing "Space-1" or "Space-4". Press "Backspace" to return to the Bluetooth Device List.

From the Bluetooth Service List, you can access the menu by pressing "Space-M" or by pressing "F2". The "Bluetooth Service List menu" contains "Open FTP", "Disconnect" and "Exit". You can navigate the menu items using "Space-1" or "Space-4".

The "Open FTP" menu item allows you to quickly open a direct FTP connection with a remote Bluetooth device without accessing the Device and Service lists. However, you must have connected the FTP service previously for the BrailleSense to allow you to transfer files. If you have not connected the FTP service, and you activate this function, the unit announces, "FTP service disconnected".

You can access "Open FTP" by pressing "Enter" on it from within the menu, or by pressing "Enter-T" directly from the Bluetooth Service List. The BrailleSense says, "Connecting service". Detailed information on connecting various service types is covered later in this chapter.

The "Disconnect" menu item allows you to disconnect services with other Bluetooth devices. You can access this item by pressing "Enter" on "disconnect" from within the menu. Or you can activate it directly from the Bluetooth Service List by pressing "Enter-D".

If there are no services connected and you activate this function, the unit says, "There is no service connected".

3.3.2.1 Connecting a Service

To authenticate a service and connect to it, press "Enter" on the service. The BrailleSense announces, "Connecting service". You are then prompted for a "PIN code:" You must use ASCII characters when you type the pin code. This edit box only appears when you have not previously connected to a service, and a pin code is required by the device. You can exit the edit box or delete text you have typed by pressing "Space-Z" and you are returned to the Bluetooth Service List.

When you finish typing the pin code, press "Enter". The unit announces, "Please wait for authentication". If the remote device is setup correctly, and has not been set up to allow this device to always connect, the remote device also asks for a pin code. Please be sure to enter the same pin code on both devices. If the remote Bluetooth device pin code matches the pin code on the BrailleSense, the service should authenticate. If connection is successful, the unit announces, "PIN code authentication accepted".

You are then prompted, "Please wait. Access permission". If the remote Bluetooth device does not give access permission, the unit says, "Connection failed", and you are returned to the Bluetooth Services List. If the remote Bluetooth device gives you access permission, the BrailleSense says "Connection succeeded". If the connection succeeds, "Connected" is displayed

after the service in the Bluetooth Service List. Some services may be connected without inputting a pin code depending on the Remote Bluetooth device settings.

3.4 Menu Manager

Using the Menu manager on the BrailleSense, you can manage what menus and/or programs are displayed in the Main Menu, sub menus, and All Apps. You can show or hide almost any program in the Main Menu with a few exceptions. You may not hide the File manager, or the Menu Manager itself.

To execute the "Menu manager", from the "program" menu, press S" to open the "Settings"" menu. Press M to open "Menu manager". The "Menu manager" dialog is displayed. You can also launch the "Menu Manager" from the Main Menu by pressing "Enter-H".

The dialog of the "Menu manager" consists of the following items: The menu/program list, Confirm button and Cancel button. To navigate among the programs, press "Space-1" or "Space-4". Press "F3" or "Space-F3" to navigate to the "Confirm" and "Cancel" buttons.

Menus are displayed if they have programs accessible inside them. If you have chosen to hide all the programs in a given menu, this menu is not displayed in the Program menu.

To hide unused programs on the BrailleSense, press "Space" on each of the programs you wish to hide in the list. Hidden programs will display an Asterisk (*) next to their names. Press "Enter" to save your settings, or tab to "Confirm" and press "Enter". After a couple of seconds, the you are returned to the program menu, and the programs you chose to hide are no longer displayed.

To unhide the programs, return to the "Menu Manager" dialog, and press "Space" on the programs you wish to unhide, and press "Enter" to save the settings.

*Note: Using the “Initialize options” function described in section 3.9, all menu items will return to their default status of being displayed.

3.5 Backup/Restore BrailleSense Settings

If you hard reset your BrailleSense or initialize the BrailleSense Options, the main option settings and settings for each program are initialized to the default values. If you have created a backup of your settings, you can restore the option settings after the hard reset.

You can run this utility by pressing K from the "Settings" menu. You can also launch the "Backup/Restore BrailleSense Settings" dialog from anywhere on the unit by pressing "F3-K".

The "backup/restore" dialog contains: the "program option check box list", a "Select all" button, a "Backup" button, a "Restore" button, and a "Cancel" button. You can move among the check boxes in the "program option check box list", by pressing "Space-1" or "Space-4". You can move among the controls in the dialog by pressing "F3" or "Space-F3".

The "program option check box list" contains "Options settings", "File manager options", "Word processor options", "E-mail Options", "Media Player options", "Radio Options", "DAISY Player options", "Address manager options", "Schedule manager options", "Web browser options", "Excel Viewer Options", "Wake Up Alarm Options" and "Bluetooth manager options".

In this list, move to the program options you want to backup or restore and check the item by pressing "Space". You can uncheck a checked item by pressing "Space" again. To select all of the program options in the list, press "Enter" on the "Select all" button.

After selecting all the options you want to backup or restore, press "Enter" on the "Backup" or "Restore" button, or press "Enter-B" to backup or "Enter-R" to

restore. If you select the "Backup" button, you are prompted to select a path for storing the backup file. You can accept the default path, or choose an alternative. Navigate to the folder in which to store the backup files using normal file/folder navigation commands. Select the folder or drive in which to store the backup by pressing "Space", and press "Enter" to start the backup process. If you select the "Restore" button, you are prompted for the path from which to restore backup files. Again, use normal file and folder navigation to select the folder containing your backup files. If you select a folder in which there is no backup, you receive a message indicating there is no backup option file. To restore your settings, select the folder containing the backup files by pressing "Space", then press "Enter" to initiate the restore process.

3.6 Change Device Name

The "Change device Name" function allows you to change the Bluetooth/network name of your unit. By default, the BrailleSense is named something like "BrailleSense035". You may wish to give it a more personalized name in order to identify it more easily when pairing with another device.

You can access the "Change device Name" dialog by navigating to the "Settings" menu, then to "Change Device Name". You can also launch the "Change device Name" dialog from anywhere on the unit by pressing "Backspace-Enter-E".

The "Change device Name" dialog consists of 3 controls: a "Device Name" edit box, a "Confirm" button, and a "Cancel" button. Use "F3" and "Space-F3" to move among the controls.

To change your device name, type a new name in the edit box and press "Enter". Or, after typing the new name, tab to "confirm" and press "Enter". If you

decide you do not wish to change your device name, tab to the "Cancel" button and press "Enter", or press "Space-E" or "Space-Z".

3.7 Quick Start Guide

The Quick Start Guide runs a getting started wizard that introduces you to BrailleSense and its functions, and lets you set basic preferences. This guide automatically launches the first time you start your unit. You can relaunch it at any time by selecting the "Quick Start Guide" option in the "Settings" menu.

When you press "Enter" on this option, you are asked, "Would you like to reboot your system and launch the Quick Start Guide?". Press "Enter" for Yes, or navigate to 'No' with "Space-4" and press "Enter" to cancel this operation.

- Note: when you launch the Quick Start Guide, all options are initialized to factory defaults, so that they can be reset using this wizard.

3.8 Password Protection

The Password Protection item allows you to set a password that must be entered when the unit is powered on, comes out of Sleep Mode, is upgraded, or the disk is formatted.

WARNING!! It is imperative that you create a password that you can easily remember, or that you store in another location for easy retrieval! Due to Android security safeguards, we are not able to send unlock codes as has been the case with older BrailleSense models. If you set a password using this function, or using the Android system settings, and you are not able to remember or retrieve it, it will be necessary to send the unit to HIMS/SELVAS for unlocking, during which, all data will be lost. As remembering and retrieving personal passwords is

the responsibility of the customer, any costs associated with transporting and/or unlocking the unit are also the responsibility of the user.

To open the "Password Protection" settings, from the Main menu, navigate to "Settings" and press "Enter". Press A to jump to and open "Password Protection". You can open "Password Protection" from anywhere on the unit by pressing "F3-N".

The "Password Protection" dialog consists of 4 items: "set/modify Password" dialog, "Password Configuration" dialog, "Set Password to factory defaults", and a "Close" button. Use "F3" or "Space-F3" to move among the controls in the dialog.

3.8.1 SETTING AND MODIFYING PASSWORDS

When you open the "Password Protection" program, you are placed on the "Set Password" option. Press "Enter" to open the "Set Password" dialog. The "Set Password" dialog consists of 4 controls: "Password" edit box, "Retype Password" edit box, "Confirm" button and a "Cancel" button".

Type the password you wish to use into the "Password" edit box. Tab to the "Retype Password" edit box, and type it again. To save the password, press "Enter", or Tab to "Confirm" and press "Enter". To cancel saving the password, press "Space-Z", or Tab to the "Cancel" button and press "Enter".

When a password is saved, the "Set Password" option is replaced with "Modify Password". Once you have entered a password, you can modify in very much the same manner. The "Modify Password" dialog consists of all the same controls, except you must type your current password, before you can type a new one. Once you have typed your current password, then the new password, you must re-type the new password. Press "Enter" on the "Confirm" button to

save the new password, or "Enter" on the "Cancel" button to cancel the modification.

3.8.2 PASSWORD CONFIGURATION

The "Password Configuration" dialog allows you to define when you are prompted to enter the password. When you press "Enter" to open the "Password Configuration" dialog, you are prompted to enter your password to gain access to this configuration. Type your password, and press "Enter".

The password configuration settings consist of 4 radio buttons: "Prompt for password on boot", "Prompt for password when in Sleep Mode", "Prompt for password when formatting" and "Prompt for password when upgrading." Use "Space-1" and "Space-4" to toggle these buttons between "Yes" and "no". When you have chosen your options, press "Enter" on the "Confirm" button to save your settings. If you want to cancel the configuration, press "Enter" on the "Cancel" button.

3.8.3 RESET PASSWORD TO FACTORY DEFAULTS

Use this option to reset the password configuration and return your unit to its factory state of no password protection.

When you press "Enter" to open this option, you are prompted, "Do you want to initialize the password settings?". Use "Space-1" and "Space-4" to choose between "yes" and "No". If you choose "yes", you are prompted for your current password. Type it, and press "Enter". The unit announces, "Password settings have been initialized" and you are placed on the "Set Password" option.

3.9 Initialize BrailleSense Options

You can use the "Initialize BrailleSense Options" item to restore the BrailleSense 6 to the factory default settings.

To launch this function, navigate to it in the "Settings" menu, and press "Enter", or press C.

You are prompted, "Do you want to initialize options?" Use "Space" to choose "Yes" or "No". Choosing "Yes" initiates the factory reset process. Choosing "No" cancels the function and returns you to the "Settings" menu. You can also perform this function by holding the Forward Media key while pressing the Power Button. Once you see "HIMS" displayed on the Braille display, you can release the Forward key.

3.10 Voice Options

The Voice options menu allows you to configure the text-to-speech engine and voice used on your BrailleSense.

You can open the Voice options dialog quickly at any time by pressing "F3-V". When the "Voice Options" dialog is opened, you are placed in a combo box with two "Voice Type" options: "BrailleSense Vocalizer" and "Android TTS". Use "Space-1" and "Space-4" to move among the choices, and press "F3" to access further options for each choice.

3.10.1 BRAILLESENSE VOCALIZER

This option allows you to install and set alternative Vocalizer Expressive TTS voices. Available voice downloads are available via the "Download" option in this dialog. To download and install directly from the BrailleSense, choose the "Download" button, and use "Space-1" and "Space-4" to choose the voice you

want and press “Enter”. The correct folder will be auto created, and the language pack will automatically be placed there and installed.

You can also download voices manually via the HIMS website.

When you have downloaded the language pack files, create a folder on your flashdisk called “TTSDATA”, and copy the files in to it.

Once done, open the “Settings” menu, and press “V” to open the Voice options dialog and press “F3” from “BrailleSense Vocalizer”. . This dialog contains a voices list, a “Download” button, an “Install” button”, and Confirm and Cancel buttons. Use “F3” and “Space-F3” to move among the controls.

Press “Enter” on the “Install” button to install any language packs available in the “TTSDATA” folder. When voices are installed, an “Uninstall” button also appears in the dialog.

When you have installed your voices, use “Space-1” and “Space-4” in the Voices list to choose your voice, and press “Enter”. Your speech should automatically begin outputting using the voice you chose.

If you wish to remove a Vocalizer voice from your unit, navigate to it and press “Space-D” to delete it.

3.10.2 ANDROID TTS

The “Android TTS” option allows you to choose the speech used on your BrailleSense from among Google TTS and any other TTS engines you have installed via the play Store or other Android packages. Press F3 on the “Android TTS” item in the combo box and any TTS engine you have installed appears in a list. Use “Space-1” or “Space-4” to choose the engine you would like to use, and press “Enter”.

*Note: While it is possible to control most voice adjustments including pitch, rate and punctuation via the various BrailleSense voice settings, it is necessary to control voice volume, the default voice for each engine, and other settings specific to the engine you are using via the Android system settings under “Text-to-speech” in either the “Accessibility” or “Language and Input” area.

3.11 BrailleSense Global Options

The "Global Options" on the BrailleSense is a large dialog box for controlling many preferences/settings. Open the "BrailleSense Global Options" from the main menu, by pressing S to open the “Settings” menu, then "O" for the "Global Options". You can also open "Global Options" from anywhere on the notetaker by pressing "Space-O".

The "Global Options" dialog is organized into groups, making it easy to jump to a particular section in the long list of settings. You can navigate among the groups with "page down ("Space-3-4-5" or "F3-F4")" or "page up ("Space-1-2-6" or "F1-F2")". When you navigate to a group, you are placed at the position where that group of options begins.

You can navigate among the individual options using "Space-1" and "Space-4" or using the scroll keys. You can also use shortcut keys to quickly move to specific items.

- Note: Often, the same shortcut key is used for multiple options. Thus, the first press of the shortcut key moves the focus to the first setting that matches; pressing the key again moves to the next setting that matches. If there are no more settings matching the shortcut, the BrailleSense wraps around to the first match.

Many of the Global Options also have global hotkeys, allowing you to adjust these settings from anywhere on the unit.

You can toggle an option on or off or cycle through the available values of a setting with the "space" and "Backspace" keys. To save your changes, press "Enter" or tab to the "Confirm" button and press "Enter". BrailleSense says, "Saved options," and you are returned to your previous position on the notetaker. To cancel without saving changes, press the "F4" key, "Space-E" or "Space-Z".

3.11.1 GLOBAL OPTIONS IN DETAIL

The "Global Options" dialog contains the following settings. For each setting, the shortcut key appears in parentheses, and the hotkey, if available, is shown in quotes.

- 1) Braille display: (B). "Backspace-F3". Toggles the Braille display On/Off.
- 2) Braille Cursor: (C). "F4-Up Scroll". Defines how the Braille cursor appears on the Braille display: "Always Up", "Blinking" or "Off". Note the cursor appears as dots-7-8 in most situations. When selecting text, it notes the end of the selection with blinking dots-1-2-3-4-5-6-7-8.
- 3) Eight Dot Mode: (E). "F4-Up Scroll". Toggles Eight Dot Mode on/off for ASCII text entry.
- 4) View/input Braille Grade: (G). "Space-G". Lets you choose what grade of Braille you want to use when viewing/entering text: Uncontracted Braille, Contracted Braille, or Computer Braille.
- 5) Braille Code: (C). "Backspace-Enter-B". Defines the rules by which Braille is translated to text and vice versa: "US", "UK", "UEB", or one of several foreign language Braille codes. When foreign language codes are set, you can read and write Braille according to that language's Braille rules. The Braille language also affects lists, combo boxes, and edit boxes. Prompts, menus, buttons and static boxes remain in English.

- 6) Message Display Time: (M). defines how the BrailleSense deals with system messages: ignore, stand by until you press a key, or, display for from 1 to 10 seconds before disappearing.
- 7) Voice: (V). "Backspace-F2". Toggles the voice output On/Off.
- 8) Punctuation Level: (P). Use this setting to choose how you want the BrailleSense to read punctuation marks and symbols: "All", "Off", "Punctuation," or "Symbols".
- 9) Keyboard Echo: (K). Defines how BrailleSense speaks as you type: "On with characters and words," "On with characters," "On with words," or "Off".
- 10) Capitalization alert: (U). Choose how the BrailleSense alerts you that a character is capitalized, either when typing or when navigating a document or edit box by character: "Off", "Say cap," or "Pitch".
- 11) Numbers: (N). Defines how BrailleSense speaks numbers: when set to "On", BrailleSense speaks numbers, and when set to "Off", it speaks digits.
- 12) Voice Volume: (L). "Backspace-F1 and Backspace-F4". Raises or lowers the volume of the text-to-speech.
- 13) Voice Rate: (R). "Space-F1 and Space-F4". Controls the rate of the speech.
- 14) Voice Pitch: (T). "Enter-F1 and Enter-F4". Adjusts the pitch of the BrailleSense voice.
- 15) Main Volume: (V). "Backspace-SPACE-F1 and Backspace-Space-F4", or the Volume Up and Down buttons. Controls the master volume of the unit, including system sounds, audio and speech.
- 16) Headphone Volume: (V). Allows you to set a separate louder volume for headphones, useful when using high-end headphones with lower output.
- 17) Scroll Voice: (S). Choose whether you want the BrailleSense to speak while using the scroll buttons as you read.

- 18) LCD: (L). "Enter-4-5-6 from main menu". Turn the LCD On, On with Backlight, or Off.
- 19) Flip LCD display: (F). Choose whether the LCD shows text in normal or reverse orientation. Reverse orientation can be useful for a sighted person viewing BrailleSense from across a table.
- 20) LCD Font Size: (F). Adjust the size of the font on the LCD.
- 21) Show Virtual Braille Display on HDMI: (B). This option determines if, when you are connected to an HDMI monitor, a visual representation of the braille display and its print translation appears at the bottom of the screen.
- 22) Audio alerts: (A). Choose whether the BrailleSense alerts you to various events using audio signals, vibration signals, both, or no signal.
- 23) Battery Alerts: (B). Choose how you are alerted when your battery is low: "Message", "Beep", "Final Message" or "Final Beep".
- 24) Play Power On/Off Sound: (P). Choose whether you hear a sound when you turn on/off the BrailleSense.
- 25) Left Scroll Buttons: (L). Define the operation of the left scroll buttons: "Scroll display", "Move by line", "Move by character", "Move by paragraph" and "Move by sentence".
- 26) Right Scroll Buttons: (R). Define the operation of the Right Scroll buttons with the same options as above.
- 27) Skip Empty Lines: (E). Choose to have the BrailleSense alert you to a blank line by announcing "Blank" or skip empty lines.
- 28) Control Information: (I). Define where information is announced regarding the type of control on which you are currently located, a list item, menu, combo box, etc. Choose from Before, After and Off.

- 29) Hide passwords: (P). Choose whether the characters of a password are hidden as you type them.
- 30) Hot Key Information: (H). Defines how hot key information is announced: "Letters," "Dot patterns," or "Off".
- 31) Announce shortcut keys: (S). Lets you choose whether the BrailleSense speaks the menu shortcut key for a function as you encounter it.
- 32) power Button operation: (P) "Backspace-Enter-P". Choose the state your BrailleSense goes in to when the Power button is pressed: "Screen Off", as in the case of a normal Android phone or tablet, or "Deep Sleep", which turns off Bluetooth, Wi-fi, etc., similar to a traditional notetaker sleep mode.
- 33) Sleep Mode: (P). Set BrailleSense to save power by automatically shutting down after a certain period of time: "On", "Off", or "Message". (Only active when using battery power.)
- 34) Sleep Mode Kick In: (K). Use this option to set the time of inactivity before entering SLEEP Mode: 10 minutes, 20 minutes, or 30 minutes.
- 35) Check Today's Schedule: (S). Set the BrailleSense to check today's schedule for appointments each time you turn it on.
- 36) Touch Gestures: (J). Turn the use of Braille display touch gestures On/Off.
- 37) One-Handed Mode: (H). This option allows users with only the use of one hand to use the BrailleSense in a specific mode adapted for that purpose. For detailed information about one-handed mode, refer to section 2.14.
- 38) Enable Spell-check: (C). You may wish to disable spellcheck in educational situations where a student needs to be tested on spelling, etc.
- Note: If "Enable Spellcheck" is turned off, Automatic Spellcheck cannot be used.

- 39) Automatic spell-check: (C). Set whether the BrailleSense automatically checks spelling as you type in "multi line edit boxes" in programs such as the Schedule Manager, Address Manager, E-mail messages, the Word Processor, etc. If the option is "On", and you enter a misspelled word, BrailleSense emits a beep sound to indicate the misspelling.
- 40) Default notepad Document Type: (Q). Set the default document type for new documents you create using the Notepad: "DOCX document", "Sense document", "Braille document,", "DOC document" or "Text document".
- 41) Automatically synchronize with time server: (T). Using this feature, you can ensure the accuracy of the time on the BrailleSense by synchronizing with an atomic clock when BrailleSense is connected to the Internet.
- 42) Time Announcement: (T). Set the BrailleSense to announce the time every 5 minutes, every 10 minutes, 30 minutes or every hour.
- 43) Progress indicator: (P). Defines how the progress of opening a document, copying files or opening a webpage is indicated: "Silent," (no progress indication) "Beep," (continuous beeps while the task is performed) and "Message" (progress is shown as a percentage).
- 44) Bluetooth: (B). "Backspace-3-4-5-6". Toggles Bluetooth On/Off.
- 45) Wireless LAN: (W). "Backspace-1-4-5-6". Toggles Wireless LAN On/Off.
- 46) Screen Reader Hints: (H). Set whether to receive spoken hints about how to interact with the screen reader as you navigate the controls of 3rd party programs: "On/Off".
- 47) Screen Reader Control Information: (I). Determines whether the screen reader speaks the type of control or field as you navigate 3rd party applications: "On/Off".

48) Notification alerts: (N). Set whether you receive push notifications from Android applications.

49) Enable Braille in standby Mode: (N). Set whether status messages are shown in Braille while in Sleep Mode.

50) Language: (L). Allows you to easily change the system language of the unit to any open firmware localization. *Several things to note: Because this is connected specifically to an Android setting, all Android languages appear in this list. If you choose a language for which we do not have a localization, BrailleSense prompts and menus appear in English, though you may operate in the chosen language on the android side. Currently Braille and TTS need to be changed separately via their respective options. Additionally, any open applications need to be closed and re-opened before they will appear in the changed language.

3.12 Android Backup & Reset

The “Android Backup & Reset” option opens the Restore and Reset options found in the Android System Settings in a more convenient location. Here, you can set options for backing up and restoring information related to your Android applications and accounts, and you can factory reset your device. **WARNING!** Performing a factory Reset in the “Android Backup & Reset” menu erases all personal data, including files, account information, installed applications, etc. This is also a long and involved process and should not be undertaken before consulting with tech support staff, as you will need special instructions to complete it properly.

3.13 Android System Settings

The "Android System Settings" is the final item in the "Settings" menu. You can launch it from anywhere using "F3-E".

This option opens the standard Android "Settings" dialog found on stock Android phones and tablets. While many settings are duplicated in the BrailleSense interface, the "Android System Settings" contains a standard Android interface, and must be navigated in the same manner as Android applications. Use "F3", and "Space-F3" to move among objects on a screen. Use "Backspace-Space-1" and "Backspace-Space-4" to adjust slider controls. Please see chapter 12 on the Mobile Screen Reader, and chapter 15, Commands Summary, for more complete information on navigating the Android interface.

4. File Manager

4.1 Overview

The functionality and structure of the "File Manager" on the BrailleSense are similar to "Windows Explorer" on a PC. To run the "File Manager", press F from the "Program" menu or press "Enter" when your cursor is placed on "File Manager". You can launch the "File Manager" from anywhere on the unit by pressing "F1-F". Close the "File Manager" by pressing "Space-Z".

The "File Manager" contains 2 basic areas: an "address window" and the content list. You can navigate between them using "F3" and "Space-F3". When you first open the "File Manager", you are placed in the drive list. By default, you are placed on the internal "flashdisk". If an SD card or USB drive is connected, it also appears in the list. You can use "Space-1" or "Space-4" or the scroll buttons to navigate the items in the drive list. Press "Enter" on the drive you want to open. The BrailleSense then displays the list of the files/folders on that drive.

The "address window" contains the path of the current directory or file.

The "File Manager" contains both menus and hot keys, allowing you to easily activate items in the program with a single keystroke as well as to access functions through the menus for commands you have not memorized. You can bring up the menus by pressing "Space-M" or "F2". For the purposes of the directions in this chapter and the rest of this manual, we will use F2. However, either keystroke will perform this function.

Use "Space-1" and "Space-4" to navigate among the available menus. To open a menu, press "Enter". Again, use "Space-1" or "Space-4" to navigate among the items in a menu, and press "Enter" to execute a menu item.

The File Manager also contains dialogs. You can navigate among the controls in a dialog box by using the equivalents of "Tab" on a PC. You can use "F3" or "Space-4-5" for "Tab" and "Space-F3" or "Space-1-2" for "Shift-Tab". For the purposes of instructions in this manual, we will use "F3" and "Space-F3", or you will be directed to "Tab" or "Shift-Tab".

4.1.1 NAVIGATING THE FILE LIST

The "File Manager" contains two kinds of lists: the "file list" and the "menu list", or rather, the list of items contained in the "File Manager" menu. The menus in the BrailleSense wrap. This means, when you reach the end of the list of items in a menu, the BrailleSense automatically brings you back to the top of the list when you press "Space-4". However, when you reach the end of a "file list," the BrailleSense remains at the end of the list, even if you press "Space-4". The following is a list of the keystrokes used for navigating lists:

- Note: the following keystrokes can be used to navigate any list on the BrailleSense.

1. Move to previous item: (Space-1) or up scroll button

2. Move to next item: (Space-4) or down scroll button

3. Move to the beginning of a list: Space-1-2-3

4. Move to the end of a list: Space-4-5-6

The following keystrokes are valid only in a "file list".

1. Jump to folder or file list: Space-5-6

2. Move between file list and address window: (Space-4-5 or F3) / (Space-1-2 or Space-F3)

3. Enter the selected folder or open the selected file: Enter

4. Move back a level in the folder structure: Backspace

5. Jump to the drive list: Space-1-2-5-6

6. Move to the previous group of 32 items: Page up (space 1-2-6 or F1-F2)

7. Move to the next group of 32 items: Page down (space 3-4-5 or F3-F4)

8. Jump to a file or folder by pressing the first character of the folder or file name. You must use ASCII characters.

4.1.2 SELECTING FOLDERS AND FILES

1. Select individual items "Space": * is displayed in front of the item to denote its selection. Pressing "Space" again on the selected folder or file cancels the selection.

2. Continuous selection "Enter-b": This selects all of the items from the point where this command is executed to the item where the cursor is located. * appears in front of the selected items.

3. Select all "Enter-a": This selects all the files and folders in the current list. You can toggle selection and de-selection of items by pressing "Space". When the

"Select All" command is executed, the BrailleSense says the number of selected files, and * appears in front of the selected files on the Braille display.

4.2 Working with Files and Folders

When viewing the contents of a drive or folder, the items in quotation marks are folders, and the items without quotation marks are files.

The items in the list are followed by numbers, such as 1/5 or 2/5. The numbers following the items indicate an item's placement in the list and the total number of items in the list. For example, "database 1/5," indicates "database" is the first folder in a list containing five items.

4.2.1 ENTERING AND EXITING FOLDERS

There are several ways to open a folder.

1. Bring up the menu by pressing "F2". Select the "Open" option from the list and press "Enter".
2. Press the hot key "Enter-O" to activate the "Open" command and open the desired folder.
3. Finally, as described above, you can simply press "Enter" on the selected folder.

To exit the current folder, use the "Backspace" key. You are returned to the previous level in the folder structure.

4.2.2 EJECTING USB DRIVES AND SD CARDS

The Android operating system requires that drives are ejected properly before removing them. If not done, re-inserting them in to the BrailleSense or

connecting to other mobile devices or computers may result in all files appearing as though they have no data, or have a size of 0 bytes. Before disconnecting a USB drive or SD card, press “Backspace-Enter-U” or “Backspace-Enter-3-6” to it from the system. If your drive is showing files with no data, please re-insert it in to the BrailleSense, and press the Unmount key to eject it properly, and the data should, be visible once again.

4.3 The File Menu

To open the "File" menu, press "F2" to bring up the menu, and press "Enter" on "File". Use normal list commands to navigate to your desired option and press "Enter".

The BrailleSense supports the following file formats:

1. Document files: hbl, brl, txt, brf, rtf, doc/docx, XLS/XLSX, PPT/PPTX, PDF and unprotected EPUB.
2. Web formats: url, and html.
3. Media formats: ac3, asf, asx, m3u, mp2, mp3, MP4, ogg, pls, wav, wax, wma, wmv and flac.
4. DAISY formats: DAISY 2.0, 3.0, BookShare.
5. Programs: APK. Press “Enter” to install.

The menu contains the following items. Shortcuts are placed in parentheses, and hotkeys are in quotes.

- 1) Open: (O) "Enter-O". Opens the selected file in the associated application for its file type.

- 2) Open With: (E) "Enter-E". Allows you to choose a compatible application with which to open the selected file.
- 3) Zip (Z) "Enter-Z". Compresses the current file or folder.
- 4) Unzip (U) "Enter-U". Appears only for compressed files, extracts the current file.
- 5) Send To: (S) "Enter-S". Lets you send the current file or folder to a different location. Targets include any connected local drives, e-mail and exchange E-mail, and "Android Program, which will allow you to send files to available 3rd party apps according to the file type you are sending.
- 6) Rename (R) "Enter-R". Rename the current file or folder.
- 7) New Document: (N) "Enter-N". Create a new document using DOCX, DOC, TXT, BRL, or HBL.
- 8) New Folder: (F) "Enter-F". Creates a new folder.
- 9) File Conversion: (T) "Enter-T". Converts selected document files to TXT, BRL, or HBL.
- 10) Find File: (F) "Space-F". Search the current drive or folder for a specific file by name.
- 11) Find Text: (F) "Backspace-F". Search Word, Text, and Braille files on the current drive or folder for specific text strings.
- 12) Merge Files: (M) "Enter-M". Select 2 or more text or Braille files to merge together.
- 13) Split Files: (P) "Enter-P". Split text or Braille files by paragraph or word count, by size, or into a certain number of files of equal size.
- 14) Information: (I) "Enter-I". Provides basic properties of the current file or folder.

15) Exit: (Z) "Space-Z". Closes the File Manager.

4.4 The Edit Menu

The Edit Menu contains the following items:

- 1) Copy: (C) "Enter-C". Copies the selected files or folders to the clipboard.
- 2) Cut: (X) "Enter-X". Cuts the selected files or folders to the clipboard.
- 3) Paste: (V) "Enter-V". Pastes the files or folders on the clipboard to the current location.
- 4) Delete: (D) "Space-D". Deletes the selected files or folders.
- 5) Select All: (A) "Enter-A". Selects all the files and folders in the current list.

4.5 The View Menu

The View Menu contains the following options:

- 1) Sort Files By: (G) "Enter-G". Determines in what order files and folders are displayed.
- 2) Set File Info: (L) "Enter-L". Determines what information is displayed about each file or folder in the list.
- 3) Display Only Files of Type: (W) "Enter-W". Filters files by file type.
- 4) Hide System Folders: (H) "Enter-H". Hides Android system folders that do not contain user content. These are hidden by default.
- 5) Document Open Settings: (D) "Enter-D". This setting lets you determine in which program DOC and DOCX files are opened by default, Word Processor or Notepad. You can also choose to "Always select", in which case you will be prompted every time.

4.6 Google drive

You can use the File Manager to interact directly with your Google drive content in a similar fashion as you do with the Flashdisk and other drives connected to your unit in terms of selecting and moving files, editing and saving documents and playing media content.

You can easily transfer files between your Google drive and BrailleSense, using the BrailleSense File Manager interface, Open and edit documents using the Word Processor and Notepad, attach files using the E-mail or Exchange E-mail programs, read spreadsheets using the Excel Viewer, and play media content in the Media player.

*Note: you must login to your Google drive account from the file Manager before the content will be available via the other applications mentioned above.

The first time you open Google drive, you are placed in a Google Sign In page. Use “F3” and “Space-F3” to navigate the page.

If you have already signed in to your Google account, it should appear in the list. press Enter on your Account, then locate the Allow Button and press Enter. Once done, you should be able to press Enter on Google drive and load your file and folders structure from the File Manager. Google drive will also appear in the Open/Attach File dialog under the drive list when using the applications mentioned previously, and you will be able to navigate and select files and folders as you would from local drives on the unit.

Please be aware of the following conditions:

- Note that the file management functionality is limited to manipulation of files and folders at this time, thus, you will find the menu items are limited to functions like delete, rename, Copy and Paste, as Google drive doesn't

support functions like Zip and Unzip, Braille and TXT File conversion, etc. Please use "Space-H" to get a complete list of the available functions.

- Only supports selecting files or folders for the purposes of opening or saving.
- The login function is not supported from programs other than the File Manager. Please login to Google drive from the file manager before trying to use additional functions or applications.
- Streaming media files is not supported. We download the file in a temporary folder in the flashdisk before playing.
- In general, it should be noted, due to transferring to and from Google Drive, opening and saving files is a bit slower than when accessing local files and folders.

4.7 Using the Address Window

This edit box lets you type the direct path of the file name you want to open. This window also contains the history list for easily accessing recently opened files or directories.

When you are in a file or folder list, you can press "tab (Space-4-5)" to navigate to the address window. The BrailleSense displays the current path including the current drive name, such as "flashdisk/download/". The address window is only visible when you are in a file or folder list. When you are in the drive list, the address window is not available.

You can edit the text by relocating the cursor with a cursor routing key, the BrailleSense inserts the text at the cursor location. If you know the exact path and file name, you can type in the path and file name to open the file with the appropriate application. For example, if you type in "flashdisk/download" in the address window and press "Enter," the BrailleSense displays the list of the files in the download folder. If you type "flashdisk/My Documents/sample.txt" in the

address window, and press "Enter," the BrailleSense loads the Word Processor, and opens the file "sample.txt".

Use "Space-1" or "Space-4" in the Address Window to review the history list (a list of file and folder paths you have recently opened). To open a file in the history list, press "Enter". The file opens with the appropriate application. (See section 4.2.3).

5. Word Processor

The BrailleSense Word Processor supports creation of formatted DOCX and DOC files, as well as opening of DOC/DOCX and PPT/PPTX/PPTM files. All other types should be opened using the notepad, described in the next chapter.

You can launch the "Word Processor" in a couple of ways:

Press "F1" to open the main "Program" menu. Navigate to "Word Processor" using "Space-4" and press "Enter". You can also press W from the main menu to quickly jump to and open the "Word Processor".

You can launch the "Word Processor" from anywhere on the unit by pressing "F1-W".

When you launch the Word Processor on the BrailleSense, it places you in a new, blank document. You can activate all the functions of the Word Processor via the Word Processor menu or by using hot keys. The menu in the Word Processor consists of File, Edit, Go To, Read, and Layout. To access these menus, press "Space-M" or "F2". Select a menu item using "Space-1" or "Space-4", and press "Enter". Or, you can press "Space-M or "F2" followed by the shortcut for the corresponding menu item.

Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab among controls in a dialog.

Press "Enter" to execute an item.

5.1 File Menu

The File menu contains the following options:

- 1) New: (N). Creates a new document, by default named "noname". You can create only DOCX files in the Word Processor. Other file types, like TXT, BRL and RTF can be created in the Notepad described in the next chapter.
- 2) Open: (O) "Enter-O". Opens an existing file. Supported file types include DOC, DOCX, PPT, PPTX, PPTM. *Note: all these files can be read, but only DOC and DOCX files can be saved. Password protected word files, such as those used in health professions, can also be read, but cannot be created or saved.
- 3) Save: (S) "Enter-S". Save the current file. Tab between name and location. Use normal file management navigation to choose a different folder.
- 4) Save As: (A) "Space-S". Save the current file under a different name or in a different location. Note: you can save both DOC and DOCX files once they are opened, but you cannot convert one to the other.
- 5) Export: (X) "Backspace-Enter-X". This function exports the current document to BRL, useful for translating a DOC or DOCX file, especially useful for saving math files to braille so that they can be embossed via the Notepad.
- 6) Print: (P) "Space-P". the document will be printed using the default print service set in the Android System Settings.

7) Settings: (E) "Enter-E". Dialog to set various document options. Tab among settings list, location Info check boxes, Default folder, and Confirm and Cancel buttons. Settings list includes View Format Characters, Reading unit (affects functions of "Space-1" and "Space-4" and "Space-2-3" and "Space-5-6"), Read Only On/Off, Automatic Save Interval, Auto Scroll Speed, Apply Settings to All Documents, Home/End movement unit, Math Braille Code, and new Document default name. Use "Space-1" and "Space-4" to move among the settings in the Settings list, and Space or Backspace to change the value of the setting.

8) File Open Setting: (T) "Backspace-Enter-O". Determines how a document is loaded: Choose "Load completely" to open the document in whole before reading, choose "Load in Background" to start reading after the first page is loaded, and continue loading in the background.

9) Dictionary: (D) "Backspace-Enter-D". Opens the Sense Dictionary installed, and inserts the word under the cursor in the lookup edit box.

10) Exit: (Z) "Space-Z". Closes the Word Processor.

5.2 Edit Menu

The Edit Menu contains the following items:

1) Start Selection: (B) "Enter-B". Marks the beginning of the selection. The end of the selection is marked by the current cursor position.

2) Copy: (C) "Enter-C". Copies selected text to the clipboard.

3) Cut: (X) "Enter-X". Cuts the selected text to the clipboard.

4) Paste: (V) "Enter-V". Pastes text from the clipboard to the current location.

5) Delete: (D) "Space-D". Deletes selected text.

- 6) Delete Blank lines: (L) "Backspace-E". Deletes blank lines from the selected text area.
- 7) Add to clipboard: (P) "Enter-P". Appends selected text to the clipboard without deleting the prior selection.
- 8) Clear clipboard: (E) "Enter-D". Empties the clipboard.
- 9) Select All: (A) "Enter-A". Selects all the text in the document.
- 10) Check spelling: (K) "Enter-K". Opens the Check Spelling dialog, and begins checking the document for errors. Use "F3" and "Space-F3" to move among the controls, and "Space-1" and "Space-4" to move through lists of options. You can also use the following hot keys:
 - A. Skip Once: Backspace-S.
 - B. Skip All: Backspace-I.
 - C. Modify Current: Backspace-C.
 - D. Modify All: Backspace-M.
 - E. Add to User Dictionary: Backspace-A.

You can also check spelling of the current word with "Backspace-K".

5.3 Insert Menu

- 1) Set Mark: (M) "Enter-M". Sets a bookmark at the current cursor position.
- 2) Jump to Mark: (J) "Enter-J". Jumps to a previously set mark.
- 3) Delete Mark: (D) "Backspace-D". Deletes a previously set mark.
- 4) Insert from File: (I) "Enter-I". Inserts information from another file.
- 5) Insert Date: (W) "Enter-W". Inserts the current date.
- 6) Insert time: (T) "Backspace-W". Inserts the current time.

7) Insert Tab: Backspace-Space-Enter-I.

8) Insert Page Break: Backspace-Space-Enter-P.

5.4 Go to Menu

1) Find: (F) "Space-F". Opens Find dialog where you can enter a text string to search, choose whether to match case, and choose the search direction.

2) Find Next: (H) "Enter-F". Finds the next instance of the previously searched string.

3) Find Previous: (V) "Backspace-F". Finds the previous instance of the search string from your current position.

4) Find and Replace: (R) "Enter-R". Dialog to search for a text string and replace with an alternative string. Choose search direction, match case, and whether to replace one instance or all.

5) Go To Location: (L) "Enter-L". Go to a location in the document according to page, paragraph, line or percentage.

6) Go To Previous Sentence: (E) "Space-2-3-6". Moves back one sentence.

7) Go To Next Sentence: (X) "Space-3-5-6". Moves forward one sentence.

8) Go To Previous Page: (B) "Space-1-2-6". Moves back one page.

9) Go To Next Page: (G) "Space-3-4-5". Moves forward one page.

10) Go To Previous Document: (P) "Shift-Tab". If multiple documents are open, moves to the previous document in the order the files were opened.

11) Go To Next Document: (N) "Tab". If multiple documents are opened, moves to the next document in the file open order.

5.5 Read Menu

- 1) Read Selected Text: (B) "Backspace-B". Reads the selected block of text.
- 2) Read beginning of Selected Text: (U) "Enter-U". Reads the first line of a selected text block.
- 3) Read from Beginning to Cursor: (T) "Backspace-G". Reads from the beginning of the document to the current cursor position.
- 4) Read from Cursor to End: (Q) "Enter-G". Reads from the current cursor position to the end of the document.
- 5) Auto Scroll: (A) "Up/Down Scroll". Starts and stops the auto scroll function. Set the speed of the auto scroll in the Settings under the File menu.
- 6) Read Current Sentence (E) "Space-1-4-5-6". Reads the current sentence.
- 7) Read Current Line: (L) "Space-1-4". Reads the current line.
- 8) Read Current Word: (W) "Space-2-5". Reads the current word. Press twice to spell.
- 9) Read Current Character: (C) "Space-3-6". Reads the current character. Press twice to hear the military alphabet equivalent.
- 10) Read Cursor Position: (R) "Space-1-5-6". Reads current cursor position as page, paragraph, word, and line.
- 11) Read Status: (S) "Space-3-4". Reads title of document and indicates Insert or Overwrite mode.
- 12) Read Character and Space Count: (I) "Backspace-I". Reads the current and total character count, and the current and total space count of the current document.
- 13) Read Font info: (F) "Backspace-Enter-N". provides font, style, and formatting information for the text at your current position.

5.6 Layout Menu

- 1) Font: (F) "Enter-backspace-F". Opens the font settings dialog. Acts on selected text. Change the default font by adjusting the setting values and choosing the "Set Current as default" button
- 2) Print Document Layout: (T) "Backspace-T". Determines the format of the pages of a printed document.
- 3) Print Paragraph Layout: (C) "Backspace-2-3-4-6". Determines the print format and style of the current paragraph.

*Note: if you find documents use incorrect word wrapping when transferred to computers or other devices, please Select All text with Enter-A, then turn Word Wrap off in the Print Paragraph Layout.

5.7 Miscellaneous Functions

The following commands can also be used in the Word Processor:

- 1) Center current line: Backspace-Space-Enter-C.
- 2) Left Justify: Backspace-Space-Enter-J.
- 3) Right justify: Backspace-Space-Enter-R.
- 4) Bold on/off: Control-B.
- 5) italics On/off: Control-I.
- 6) Underline On/Off: Control-U.
- 7) Start/End Math Mode: Backspace-M.

- 8) Insert math symbol: Press Space-H in Math Mode, and press Enter on the symbol in the list.
- 9) Next slide in PowerPoint file: Space.
- 10) Previous Slide in PowerPoint file: Backspace.
- 11) Go to slide in Power Point File: Enter-L.
- 12) Move back by double carriage return: Backspace-Space-2-3.
- 13) Move forward by double carriage return: Space-Enter-5-6.
- 14) Move forward by heading: space-dots-2-5-6.
- 15) Move back by heading: Space-dots-2-3-5.

6. Notepad

The BrailleSense Notepad supports reading and creation of RTF, TXT, HBL, BRL and BRF files as well as unformatted DOCX files, and will open PDF, EPUB, DOCX and DOC files. To read PowerPoint files, or to create and/or preserve formatting in Word documents, or to create mathematical expressions, please use the Word Processor described in the previous chapter.

*Note: though you can open Word documents in the notepad, all document formatting will be stripped. Response time for navigation and searching will be reduced in the Notepad, but if a Word document is saved from within it, all formatting will be lost.

You can launch the "Notepad" in a couple of ways:

Press "F1" to open the main "Program" menu. Navigate to "notepad" using "Space-4" and press "Enter". You can also press N from the main menu to quickly jump to and open the "notepad".

You can launch the "Notepad" from anywhere on the unit by pressing "F1-N".

When you launch the Notepad on the BrailleSense, it places you in a new, blank document. You can activate all the functions of the Notepad via the Notepad menu or by using hot keys. The menu in the Notepad consists of File, Edit, insert, Go To, Read, and Layout. To access these menus, press "Space-M" or "F2". Select a menu item using "Space-1" or "Space-4", and press "Enter". Or, you can press "Space-M or "F2" followed by the shortcut for the corresponding menu item.

Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab among controls in a dialog.

Press "Enter" to execute an item.

6.1 File Menu

The File menu contains the following options:

- 1) New: (N). Creates a new document, by default named "noname". You can create a DOCX or DOC file without formatting, or a TXT, BRL or HBL file.
- 2) Open: (O) "Enter-O". Opens an existing file. Supported file types include all those listed at the beginning of this chapter.
- 3) Save: (S) "Enter-S". Save the current file. Tab between name, location and file type. Use normal file management navigation to choose a different folder.
- 4) Save As: (A) "Space-S". Save the current file under a different name, as a type or in a different location. You can choose to password protect files by choosing the HPF file format.
- 5) Print: (P) "Space-P". Supports only Braille embossing: the document will be printed using a connected embosser.

6) Settings: (E) "Enter-E". Dialog to set various document options. Tab among settings list, location Info check boxes, Default folder, Set Password, and Confirm and Cancel buttons. Settings list includes View Format Characters, Reading unit for Space-1 and Space-4, Read Only On/Off, Automatic Save Interval, Auto Scroll Speed, Apply Settings to All Documents, Home/End movement unit, and new Document default name. Use "Space-1" and "Space-4" to move among the settings in the Settings list, and Space or Backspace to change the value of the setting.

7) Dictionary: (D) "Backspace-Enter-D". Opens the Sense Dictionary installed, and inserts the word under the cursor in the lookup edit box.

8) Exit: (Z) "Space-Z". Closes the Notepad.

* The BrailleSense allows you to encrypt and password protect files which you do not wish for anyone else to be able to open. Before you save a document as a protected file, you must set a password in the "Settings" option under the "File" menu. The "Set Password" dialog consists of 5 items: "Delete the original file" checkbox, ?, a "Password?" edit box, a "Retype Password" edit box, a "confirm" button, and a "cancel" button. Once the password is set, use the save as dialog, and choose HPF as your file type.

6.2 Edit Menu

The Edit Menu contains the following items:

1) Start Selection: (B) "Enter-B". Marks the beginning of the selection. The end of the selection is marked by the current cursor position.

2) Copy: (C) "Enter-C". Copies selected text to the clipboard.

- 3) Cut: (X) "Enter-X". Cuts the selected text to the clipboard.
- 4) Paste: (V) "Enter-V". Pastes text from the clipboard to the current location.
- 5) Delete: (D) "Space-D". Deletes selected text.
- 6) Delete Blank lines: (L) "Backspace-E". Deletes blank lines from the selected text area.
- 7) Add to clipboard: (P) "Enter-P". Appends selected text to the clipboard without deleting the prior selection.
- 8) Clear clipboard: (E) "Enter-D". Empties the clipboard.
- 9) Select All: (A) "Enter-A". Selects all the text in the document.
- 10) Toggle Insert/overwrite Mode: (I) "Space-I". determines whether, when you type text, if it is inserted at the current position, or overwrites the current text.
- 11) Check spelling: (K) "Enter-K". Opens the Check Spelling dialog, and begins checking the document for errors. Use "F3" and "Space-F3" to move among the controls, and "Space-1" and "Space-4" to move through lists of options. You can also use the following hot keys:
 - A. Skip Once: Backspace-S.
 - B. Skip All: Backspace-I.
 - C. Modify Current: Backspace-C.
 - D. Modify All: Backspace-M.
 - E. Add to User Dictionary: Backspace-A.

You can also spellcheck the current word with "Backspace-K".

6.3 Insert Menu

- 1) Set Mark: (M) "Enter-M". Sets a bookmark at the current cursor position.

- 2) Jump to Mark: (J) "Enter-J". Jumps to a previously set mark.
- 3) Delete Mark: (D) "Backspace-D". Deletes a previously set mark.
- 4) Insert from File: (I) "Enter-I". Inserts information from another file.
- 5) Insert Date: (W) "Enter-W". Inserts the current date.
- 6) Insert time: (T) "Backspace-W". Inserts the current time.

6.4 Go to Menu

- 1) Find: (F) "Space-F". Opens Find dialog where you can enter a text string to search, choose whether to match case, and choose the search direction.
- 2) Find Next: (H) "Enter-F". Finds the next instance of the previously searched string.
- 3) Find Previous: (V) "Backspace-F". Finds the previous instance of the search string from your current position.
- 4) Find and Replace: (R) "Enter-R". Dialog to search for a text string and replace with an alternative string. Choose search direction, match case, and whether to replace one instance, from cursor to end, or all.
- 5) Go To Location: (L) "Enter-L". Go to a location in the document according to page, paragraph, line or percentage.
- 6) Go To Previous Sentence: (E) "Space-2-3-6". Moves back one sentence.
- 7) Go To Next Sentence: (X) "Space-3-5-6". Moves forward one sentence.
- 8) Go To Previous Page: (B) "Space-1-2-6". Moves back one page.
- 9) Go To Next Page: (G) "Space-3-4-5". Moves forward one page.
- 10) Go To Previous Document: (P) "Shift-Tab". If multiple documents are open, moves to the previous document in the order the files were opened.

11) Go To Next Document: (N) "Tab". If multiple documents are opened, moves to the next document in the file open order.

6.5 Read Menu

- 1) Read Selected Text: (B) "Backspace-B". Reads the selected block of text.
- 2) Read beginning of Selected Text: (U) "Enter-U". Reads the first line of a selected text block.
- 3) Read from Beginning to Cursor: (T) "Backspace-G". Reads from the beginning of the document to the current cursor position.
- 4) Read from Cursor to End: (Q) "Enter-G". Reads from the current cursor position to the end of the document.
- 5) Auto Scroll: (A) "Up/Down Scroll". Starts and stops the auto scroll function. Set the speed of the auto scroll in the Settings under the File menu.
- 6) Read Current Sentence (E) "Space-1-4-5-6". Reads the current sentence.
- 7) Read Current Line: (L) "Space-1-4". Reads the current line.
- 8) Read Current Word: (W) "Space-2-5". Reads the current word. Press twice to spell.
- 9) Read Current Character: (C) "Space-3-6". Reads the current character. Press twice to hear the military alphabet equivalent.
- 10) Read Cursor Position: (R) "Space-1-5-6". Reads current cursor position as page, paragraph, word, and line.
- 11) Read Status: (S) "Space-3-4". Reads title of document and indicates Insert or Overwrite mode.

12) Read Character and Space Count: (I) "Backspace-I". Reads the current and total character count, and the current and total space count of the current document.

6.6 Layout Menu

1) Braille Paragraph Layout: (B) "Backspace-1-2-4-6". Determines the Braille format of the current paragraph.

2) Braille Document Layout: (P) "Backspace-P". Determines Braille document format for embossing.

7. E-mail

To use the e-mail program, you must have an e-mail account, provided to you by your Internet service provider or sign up for a free account such as Gmail or Hotmail. Your e-mail account needs to support POP3 (Post Office Protocol 3) or IMAP (Internet Message Access Protocol) and SMTP (Simple Mail Transfer Protocol) to be accessed with the BrailleSense.

Before you use the e-mail program, you must also configure the BrailleSense to access the Internet, (see section 3.2 "Internet Setup").

- Note: The menus in the BrailleSense's e-mail program are specific to the task you are performing: for example, when checking the "Inbox" in the e-mail program, the menu items are different from the available items when you are writing an e-mail or viewing mailboxes. Generally, the available e-mail program menus are "File", "Edit", "Tools" and "Message".

To access these menus, press "Space-M" or "F2". Select a menu item, and press "Enter". Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab among controls in a dialog.

7.1 Executing E-mail

To use the e-mail program, you must configure the BrailleSense to access each of your e-mail accounts by entering your e-mail account information.

To execute the e-mail program, navigate to "E-mail" in the "program" menu, by pressing "Space-1" and "Space-4" and press "Enter", or just press E from the main program menu. You can launch the "E-mail program from anywhere on the unit by pressing "F1-E".

When you launch the e-mail program for the first time, the BrailleSense says, "You must create at least 1 mail account from the accounts manager in the Tools menu". Then, the "Inbox" is opened.

The "Inbox" is the main window of the E-mail program. When you run the e-mail program for the first time, this "Inbox" is empty and the BrailleSense displays "no items".

If there are e-mail messages in the "Inbox", "Opening inbox." is announced; and "xx% processed" is displayed on the Braille display. Once progress has reached 100%, the "Inbox" is opened and displays the received e-mail in the message list.

Press "Shift-Tab (Space-1-2)", and you are placed in the mailbox list. By default, the BrailleSense contains the following mailboxes:

- 1) Inbox: received mail that has not been deleted or saved to a different folder.
- 2) Sent: sent messages you have chosen to store
- 3) Outbox: messages waiting to be sent, either when you save mail in "Outbox" or sending mail fails.

7.2 Setting Up an Account

To access the "Accounts Manager", Open the menu with "F2", navigate to "Tools" and press "Enter". "Accounts Manager" is the first item in the "Tools" menu, thus, you can just press "Enter" again to open it. You can also launch the "Accounts Manager" from anywhere in e-mail by pressing "Enter-M".

"Account name: no items" is displayed.

The "Accounts manager" dialog contains the following 6 items: "Accounts" list, "Information" button, "Add" button, "Modify" button, "Delete" button and "Close" button.

To add an e-mail account, follow these steps:

- 1) From the "Accounts Manager", press "F3", to move to "Add" and press "Enter".
- 2) The dialog is opened and "Server Type" is displayed. This is a combo box in which you can choose to manually setup your account, or ask BrailleSense to auto fill in server information from known server types like Gmail and iCloud. Use "Space-1" and Space-4" to navigate the Server types. If your server is not in the list, select "Manual". Press F3 to move to the incoming server type.
- 3) "Default mail server" is displayed. Use Space to choose between POP3 and IMAP. Press "F3" to move to the account settings list.
- 4) "Account name:" is displayed. Type the name by which you want to identify the e-mail account: (example, "Gmail" or "work e-mail").
- 5) Press "Space-4", to move to "Display name". Type the "Display name" in the edit box. This name is the name people will see when they receive an e-mail from you.
- 6) Press "Space-4", to move to "Logon username". Type the "Logon username" in the edit box (using computer Braille if writing in US Braille). (Ex: Sense or sense@himsintl.com).
 - Note: some providers require you to use the entire e-mail address as the username, while others may use the part of the address before the at-sign. BE sure you type the username correctly for your e-mail provider.
- 7) Press "Space-4", to move to "Password". Type the "password" in the edit box (computer Braille if using US Braille).

- Note: By default, the password is hidden, and thus displays as asterisks. Be sure to type carefully as you cannot review your password unless you choose not to hide passwords in the "Global Options".

8) Press "Space-4", to move to "E-mail address". Type your "E-mail address" in the edit box (again, use computer Braille if entering US Braille). (Ex: sense@himsintl.com)

9) Press "Space-4", to move to "Incoming Server?". If you have selected "IMAP" as your incoming server type, "Incoming IMAP server?" is displayed. If you have chosen "POP3" as your incoming server type, "Incoming POP3 server" is displayed. Type the "Incoming server" in the edit box (in computer Braille for US Braille). (Ex: imap.himsintl.com, or pop.himsintl.com). Check with your account provider for your specific incoming server name.

10) Press "Space-4", to move to "Outgoing SMTP server". Type the "Outgoing SMTP server" in the edit box (computer Braille if using US Braille). Check with your account provider for the exact name of your outgoing SMTP server.

11) "Tab to the "Confirm" button and press "Enter".

12) The BrailleSense exits to the "Account name list" and shows the added account name (Ex: Account name: BrailleSense 1/1).

13) When you finish adding accounts, press "Space-Z" to return to the "Inbox".

- Note: Many e-mail service providers use secure servers with different port numbers. If your service provider requires you to use encryption, you must set these options in the "Advanced" dialog explained below:
- Note: If you are not using UEB, several of the fields for setting up an e-mail account require you to use computer Braille. To type your ID, Password, and address,

While adding your account, you can create a signature and set advanced options in the "Advanced" dialog.

- 1) Open the "Accounts Manager" as described previously.
- 2) If you're creating a new account, "Tab to the "Add" button and Press "Enter". If you need to add advanced settings to an account you've already created, Tab to the "Modify" button and press "Enter".
- 3) When you have filled in everything in the main account settings list, press "F3" to move to "Advanced" and press "Enter".
- 4) BrailleSense displays "Use secure POP3 (or IMAP) (POP-SSL)". The default value of this check box is unselected. If the POP3 or IMAP server uses "SSL", select the check box by pressing "Space".
- 5) Press "F3" to move to "POP3 (or IMAP) port number": POP3 is set to 110 by default, and IMAP is set to 143 by default. Type a different POP3 or IMAP port number in the edit box if your provider requires a different port number.
 - Note: if you check "SSL" above, the pop port number is automatically changed to 995 and the IMAP port to 993, as most servers with SSL require.
- 6) Press "F3" to move to "SMTP encryption type". The setting values are "None", "SSL" and "TLS". If your Outgoing SMTP server uses "SSL" or "TLS", change the value by pressing "Space".
- 7) Press "F3" to move to "SMTP port number": set to 25 by default. If you have chosen "SSL" as the encryption type, the default port number is set to 465 as is most commonly used for SMTP servers using SSL. If you have chosen TLS, the default SMTP port number is set to 587. Type a different value in the edit box if your provider uses a different SMTP port number.

- 8) Press "F3" to move to "SMTP username". If your Internet account provider requires you to use a different SMTP username, enter it in this edit box.
- 9) Press "F3" to move to "SMTP password". Type the password that corresponds to the above ID (using computer Braille if entering US Braille). If your SMTP username and password are not different, leave these items blank.
- 10) Press "F3" to move to "Use as default send-from account" To use the username and password that were set in steps 7 and 8, select the check box by pressing "Space".
- 11) Press "F3", move to "Use signature". The default value is "No". To attach a signature to your mail, press "Space" to change "No" to "Yes".
- 12) If you chose yes, press "F3", to move to "Signature" and press "Enter".
- 13) You are placed in the "Signature:" multi edit box. Type the information for the signature you want to attach to your e-mails.
- 14) Tab to "Confirm" and press "Enter".
- 15) Tab to the "Confirm" button and press "Enter" to save the settings in the "Advanced" dialog.
- 16) Tab to the "Confirm" button one more time to save the settings in the main "Account information" dialog.

7.3 Managing Accounts

To view the information about a stored e-mail account, open the "Accounts Manager" as described previously. Navigate to the account for which you want information and press "Enter" or Tab to "Information" and press "Enter".

In this dialog you can read all information associated with an e-mail account. Use "Space-1" and "Space-4" to navigate the list of settings for the account. Press "Space-E" to return to the accounts list.

To modify the information for an e-mail account, open the "Accounts Manager" and navigate to the desired account in the list. When the desired account is displayed, press "F3" to move to the "Modify" button, and press "Enter". Or, press Enter-M to activate the "Modify" button.

You are placed in the "Account name" edit box. You can now make changes to the information presented. The "Modify" dialog is exactly like the "Account setup" dialog, except that all the information is already filled in. Use normal editing and navigation commands to navigate to the information and make your desired changes.

When you have finished making modifications, press "F3" to move to "Confirm" and press "Enter".

Use the following steps to delete an E-mail account.

- 1) Open the Accounts Manager in the manner previously described.
- 2) Press "Space-4" or "Space-1" to move to the desired account in the account list.
- 3) When the desired account is displayed, press "Space-D" Or, press "F3" to move to the "Delete" button, and press "Enter".
- 4) BrailleSense prompts, "Are you sure you want to delete the (added account name) account? Yes".
- 5) Press "enter" To delete it. Press "Space-Z" to cancel the deletion.

7.4 Receiving E-mail

To check for new mail, press "Enter-N". You can also choose "Check for New Mail" from the "Message" menu. For POP3 accounts, the BrailleSense displays "There are xx messages to receive. Now receiving." "x/y mail received;" "Successfully received x out of y message" is displayed. When the e-mail messages are downloaded from the server, the e-mail message list is displayed in the "Inbox".

To cancel the download, press "F3" to move to the "Cancel" button, and press "enter", or press "Space-E".

When receiving messages via an IMAP server, you can both check the newest mail, according to the number of messages you have it set to automatically receive in the "Set options" dialog, or you can ask the BrailleSense to load older e-mail messages. To receive the newest e-mail messages, press "Enter-N" from the message list of the mailbox on which you are located. To receive previous mail, press "Backspace-P". You can also access "Get new mail" and "Get Previous mail" from the "Message" menu.

In addition, when using IMAP, as you navigate the various mail boxes in an account, new mail is automatically received for each mail box as you open it.

7.5 Reading Messages

E-mail messages are displayed in the Inbox and other mailboxes with the following information: "Subject", "Date", "From" (Sender), "CC", "Message" (main body of the e-mail) and "Attach" (displayed only for an e-mail with one or more attached files) you can move among the items by pressing "F3" or "Space-F3".

If there is a file attached to the e-mail, an A is shown before the subject, and the number of attachments is announced as you navigate the message list. The BrailleSense also indicates which e-mail messages have been read, and which e-mail messages are unread. E-mail messages that have not yet been read have a dash placed in front of the subject.

"Date" shows the date and time when the e-mail was received in the following order: day, month, date, and year. The time is displayed in the following order: hour, minute, and second using 24 hour time format.

"From" shows the sender information; "From: (sender's e-mail address) you can save the e-mail id of the sender to your contacts in the "Address Manager" using the following steps:

- 1) Press "Enter-I" when the sender is displayed.
- 2) The "add address" dialog is opened, and "Name: (sender's user name or e-mail address)" is displayed.
- 3) Type the sender's first and last name, or, if what is automatically filled in is acceptable, press "F3" to move to "confirm", and press "enter". The contact is saved, and you are returned to your previous position in the message list. If the contact already exists, you will be prompted, and given the option to modify the contact.

You can also save the e-mail id of the carbon copy recipient (CC) to the Address Manager using the process described above.

*Note: You can choose the content that is displayed in the Message List as well as the order in which it appears in the options menu described later in this chapter.

You can move to the body of a message by pressing "F3". Or, you can press "enter" when "Subject", "Date", or "From" is displayed. When you press "enter", the entire message is automatically spoken. To interrupt reading, press "Backspace-enter". Navigate the message as you would a document in the "Word Processor". To return to the "Subject", press "Space-E".

- Note: Messages in the BrailleSense are opened as plain text. If you receive an html e-mail, you can access the links and other HTML content by viewing the message in the BrailleSense's Web Browser. Open the message with the Web browser by pressing "Backspace-O", or by selecting "Open Message in Web Browser" from the "Message" menu.

You can find and download attached files by tabbing to "Attach" on messages containing attachments.

To save an attachment, move to the desired file name in the attachments list and press "Enter". You are prompted, "Attachments will be saved to /flashdisk/download. Do you want to change this location? No". Press "enter". The selected file is saved in the "download" folder on the "flashdisk". If you have more than one attachment you want to download, you can press "Enter-A" to select all attachments in the list, or press space on any number of individual files to select them.

To change the download path: When you are prompted, "Attachments will be saved to /flashdisk/download. Do you want to change this location?", navigate to "Yes" and press "Enter". You are placed in the file list of the "Downloads" folder on the "flashdisk". Use normal file and folder navigation to locate the folder into which you want to save the attachments. Press "Space" to select the folder in which to save the attachments, and press "Enter". * Note: when using IMAP, the message must be opened before you can navigate to the attachments.

You can delete e-mail messages individually, or you can select multiple messages and delete them simultaneously. Press "Space-D" to delete the

current message, or select multiple messages with "Space", then press "Space-D" to delete all of them.

When you delete messages, they are automatically moved to the "Trash" folder, where they remain until you delete them from there, or until you empty the "Trash" folder. You can empty the "Trash" folder at any time by pressing "Enter-Backspace-Q" or choosing 'Empty Trash' from the "Edit" menu.

- Note: When messages are deleted from the "Trash" folder, they are permanently disposed of, and cannot be retrieved.

7.6 Writing E-mail

To write a new message, choose "Write New Message" from the "File" menu, or press "Enter-W" from the message list. "To:" is displayed on the Braille display. If you wish to use a different account from the default, Shift-Tab to the "Account" combo box, choose your desired account, and Tab back to the "To" field.

Type the e-mail address of the recipient, or rather, the person to whom you are sending the message. If using US Braille, you must use computer Braille. To send to multiple recipients, place a semicolon between the e-mail addresses.

You can also add recipients from the address list by selecting "Look up Address" from the "File" menu or pressing "Enter-L". Type the First name, last name or part of the e-mail address of the desired recipient, and press "enter".

Addresses matching your search are displayed. If you press "enter" without typing in the desired name, all of the addresses in your address list are displayed. If there is no matching address, "No match" is announced. Move to the desired address, and press "enter". You can select more than one address by pressing "Space", then enter to input all of them.

You can also type any of the information above directly into the To or CC fields, and the first match will automatically be filled in when you Tab to the next field.

When you are finished entering recipients in the "To" field, press "F3" to move to "CC:" and "BCC", and input any copy or blind copy recipients as described above. If you do not need to carbon copy anyone, you can skip to the next step.

Press "F3" to move to "Subject", and type the subject of the e-mail message. Finally, press "F3" to move to "Message body:", and type the text of your e-mail message. Use the same input and editing methods as are used in the Word Processor.

When you have finished writing your e-mail message, select "Send" from the "File" menu, or press "Enter-S" directly from your message. When the e-mail is finished sending, "Successfully sent new message" is displayed. If sending fails, the BrailleSense displays "Unable to send message", often with a specific error explaining the reason for the inability to send the message. If this occurs, the e-mail is saved in the "Outbox".

To cancel e-mail message creation or sending, press "Space-E" or "Space-Z" and you are returned to the "Inbox".

7.6.1 ATTACHING A FILE

To attach a file to a message, select "Attach File" from the "File" menu, or press "Enter-F" while writing an e-mail message. Use normal file and folder navigation keys to locate the file or files you want to attach.

Press "enter" on the desired file to attach it. To attach more than one file, press "Space" to select each file you want to attach. * is displayed in front of the selected file names. Press "enter" To attach the selected files. Now, the "Attach" item is displayed in the tab order after the "Message body".

7.6.2 SAVING E-MAIL TO THE OUTBOX

Messages are saved in the "Outbox" under 2 conditions:

1. The user manually saves the e-mail to the "Outbox" for later sending.
2. An e-mail is automatically saved to the "outbox" after it fails to send due to a network error or other reason.

While writing a message, you can save by selecting "Save Mail in Outbox" from the "File" menu, or pressing "Space-S". A default file name is shown titled with the subject of the email. Press "Enter" to save the e-mail under the current file name, or type a new file name and change the path where the file is stored using the same method as in the "Save As" dialog of the Word Processor.

- Note: If you choose to save the e-mail in a location other than "flashdisk/email/outbox", you cannot access the e-mail message in the "Outbox". Thus, you cannot send the e-mail unless it is returned to the "Outbox" folder.

While creating a "message", press "Space-E" or "Space-Z" To cancel writing the e-mail. The BrailleSense displays "Currently writing mail. Save? Yes". Press Enter to save the message in the Outbox. To return to the "Inbox" without saving the e-mail, press "Space" to change "Yes" to "No", and press "Enter".

To send a message from the Outbox, "Shift-Tab" to the mailbox list, navigate to the Outbox, and Tab to the message list. Press "Space-1" or "Space-4", to move to the message you want to send.

If you want to send all the e-mails in the list: Press "Enter-a (dot 1) to select all. If you want to send individual messages, use normal selection commands to select them. Press "Enter" on "Send" from the "File" menu, or just press "Enter-S" from the outbox, and the messages are sent.

7.7 Navigating Accounts and Mailboxes

There are two ways to move to an account or mail box.

You can access the accounts list or mailbox list by moving through the tab order until you reach "mailboxes" or "Accounts". Use "Space-1" or "Space-4" to move among the items in the list, and press "F3" to open the Account or mailbox.

You can also use the following hot keys to navigate accounts and mailboxes:

Go to next mailbox: Enter-3-4-5.

Go to previous mailbox: Enter-1-2-6.

Next account: Backspace-dots-3-4-5.

Previous account: Backspace-1-2-6.

- Note: These hot keys are not available from the account list or mail box list.

7.7.1 MANAGING MAILBOXES

You can create additional mailboxes in the "E-mail" folder for sorting and storing your messages.

To create a new mailbox, from the Mailbox list, open the "Edit" menu and select "Create New Mailbox", or press "Enter-N" to activate this function directly. Type the mailbox name you want to create, and press "Enter".

The BrailleSense does not allow you to delete the "Inbox", "Sent", or "Outbox". However, you can delete folders/mailboxes you have created.

To delete a folder/mailbox, navigate to the mailbox you want to delete in the Mailbox list, open the "Edit" menu and choose "Delete Mailbox". You can also delete the current mailbox in the list by pressing "Space-D".

You can also rename mailboxes you have created. To do this, from the mailbox list, navigate to the mailbox you wish to rename. Open the Edit Menu and select "Rename mailbox", or press "Enter-R" to open the "rename mailbox" dialog directly. Type the new name for the folder mailbox, and press "Enter".

7.7.2 COPYING AND MOVING TO MAILBOXES

You can copy and move e-mails using the "Copy to Mailbox" or "Move to Mailbox" commands in the "Edit" menu.

The "move/Copy dialog" contains the following:

- 1) "Folder" list.
- 2) "Create mailbox" Button
- 3) "Delete mailbox" Button
- 4) "Rename mailbox" Button
- 5) "Confirm" Button
- 6) "Cancel" Button

To copy and move an e-mail to another folder, follow these steps:

- 1) Select individual e-mails by pressing "Space", select a continuous group with "Enter-B" or select all with "Enter-A".
- 2) Open the "Edit" menu, and press "SPACE-4", to move to "Copy to Mailbox" or "Move to Mailbox" and press "Enter". You can activate these functions directly from the message list by pressing "Backspace-C" for "Copy" and "Backspace-X" for "Move to".
- 3) BrailleSense displays "Folder list".

4) Press "Space-4", to move to the folder into which you want to copy or move messages and press "Enter".

5) The e-mail is moved or copied to the selected folder, and you are returned to the mailbox from which you activated the "Move" or "Copy" command.

7.7.3 IMPORTING EMAIL

You may wish to access e-mail on your BrailleSense from older Braille Sense devices or other units that use EML message files. You can do this using the Import function in the File Menu, or by pressing "Backspace-I" from the message or mailbox list.

*Note: files and folders selected in this dialog will import to the currently open or focused mailbox. So, if you wish to import to a clean folder, you must create a folder as described previously, and be located within it or focused on it before you start this process.

When you activate the import function, you are placed in an "Open File" dialog. Use normal file and folder navigation to locate the folder where your messages are stored. Use File management selection commands to select the folders and files you want to import, and press Enter. Import progress is shown as a percentage.

7.8 Advanced Message Features

7.8.1 REPLY AND REPLY ALL

When you reply to an e-mail, you can edit or add a message to the original message, or add another recipient. Choosing "Reply" sends an e-mail to only

the original sender. Choosing "Reply To All" sends an e-mail back to the original sender as well as all other recipients of the original e-mail.

1) From "Subject", "Date", or "Message body" of a message in the "Inbox", select "Reply" or "Reply All" from the "Message" menu, or press "Enter-R" or "Backspace-R" respectively.

2) You are placed in the message body, which already contains the original message body of the received e-mail; below "--- Original Message ---".

3) You can type additional text, edit the original message, and/or edit any of the fields associated with the message including the subject and recipient fields.

4) When you have finished, press "Enter-S" to send the e-mail.

You can save replies just as you can save new messages in the "Outbox" for sending later.

7.8.2 FORWARDING A MESSAGE

"Forwarding" refers to sending an e-mail you received to another person.

1) From the "Subject", "Date", or "Message body", open the "Message" menu and select "Forward", or press "Enter-F".

2) BrailleSense displays "To:". Input the e-mail address of the new recipient, or multiple recipients, using computer Braille.

3) Press "F3", to move to "CC", "BCC", and "Subject" and edit them as you wish. The subject will appear as "FW: (original subject)".

4) "Tab" to the message body. The original e-mail message is shown following, "----- Forwarded Message -----".

5) Type any additional text you wish to add to the message, or make any edits to the forwarded text, and press "Enter-S" to send.

If you don't want the e-mail to appear in the same message body as the e-mail you are writing, you can also forward the current e-mail as an attachment. Use "Backspace-W" to send the current e-mail as an attachment, then use the process described above.

7.8.3 SAVING AN E-MAIL

There may be times when you wish to save a received message as a file. Please note that there is a small difference between IMAP and POP3.

- 1) Using Pop3: when placed on one of the fields of an e-mail message you want to save, select "Save as Text" from the "File" menu, or press "Space-S".
- 2) BrailleSense displays "File name: (subject of e-mail being read.txt)".
- 3) Press "Enter" to save the message under the current file name. If you wish, you can rename the file using the same method as in the "Save As" dialog of the "Word Processor".

If you are using IMAP, you must first navigate to and open the message before you can save it. Otherwise the steps are exactly as those above.

7.8.4 FINDING MESSAGES

The "Find" command is used to search for messages in the current mailbox. The "Find" function looks for text that matches the text you enter in "Subject", "Date", and/or "From". This function only works in the "Subject", "Date", or "From" controls. Use "Space-F" to open the "Find" dialog, or open the "Edit" menu and press "Enter" on "Find".

Use "F3" and "Space-F3" to move among the controls. Type the text to search in the Edit box. Choose what parts of the message to "Search". Choose the direction to search, and press "Enter".

The "Find Again" command finds the next occurrence of the text according to the search text and direction specified in the "Find" dialog. As with "Find", the "Find Again" function only works in the "Subject", "Date", or "From" fields. Open the "Edit" menu and press "Enter" on "Find Again", or press "Backspace-F" from the message list.

7.8.5 MOVE TO NEXT UNREAD MESSAGE

The "Move To Next Unread Message" command allows you to move to the next unread message. If you use this function on the last unread message in a list, you are taken to the first unread message in the list.

To activate this function from the "Subject", "Date", or "From" field, open the "Edit" menu and select "Move to next Unread Message" or press "Enter-U" to activate this function directly.

7.8.6 ACTIVATING LINKS

Many e-mail messages contain web links, either to general websites, or to calendar/online events.

You can open a link in the BrailleSense Browser by pressing "Enter-L" when your cursor is placed on the Web address.

If you are activating a link to a Zoom meeting, Google Meet invitation, a Teams live event, or other item that requires launching of an installed Android ap, you must open this link in an Android browser, as the BrailleSense browser does not automatically launch other programs.

To open a link located in the current paragraph in the default Android browser, press "Backspace-Enter-L". The browser will launch, which will then, in turn, activate the associated Android program, provided that program is installed on

your BrailleSense. For example, if you want to launch a Zoom meeting, you need to install the Zoom application on your BrailleSense from the Play Store.

7.9 Using the Tools Menu

In addition to the "Accounts Manager", the "Tools" menu contains options for setting the save path, spam filters and other e-mail options.

7.9.1 SET PATH

"Set Path" sets the default download path where file attachments and e-mail are stored. To set the download path, open the "Tools" menu and press "Enter" on "Save Path", or press "Enter-P" from the message list.

The "Set Path" dialog contains three settings:

- 1) Save attachments path
- 2) Send attachments path
- 3) Disk to save mail in

You can move among these items by pressing "Space-4" and "Space-1". Each item consists of "Path", "Modify", and "Close". At "Path", the default path is displayed. You can move among this information by pressing "F3" or "Space-F3". If you choose to modify any path, use normal file management navigation to locate your desired folder, and press "Enter".

7.9.2 SET OPTIONS

To open the "Set options" dialog, press "Enter" on "Options" from the "Tools" menu, or press "Enter-O" to activate this item directly.

The first item in the "Set options" dialog is "Default server?" You can set the default server to "IMAP" or "POP3". Use "Space" to change the value of the setting.

The next item in the "Set Options" dialog is "Use inbox as default mailbox when using POP3 server? Yes.". If you want the e-mail program to always place you in the "inbox" when starting e-mail using POP3, leave this setting as is. If you would like to use a different mailbox, press "Space" to change this option to "no".

The next option in the dialog is "Get the number of mail at once when using IMAP". The default is set to 64. Use "Space" or "Backspace" to cycle through the various options. The available values are All, 32, 64, 96, and 128.

The next 4 options relate to how information appears in the Message List. For each of the 4 message headers, you can choose to display the sender, (From, the recipient (To), subject of the message (Subject), or the date and time the message was received (Date). For the 2nd, 3rd and 4th headers, you also have the option not to display them at all (Off). This option is not available for the 1st header as, if all headers were turned off, the Message list would display no content. This allows you to show only the content you want to see in the Message list, as well as to determine in what order it appears.

As you make choices for each header, these disappear from the options for the remaining headers. For example, if you choose "Subject" for the 1st header, you will not see this as an option for your 2nd, 3rd and 4th headers. But, if you change the first header to "From", the "Subject" option will re-appear, and the "From" option will not appear for the rest of the headers.

When you have finished adjusting the settings, press "Enter" to save the options, or tab to the "Confirm" button and press "Enter". If you do not wish to save your changes, tab to the "cancel" button and press "Enter" or press "Space-Z".

7.9.3 SPAM SETTINGS

When the BrailleSense downloads e-mail from the e-mail server, there may be a number of e-mail messages that are spam. In this case, you can avoid downloading these spam e-mail messages by identifying specified words in the subject or specifying e-mail addresses you know are from spammers.

The spam e-mail messages that are identified by your "Spam Settings" are not downloaded into the BrailleSense, and remain on the e-mail server.

Use the following steps to set spam e-mail filters:

- 1) Open the "Tools" menu and select "Spam Settings", or press "Enter-E" to activate this item without accessing the menus.
- 2) The "Spam Settings" dialog contains 6 items: "Type" combo box, "List", "Add" button, "Modify" button, "Delete" button, and a "Close" button.
- 3) "Type" can be set to 1 of 3 options: "Subject", "E-mail address string", and "Host". You can move among these options by pressing "Space-1" or "Space-4". The "list" is displayed according to the "Type" you choose.

— If you have selected "Subject", the list is displayed as "Subject: (Registered word) 1/xx". If you selected "E-mail address string", it is displayed as "E-mail address string: (Registered e-mail address string) 1/xx". It is displayed as "no items" if there is no registered word or e-mail address string.

How to register Spam Settings:

- 1) Press A when "Type: Subject 1/3" is displayed. Or, press "F3" to move to the "Add" button, and press "enter".
- 2) The "add dialog" is opened and "Subject add:" is displayed.

3) Type the words, by which you want to identify spam messages and press "Enter". Or, "Tab" to the "Confirm" button, and press "enter".

4) "Successfully added spam data". is announced and you are returned to the "list". "Subject: (registered word) 1/1" is displayed.

By adding the word in the manner just described, if there are e-mail messages that include the words you entered in the subject of the message, the BrailleSense downloads those e-mails from the e-mail server, but deletes them automatically.

You can also avoid receiving e-mail messages from a specific sender or host by registering it in the same manner.

To cancel setting spam filters from the "Add dialog", press "F3" to move to the "Cancel" button, and press "Enter". Or, press "Space-E" or "Space-Z".

To modify a spam filter you have set, do the following:

1) Access the "Spam Settings" dialog as described previously, and navigate to the "Type" of spam setting you want to modify by pressing "Space-1" or "Space-4".

2) Press "F3" to move to the list of registered items in that category.

3) Navigate to the item you want to modify using "Space-1" or "Space-4".

4) Press M, or press "F3" to move to the "Modify" button, and press "Enter".

5) Make the modifications in the edit box.

6) Press "F3" to move to the "confirm" button, and press "Enter" to save the changes.

You can also delete an item that has been registered in the spam list.

- 1) Access the "Spam Settings" dialog as described previously, and navigate to the "Type" of spam setting you want to delete by pressing "Space-1" or "Space-4".
- 2) Press "F3" to move to the list of registered items in that category.
- 3) Navigate to the item you want to delete using "Space-1" or "Space-4".
- 4) Press D or press "F3" to move to the "Delete" button, and press "Enter".

8. Exchange E-mail

The Exchange E-mail program has much in common with the E-mail program described in the previous chapter in terms of features and operation. Thus, this chapter will not describe it in detail, rather, it will focus on features that differ from the original E-mail program which supports IMAP and POP3.

For more complete and detailed information on supported features and functions, please check the previous chapter, press Space-H within the Exchange application for keystrokes, or press F2 to explore the menus.

The purpose of the Exchange E-mail program, as its name implies, is to support accounts using the Microsoft Exchange service. Currently, the BrailleSense programming supports only the syncing of e-mail content. However, appointments and contacts can also be synced simply by installing an Android app such as Outlook or AquaMail, and setting up your account. You do not have to use the app, but if it resides on the unit with your account set up, it will sync with the Android Contacts and Calendar apps which, in turn, sync with the BrailleSense Address Manager and Schedule Manager.

8.1 Executing Exchange E-mail

To execute the Exchange E-mail program, navigate to "Exchange E-mail" in the "program" menu, by pressing "Space-1" and "Space-4" and then "Enter", or just press L from the main program menu. You can launch the "Exchange E-mail program from anywhere on the unit by pressing "F1-L".

When you launch the Exchange E-mail program for the first time, the BrailleSense says, "You must create at least 1 mail account from the accounts manager in the Tools menu". Then, the "Inbox" is opened.

The inbox will display, "No items" as there will be no e-mail to receive. However, when an account is set up, and Exchange E-mail is launched, the most recent messages will be automatically retrieved according to the default mail server and number of messages you have set in the Options dialog.

Operation and features of the Exchange E-mail mailboxes, messages and accounts is very similar to IMAP operation and navigation in the original e-mail program described in the last chapter, but for the exceptions described in the rest of the current chapter.

8.2 Setting up an Exchange E-mail Account

Accounts are controlled in the Exchange app using the Accounts Manager, just as with the original e-mail program. However, Exchange accounts require much less information.

To set up a new account, press "Enter-M" to open the Accounts Manager, then "Enter-A" to add a new account.

The Exchange account setup only supports the entry of Account name, server address, username and password, as this is all that is required. Use "Space-1" and "Space-4" to move among the edit boxes.

When you have finished filling in the correct information, Tab to the "Signature" yes/no list box, and press "Space" to make your choice. If you chose to add a signature, Tab to the edit box and enter the necessary content.

Once you have entered all the necessary information, press "Enter" to save the account, or Tab to the Confirm button and press "Enter". Once saved, BrailleSense will automatically try to retrieve the most recent messages for the Inbox of the account you just created. If you entered something in error, you will receive the prompt, "Invalid account information", and will be placed in the Accounts list from where you can return to the Accounts Manager and modify the account information.

8.3 Saving Exchange messages offline

The "File" menu of the Exchange e-mail program parallels the original e-mail program, but for one exception: just above the "Exit" item is an option to "Save Offline to BrailleSense Mail".

As Exchange is a constantly syncing protocol, it's content changes dynamically as you send and receive messages. There may be times you wish for a message to remain in a fixed position, locally on your unit. For this reason, we allow you to save a message offline.

When you activate this function, you are given a choice as to what account you want to save it to from the original BrailleSense E-mail program. Once you choose an account to save to, the message will appear in the Inbox of that account as though you received it there. Please note: if the mailbox is currently open, you will need to refresh the Message List to see the saved messages.

8.4 Navigating Mailboxes

In the Exchange application, due to the ability to use and retrieve sub folders in Exchange accounts, a multi-level directory structure must be supported, unlike the original e-mail program, which supports only a single folder level. For this reason, navigation of mailboxes is slightly different in the Exchange program.

If a mailbox contains sub folders, press "Enter" and "Backspace" to navigate in and out of them respectively, just as you would do in the folder structure of the File Manager.

Because "Enter" and "Backspace" are used to navigate folder levels, you cannot use "Enter" to navigate forward from accounts, to folders, to messages as you can do in the original e-mail program. You must now always use the Tab function (Space-4-5, or F3) to perform this function.

8.5 Exchange Options

The Options dialog in the "Tools" menu contains all the same items as does the original e-mail program with one addition. Just before the 1st e-mail header setting in the tab order is an option for choosing the "Message List Type".

This is related to the speed at which e-mail messages are received. If you choose to display less header information, e-mail is received faster. The settings are "Detailed" and "Simple".

The "Detailed" setting displays the Attachment count, title, sender name, sender email, and the date. The "Simple" view displays the Attachment count, title, sender name, and the date.

This setting also affects the Spam Settings in the "Tools" menu. If you select the "Detailed" option, you can add the subject, address or host to the spam filter. If you select the "Simple" option, you can add only the subject to the spam filter.

9. Media

The "Media" menu on the BrailleSense 6 consists of programs related to media playback: specifically music, videos, and DAISY content. The "Media" menu contains 3 items: "Media Player", "FM Radio", and "DAISY Player".

You can open the "Media" menu by pressing "F1" to bring up the main "Program" menu, then navigating to "Media" using the DOWN scroll key and pressing "Enter". Or, you can quickly jump to and open the "Media" menu by pressing M from the main menu.

9.1 Media Player

You can use the Media Player to play audio files as well as listen to the audio for some video file formats. The Media Player plays the following formats, ac3, asf, asx, m3u, mp2, mp3, MP 4, mpa, mpg, ogg, pls, wav, wax, wma, flac, midi and wmv.

You can control the playing of media files using either Braille keyboard commands or the media buttons on the front panel of the BrailleSense. To launch the "Media Player", navigate to it in the "Media" menu and press "Enter". Or, launch it from anywhere on the unit by pressing "F1-M".

9.1.1 USING THE MEDIA BUTTONS

Near the center of the front panel on the BrailleSense is a group of 5 buttons. These buttons can be used to control the Media Player. Moving from left to right, the buttons are: "Back", "Record", "Stop", "Play/Pause", and "Forward".

The media buttons can control media playback, DAISY content playback, or application operation, depending on the position of the Media Mode switch,

located to the left of the media buttons. Slide this switch to the right-most position, "Media Mode", to control media playback.

If the "Play/Pause" button is pressed while you are using another program on the BrailleSense, the unit opens the Media Player and begins playing the files in the most recently played list. If there are no files in the playlist, the BrailleSense says, "There are no items to display". "Back", "Stop", and "Forward" are available after audio playback is started.

- Note: As the BrailleSense is able to multi-task, initiating audio playback with the "Play/Pause" button does not close the program you were running when you pressed it. To return to your former position, press "F2-F3" to cycle through the running programs until you reach your former location.

A playlist can contain only 1 file, or many files. If there is more than one file in the playlist when you press the "Play/Pause" button, the BrailleSense plays all the files in the list one after another. If you press the "Play/Pause" button during playback, audio playback is paused. To resume playing, press the button again. If you press the "Play/Pause" button while you are in the "Record" dialog, the most recently recorded file is played.

If you press the "Forward" button, the next file in the playlist is played, and pressing the "Back" button plays the previous file. If you press the "Forward" button (while the last file in the list is playing, (or the "Back" button (while the first file in the list is playing), it has no effect, as there is no file to navigate to.

You can use the "Record" button to create a recording using the internal microphone or an external microphone from anywhere on the unit provided the Media Mode switch is set to the right-most position. Pressing the "Record" button once opens the "Record" dialog. Press the "Record" button again to start recording. Press the "Record" button while you are recording to pause recording. Resume recording by pressing the "Record" button one more time.

Press the "Stop" button to stop and save your recording. The recorded file is saved in the "flashdisk/music/record" folder.

The "Stop" button is located in the center of the group and has a dot on it. It is used to stop playback or recording. If you press "Stop" during audio playback, the next time media playback is launched, the BrailleSense starts playing the first file in the playlist. If you press this button while recording, the BrailleSense stops and saves the recording. If you press the "Stop" button in the "Record" dialog when not recording, the dialog closes.

9.1.2 MEDIA PLAYER LAYOUT

You can use the Braille keyboard to perform the same functions as the media buttons as well as more advanced features in the Media Player.

The playback dialog contains two information tabs: "playback information" and "playlist". Press "F3" or "Space-F3" to move between these tabs.

The "playback information" tab displays the track number, the file name, and the current playback status: (play, pause, or stop). If there are no files in the playlist, the BrailleSense displays "Title: There are no items to display". If you press "Space-4", the BrailleSense displays playback time information.

When in the Playback tab, you can use the cursor routers to jump to specific areas of the media file. For example, if you want to move to a place near the middle of the file, press cursor router 16 or 17. If you want to move to a place near 75%, press cursor router 24 or 25.

The "playlist" tab displays all the files in the playlist. For each file in the playlist, it displays the track number, file name, the location of the file in the list, and the total number of files in the list. If there is no file in the playlist, the BrailleSense displays the message "no items".

The BrailleSense's Media Player supports the viewing of lyrics for MP3 files which contain them. The "Lyrics tab appears in the tab order only if the file you are playing contains lyrics in its file information. The lyrics are displayed in a multi edit box which you can read and navigate using normal document navigation commands.

The Media Player menu contains five items: "File", "Play", "Record", "Position", "Mark" and "Settings".

To open the Media Player menu, press "Space-M" or "F2". Select a menu item using "Space-1" or "Space-4" and press "Enter". Or, you can press "Space-M" or "F2" followed by the shortcut for the corresponding menu item. For example to open the Record menu, press "Space-M" or "F2" to open the menu, and press R.

The Media Player also contains various lists and dialog boxes. Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab among controls in a dialog. For the purposes of these instructions, we will generally refer to using "F3" and "Space-F3" or simply "Tab". However, in these instances, "Space-4-5" and "Space-1-2" can also be used. Press "Enter" to execute an item.

9.1.3 THE FILE MENU

- 1) Open File: (O) "Enter-O". Opens a media file and creates a new playlist.
- 2) Open Folder: (F) "Enter-F". Creates a new playlist with all the media files in the selected folder.
- 3) Add File: (A) "Backspace-O". Adds the selected files to the current playlist.
- 4) Add Folder: (I) "Backspace-F". Adds all the media files in the selected folders to the current playlist.

- 5) Save Playlist: (S) "Space-I". Saves the current playlist.
- 6) Save As Playlist. Saves the currently playing files as a playlist.
- 7) Delete Item: (D) "Space-D". Deletes the current item from the playlist.
- 8) Tag Information: (T) "Enter-T". Opens the properties for the current media file.
- 9) Open URL: (U) "Enter-U". Opens a website for streaming playback.
- 10) Exit: (Z) "Enter-Z". Exits the Media Player.

9.1.4 THE PLAY MENU

- 1) Play (P) "Enter". Initiates playback. You can also press the "Play" button.
- 2) Previous Track: (B) "Dot-3". Moves back 1 track. You can also press the "Back" button.
- 3) Next Track: (N) "Dot-6". Moves forward 1 track. You can also press the "Forward" button.
- 4) Back 5 Tracks: (D) "Dots-2-3". Moves back 5 tracks.
- 5) Forward 5 tracks: (U) "Dots-5-6". Moves forward 5 tracks.
- 6) First Track: (T) "Dots-1-3". Moves to the first track in the playlist.
- 7) Last Track: (E) "Dots-4-6". Moves to the last track in the playlist.
- 8) Volume Up: (H) "Space-Up Scroll". Increases the volume.
- 9) Volume Down: (L) "Space-Down Scroll". Decreases the volume.
- 10) Speed Up: (K) "Dots-3-4-5". Increases the playback speed.
- 11) Slow Down: (M) "Dots-1-2-6". Decreases playback speed.
- 12) Pause: (A) "Space". Pauses playback. Press again to resume. You can also use the "Play" button.

13) Stop: (S) "Backspace". Stops playback. You can also use the "Stop" button.

9.1.5 MAKING RECORDINGS

1) Record: (R) "Enter-R". Places you in standby mode. Press again to initiate recording. Press "Space-Z" when finished. You can also use the "Record" button to start recording and the "Stop" button to finish.

9.1.6 THE POSITION MENU

- 1) Mark Position: (M) "Enter-M". Marks the current position for later use.
- 2) Delete marked Position: (D) "Enter-D". Unmark the marked position.
- 3) Jump to Marked Position: (J) "Enter-J". Go to the position previously marked.
- 4) Go to Time: (G) "Enter-G". Go to a specific time in the file.
- 5) Go to Percent: (E) "Backspace-P". Move to a place in the file according to its percentage.
- 6) Set Start Point: (L) "Enter-P". Set a start point for repeat play.
- 7) Release Point: (R) "Space-E". Set the end point for repeat playback.

9.1.7 THE MARK MENU

- 1) Set Mark: (S) "Backspace-M". Bookmark a specific point the media file.
- 2) Jump to Mark: (M) "Backspace-J". Move to a previously set mark.
- 3) Previous Mark: (P) "Backspace-dot-3". Moves back one mark from your current position.
- 4) Next Mark: (N) "Backspace-dot-6". Moves to the next mark after your current position.

5) Delete Mark: (D) "Backspace-D". Deletes a mark.

6) Mark Manager: (K) "Backspace-K". View and modify all marks in the current file.

9.1.8 THE SETTINGS MENU

1) Configuration: (C) "Backspace-C". Lets you configure file information display, and exit position saving.

2) Playback Settings: (P) "Enter-S". Settings for Repeat Shuffle and other playlist settings.

3) Record Settings: (R) "Backspace-S". Choose recording source, file type, sample rate, bitrate, etc.

4) Effect Settings: (E) "Enter-E". Set Equalizer and bass boost.

9.2 FM Radio

To use the "FM Radio" on the BrailleSense, you must connect headphones to the headphone jack on the unit, as the headphone cord serves as the antenna, and without it, radio reception will be poor to nonexistent.

From the Media menu, use the scroll buttons to navigate to "FM Radio" and press "Enter". You can also press R to quickly jump to and launch the "FM Radio". You can launch the "FM Radio" from anywhere on the unit by pressing "F1-R".

When the FM Radio is launched, your BrailleSense announces the radio frequency. If you have used the FM Radio program previously, the BrailleSense tunes to the frequency that you last listened to.

- Note: the frequency range of the FM Radio is 87.5MHz to 108MHz.

9.2.1 LISTENING TO FM RADIO VIA THE INTERNAL SPEAKERS

When you execute the FM Radio after connecting headphones, sound from the radio is directed through the headphones. As previously explained, it is necessary to connect the headphones to provide an antenna for radio reception. However, there may be times when you wish for the sound from the radio to output through the speakers on the BrailleSense. To listen to the radio through the internal speakers, press X, to toggle internal speaker output on/off.

You can adjust the radio volume using Space with the Up and Down scroll buttons.

9.2.2 FREQUENCY CONTROL

To tune to another frequency, use the following commands:

Press "dot 5" to scan forward for the next radio station, and "dot 2", to auto scan backward. Press "dot 6", to increase the frequency by 0.1MHz, and "dot 3", to decrease the frequency by 0.1MHz. Press "Space-F" to input the frequency directly. Type the frequency in numbers in the edit box and press "Enter".

You can also use the media buttons on the front panel to control the FM Radio. The Media mode switch should be moved to the left-most position, "Application Mode." Pressing the "Back" button scans backward, and pressing the "Forward" button scans forward.

You can also activate frequency movement commands from the "Frequency" menu.

9.2.3 ADDING AND NAVIGATING PRESETS

To add the currently selected frequency as a preset, select "Add Preset" from the "Presets" menu, or press "Enter-S".

The "add preset" dialog is opened and "Frequency: (current frequency)" is announced. "Tab" to the "Label" edit box. Type the name by which you want to identify the preset. Press "Enter" to save the preset.

To add a frequency other than the one you're currently listening to, you may type the number of the frequency you wish to add in the "Frequency" edit box of the "add preset" dialog.

You can navigate your presets by using "dot 1" to move back, and "dot 4" to move forward a preset.

To delete a preset, navigate to the preset and press "Space-D".

- Note: when this command is executed, the preset is deleted instantly without any warning or verification.

You can also add and delete presets using the Preset Manager dialog. Select "Preset Manager" from the "Presets" menu, or press "Enter-M".

The Preset Manager dialog contains the "Preset list", an "Add" button, a "Delete" button, a "Modify" button and a "Close" button.

Pressing "Enter" on the "add" button opens the "add preset" dialog explained above. The "Modify" dialog lets you change the label of the current preset in the list. Use the "Delete" button to delete a preset from the preset list. To exit the Preset Manager" dialog navigate to the "Close" button and press "Enter" or press "Space-E".

You can also use the media buttons on the front panel to add and move among presets. Press the "record" button for more than 2 seconds, to open the "add preset" dialog. To move to the previous preset, press the "Back" button for more

than 2 seconds. Pressing the "Forward" button for more than 2 seconds moves to the next preset. Pressing the "stop" button for more than 2 seconds deletes the selected preset.

9.2.4 RECORDING FROM THE FM RADIO

To record the audio of the radio station to which you are currently listening, Select "Record" from the radio's menu, or press "Enter-R". The "record" dialog is opened and the BrailleSense begins recording.

While recording the radio, "Tab" to the "Pause" button and press "Enter", or press A to pause recording. The name of this button is changed to "Continue". Activate the "Continue" button or press G, recording resumes. To stop the recording, press S. Or, "Tab" to the "Stop" button and press "Enter".

To check the duration of the current recording press "Space-4". The unit announces the current recording run time and total possible recording time.

Recorded files are saved in the "flashdisk/music/radio" folder. The files are named according to the preset label and frequency. If you are recording a radio station that is not in your preset list, the recording is named "radio" followed by the frequency.

You can also instantly start recording by pressing the "record" button on the front panel. Press the "Stop" button to stop and save the recording. Press the "Record" button again to pause and resume recording. Pressing the "Back" and "Forward" buttons moves among the controls in the "record" dialog.

9.2.4.1 Changing record folder

To change the folder where recorded files are saved, select "Recording Settings" from the "File" menu or press "Backspace-S".

You are placed in the "Recording Settings" dialog on the first item, "Recording type: mp3". "Tab" to the "Record folder" setting.

"Record folder: /flashdisk/music/radio" is displayed. Press "Enter". Use normal folder navigation to locate your desired folder for storing recorded files. Press "Space" to select it and press "Enter", and you are returned to the "Record Settings" dialog. Press "Enter" again to save the settings and exit the dialog.

In the Record Settings dialog, you can also set recording type, sample rate, and bit rate. To change the recording type, press "Space" to toggle between "wav" and "mp3".

9.2.5 ADDITIONAL FEATURES

1) Sensitivity adjustment

If the BrailleSense cannot tune to the broadcast frequency you desire, adjust the scanning sensitivity. Open the "Settings" dialog. Press "Space-4" to navigate to "Sensitivity" and press "Space" or "Backspace" to adjust the sensitivity. Higher values increase the scanning sensitivity of the radio. If the sensitivity is increased, you can fine tune to more frequencies.

2) Mute

While listening to the radio, press "Space" to mute the audio. Pressing "Space" once again unmutes the radio. You can also mute the radio by pressing the "stop" button on the front panel, and unmute by pressing the "play" button.

9.3 DAISY Player

Using the DAISY Player on the BrailleSense, you can listen to files and read books that use the DAISY format. DAISY is an acronym for (DIGITAL ACCESSIBLE INFORMATION SYSTEM). One advantage to DAISY books is

that you can listen to the audio as well as read the associated text with the audio in DAISY books where both text and audio are present.

The BrailleSense supports DAISY books that are version 3.0 or earlier. The DAISY Player supports the following book types:

DAISY 2.0/2.02/3.0, DAISY XML/Digital Talking Book.

ANSI/NISO Z39.86 2002, 2005.

Bookshare.org DAISY content,

NIMAS 1.1

DRM: supports 2006 PDTB2 authentication.

9.3.1 ARRANGEMENT OF THE DAISY PLAYER

To launch the DAISY Player, press "Enter" on "DAISY Player" in the BrailleSense "Media" menu. You can also launch the DAISY player from anywhere on the unit by pressing "F1-D".

When you launch the DAISY Player, you are placed in BrailleSense's drive list. Press "Enter" on the drive containing the DAISY book you want to open.

Use normal folder navigation to locate the folder containing a DAISY book you want to open. Press "Space" to select it, then press "Enter" to open it. You must first select the folder containing the book, because a DAISY book contains multiple files, thus, BrailleSense needs to know exactly from which folder to get all the book text and information.

If you replay a book that you have listened to before, playback resumes from the location where you last exited. To stop playing a DAISY book, press "Space-Z". A prompt box appears asking if you would like to exit the DAISY Player. Select "Yes" or "No" with "Space" or "Backspace" and press "Enter".

You can also control DAISY playback using the media buttons on the front panel. The Media Mode switch must be set to the center position, "DAISY Mode". Start DAISY playback by pressing the "play" button on the front panel. Press "play" again to pause playback. Press the "Back" button to move back 1 navigational element, and press the "Forward" button to move forward by 1 navigational element. Use the "Record" button to set the navigational element by which the "Back" and "Forward" buttons move.

Open the DAISY Player menu by pressing "Space-M" or "F2". The menu contains "File", "Document", "Mark", and "Heading". You can navigate the menu items using "Space-1" and "Space-4". To activate a menu item, press "Enter".

Or, you can press "Space-M" or "F2" followed by the shortcut for the corresponding menu item. For example to open the "Heading" menu, press "Space-M" or "F2" to open the menu, and press H.

The DAISY Player also contains various lists and dialog boxes. Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab among controls in a dialog. For the purposes of these instructions, we will generally refer to using "F3" and "Space-F3" or simply "Tab". However, in these instances, "Space-4-5" and "Space-1-2" can also be used. Press "Enter" to execute an item.

While playing DAISY content, you are placed in a dialog in which four controls/components are available: "heading", "Information", "Play" and "Pause". "Tab" through this information using "F3".

"Heading" shows you a prefixed heading or navigation element as well as the text under that element if text exists. In "information", you can find the current page, level, phrase, elapsed time, and running time. You can move among these items with "Space-1" or "Space-4". You can press "Enter" on "Pause" or "Play",

to pause or play the audio. From anywhere in the dialog, you can also toggle play/pause with "Space".

9.3.2 THE FILE MENU

- 1) Open DAISY: (O) "Enter-O". Calls the "Open DAISY" dialog.
- 2) Voice Settings: (S) "Enter-S". Set auto playback settings as well as rate, pitch and volume of the reading voice.
- 3) Book Info: (I) "Enter-I". Obtain information about the currently playing DAISY book.
- 4) Exit: (Z) "Space-Z". Closes the DAISY Player.

9.3.3 THE DOCUMENT MENU

- 1) Play/Pause: (P) "Space-P". Pause and resume playback. You can also use the "Play" button.
- 2) Previous Page: (U) "Dots-1-2-6". Moves back one page.
- 3) Next Page: (D) "Dots-3-4-5". Moves forward one page.
- 4) Go To Page: (G) "Enter-G". Enter the number of the specific page you want to go to.
- 5) Move by Previous Movement Unit: (H) "Dot-3". Move back by specified movement type.
- 6) Move by Next Movement Unit: (L) "Dot-6". Move forward by specified movement type.
- 7) Change to Previous Movement Unit: (I) "Dots-2-3". Cycle backward one movement type.

8) Change to Next Movement Unit: (E) "Dots-5-6". Cycle forward one movement type.

9.3.4 THE MARK MENU

- 1) Set Mark: (M) "Enter-M". Bookmark a position.
- 2) Jump to Mark: (J) "Enter-J". Move to a previously set mark.
- 3) Delete Mark: (D) "Space-D". Delete a previously set mark.

9.3.5 THE HEADINGS MENU

- 1) Headings List: (L) "Enter-H". Open the list of headings in the book.
- 2) Scan for Next Heading: (N) "Enter-N". Move to the next available heading.
- 3) Read Current Heading: (W) "Enter-W". Read the heading under which the current text is located.
- 4) First Heading: (T) "Dots-1-3". Move to the first heading.
- 5) Last Heading: (B) "Dots-4-6". Move to the final heading of the book.
- 6) Find: (F) "Space-F". Search for text.
- 7) Find Again: (R) "Enter-F". Find the next instance of the search string you specified in the "Find" dialog.

9.3.6 THE MEMO MENU

- 1) Insert Memo: (M) "Backspace-M". Insert an annotation.
- 2) Read Memo: (R) "Backspace-R". Read the current memo.
- 3) Previous Memo: (P) "Backspace-P". Move back one memo from your current position.

4) Next Memo: (N) "Backspace-N". Move to the next memo from your current position.

5) Delete Memo: (D) "Backspace-D". Delete the current memo.

10. Organizer.

The "Organizer" menu contains items to help you organize and synchronize appointments and contacts. It contains the "Address Manager", and the "Schedule Manager".

Open the "Organizer" menu by navigating to "Organizer" in the main "Program" menu, or just by pressing "O" from the main menu.

10.1 Address Manager

The "Address Manager" is a program you can use to keep track of your contact information for friends and colleagues, such as company addresses, home addresses, e-mail addresses, as well as other information. You can also synchronize your contacts with your Google account under Accounts in the Settings menu of All Apps.

The "Address Manager" consists of two main dialogs: "Add Address" for entering contact information, and "Search Address" for retrieving this information.

In the "Organizer" menu, you can quickly jump to an open the "Address Manager" by typing "a" or by navigating to the "Address Manager" and pressing "Enter".

When the "Address Manager" is opened, the "Search Address" dialog is displayed. The BrailleSense prompts, "Last name:"

You can use the menus to conveniently browse and execute all of the available commands. To open the menu, press "Space-M" or "F2". Select a menu item using "Space-1" or "Space-4", and press "Enter". Or, you can press "Space-M" or "F2" followed by the shortcut for the corresponding menu item. Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab and Shift-Tab among controls in a dialog.

10.1.1 ADDING AN ADDRESS

This dialog allows you to enter new contacts. Select "Add Address" from the menu, or press "Enter-I" directly from in the "Address Manager".

The "Add address" dialog contains a list of 22 input fields, the "Memo" field, a "Confirm" button, and a "Cancel" button. In the input fields, enter the information corresponding to the 22 fields of a record. You can move from one field to the next using "Space-1" and "Space-4" or the up and down scroll keys. You do not have to fill in all the fields, simply leave blank those you do not need to use.

If using US Braille, you can enter telephone numbers using computer Braille (without a number sign) or regular grade 1 Braille (with or without a number sign). To enter a dash, press "Space" or "dots 3-6". To put an extension number after any phone number, enter the phone number as described above, and then type a comma (,), followed by the extension number, as in "000-000-0000,100." In this case, Press "dot 6" to enter the comma in computer Braille or "dot 2" in grade 1 Braille. When you enter a home page address or an e-mail address, you should also use computer Braille.

After filling in the desired information, Tab to the "Memo" field. You can type in multiple lines of text in the "Memo" field. If you do not wish to enter a memo, leave it blank.

When you have entered all the relevant information, Tab to the "Confirm" button and press "Enter". Or, just press "Enter" from any field. To cancel the contact entry, Tab to "Cancel" and press "Enter". After canceling or adding an entry, you are placed in a new blank record.

10.1.2 SETTING ADD ADDRESS FIELDS

There are 22 fields in a record, but you may not want to enter information in every one of the 22 fields. You can use "Set Address Fields" to select only the fields that you need.

To use this function, Select "Set Address Fields" from the menu, or, press "Enter-O" from the "Add Address" dialog. On the right side of each field name, "used" or "unused" is displayed. The default value is set to "used". To toggle the value, press "Space".

After setting all the fields you want to change, Tab to "Confirm" and press "Enter". To cancel changes, press "Space-E".

10.1.3 SEARCHING FOR AN ADDRESS

The "Search Address" dialog enables you to search for specific records in the "Address Manager". The "Search Address" dialog is opened automatically when the "Address Manager" is launched. To open "Search Address" from "Add Address", you can either select it from the menu, or press "Enter-S" from the "Add Address" dialog. "Last name" is displayed as this is the first field in the address data.

Type a last name to search for and press "Enter". Press "Enter" without entering text to search, and all records are shown. To search by a different field, navigate to it using "Space-4", type the search text in the edit box, and press "Enter".

You can move among found records using "Space-1" and "Space-4". Press F3 on the record you wish to view. The information appears in a multi-line read box which you can navigate using normal text navigation commands.

If a record contains an e-mail Address, you can press "Enter" to launch the e-mail program and send an e-mail to that person. If a field contains a web page, you can press "Enter" on it to open the page in the Web browser.

When you enter text to be searched, you do not have to type the entire content of a field. For example, to find a record about "Edward", type in "ed" in the "search name" field. Then, all the records whose name field contains "ed" are found.

10.1.4 SETTING SEARCHING ADDRESS FIELDS

This option enables you to select the fields that you want to search. Select "Set Address Search Fields" from the menu, or press "Enter-O" from the "Search Address" dialog.

All the address fields are displayed. Press "Space", to toggle the value of a field between "used" and "unused".

10.1.5 BACKING UP AND RESTORING THE ADDRESS LIST

If you perform a hard reset, or if your battery dies, all of the data in the "Address Manager" is lost. Thus, it is very important that you make a backup file every time you make a change to the address list. Select "Backup Address List" from the menu, or press "Enter-U" from anywhere in Address Manager.

If you already have a backup file, you are prompted, "Backup file already exists. Continue to backup? Yes". If you have made changes to your address list, press "Enter" to create a new backup file. To keep the previous backup file press

"Space-4", to move to "No", and press "Enter". When you create a backup file in your address list, you will have a new backup file in the database folder on the Flashdisk.

To restore the address list after a hard reset, select "Restore Address List" from the menu, or press "Enter-R" from anywhere in Address Manager.

If you already have an address list, you are prompted, "Address file already exists. Which do you prefer? Append". If you press "Enter", the backup address list is appended to the current address list. If you press "Space-4" once and press "Enter" on "Overwrite", the new backup file is restored, erasing the old backup file.

You can backup your address list automatically using the "Set Backup Options". Select "Set Backup Options" from the menu, or press "Enter-E".

The "Backup mode" has three radio buttons: "Always backup changes when exiting", "Manual backup", and "Prompt backup changes when exiting". Move through the radio buttons using "Space" and "Backspace". Press "Enter" to save your setting.

10.1.6 IMPORTING FROM CSV

"CSV" is the acronym for Comma Separated Value. A CSV file is used for the digital storage of data structured in a table form, with the information in the various columns separated using commas. The "Address Manager" can import a CSV contact file created using MS outlook or some cell phones. You can also export a CSV file of your contacts for use on other devices.

To import the data from a CSV file, select "Import from CSV" from the menu, or press "Backspace-I" from anywhere in the Address Manager. Type the name of the file you want to import in the edit box. To browse for a file, "Shift-Tab to the

path search list, and use normal file navigation commands to choose your file and press "Enter".

When the CSV file is imported, "Last name: unused 1/xx" is displayed. You can choose whether the data from the CSV file for each field should be imported. Some CSV files may contain data that is not equivalent to any of the fields in the "Address Manager", or you may simply not wish to import certain pieces of information.

This dialog contains all the fields in the "Address Manager", a check box for allowing fields in the first record, and "Confirm" and "Cancel" buttons. Tab and Shift-Tab among these items. For each "Address Manager" field, there is a list box containing "Unused" and all of the fields of the CSV file being imported. Use "Space-1" and "Space-4" to move among the fields. If there is no equivalent field in the CSV file for an "Address Manager" field, select "unused".

Often, the first set of fields in a CSV file represent field titles. For this reason, there is a check box for allowing fields in the first record to be imported or not. If the first set of fields does contain title data, make sure that the "allow fields in first record" item is not checked.

To complete the import, Tab to "Confirm" and press "Enter". You are prompted, "Address file already exists. Which do you prefer?" The default value is "Overwrite". The setting values are "Overwrite" and "Add". You can either replace the existing file with the data imported from the CSV file, or you may add it to the existing address list. To change the setting value, press "Space", and press "Enter" to save the data.

10.1.7 EXPORTING TO CSV

To export address data to CSV, select "Export to CSV" from the menu, or press "Backspace-X". Type a file name for the exported data and press "Enter". To

change the location where the file is stored, Shift-Tab to the file list and use normal file and folder navigation to select the location.

10.1.8 EDITING, SAVING, AND DELETING ADDRESSES

To edit information in a record, find the desired address by using the "Search Address" function. Select "Modify Address" from the menu, or press "Enter-M" to activate this command directly. You can now enter new information in any of the fields. When you have modified the information, press "Enter". The BrailleSense saves the changes, and you are returned to your previous position. You can press "Space-E" to cancel the modification.

To delete the record you are currently reading, select "Delete Address" from the menu, or press "Space-D" while viewing the record. You can also delete more than one record at a time by first selecting individual records with "Space", or by selecting all records with "Enter-A".

You can also save selected records to a file. Select "Save addresses As File" from the menu, or press "Enter-T" from the "Search Address" dialog. The "file save" dialog is displayed and you are asked to enter the file name and select a file format. After you type the file name, and select one of the available file formats, press "Enter" on "Confirm", and the selected records are saved.

10.1.9 SYNCING CONTACTS

Contacts entered or imported into the BrailleSense can be synchronized with your Google Account, and thus also, any other accounts that synchronize with your Google Account.

You can set your sync preferences in the Android System Settings, under the accounts item, then Google. All of your Google accounts are listed and identified whether Sync is enabled. Press Enter to open a dialog in which you can choose

what items you would like to have synced. If you choose to sync your contacts, they are automatically synchronized when you make changes. You can also manually "Sync Now" in the More options menu of the Accounts dialog.

Note: if you make too many changes, such as when importing a CSV, or synchronizing many new contacts, you may receive "Sync Errors". If this occurs, press Space-1-2-3-4-5-6 to open the Notification Shade, and press Enter to open the error. From there, you are given options as to how you would like to handle the errors.

10.2 Schedule Manager

The "Schedule Manager" is a program used to manage information about appointments, anniversaries, and events. It can also inform you of appointments at a specified time using alarms. You can start the "Schedule Manager" from the "Organizer" menu, by navigating to it with "Space-4" and pressing "Enter". You can launch the "Schedule Manager" from anywhere on the unit by pressing "F1-S".

The "Schedule Manager" has two main dialogs: "Add appointment" and "Search appointments". In "Add appointment", you can enter information for a new appointment. Using "Search appointments", you can look for information about a specific appointment and edit or delete the information.

When the "Schedule Manager" is opened, the "Search appointments" dialog box is displayed. You are prompted for a "Find date:" with the current date filled in as the default. To add a new appointment, select "Add appointment" from the menu, or press "Enter-I" from the "Search appointments" dialog.

- Note: Due to the nature of appointments, it is very important that dates and times are entered correctly to ensure that your schedule is accurate and you are alerted to your appointments at the proper times through the

"Check Today's SCHEDULE" option as well as the alarms that can be associated with appointments. Please be sure your time and date are set correctly. (See section 3.1)

10.2.1 ADDING AN APPOINTMENT

To add a new appointment, press "Enter-I" from the "Search appointments" dialog. The "Add appointment" dialog has a set of input fields, a "Confirm" button, and a "Cancel" button. Use "F3" and "Space-F3" to move through the controls.

When you start "Add appointment", you are prompted to choose the appointment type. Use Space-1 or Space-4 to choose between "Anniversary" or "Appointment".

The next field is the "Start date" edit box. To use the date displayed, Tab to the next field. To enter a new date, type month/day/year.

- Note: If you have not searched for an appointment, or have searched by subject, the default "Start date" is today's date. If you have searched appointments by date, it is the last date for which you searched.

Tab to the "Start Time" field. By default, this field shows the current time. Enter the time as hhmm or use navigation keys to select your appointment time. If you're using "12 hour" format, press "Space-X" to change between am and pm.

The next field is the end date. By default, the end date is the same as the "Start date". To use this date, Tab to the next field. To enter a new date, type month/day/year.

By default, the end time is the same time as you set in the "Start time" field. To change the time, type the time as hhmm.

The next field is the subject of the appointment. If you do not enter anything in this field, you cannot save the record. If you press "Enter" after typing in the subject, the record is saved, leaving the "Location" and "Memo" field empty, and the "Alarm time" field set to its default value ("Time of appointment"). Type your subject and Tab to the next field.

Type the location of your appointment. Press "Enter" to save the appointment, or Tab to the next field.

In the "Memo" field, you can enter detailed information about an appointment. This is a multi-line edit box, meaning you can type more than one line of text. You can use most of the editing commands that are used in the "Word processor". You can leave this field blank if you wish.

The "Alarm time" function allows you to set when you are reminded about an appointment. The alarm is set relative to the appointment time. For example, you may want the alarm to sound 30 minutes before your appointment, so that you are not late. The default value for this field is set to "Time of appointment".

The time can be set anywhere from one minute to two weeks before the appointment. Use the same navigation keys as when setting the time and date. If you do not wish to have an alarm reminder, press "Space-1" to "Alarm off". After setting the "Alarm time", press "Enter" to save the appointment.

10.2.2 SETTING A RECURRING APPOINTMENT

Set this option when the appointment you are creating recurs at regular intervals. If it is unchecked, the appointment is set to occur once. To set a recurring appointment, check this box with "Space", then Tab to the resulting combo box for setting the recurrence pattern. You can select from "Daily", "Weekly", "Monthly" and "Yearly".

If you select "Daily", Tab to the "Recurring interval of day" edit box. You can type a number between 1 and 999 to represent the number of days between recurrences. If you enter 1, the appointment recurs every day, and if you enter 2, it recurs every other day. Be sure the difference between the start and end date is shorter than the recurring interval value.

Tab to the "Set recurrence end date" check box. By default, it's unchecked, which indicates that the appointment recurs indefinitely. If you check this box, an edit combo box for entering the end date appears in the tab order.

If you set the "Recurrence" to weekly, Tab to the "Recurring interval" edit box, and type the number of weeks between appointments in numbers from 1 to 999. As with daily, the appointment duration should be shorter than the recurring interval.

Tab to a list box where you can select the day of the week on which the appointment recurs. To select or unselect the current item, press "Space". You can make more than one selection, thus the appointment recurs on more than one day during the week. If you don't select a day, it is set to the today's week day.

If you set the "Recurrence to monthly", Tab to the "Recurring type" combo box. You can set the recurring type as "Date" or "Day of the week". "Date" is used when you want to set an appointment recurring on the same day of a month, for example, on the 17th of every month, or every other month. On the other hand, "Day of the week" is used when you want to set an appointment recurring on the same week and day of the week of a month, for example, on Wednesday of the second week of every month. After selecting the "Recurring type", Tab to the "Recurring interval", and type a number of months between appointments in numbers from 1 to 999.

Different fields are shown in the Tab order depending on what you have selected in "Recurring type". If you set the "Recurring type" to "Date", an edit box

appears for typing the day of the month for the recurrence. By default the "Start date" is shown. Tab to "Set recurrence end date" and make your choices.

If you set the "Recurrence" to "Yearly", Tab to the "Recurring type" combo box. Select either "date" or "day of the week". If you set the "Recurring type" to "Date", you can set the month and the day of the month when the appointment recurs. If you set it to "Day of the week", you can set the month, the week, and the day of the week when the appointment recurs every year. After setting "Recurring type", Tab to the "Month for recurrence" combo box. Select the month of the year using "Space-1" or "Space-4". From here, different fields appear in the tab order depending on how you have set the "Recurring type".

If you have set the "Recurring type" to "Date", the focus moves to "Day of recurrence" edit box. You can type a number between 1 and 31. Then, Tab to the "Set recurrence end date" check box. If the "Recurring type" is set to "Day of the week", you have to set "recurring week" and "Day of week for recurrence".

10.2.3 SEARCH APPOINTMENTS

When you start "Schedule Manager", the "Search appointments" dialog is opened. You can also open this dialog from the "Add appointment" dialog, by pressing "Enter-S" or selecting "Search appointments" from the menu. There are two methods for searching an appointment: "searching by subject" and "searching by date".

When "searching by subject", you can find an appointment using keywords in the subject field. When "searching by date", you can find appointments according to a specified date. To toggle the search mode, Select "Toggle Search Mode" from the menu, or press "Enter-F". The last search mode you used is saved when you exit the Schedule Manager.

When you search by date, by default, the BrailleSense searches for appointments occurring today. To search for another date, type the date in the format "mm/dd/yyyy", or use the "move by date" hotkeys. Press "Enter", and the appointments occurring on the chosen day are shown. If there is no appointment on that day, the unit announces, "There is no appointment registered".

The first appointment is announced in the form of "date, time, and subject". If the appointment lasts for more than one day, "continuous schedule" is announced. If the appointment recurs, "recurring schedule" is announced. If you scheduled more than one appointment for that day, move among the appointments using "Space-1" or "Space-4". To move to the first appointment, press "Space-1-3", and to move to the last appointment on that day, press "Space-4-6".

To move to the appointments on another day, press "Space-2" or "Space-5". The previous/next appointment occurring within four months is displayed. To move to the first appointment stored in the Schedule Manager, press "Space-1-2-3", and to move to the last appointment stored in the Schedule Manager, press "Space-4-5-6". To obtain details on an appointment, press "Enter".

To search by subject, type in the keywords to search and press "Enter". All appointments with the keywords are displayed. If you press "Enter" without typing text, BrailleSense displays all appointments in the "Schedule Manager".

To move to the next or previous appointment, press "Space-4" or "Space-1" respectively. Pressing "Space-1-3" and "Space-4-6" move to the first and last items in the appointment list.

For recurring appointments and multiple day appointments, the BrailleSense only shows the appointments of the first day if you search the schedule by subject. Press "Enter" on an appointment to read the details. Press "Space-E" to return to the appointment list. To search other keywords, Tab to the "Search subject" edit box, and type new search text.

Some appointments you enter or receive may contain links to a Zoom meeting, Google Meet invitation, a Teams live event, or other item that requires launching of an installed Android ap. These are generally located in the Memo or location fields. You can open these links in an Android browser, as the BrailleSense browser does not automatically launch other programs.

To open a link located in the current paragraph in the default Android browser, press "Backspace-Enter-L". The browser will launch, which will then, in turn, activate the associated Android program, provided that program is installed on your BrailleSense. For example, if you want to launch a Zoom meeting, you need to install the Zoom application on your BrailleSense from the Play Store.

10.2.4 EDITING, SAVING, AND DELETING APPOINTMENTS

To modify an existing appointment, move to the appointment and select "Modify Appointment" from the menu, or press "Enter-M" from the search results list. If the appointment is a recurring schedule, a prompt, displaying "Recurrence appointment. Open type?" appears. Choose whether you want to modify the recurring appointment or the appointment for a specific day. If you select "This occurrence", your changes affect the appointment of the day, but the recurring appointments on other days are not changed. If you select "The series", your changes affect all the recurrences.

The "Modify appointment" dialog contains the same controls as in the "Add appointment" dialog. The only difference is that each field has the value entered previously. If you have made changes you wish to save, tab to "Confirm" and press "Enter".

To delete an appointment, move to the appointment in the search results list. Select "Delete Appointment" from the menu, or press "Space-D" from the search results list. If the selected appointment is a recurring appointment, you are

prompted, "Recurring appointment. Delete type?" is displayed. You can choose whether you want to delete the recurring appointment, or just the appointment for that day. If you select "This occurrence", only the current appointment is deleted. If you select "The series", all recurrences of the appointment are deleted.

To completely initialize the Schedule Manager, select "Delete All Appointments" from the menu, or press "Backspace-Enter-q" from anywhere in the Schedule Manager.

You can also save selected appointments as a document. Select "Save Appointments as File" from the menu, or press "Enter-T" from the search results list. The BrailleSense asks you to specify the beginning and end dates of the appointments to be saved. After setting the beginning and end dates, press "Enter", and you are prompted to enter the name and the format of the file to be saved. This dialog is the same as the "Save As" dialog in the "Word processor".

10.2.5 BACKING UP AND RESTORING YOUR APPOINTMENTS

If you perform a hard reset, or your battery dies, all of the data in the Schedule Manager is lost. Thus, it is very important to create a backup file each time you change your schedule. To backup your schedule, Select "Backup Appointments" from the menu, or press "Enter-U" to activate this function directly.

If you already have a backup file, you are prompted, "Backup file already exists. Backup anyway? Yes". If you have made changes to your schedule, press "Enter" to create a new backup file. To keep the previous backup file, press "Enter" on no. If you make a backup file again in the Schedule Manager, a new backup file is created with the extension DAT, and another file is created with the extension BAK containing your previous backup file.

To restore your schedule after a hard reset, Select "Restore Appointments" from the menu, or press "Enter-R" to activate this function directly.

If you have an existing schedule, you are prompted, "Schedule file already exists. Which do you prefer? Append" or, press "Space-4" to move to "Overwrite". If you choose to append, and the backup file contains appointments that also exist in your current file, you will have redundant appointments. If you choose to overwrite and your current schedule contains appointments the backup file does not, these appointments are lost.

You can back up the schedule automatically by activating the "Set Backup Options" function. Select "Set Backup Options" from the menu, or press "Enter-E".

The "backup Options" dialog has three controls, "Backup mode" radio button, a "Confirm" button, and a "Cancel" button. To move between the radio buttons, press "Space and Backspace".

The options are: "Always backup changes when exiting", "Manual backup", and "Prompt backup changes when exiting".

If you select the "Always backup changes when exiting", a new backup file is created automatically if you have modified or added to your schedule.

If you select the "Manual backup", a new backup file is not created, even if there are changes to your schedule. If you select this option, you can only create a new backup file by using the "Backup Schedule" function.

If you select "Prompt backup changes when exiting", you are prompted, "Backup changes? Yes" if you have modified or added to your schedule. Press "Enter" on "Yes", and a new backup file is created before exiting the Schedule Manager.

Choose your setting, and press "Enter".

10.2.6 MULTIPLE ACCOUNTS AND SYNCING

BrailleSense 6 allows you to use multiple calendars, and automatically makes associations with any Google account connected to BrailleSense. To view available calendars, choose the “Calendar Settings” option from the Schedule Manager menu.

By default, there is an account called “BrailleSense 6”. In addition, any Google account you have signed in to with BrailleSense is also available in this list.

The dialog contains three settings: “calendar to add appointment”, “Calendar to print appointments”, and “Calendar Type”. This allows you to set appointments in multiple calendars, and view, add and output appointments from your chosen calendars. This can be useful for creating separate calendars for work and personal use.

Use the “Calendar Type” setting to determine whether the Calendar view shows the Solar Calendar, lunar Calendar, or both. By default, this is set to show the Solar calendar only. If both are shown, use “F3” and “Space-F3” to toggle the dates between lunar and Solar on the current day.

In addition, you can synchronize these calendars with Google using the “Accounts” dialog of the Android Settings under All Apps. See section 10.1.9 for more information on syncing using the Android System Settings.

You can also sync directly with the cloud from the BrailleSense Schedule Manager, using the “Refresh” command in the menu, or by pressing “Backspace-R”.

Note: as with the Address Manager, if too many items are changed, added or deleted, you may experience sync errors. If you receive a Sync Error notification, press Space-1-2-3-4-5-6 to open the Notification Shade, and press Enter on the Sync Error to see your options.

10.2.7 SETTING ALARM OPTIONS

You can open the "Set Alarm Options" dialog by pressing "Enter-O" or by pressing "Enter" on "Set Alarm Options" in the menu. Alarm option fields include "type of echo", "Alarm duration", "Alarm repeat interval" and "Repeat". You can move between these fields using "Space-1" or "Space-4".

The "type of echo" option lets you set what type of alarm you would like to use to alert you to your appointment: Choose from several audio alarm types, including a media file, the FM Radio, or 1 of 3 alarm melodies. You can also choose a vibration, or both (All). Press "Space" to cycle among the options.

You can change the duration of the alarm sound. You can set this option to a value that is between 5 and 600 seconds. The default value is set to 30 seconds.

When the alarm goes off, if you do not stop it with "Backspace-Enter", the BrailleSense keeps giving you a notice about the appointment after a specified time interval. The "Alarm repeat interval" option allows you to change this interval. You can set the value anywhere from 1 to 60 minutes, the default value is set to 5 minutes.

The "Repeat" field allows you to decide how many times the alarm will sound. You can set the value from 1 time to 10 times; the default value is set to 3. To change the value, type a number between 1 and 10.

Press "Enter" on the "Confirm" button to save your settings. Cancel saving the alarm options by pressing "Enter" on "Cancel" or pressing "Space-e".

11. Web Tools

The "Web Tools" menu consists of programs used to assist in the searching and downloading of web content. Currently, this menu contains only the Web Browser.

11.1 Web browser

You can use the "Web Browser" to read pages on the web, as well as to read html documents you have saved to disk. You can also download files and stream audio content.

To launch the "Web Browser, from the "Web Tools" menu, navigate to "Web Browser" and press "Enter". Or, press B. You can launch the "Web Browser" from anywhere on the unit by pressing "F1-B". The BrailleSense opens the "Web Browser" and connects to the "home page".

You can also launch the Web Browser" by using the global hot key for opening a web page: Press "Space-W" from anywhere on the unit. The BrailleSense, opens up a computer edit box into which you can type in the URL you want to go to and press "Enter". The Browser is launched, and takes you to the web page you requested.

You can access the Web Browser's functions using hot keys, or via its menus. To access these menus, press "Space-M" or "F2". Select a menu item using "Space-1" or "Space-4", and press "Enter". Or, you can press "Space-M or "F2" followed by the shortcut for the corresponding menu item. Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab among controls in a dialog. For the purposes of these instructions, we will generally refer to using "F3" and "Space-F3" or simply "Tab". Press "Enter" to execute an item.

11.1.1 THE FILE MENU

- 1) Open URL: (U) "Enter-U". Type the web address of the page you wish to go to and press "Enter".
- 2) Open: (O) "Enter-O". Open an HTML file or URL stored on your BrailleSense.
- 3) Save As: (S) "Space-S". Save the current webpage to your BrailleSense.
- 4) Copy URL: (D) "Backspace-D". Copy the URL of the current page to the clipboard.
- 5) Copy Link: (C) "Backspace-L". Copy the address of the focused link to the clipboard.
- 6) Page Information: (I) "Enter-I". Opens a dialog of general information about the current webpage.
- 7) File Download Dialog: (L) "Backspace-3-4". Opens a dialog to show current download progress as well as file download history.
- 8) Exit: (Z) "Space-Z". Closes the Web Browser.

11.1.2 THE EDIT MENU

- 1) Start Selection: (B) "Enter-B". Begin selecting a block of text. The end of the selection is determined by your cursor position.
- 2) Copy: (C) "Enter-C". Copy selected text to the clipboard.
- 3) Add to clipboard: (P) "Enter-P". Append selected text to the clipboard.
- 4) Select All: (A) "Enter-A". Select all the text on the current page.

11.1.3 THE GO TO MENU

- 1) Go To Home page: (S) "Backspace-H". Return to your home page.
- 2) Go To Previous Page: (P) "Backspace-P". Go to the page you were viewing before the current one.
- 3) Go To Next Page: (N) "Backspace-N". Move forward one page in your viewing history.
- 4) Go To Previous Heading: (U) "Backspace-B". Move back one heading.
- 5) Go To Next Heading: (D) "Backspace-F". Move forward one heading.
- 6) Go to Previous Text: (B) "Dots-2-4-6". Go to the block of text before your current position.
- 7) Go To Next Text: (P) "Dots-1-2-4-5-6". Go to the next block of text below your current position.
- 8) Refresh: (R) "Enter-R". Reload the current page.
- 9) History: (H) "Enter-H". Open a list of pages you have recently viewed. Press "Space-D to delete the current item, or use the Clear All button to clear the entire history.
- 10) Find: (F) "Space-F". Find text on the current page.
- 11) Find Again: (E) "Enter-F". Search for the next instance of the text string you entered in the "Find" dialog.
- 12) Links List: (I) "Backspace-I". Displays all the links on a page in a list view.
- 13) RSS Feed list: (J) "Enter-J". Displays any RSS feeds on the current page.

11.1.4 THE READ MENU

- 1) Read from Beginning to Cursor: (T) "Backspace-G". Reads the text from the top of the page to your current position.

2) Read from Cursor to End: (G) "Enter-G". Reads the text from your current position to the bottom of the page.

3) AutoScroll: (A) "Up and Down Scroll". STARTS and stops the auto scroll function.

11.1.5 THE FAVORITES MENU

1) Set Current as Home Page: (S) "Enter-S". Set the current page as your home page.

2) Add to Favorites: (A) "Backspace-A". SAVE the current page as a favorite.

3) Favorites List: (L) "Enter-L". Open the list of favorites you have saved previously.

11.1.6 OPTIONS

Use the "Options" dialog to manage Web browser preferences. Navigate the settings with "F3" and "Space-F3".

1) User Agent: use "Space" to choose between Mobile and desktop.

2) Show visited links: Use "Space" to choose On or Off.

3) Show images: Use "Space" to choose On or Off.

4) Show cell information: (in tables) Use "Space" to choose On or Off.

5) Default Download Folder: Press Enter to open folder list and choose desired download folder.

6) Clear Cookies: press "Enter" to clear all cookies.

11.1.7 NAVIGATING PAGE ELEMENTS

On a web page, there are links, controls, and other various elements, as well as text. To read web pages properly, the "Web Browser" should classify each element, and notify the user with its information.

When reading a web page by moving with "Space-4" or "Space-1", one element is displayed on one line. The links and controls are displayed with their respective symbols so the user can distinguish each element. To move among controls without viewing the text between them, use "F3" and "Space-F3".

A control is a link, an edit box, a radio button, or a combo box. Each control has different purposes. Therefore, the user has to know exactly what type of control is being used.

When you find controls in web pages, the BrailleSense displays control symbols ahead of the contents of the control. The following is a list of control symbols.

Control name

Symbol

Link

LN

Visited link

LVN

Anchor

ANC

Edit box

EB

Multi edit box

MEB

List Box

LB

Combo box

CB

Radio button select

SRB

Radio button unselect

URB

Check box check

SCHB

Check box uncheck

UCHB

Button

BT

To change a selection in the combo box, press "Backspace-Space-4" or "Backspace-Space-1". You can move to the next item by pressing "Space-4". The page is renewed automatically when you change a selection in a combo box.

To select or unselect a checkbox or radio button, press "Space".

You can move to the previous form field: with Space-1-2-3-4-6, and to the next form field: with Space-1-3-4-5-6.

Move to the previous frame with Space-3-4-6, and to the next frame with Space-1-4-6.

11.1.8 TABLE NAVIGATION

At the beginning position of the table, the BrailleSense shows the table number, and the number of rows and columns on the table.

For example, there are two large tables (we call these "upper level tables") on one web page. Each table has 3 sub tables (we call these "lower level tables") with 3 rows and 2 columns on each. In this case, when the BrailleSense shows the first sub table on the second large table, it displays "table 2-1 3 rows, 2 columns". At the end of the sub table, the BrailleSense displays "table 2-1".

If the table consists of only 1 row and 1 column, or empty space that has only border or background, such tables are not considered tables in the BrailleSense.

The BrailleSense provides a way to move table by table, and cell by cell, in order for the user to comprehend the structure of the table. To move cell by cell, the cell position and contents of the cell are displayed. However, the position information is only announced with speech.

For example, if you have move to a cell that is positioned on the second row and third column, and the content of the cell is "news", the BrailleSense announces, "row 2 cell 3 news". In this case, "row 2 cell 3" is announced with speech only, while "news" is displayed in Braille.

If a table is included in the cell, the BrailleSense announces "table cell". If there is not a table in the cell, you hear "empty cell".

In the "table cell", To move to any cell within the lower level table, press "Space-4" to move to the beginning line of the lower level table.

Moving by cell is only valid in the table. If this command is executed outside of the table, a warning sound is played.

1. Move to the previous table: Space-3-4-5-6
2. Move to the next table: Space-1-4-5-6

3. Move to the previous cell: Space-2-3
4. Move to next cell: Space-5-6
5. Move to upper cell: Space-3-5
6. Move to lower cell: Space-2-6
7. Read current cell: Space-2-3-5-6
8. Move to the previous cell in the upper level table: Backspace-2-3
9. Move to the next cell of the upper level table: Backspace-5-6
10. Move to the upper cell of the upper level table: Backspace-3-5
11. Move to the lower cell of the upper level table: Backspace-2-6
12. Check the current position: Space-1-5-6

11.1.9 PLAYBACK OF STREAMING AUDIO FILES

While navigating a web page, if you come across a link for a streaming audio file, you can simply press "Enter" on it. The Media player program is launched with lists of address inside of the streaming file, and the streaming audio file on the first address is played. The supported streaming file formats are "m3u", "pls", "asf", "asx".

For other formats, if you press "Enter" the "Web Browser" program plays the streaming audio file directly. When you play the streaming audio file in the "Web Browser", the "Add To Favorites" dialog appears. You can add the streaming address to the "Favorites list" and the streaming file is saved with "ASF-WAS" at the beginning of the file name. "ASF" denotes that the link is a streaming audio file.

While you play a streaming audio file in the "Web Browser", if you exit the "Web Browser" program or you lose the internet connection, the streaming audio file stops playing.

You can use the following playback commands in the "Web Browser" while playing a streaming audio file.

1. Play: "play" button on the front panel
2. Stop: "stop" button on the front panel
3. Turn up the volume: Space-up scroll button
4. Turn down the volume: Space-down scroll button

Playback of a streaming audio file does not stop if you switch to another program, unless you press the "stop" button on the front panel after returning to the "Web Browser".

- Note: Real Audio files are not supported in the "Web Browser".

11.2 Google Search

The Google Search program allows you to easily perform a search using Google's Internet search service. To Open Google Search, from the "Web Tools" menu, move to "Google Search" and press "Enter", or press G". You can launch Google Search from anywhere on the unit by pressing "F1-U".

11.2.1 THE SEARCH DIALOG

The "Search" dialog contains 3 items: a "Search Term" edit box, a "Search" button, and a "Close" button. To move through the items in the dialog, press "Tab (F3) or "Shift-Tab (Space-F3).

When the Google Search application is launched, you are placed in the "Search Term" edit box. Type the term you wish to search for, and press "Enter" to launch the search; or Tab to the "Search" button and press "Enter".

To exit the program, Tab to the "Close" button and press "Enter", or press "Space-Z".

11.2.2 THE RESULTS DIALOG

After a search is performed, you are placed in the "Results" dialog. The "Results" dialog contains 5 items: the "Search Term" edit box, the "Title" list, the "Contents" text box, a "Search" button, and a "close" button. When the Braille Sense has finished searching the term you entered, you are placed in the "Title" list.

The "Title" is the name of the web result that matches your search term. The "Title" list can contain up to 64 items. To go to the current search result on the web, simply press "Enter" and the webpage for the current title is opened in your chosen Browser. When you have finished viewing the webpage, press "Space-Z" to close the Browser, and return to your results list in the Google Search program.

The "Contents" text box contains information about the current title, just as you would receive if you were performing a search on the Google website. This information usually includes a quick preview of the content on the web page, and the URL of the web result.

Use the "Search term" edit box, and "Search" button to launch a new search as described in the previous section.

11.2.3 CHOOSING YOUR BROWSER

When the Google search program is opened, press F2 to open the menu. It contains only "Settings", and "Exit".

Press Enter on "Settings" to choose which browser to use when opening Google results. By default, Google search uses the native BrailleSense Web browser. However, you may also choose to use the "Default Browser", which will open results in the default Android browser as set in the Android System Settings.

Use "Space-1" and "Space-4" to move between the browser options, and press "Enter" to save your choice.

12. EXTRAS

12.1 Excel Viewer

The "Excel Viewer" allows you to read and navigate XLS and XLSX files by cell, column, row, group, worksheet, etc. To open the "Excel Viewer", from the Main menu, press X to open the "Extras" menu, then press X again to open the "Excel Viewer". You can launch the "Excel Viewer" from anywhere on the unit by pressing "F1-X".

When the "Excel Viewer" is launched, you are placed in an "Open File" dialog. Use normal "File List" navigation commands to locate the XLS or XLSX file you wish to open, and press "Enter". When the file is loaded, you are placed at the top of the first worksheet. Cells are displayed with the coordinates followed by the text of the cell. For example, the first cell in a column containing the date might appear as "a1 Date".

12.1.1 GENERAL NAVIGATION

Use the following commands to navigate an Excel file in the "Excel Viewer".

Move to next row: Space-Dot-4

Move to previous row: Space-Dot-1

Move to next column: Space-Dot-6

Move to previous column: Space-Dot-3

Move to beginning of row: Space-Dots-1-3

Move to end of row: Space-Dots-4-6

Move to top of column: Backspace-Dot-2

Move to bottom of column: Backspace-Dot-5

Move to top of worksheet: Space1-2-3

Move to bottom of worksheet: Space-4-5-6

Move to next worksheet: Space-3-4-5

Move to previous worksheet: Space-1-2-6

Move to next group: Backspace-Dot-5-6

Move to previous group: Backspace-Dot-2-3

Move to next group in row: Space-Dot-5

Move to previous group in row: Space-Dot-2

Move to next group in column: Space-Dot-5-6

Move to previous group in column: Space-Dot-2-3

You can access functions of the Excel Viewer using hot keys, or via its menus.

To access these menus, press "Space-M" or "F2". Select a menu item using

"Space-1" or "Space-4", and press "Enter". Or, you can press "Space-M" or "F2" followed by the shortcut for the corresponding menu item.

The Excel Viewer also utilizes lists and dialogs. Use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab and shift-tab among controls in a dialog. For the purposes of these instructions, we will generally refer to this as "Tabbing".

Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Press "Enter" to execute an item.

12.1.2 THE FILE MENU

- 1) Open: (O) "Enter-O". Open Excel Viewer compatible files.
- 2) Open Hyperlink: (H) "Enter-H". Open a link contained in an Excel file in the Web Browser.
- 3) Set User Table: (T) "Enter-T". Set the parameters for columns and rows in order to get accurate information for headers and rows of sub tables.
- 4) Cell Properties: (I) "Enter-I". View format information for the cell.
- 5) Quick Setup: (Q) "Enter-X". Operates similar to the "Set User Table" function, except instead of defining the boundaries manually, the beginning cell is automatically set as the cell at your current cursor position, and the end cell is automatically set as the last cell containing data.
- 6) Save As: (S) "Space-S". Lets you save an Excel file as a CSV or TXT separated by tabs or commas.
- 7) Exit: (Z) "Space-Z". Close the Excel Viewer.

12.1.3 THE EDIT MENU

- 1) Start Selection: (B) "Enter-B". Begin selection of a text block. The end of the selection is determined by your current cursor position.
- 2) Select All: (A) "Enter-A". Select all the data on the current sheet.
- 3) Copy: (C) "Enter-C". Copy the selected text to the clipboard.

12.1.4 THE GO TO MENU

- 1) Find: (F) "Space-F". Search for text on the current sheet.
- 2) Jump to Cell: (G) "Space-J". Move to a cell using its coordinates.
- 3) Next Sheet: (N) "Space-Dots-3-4-5". Move to the next sheet in the workbook.
- 4) Previous Sheet: (P) "Space-Dots-1-2-6". Move to the previous sheet in the workbook.
- 5) Sheet List: (L) "Enter-L". Bring up a list of all the sheets in the workbook.
- 6) Set Mark: (M) "Backspace-M". Bookmark your current position.
- 7) Jump to Mark: (J) "Backspace-J". Move to a previously marked position.
- 8) Delete Mark: (D) "Backspace-D". Delete a previously set mark.
- 9) Move to Previous Group: (R) "Space-dots-2-3". Move to previous data filled area.
- 10) Move to Next Group: (E) "Space-dots-5-6". Move to next data filled area.

12.1.5 THE READ MENU

- 1) Sheet Title: (T) "Space-Dots-1-5-6". Read the name of the current sheet.
- 2) Read from Beginning of Row to cursor: (P) "Backspace-Dot-3". Read from the left most cell to the current position.

- 3) Read from cursor to end of Row: (N) "Backspace-Dot-6". Read from your current position to the right-most cell.
- 4) Read from Top of Column to Cursor: (U) "Backspace-Dot-1". Read from the top of the column to the current cursor position.
- 5) Read from Cursor to Bottom of Column: (D) "Backspace-dot-4". Read from your current position to the bottom of the column.
- 6) Read Current Row: (C) "Space-dots-1-4". Read all cells in the current row from left to right.
- 7) Read Current Column: (R) "Space-Y". Read all cells in the current column from top to bottom.
- 8) Read Header Cell of Row: (Y) "Backspace-Y". Read the title of the current row.
- 9) Read Header cell of Column: (X) "Backspace-C". Read the title cell of the current column.
- 10) Read File Name: (F) "Space-dots-3-4". Read the name of the workbook.

12.1.6 THE VIEW MENU

- 1) Set Time and Date: (D) "Enter-D". Determine how time and date are displayed.
- 2) Show Hide Hidden sheets: (S) "Enter-S". Toggle whether hidden sheets are viewable.
- 3) Show Hide Hidden Rows: (R) "Enter-R". Toggle whether hidden rows are viewable.
- 4) Show Hide hidden Columns: (Y) "Enter-N". Toggle whether hidden columns are viewable.

5) Cell Activation: (A) "Enter". Opens the cell as a static box so you can review or select text.

12.2 Online DAISY

The Online DAISY program allows patrons of the Canadian National Institute for the Blind and Vision Australia libraries to access and download DAISY books available through their online bookshelf.

To execute the Online DAISY program, Open the "Extras" menu, navigate to "Online DAISY" and press "Enter".

When you execute the Online DAISY program for the first time, you are asked for your login information. Use "Space-1" and "Space-4" to navigate to the server you wish to log in to. Once done, Tab to the ID field and enter it in computer Braille. If accessing the CNIB library, type "CNIB_" followed by your user number in the ID field. For Vision Australia, type just your user ID number. Tab to the "Password" field and type it in computer Braille. Press "Space" to toggle the check boxes for "Remember Me", and "Save password". When you have chosen your login settings, Tab to "Login" and press "Enter".

You are presented with a combo box with 3 items: "Expired", "New", and "Issued". Press "Tab (F3 or Space-4-5)" to access the list in your desired category. You can navigate the book list using "Space-1" and "Space-4".

To download a book, navigate to it and press "Enter". The unit announces "Download start" and shows progress indicators to let you know the progress of the download process. When the book is finished downloading, you may play it using the DAISY Player.

Books downloaded using the Online DAISY program are available in the "Online DAISY" folder from the "Open DAISY" dialog of the DAISY Player. For reasons

of copy protection, these books are only visible from the DAISY Player and cannot be accessed from the File Manager.

Vision Australia users can also download BRF books. When a BRF book is downloaded, you have the option of opening it in the Word Processor, from which you can copy the text to another document, or save it to a different location where it can be more easily accessed.

When you wish to return a book, navigate to it in the list, and press "Enter-R" This returns the book to the virtual library so that others can be issued to you.

12.3 Sense Dictionary

The Sense Dictionary is an add-on piece of software for the Sense notetakers available for purchase from HIMS. The basic package includes an English dictionary and Thesaurus, and one foreign language dictionary. Available dictionaries are as follows:

US English Dictionary: Webster's New World College Dictionary

US English Thesaurus: Collins Thesaurus A-Z 3rd

UK English Dictionary: Collins Comprehensive English Dictionary

Spanish/English Dictionary: Collins Spanish Dictionary – Complete and Unabridged

Italian/English Dictionary: Collins Italian Dictionary

French/English Dictionary: Collins English French Electronic Dictionary

- Note: If you have not purchased the dictionary package containing the dictionary data and license key, pressing "Enter" on the Sense Dictionary option in the "Extras" menu results in the message, "Authentication

failed". For pricing and purchasing information, please contact HIMS or your BrailleSense dealer.

13. Utilities

The "Utilities" menu provides several special functions and settings for the BrailleSense. Move among the items in the "Utilities" menu using "Space-1" and "Space-4" and press "Enter" to open an item.

You can access functions for each program using hot keys, or via that program's menus or dialogs. If a program has menus, press "Space-M" or "F2" to open the menu. Select a menu item using "Space-1" or "Space-4", and press "Enter". Or, you can press "Space-M" or "F2" followed by the shortcut for the corresponding menu item.

In dialogs, use "Space-4-5" and "Space-1-2" or "F3" and "Space-F3" to tab and shift-tab among controls. For the purposes of these instructions, we will generally refer to this as "Tabbing".

Use "Space-1" and "Space-4" or the Up and Down scroll keys to navigate menus, settings or file lists. Press "Enter" to execute an item.

13.1 Calculator

Using the calculator on the BrailleSense, you can do simple math equations as well as complex scientific calculations. The results and expressions are displayed in Braille, and spoken by the BrailleSense.

To use the calculator, press C from within the "Utilities" menu. You can launch the "Calculator" from anywhere on the unit by pressing "F3-C". When the calculator is opened, 0 is displayed in computer Braille.

You may type using computer Braille, UEB, or Nemeth Braille when inserting expressions. Press "Backspace-G" to cycle the Braille codes. Clear the calculator with "Backspace-C". If you insert numbers or a function into a result, the result disappears; and if you enter operators, you move to the next step. In addition, you can keep your result in the calculator's memory, and recall it later. You can activate the functions and operators via the calculator menu or you can use the hot keys associated with the various functions.

13.1.1 GENERAL FUNCTIONS

To access general mathematical operations, open the menu by pressing "Space-m" or "F2" and press "Enter" on "General Functions". From the calculation line, access the "General Functions" by pressing "Enter-G".

"General functions" is a dialog box that contains the "general function list", a "Confirm" button, and a "Cancel" button. You can navigate the operators using "Space-1" or "Space-4". Or, you can press the first letter of the operator you want to access.

Select your desired operation and press "Enter" to perform it. You can also use hot keys to quickly execute operations. Press "Space-H" for a list of hot keys in the Calculator.

To enter negative numbers type minus (dots 3-6) and the number in parentheses. For the percent operation, type the numbers first followed by the percent sign. Immediately after you type the percent sign (%), the number you have typed in is converted into a percent value.

13.1.2 FRACTIONAL ARITHMETIC

The BrailleSense can perform fractional operations such as simplifying and multiplying fractions as well as converting fractions to decimals.

You can also convert fractions to decimals, decimals to fractions, improper fractions to mixed numbers, or mixed numbers to improper fractions. When your result is fractional, you can bring up the "fraction options" by pressing "Space-3-4". Press "Space-1" and "Space-4" to move among the available options. If the fraction is a simple fraction, your options are ""fraction to decimal" and "return original fraction". If the fraction is an improper fraction, you also have an option to return "improper fraction to mixed number". If your fraction is a mixed number, the "mixed number to improper fraction" appears. Finally, if your result is a decimal and you press "Space-3-4", the decimal is converted to a fraction.

13.1.3 MEMORY FUNCTIONS

The "Memory Functions" act on stored answers and calculations in memory. To access these functions, press "F2" to open the menu, and press "Enter" on "Memory Functions".

"Recall Items" is a dialog, containing "Variable list", "Delete", "Display detail", "Confirm", and "Cancel". You can access the dialog by pressing "Backspace-R" in the calculation line or by selecting it from the "Memory Functions" menu...

The "Variable list" displays "item name: value". Insert the value into your calculation by pressing "Enter" on the item name. You can also copy items in the "Variable list". Press "Space" to select an item in the "Variable list". Then, press "Enter-I" to copy the selected items to the clipboard.

The "Display detail" shows the item name and its result with the default set to "No". To review the detailed information, press "Space" to switch to "Yes". Then, it displays "item name: the expression = its result value" in Braille.

"Delete" is used to remove a variable item from the list.

You can delete all of the saved items stored in the "Recall Items". Select "Delete All Saved Items" from the "Memory Functions" menu, or press "Enter-D" from the calculation line.

"Save Items" is a dialog that allows you to define variables according to the current expression or result in the calculator and save them for future use. The dialog contains an edit box for typing in the variable name, a "Confirm" button and a "Cancel" button. SELECT "Save Items" from the "Memory Functions" menu, or press "Backspace-S" from the calculation line.

When the dialog is opened, the "item name" edit box is focused. Type a name for the variable, and press "Enter" or move to "confirm", and press "Enter". The current expression or result is then saved under the variable name you just specified.

If you type names under variables which have already been created, you are prompted, "Replace variable xx with current value? Yes". If you type function names like "sin", "cos", or "pi" as a variable name, you are warned, "Invalid item name", as you cannot use these functions in variable names.

You can also define variables directly from the calculation line by typing "variable=expression" and press "Enter" to save the variable.

If you select the Statistics function, a dialog opens, which consists of a list box containing all the stored variables and their values, a list box displaying the results of several functions, and a "Close" button. Press "Backspace-T" to activate this function from the calculation line,

Press "Space" to select each variable in the list that you want included in the data set for which the statistics are calculated. TAB to the "result" list to view the sum, average, variance, and standard deviation of the selected variables. You can also select items from the result list and copy them to the clipboard.

13.1.4 SINE FUNCTIONS

Select "Sine Functions" in the Calculator's menu, or press "Enter-S" in the calculation line. The "sine function" list consists of "Sine", "Arc sine", and "Hyperbolic sine". If you press "Enter" on one of the three functions you can insert it into your calculation.

13.1.5 COSINE FUNCTIONS

Select "Cosine Functions" from the Calculator's menu, or press "Enter-I" from the calculation line. The "cosine function" list consists of "Cosine", "Arc cosine", and "Hyperbolic cosine". If you press "Enter" on one of the three functions, the calculation is performed. Or, you can activate one of the three cosine functions by pressing the first letter of each from the list.

13.1.6 TANGENT FUNCTIONS

You can open the "Tangent Functions" by pressing "Enter-T" from the calculation line, or by selecting "Tangent Functions" from the Calculator's menu. The "tangent functions" list consists of "Tangent", "Arc tangent", and "Hyperbolic tangent". If you press "Enter" on one of the three functions, the calculation is performed.

13.1.7 LOGARITHM FUNCTIONS

You can open the "Logarithm Functions" by pressing "Enter-L" from the calculation line, or by selecting "Logarithmic Functions" from the Calculator's menu.

The "Logarithm Functions" is a dialog, which contains a "function list", a "Confirm" button, and a "Cancel" button. The function list consists of "Common logarithm" and "Natural logarithm". If you press "Enter" on one of the logarithm functions, the calculation is inserted into your calculation line.

13.1.8 CONVERT UNIT

The "Convert Unit" dialog allows you to convert measurements of mass, area, volume, distance, and temperature from metric measure to imperial measure as well as other standards of measurement and units of measurement within the same system. It also converts several units of measurement related to programming and computers.

To open the "Convert Unit" dialog, Select "Convert Unit" from the Calculator's menu, or press "Enter-U" from anywhere in the Calculator.

The "Convert Unit" dialog contains the following items: "Unit Category" combo box, "Current Unit" combo box, "Change Unit" combo box, "Input Number" edit box, "Convert" button, "Cancel" button and the "Result" static box. You can move among the controls in the dialog using "Tab (F3 or Space-4-5)" or "Shift-Tab (Space-F3 or Space-1-2)". Use "Space-1" or "Space-4" to move among the items in the combo boxes.

The available unit categories are Distance, Area, Mass, Volume, Temperature, Data, numbers and Text. When you have chosen your measurement category, the "Current Unit" and "Change Unit" combo boxes will contain measurement units according to your chosen category. The available distance units are: Millimeters, Centimeters, Meters, Kilometers, Inches, Feet, Yards, and Miles. The available units of area are: Square Meters, Are, Square feet, Square Yards, Acres, and Hectares. The available units of mass are: Grams, Kilograms, Tones, Grains, Ounces, and Pounds. The available units of volume are: Deciliter,

Milliliter, Liter, Cubic Centimeter, Cubic Meter, Cubic Inch, Cubic Foot, Cubic Yard, Gallon, Fluid oz, Cups and Quarts. The available units of temperature are: Celsius, Fahrenheit and Kelvin.

The available data units are bit, byte, kilobyte, megabyte and gigabyte. The available numeric units are Hex, Dec and binary. Finally, the text conversion allows you to discover the ASCII equivalent to any text character.

Choose your unit to convert from in the first combo box. Choose your unit to convert to in the "Change" combo box. Type the number of the measurement you want to convert in the "Input Number" edit box. Finally, to convert the measurement, Tab to the "Convert" button and press "Enter". If you wish to cancel the conversion, Tab to the "Cancel" button and press "Enter". When you activate the "Convert" button, the conversion result is displayed in the "Result" static box.

13.1.9 COPY TO CLIPBOARD

The "Copy to Clipboard" function is used to copy your expression or result to the clipboard for pasting your expression or result in a document. You can also access up to 20 lines of calculation history and copy them as well. Press "Enter" on the "Copy to Clipboard" option in the menu, or press "Enter-C" in the calculation line to copy the current expression. You can now paste the calculation to a document in the Word Processor or to another application on the unit.

To access the cumulative calculation history, press "Enter-H". You can then use normal selection commands to select the lines of the history you wish to copy. Press "Space-D" to delete an item from the cumulative history.

13.1.10 RETURN FORMULA

There may be times, after viewing a result, that you wish to be reminded what expression you used to obtain that result. The "return formula" function allows you to return the expression you just entered

When your result is displayed, press "Enter-R", or select "Return Formula" from the menu.

The expression you entered to obtain your current result is displayed.

13.1.11 OPTION SETTINGS

You can activate the "Option Settings" by pressing "Enter" on the menu option, or from your calculation line, press "Enter-O".

The "Option Settings" dialog contains combo boxes for changing various settings, an "OK" button, and a "Cancel" button. The "available settings are "Angle Unit", "Calculator Braille Code", "decimal places to display" and "Display Using Scientific notation". You can navigate among the controls using "F3" or "Space-F3". Use "Space" to change the value of each setting.

When setting the angle unit, if you select "Radian", you can get the PI value while using the trigonometric function. If you select "Degree" and activate PI, you will get a wrong answer as this is not a valid calculation.

You can set the Calculator Braille code to computer Braille, Nemeth or UEB math Braille as described previously. You can use "Space-G" to change this setting from anywhere in the calculator.

13.2 BrailleSense Math

This program allows you to create equations, or lists of equations, and perform advanced graphing functions. Be aware, because this program needs to do specific math related functions, it does break some of the usual interface rules.

When you first open it, you see an empty list, with the index 1/1. We use a list view here for the purposes of being able to select math problems in the same way you can select files or folders in a list, so that you can perform a single action on all selected items. One difference though, because you will use Space when writing math expressions, you will use Space-Enter to select the current item.

To enter a math problem, simply begin typing. When you press Enter after completing it, you will see an empty list item labeled 2/2. This will continue happening as you enter subsequent math expressions and press enter.

As you enter math expressions, graphs are automatically created when you press Enter, navigate away from the edit box, etc.

As you add expressions and graphs, they are added to the same image, the same x and y axis, with each subsequent graph appearing in a different color. If you graph more than 6 expressions, colors will be repeated, as there are only 6 different colors.

From the Word Processor, you can press Backspace-Enter-G to open the graphing function, this automatically opens BrailleSense Math, and inserts the current equation in to the list.

As with most other programs on BrailleSense, press F2 or Space-M to open the menus. Menu functions are outlined below:

13.2.1 THE FILE MENU

- 1) Export: (E) “Enter-E”. Saves the current BrailleSense math file (.mth). You are not able to export this to other formats, just save it for later use, or for sharing with another BrailleSense user.
- 2) Import: (I) “Enter-I”. Allows you to open a previously saved BrailleSense Math (.mth) file.
- 3) Save to History: (S) “Space-S”. Saves the current selection to your history.
- 4) History: (H) “Enter-H”. Opens a list of equations you saved to your history. You can save an expression to your history by pressing Space-S.
- 5) Save Graph As Braille: (F) “Enter-F”. Saves the tactile rendering of your currently selected graph as a Braille file.
- 6) Exit: (Z) “Space-Z”. Performs the obvious function.

13.2.2 THE EDIT MENU

- 1) Copy: (C) “Enter-C”. Copies the current or selected math expressions to the clipboard. This function does not copy graphical images. Use the copy Graph function to do this.
- 2) Cut: (X) “Enter-X”. Cuts the current selection to the clipboard.
- 3) Paste: (V) “Enter-V”. Pastes clipboard content can currently paste only from other BrailleSense Math areas, such as the history or a math file.
- 4) Delete: (D) “Space-D”. Deletes the current or selected expressions.
- 5) Select All: (A) “Enter-A”. Selects all expressions in the current list.
- 6) Start Selection: (B) “Enter-B”. The same Start selection command that exists in File Manager and other list areas, selects continuously to your current cursor position.

- 7) Copy Graph: (G) “Enter-G”. Copies the graph image and associated equations to the clipboard.
- 8) Symbol picker: (S) “Backspace-S”. Allows you to search for and select math symbols to enter into your math expression.
- 9) Set Math Code: (M) “Backspace-G”. Choose between UEB and Nemeth.

13.2.3 THE GRAPH MENU

- 1) Axis: (X) “Backspace-X”. Allows you to set the limits of the axes, label them, and set the value each point represents.
- 2) Graph label: (L) “Backspace-L”. Lets you label the graph as a whole.
- 3) Graph Style: © “Backspace-C”. Lets you set color and shape of the graph.
- 4) Show/Hide graph: (H) “Backspace-H”. This allows you to hide the graph for the equation on which you are focused. This menu item will then become “Show Graph” which will restore its view.

13.2.4 THE OUTPUT MENU

- 1) Display Tactile Graph: (T) “Enter-T”. Renders a tactile representation of the current graph on the Braille display. Scroll Quickly to easily determine its shape. Start and stop quick auto scroll by pressing the Up and Down Scroll keys simultaneously.
- 2) Send to Embosser: (P) “Enter-P”. Does just what it implies, allows you to emboss a tactile version of the graph.

13.3 Display Compass Heading

Use this option to display the direction of the compass on the BrailleSense. You can display the compass heading from anywhere on the unit.

To display the compass heading, press "Space-F3-F4" from anywhere on the BrailleSense.

The current status of the compass is displayed.

You can also access compass status in the "Utilities" menu. Move to "Display Compass Heading" with "Space-4" and press "Enter".

As you change direction, to check the current compass status, press "Space". The current direction is announced. To exit this function, press "Space-E" or "Space-Z".

- Note: It will take about 15 minutes to initialize the unit should the following situations occur:

1. The unit discharges a great distance from the location where the compass was last used.
2. The unit remains discharged for more than a month.

13.3.1 OPTIONS OF GYROCOMPASS

There are 2 option settings available when viewing compass status: "Refreshing time" and "Compass display type".

Refreshing time is the interval of the automatic announcement of the compass status. To set this option, from the compass status, "Tab" to the "Refreshing time" edit box. Type the number of seconds between compass status announcements. By default, it is set to 0. You may type a number between 0 and 120.

You can set the Compass Display Type by tabbing to this option from the compass status. The setting values are "cardinal and degrees", "cardinal only" and "degrees only". By pressing "Space", you can change the value of the setting. By default, it is set to "cardinal and degrees".

13.4 Display Time and Date

Use this function to check the current date and time. Navigate to "Display Time and Date" in the "Utilities" menu, and press "Enter". You can also query the time and date from anywhere on the BrailleSense with "Space-T". You can Tab to display today's date.

13.5 Wake Up Alarm

You can use this function to set multiple alarms to remind you of recurring events, or wake you up at different times on varying days of the week. To open "Wake up alarm", from the "Utilities" menu, use "Space-1" or "Space-4" to navigate to "Wake up alarm", and press "Enter". You can launch the "Wake Up Alarm" from anywhere on the unit by pressing "F3-A". If you have not yet set the time or date in the BrailleSense, the unit prompts, "Set time:". After you set the time, the BrailleSense displays the "Wake up alarm" dialog.

The "Wake Up Alarm" dialog contains 5 items: Alarms list, an "Add" button, "Modify" button, "Delete" button, and a "Close" button. Use "F3" and "Space-F3" to move among the controls. Use "Space-1" and "Space-4" to navigate the Alarms list. You can also add an alarm by pressing "Enter-A", modify with "Enter-M", and delete an alarm with "Space-D".

The "Add alarm" dialog contains: "Set time", "Bell", "Term", "Alarm duration", "Repeat interval", "Wake up call times", a "Confirm" button, and a "Cancel" button.

When the alarm sounds, the Braille display displays the message "Wake up call. Press Backspace-enter to stop the alarm".

When you open the "Wake up alarm" dialog, the BrailleSense displays the "Set time" field. The value is set to the current time or the time specified previously. Here you can directly type the time, or set the time using the shortcut keys, as explained in section 8.1.3.

Tab to the next field and select from one of 3 alarm sounds, a vibration, a media file or an FM Radio frequency for the wake up alarm. Press "Space-1" or "Space-4" to change the alarm sound. As you move among the sounds, the various alarm sounds are played. When you reach the 4th option, "Vibrate motor", the BrailleSense produces a long vibration representing the one you would feel should you choose to use this option to alert you of your alarm.

If you choose "Media File", Tab to the "File Name" button and press "enter" to browse for the file you wish to use as your alarm sound. You are placed in a File Manager control with the focus inside the "Media" folder. Locate the file you wish to use, and press "Enter". If you choose "FM Radio", tabbing once places you in an edit box in to which you can type the frequency of the radio station you wish to wake up to.

When you have selected your desired alarm type, Tab to the next field.

In the term field, you can set which days of the week the alarm sounds. There are four options: "Off", "Once", "Week days", and "Every day". If the "Term" is set to "Off", the alarm does not go off at the specified time, no matter how the other options in the dialog are set.

The duration option sets the amount of time that the alarm sounds. The options are 1 minute, 2 minutes, or 3 minutes. If you set the ringing duration to "1 minute", the alarm sounds for one minute, and then is silent. To stop the alarm before a minute has passed, press "Backspace-enter".

The repeat interval defines the amount of time that is between the times that the alarm sounds. For example, if you set the repeat interval to three minutes, after the alarm has stopped sounding, it rings again in 3 minutes. The available settings are 1, 3, 5, 10, 15, or 20 minutes.

If the alarm is not turned off by pressing "Backspace-enter", it repeats according to the number of times you set the Wake Up Call Time option. The available settings are 1 to 10.

Press "Enter" to save your setting, or "Space-Z" to cancel setting the alarm.

13.6 Stopwatch

You can launch the "Stopwatch" from anywhere on the unit by pressing "F3-W". Or, you can navigate to it in the "Utilities" menu and press "Enter".

"00:00:00" is displayed. Press "Enter", to start the stopwatch running. Press "Enter" again, to pause, and the BrailleSense announces the elapsed time, and displays it in Braille. Pressing "Enter" again restarts the stopwatch running.

If you press "Backspace" while the stopwatch is paused or running, The BrailleSense announces the elapsed time and clears to "00:00:00". While the stopwatch is running, pressing "Space" gives you the elapsed time. As the stopwatch is still running, the Braille display continues displaying the time second by second as it passes.

Press "Space-F2" to stop displaying the time. If you press "Space-F2" once more, the time is displayed again.

Tab to switch to the "Countdown timer". You are asked to type hours, minutes, and seconds. Type a two-digit number for the hour, minute, and the second using Computer Braille or grade 1. You do not need to use a number sign. If you press "Enter" without typing any numbers, the "Countdown timer" starts counting down by the default time, which is set to one minute. When started, the "Countdown timer" displays "countdown xx minute", and begins running.

Press "Enter" on the "Countdown timer", and the BrailleSense tells you the remaining time, and pauses countdown. Press "Enter" again to restart the timer. Pressing "Backspace" clears the "Countdown timer". If the timer reaches zero, a 10-second alarm sounds, and the "Countdown timer" clears. If you press "Backspace-enter" while the alarm is sounding, the alarm stops and your timer clears, and is ready to accept a new time.

To exit the "Stopwatch" program, press "Space-Z".

13.7 Terminal for Screen Reader

The "Terminal for screen reader" function allows the BrailleSense to be used as a Braille display when connected to a computer or cell phone that uses a compatible screen reader such as JAWS, Supernova, Android's TalkBack or Apple's VoiceOver. Before using this function, make sure that the BrailleSense is connected to your personal computer or smart phone via a Bluetooth serial connection, or to the USB client port. To turn on this function, from the "Utilities" menu, navigate to "Terminal for screen reader", and press "Enter". To launch "Terminal for Screen Reader" from anywhere on the unit, press "F3-S".

The list of connection options is displayed: "Bluetooth serial port" and "USB port". Press "Enter" on your desired connection type, and the unit announces, "Terminal mode". If you are connecting using the Bluetooth serial connection

and Bluetooth has not been activated, the unit automatically activates Bluetooth before entering terminal Mode.

Once in Terminal Mode, the BrailleSense no longer speaks as it is now simply acting as a Braille display for your computer or smart phone. If the connection fails, you are returned to the list of connection methods.

You can connect a Bluetooth and USB device to the Terminal for Screen Reader simultaneously. Once you have established the first connection, press Backspace-Space-Enter-P to initiate the second connection. When 2 devices are connected, press Backspace-Space-Enter-C to toggle between the connections.

To exit the "Terminal for Screen reader" press "Backspace-Space-Z".

13.7.1 TERMINAL CLIPBOARD

Terminal Clipboard allows you to create and edit text on the notetaker before sending it via the Terminal for Screen Reader to the connected device. This can be useful in instances like when using VoiceOver on an i-device, in which characters are translated as they are sent to the iPhone or iPad. Or, you may simply find it more comfortable to edit in the notetaker environment.

As explained above, the Terminal Clipboard allows you to create and edit text on the notetaker, and send the completed text to the connected computer or Smartphone via the Terminal for Screen reader.

To enter "Terminal Clipboard", press "Space-Enter-I". The BrailleSense announces, "Terminal Clipboard". Type and edit your text using normal text entry and edit commands. In "Terminal Clipboard", the Braille display and keyboard are temporarily disabled from sending/receiving information to/from your connected computer or i-device, so that you may use them for creating/editing your text. For more information on text entry and editing, please see section 5 of

this manual on the Word Processor. When you are finished typing the text you wish to send to your computer or mobile device, press "Enter-S" and the text is quickly inputted to the device via the Terminal for Screen reader, and your BrailleSense returns to sending/receiving information to and from the connected device.

"Terminal Clipboard" can also be useful for transferring text from a document or e-mail in your notetaker, to a document or e-mail you are creating on your connected computer or Smartphone. As the "Terminal Clipboard" is a standard edit box on the notetaker, you can paste text in to it from the notetaker's clipboard. (see the sections of this manual regarding selecting text as it applies to supported programs such as the Word Processor, E-mail, Web Browser, etc.)

To send text from your notetaker:

- 1) Press "Space-Enter-I" to activate "Terminal Clipboard".
- 2) Press "Enter-V" to paste the text from the notetaker's clipboard to the edit box.
- 3) Press "Enter-S" to send the text via the Terminal for Screen Reader.

13.8 Display Network Status

This utility provides information about the current network connection type and status, the current IP of the unit, subnet mask, gateway, and MAC address. To check network status, press "Enter" on "Display Network Status". You can also query network status from the "program" menu by pressing "Space-N".

The information about the current connection is displayed. You can Tab to a "Renew" button and a "Close" button.

The first item in the information list (status) indicates whether the BrailleSense is currently connected to the Internet (online/offline), and if so, the current

connection type (LAN, Modem, ADSL, or Wireless LAN). The next item is the "current IP address", followed by "Subnet mask", and "Gateway", and finally "MAC address". You can move among these items by pressing "Space-4" or "Space-1". If the BrailleSense is not currently connected to the Internet, the status information shows "offline", and the IP address, subnet mask, and gateway display 0's.

13.9 Display Power Status

Press "Enter" on "Display power status" from the "Utilities" menu. Or, query power status from anywhere on the BrailleSense by pressing "Space-1-6". Two pieces of information are displayed: the percentage of charge remaining in the battery, and whether the BrailleSense is using the AC adapter or the battery to receive power. Exit by pressing "Space-Z". And you are returned to your previous location on the unit.

13.10 Format

WARNING!! The BrailleSense makes it possible to format the flashdisk to solve corruption problems and/or start over with a clean internal drive. Formatting the flashdisk erases everything on the flashdisk. Be extremely cautious when executing this command. If you execute this command, you will lose all of your data. You will NOT be able to retrieve this data once the command is executed. To format the flashdisk, press and hold "Space-1-2-3". While holding down "Space-1-2-3," quickly press and release the reset button. Continue holding "Space-1-2-3" until you hear the "format start" announcement.

You can also access the "format" function by pressing F from the "Utilities" menu. Or, you can launch it from anywhere on the BrailleSense by pressing "F3-F".

When you format any disk on the BrailleSense, all information is erased. Therefore, before you format, you are advised to backup any important data if possible.

- 1) From the Utilities menu, press "Enter" on "Format" and "flashdisk" is focused. Press "Enter", or choose another disk from the list, and press "Enter". *Note: you can choose the format type between FAT32 and exFAT by pressing F3 from the disk list.
 - 2) You are prompted, "Format flashdisk?" Yes". Press "Enter"
 - 3) You are prompted, "All data will be lost. Continue? Yes". Press "Enter".
 - 4) The unit announces, "Now formatting flashdisk" and the format is started.
 - 5) When the format is complete, you are returned to the "Disk" list.
- To cancel the format, press "Space", or press "Enter" on "No" in step 2 or 3.

13.11 Set Sleep Timer

You can set the sleep timer to turn the BrailleSense off after a specified amount of time regardless of what you are doing.

To set the Sleep Timer, navigate to "Sleep Timer" in the Utilities" menu and press "Enter". Or press "Space-J" from anywhere on the unit.

You are prompted, "Sleep timer: 0". Enter the number of minutes for the sleep timer in the edit box. The available settings range from 0 to 180. Press "Enter".

When the sleep timer is set, even if the device is active, it powers off after the specified amount of time. If you set the sleep timer for "5" minutes and press "Space-J (dots 2-4-5)" after 2 minutes, "3" is displayed in the edit box, indicating the time remaining before the sleep timer powers the unit off.

13.12 Using Macros

Macros allow you to automate functions of the Braille Sense for ease of use and efficiency, especially in cases where you perform a similar action often. For example, you may wish to set a macro to open a set of programs and documents when you come to work each morning. Or, perhaps, you want to set a macro to automate insertion of text and formatting of a document. You may even want to create a macro to automatically set an alarm for 6 in the morning the next day.

13.12.1 RECORDING A MACRO.

To create a macro, you simply ask the unit to record a set of functions as you perform them, and save it. To begin recording a macro, press "F2-R" from anywhere on the unit. Perform all the functions you wish to automate later, then press "F2-R" again to stop recording. For example, if you wish to set a macro to set an alarm, press "F2-R", go to the Utilities menu, and the Wake Up Alarm function. Go through the process of setting the alarm. After you have pressed "Enter" to complete the alarm setting, press "F2-R" to stop macro recording. You are asked to enter a name under which to save the macro. Type it in the edit box, and press "Enter". If you have made a mistake, and do not wish to save the macro, press "Space-Z" to cancel saving the macro.

There may be times when it is necessary to insert some wait time in your macro to account for loading time of a file or website, etc. You can insert delay time while recording a macro by pressing "F2-D". Type the delay time in milliseconds and press "Enter". Continue recording your macro as normal.

13.12.2 EXECUTING A MACRO.

To execute a previously created macro, press "F2-E" from anywhere on the Braille Sense. You are prompted for the macro name, and the last macro you created is automatically displayed. Type the macro name and press "Enter". The unit announces, "Executing macro, please wait,,,". The macro takes several seconds to execute as it must perform all the functions you previously recorded. When the macro is finished running, you are placed in the position you would normally be placed in if you had performed the functions manually. For example, if you set a macro to open your e-mail and check for new mail, when it has finished executing, you are placed in your inbox with the focus on the first unread message.

You may also create a macro to define a task you wish to perform multiple times. You can set the macro to repeat any number of times from the "Execute Macro" dialog. After pressing "F2-E", press "Space-4" to navigate to the "Repeat" edit box, and type the number of times you wish for the macro to repeat. The repeat count is remembered until you change it again or reset the unit.

13.12.3 THE MACRO MANAGER

The "Macro Manager" allows you to execute and manage macros you have previously created. It is located in the "Utilities" menu, just above the "Upgrade" option. Navigate to "Macro Manager" and press "Enter". You can also open the "Macro Manager" from anywhere on the unit using "F2-L".

The "Macro Manager" dialog contains: Macro Name List, "Repeat Macro", "Delete Macro", "Rename Macro", "Execute Macro", "Import Macro", "Export Macro", and a "Close" button.

The Macro Name List is a list of all the macros you have created. To execute a macro, press "Enter" on the macro name, or Tab to "Execute Macro" and press "Enter". As previously described, you can also execute a macro from anywhere

on the unit by pressing "F2-E". To delete a macro, navigate to the macro you wish to delete in the list, Tab to "Delete Macro" and press "Enter", or press "Space-D". You can rename a macro by focusing the macro you wish to rename, and selecting the "Rename Macro" option. You are placed in an edit box in which you can type the new macro file name, with the current name displayed. To rename the macro, edit the current name or type a new one, and press "Enter".

You can also import and export your Macro list from the Macro Manager, either to share with others, to backup Macros you have created, or to share among multiple units.

13.13 Upgrade BrailleSense 6 Firmware

Using this utility, you can update the BrailleSense either via an Internet connection or from external media containing the upgrade files. If you choose the "Online" method, be sure you have an active Internet connection before attempting the upgrade. When you update the BrailleSense, settings are usually maintained. However, if there are changes to setting values in the firmware, all settings will be initialized. Therefore, it is recommended that you back up your settings using the backup/restore utility in the "Settings" menu. Data in the flashdisk is protected. However, we recommend you backup your files before updating the BrailleSense Firmware as unexpected problems can occur.

- Note: You cannot cancel the update once it has started. You must not reset the unit during an upgrade. You must keep the unit connected to AC power throughout the upgrade process. We recommend you do not touch the BrailleSense while it is updating. It may take up to 60 minutes to update the BrailleSense, possibly even longer depending on the speed of your Internet connection and how many Android apps you have installed.

- Also note: the upgrade files are temporarily stored on the flashdisk for installation. If you do not have enough memory available on the flashdisk, the BrailleSense is unable to update until you make room for the upgrade files.

You can access the "Upgrade Firmware" option via the "Utilities" menu, or launch it from anywhere on the unit using "F3-U".

When you have chosen your upgrade method, and the upgrade files are downloaded or detected, BrailleSense will extract all of the necessary program files. Progress is announced every few seconds.

Once done, the BrailleSense will reboot, and the system files will be upgraded. During this process, there is no speech, but the progress will be shown in Braille as a percentage.

Once this has completed, a Braille message displaying "Initialization" will appear, followed by a numeric count of application files that have been initialized. Note that these application files include all BrailleSense and Android packages, and the number will be somewhere around 180 even if you have not installed any additional Android apps.

When the system files have been upgraded and the applications initialized, the software will load and boot. During this process, the BrailleSense displays "Starting BrailleSense" in Braille, and a progress indicator of changing dots appears on the right.

When the system reboots, if there are no new or changed setting values in the firmware, the Main Menu loads, and your settings are restored. If setting values have been changed, all Settings are initialized, and the system boots and runs the Quick Guide.

When the system boots for the final time after the upgrade, the release Note is displayed. Press "F4" to exit to the Main Menu. If you want to save the release

Note to the Documents folder, Tab to save and press "Enter". To display it again at any time, press "F2-F3-R" from the main Menu.

13.13.1 UPGRADING THE BRAILLESense FIRMWARE USING THE INTERNET

To upgrade the BrailleSense Firmware via the Internet, follow these steps:

- 1) Make sure you are connected to the Internet and plugged into AC power. Press "Enter" on "Upgrade BrailleSense Firmware" in the "Utilities" menu.
- 2) Press "Enter" on "Upgrade: Online". The BrailleSense connects to the HIMS server, and your current software version is compared to the HIMS server version. You are prompted to continue with the upgrade.
- 3) The BrailleSense begins downloading the upgrade files. The upgrade is around 1.3 GB. Thus, depending on the speed of your Internet connection, it may take a long time to download. If the upgrade fails, you may receive the message, "Internet connection is not available. To try the upgrade again, press "Space-Z" to cancel, and activate "Upgrade BrailleSense Firmware" again. If the connection is successful, the upgrade files are downloaded.
- 4) After downloading the upgrade files, BrailleSense automatically starts the upgrade process. Please remember you must not touch the BrailleSense until the upgrade is complete, as pressing keys, resetting, or unplugging the unit can interrupt the upgrade process, thus resulting in an incomplete installation. When the upgrade is complete, you are returned to the "program" menu, and the unit announces, "File manager".
- 5) After the upgrade is complete, check the software version by pressing "Space-V" while in the "program" menu.

13.13.2 UPGRADING THE BRAILLESense FIRMWARE FROM A DISK

To upgrade the BrailleSense Firmware from a disk, follow these steps:

- 1) Download the upgrade file from the HIMS website. Note: the file is a .bin file, however, some versions of Windows may rename it as a .zip file. Do not extract this file. Simply rename the extension back to .bin. Copy the file to the root of an external storage device such as an SD card or USB thumb drive.
- 2) Connect the SD card or USB drive to the BrailleSense, and be sure the BrailleSense is connected to AC power. Then, press "Enter" on "Upgrade BrailleSense Firmware" from the "Utilities" menu.
- 3) "Upgrade: Online" is displayed. Press "Space-4" to go to the "Upgrade: Offline" option. Press "Enter".
- 4) If you are already running the latest version of the software, you are asked to continue with the upgrade. If you press "Enter" on "No", the upgrade is cancelled. Press "Space" to toggle between "Yes" and "No".
- 5) The BrailleSense starts the upgrade process. Please remember you must not touch the BrailleSense until the upgrade is complete. When the upgrade is complete, you are returned to the "program" menu, and "File manager" is displayed.
- 6) When the upgrade is complete, check the software version by pressing "Space-V" while in the "program" menu.

14. Installing and Using Native Android Apps

While the BrailleSense has many powerful programs created specifically for the visually impaired, one of its most powerful features is the ability to install and

use native Android applications directly from the Google Play Store. Before you can install native Android applications, you must have a Google Play Store account.

You can run the Play Store by navigating to it from the Main menu, and pressing "Enter". You can also press the letter "P".

When you first open the Play Store application, you will need to sign into your Google account. It is important to note: when you use the Google Play Store and any applications you install from there, you are utilizing native Android apps. Thus, you are no longer accessing the BrailleSense interface, and interaction and navigation is slightly different.

BrailleSense 6 contains a mobile screen reader that is automatically executed when you use any native Android app. It is somewhat similar to TalkBack for Android with some notable enhancements. Because you must navigate the graphical Android screen, it can sometimes be difficult to locate what you need efficiently. For this reason, the BrailleSense screen reader allows you to locate icons according to their first letter. So, if you're looking for a Search icon, press the letter S. Pressing a given letter will take you to the first icon that matches. Subsequent presses of the same letter will jump to additional icons on the screen beginning with that letter.

Use "F3" and "Space-F3" to move among individual objects on the screen. The Control and Alt keys will often take you to different content areas on the screen. In programs like Docs, Sheets, and Slides, keyboard shortcuts are available for QWERTY keyboards, and the Control and Alt keys allow you also to utilize these functions. Use Backspace when Shift is required.

Use "Space-1-2-3" to move to the top of the current content area, and "Space-4-5-6" to move to the bottom of the current content area. Use "Enter" to select or run the current item, and "F4", "Space-E", or "Space-Z" to move back one screen.

To type in an edit box, you must press "Enter" to activate Edit Mode before you can input text. To adjust slider controls, press "Backspace-Space-4" to increase the slider value, and "Backspace-Space-1" to decrease a slider value".

You can also navigate Android apps using the Media keys on the front panel. Use "Back" and "Next" to move to the previous and next object. "Record" serves as the android Back button. "Play" opens Recent Apps, and "Stop" returns you to the main Menu. When playing media with an Android app such as Apple music or Spotify, hold the "Back" button to move to the previous item, hold the "Play" button to play/pause, and hold the "Next" button to move to the next item or track.

If you prefer to use more traditional keystrokes for navigating objects, you can change the method of navigation to emulate Swipe gestures as does TalkBack, using Space-1 and Space-4. To turn on the gesture method, press Backspace-Enter-H while using the Mobile Screen reader. Use the same keystroke to change back to the original mobile Screen Reader method.

To emulate a Double tap directly, regardless of mode, press "Space-Enter". This is useful in apps like Tap Tap See, where some buttons have no keyboard equivalents. To emulate Double Tap and Hold, press "Backspace-Space-Enter". This is useful in apps like Amazon, that offer additional menus of functions when you are placed on a shopping item, a song in Amazon Music, etc. Finally, to emulate a hold, you can use "Backspace-Space-Dot-2" to start and stop the hold. This is necessary in apps like WhatsApp Messenger for activating certain functions.

It is also possible to operate Android apps in a more traditional manner using a keyboard and/or mouse, or portable touch monitor. You can turn accessibility features on and off by pressing the "Stop" and "Next" media buttons together on the front panel of the device. When the Mobile Screen Reader is on, these accessories will operate similar to how they would when using TalkBack. When

the mobile Screen Reader is off, they will operate as they would traditionally operate in Android.

Once you have signed into your Google account, you may search for applications using the "Search" function. If you have other Android devices, and have already purchased apps from the Play Store, open the Navigation drawer, and choose the "My Apps" option. By default, the Play Store displays only what is currently installed on this unit. Select the "All" button to show all apps you have purchased or downloaded.

From an Apps list, press "Enter" to view the details of an app. If you wish to install it, locate the "Install" button and press "Enter". You are presented with a dialog showing the necessary app permissions. Locate the "Accept" button, or press the letter A to jump to it, and press "Enter".

When apps are installed, they will appear in the "All Apps" list at the end of the main program menu. When BrailleSense ships, this list contains stock Android apps and Google basics. The list is displayed in alphabetical order and can be navigated with first letter navigation.

At the bottom of the Apps list is an "App Manager. The App Manager allows you to uninstall any native Android app on your BrailleSense. You can also access the traditional Android Actions Menu for each app by pressing "Enter-I" when focused on the app you want to act on.

When you install Android apps, you may begin to hear sounds and announcements related to these apps. Many apps send "Notifications" to your BrailleSense in real-time, just as happens on smartphones and tablets. Press "Space-1-2-3-4-5-6" from anywhere on the unit to see a list of these notifications.

Press "Space-1" and "Space-4" to navigate the list, and press "Enter" on a notification to launch its associated application. Press "Space-D" to delete a

notification from the list. Press "Space-Enter-D" to clear all notifications. * Note: Google does not allow the deletion of some notifications.

- Note: While there are many native Android apps that work well, HIMS cannot and does not guarantee the accessibility or usability of any application that is not developed specifically for BrailleSense. Apps should generally work well if they meet Google accessibility standards, however many apps are not made to meet these standards and are not easily usable on any Android device running a screen reader.

15. Using Help

The help section consists of a navigable version of the BrailleSense 6 user Manual. To access this help file, press "F1" to go to the "program" menu, and use "Space-4" repeatedly until you locate "Help". Or press the letter h to quickly jump to and open "Help". You can also launch "Help from anywhere on the unit using "F1-H".

The User Manual will load, and you will be placed in the Navigation menu. Use "Space-dot-1" and "Space-dot-4" to navigate to the section you wish to read. To navigate to sub sections for the focused section, press F3. Then, use "Space-1" and "Space-4" once again to navigate the sub sections. To move back a level in the structure, press "Space-F3".

Press "Enter" on the focused section to open the manual at that location and start reading. Keys for navigating are the same as those in the Notepad. For a list of navigation keys, please see chapter 6. You can return to the Navigation menu at any time by pressing "Space-N".

As you read the manual on the BrailleSense 6, you can search for text by pressing "Space-F". Type the text you wish to find, and press "Enter". You can

find the next occurrence of the particular text you searched for by pressing "Enter-F".

The Introduction section gives a physical orientation to the BrailleSense 6 and an overview of its functionality.

The section, "Basic Functions", gives a basic overview of the programming and operation of the BrailleSense 6

The section, "Customizing your notetaker" provides an explanation of time and date setup, internet Setup, Bluetooth connectivity, and the "Global Options" and how to use them to configure your BrailleSense 6 according to your preferences.

Most of the remaining sections cover each of the functions of the BrailleSense 6 in detail.

The "Commands Summary" section is a quick reference of hot keys for all the programs on the BrailleSense 6, useful when you simply want to know the keystroke for performing an action without reading the entirety of the instructions on a given program.

You can also get help related to the specific task you are performing. Use "Space-H" within any program to get a list of keystrokes available in that program.

16. Troubleshooting Guide

If you are encountering problems with the BrailleSense 6, please take a look at the following suggestions for solving many common problems:

16.1 The BrailleSense 6 Will Not Turn On

- 1) Check to make sure that the battery is inserted in to its compartment properly.

2) If the battery is inserted properly, connect the AC adapter, and plug the AC adapter in to an electrical outlet. The battery may need to be charged.

3) Or, Check the location of the keyboard lock switch. If this switch is placed in the "lock" position, the power switch is not functional.

16.2 The Voice Cannot Be Heard

1) Make sure the volume is loud enough to be heard. Press "backspace-F4" to raise the volume.

2) Make sure the voice is turned on. Press "Backspace-F2" to toggle the voice on/off.

16.3 The Voice is Too Fast to Understand

1) Press "space-F1" to lower the voice rate.

16.4 The Voice Pitch Is Too Low or Too High to Understand

1) Press "enter-F1" to lower the pitch, or press "enter-F4" to raise the pitch of the voice.

16.5 You Don't Know Where You Are

1) If you have become disoriented as to your location on the BrailleSense 6, and cannot remember where you are, press "F1" to access the "program" menu. Then, press "F2-F3" to switch among the open programs. If you are in the Word

Processor, pressing "space1-5-6" gives you your current location in the open document.

16.6 No Internet Connection

- 1) If you are using a Wi-Fi connection, ensure wireless is turned on. Use "Backspace-dots-1-4-5-6" to toggle wireless on/off.
- 2) If using an Ethernet connection, check to ensure that the Ethernet cable is connected securely with the USB to Ethernet adapter, and that the network settings are correct.

16.7 E-mail

- 1) If you are having trouble downloading email, please contact your Internet service provider to make sure that you have the correct POP3/IMAP and SMTP settings.
- 2) If you cannot send or receive mail, make sure you have an active connection to the Internet.

16.8 The Braille display Does Not Display Braille

- 1) If the Braille display does not display Braille, make sure that the Braille display is turned on. Use "Backspace-F3" to toggle the Braille display on/off.

16.9 The BrailleSense 6 Does Not Speak when using Play store or Android Apps

If you do not have speech when you open the Play Store, or an Android app in your All Apps list, make sure the Mobile Screen Reader is on. You can toggle the

mobile Screen Reader on and off by pressing the “Stop” and “Next” media buttons on the front panel of the unit.

16.10 The BrailleSense Menus are Not Displayed or Spoken

At times, you may receive a prompt to set the default launcher between Google Now and the BrailleSense Launcher. If it is accidentally set to use the Google Now Launcher, the BrailleSense menus are not displayed, and accessibility features are not activated. In this case, System notifications may still be heard, but no Braille will appear, and no speech is heard.

To reset the BrailleSense Launcher, press “F2-F3-1-2-3-4-5-6-7-8”.

16.11 The BrailleSense 6 Does Not Respond

1. If the BrailleSense 6 does not respond to keystrokes or commands, it may be necessary to reset the unit. To reset the BrailleSense 6, press "1-2-3-4-5-6-F2-F3."
- 2) If the soft reset does not revive the BrailleSense 6, you may need to completely shut down the unit. The easiest way to do this is to remove the battery, leave it out for a few seconds, re-insert it, and power on the unit.
- 3) Initializing BrailleSense Options: if you are not able to solve your problem with the above methods, you may need to reset your BrailleSense options back to factory defaults. You can do this in two ways: if your unit is operational, you can launch the “Initialize BrailleSense options” item in the “Settings” menu. You may also hold the Forward Media key while pressing the Power On Button. Continue to hold Forward until “HIMS” is displayed on the braille display.

4: Hard resetting the Android system: In very serious cases, it may be necessary to perform a full system reset. However, this should not be attempted unless all other options have been exhausted, or unless you wish to completely erase all data, applications and settings. In addition, this process is complex and unique, and should only be done under the direction of an authorized support staff member. If you believe a system reset is necessary, please contact technical support for consultation and full instructions.

16.12 The LCD Does Not Come On

1) If the LCD screen does not come on, make sure that it is turned on. Press "space-o (dots 1-3-5)" to open "Global Options". Press "F3-F4" twice. Press "space" to turn the LCD screen option on, and press "enter" to save the option settings.

16.13 USB or SD Files Seem to Contain 0 Bytes

The Android operating system requires that drives are ejected properly before removing them. If not done, inserting them in to the BrailleSense or other devices may result in all files appearing as though they have no data, or have a size of 0 bytes. Before disconnecting a USB drive or SD card, press "Backspace-Enter-U" or "Backspace-Enter-3-6" to unmount it properly. If your drive is showing files with no data, please re-insert it in to the BrailleSense, and press the Unmount key.

17. Command Summary

17.1 Common Combination Keys

Open the main program menu: F1

Open the Running Tasks list: F1-F4

Switch to next running task: F2-F3

Switch to previous running task: Space-F2-F3

Open program specific menu: Space-m (dots 1-3-4) or F2

Exit: Space-z (dots 1-3-5-6)

Escape: Space-e (Space-1-5) or F4

Move to the next control in a dialog box: Tab (Space-4-5 or F3)

Move to the previous control in the dialog box: Shift-tab (Space-1-2 or Space-F3)

Move to the previous character: Space-3

Move to the next character: Space-6

Move to the previous line/item: Space-1

Move to the next line/item: Space-4

Move to the beginning of line/item: Space-1-3

Move to the end of line/item: Space-4-6

Move to the top of a document or list: Space-1-2-3

Move to the bottom of a document or list: Space-4-5-6

Move to the previous page or to the first item of the previous 32-item group:
Space-1-2-6 or F1-F2

Move to the next page or to the first item of the next 32-item group: Space-3-4-5
or F3-F4

Scroll back: Up scroll button

Scroll forward: Down scroll button

Move the cursor to a specific cell or list item: press the corresponding cursor routing key

Find: Space-f (dots 1-2-4)

Upper case: Space-u (dots 1-3-6)

Repeat current item: Space-r (dots 1-2-3-5)

Open web Page: Space-w (dots 2-4-5-6)

Say current time: Space-t (dots 2-3-4-5)

Display power status: Space-1-6

Open "Global options": Space-o (dots 1-3-5)

Online help: Space-h (dots 1-2-5)

Check software version: Space-v (dots 1-2-3-6) only from Main menu

Display network status: Space-n (dots 1-3-4-5) from Main menu

Display flash disk empty space: Space-i (dots 2-4) from Main menu

LCD on/off: Enter-4-5-6 from Main menu

Voice on/off: Backspace-F2

Braille on/off: Backspace-F3

Increase voice volume: Backspace-F4

Decrease the voice volume: Backspace-F1

Increase the voice rate: Space-F4

Decrease the voice rate: Space-F1

Increase the voice pitch: Enter-F4

Decrease the voice pitch: Enter-F1

Increase the main volume: Backspace-Space-F4

Decrease the main volume: Backspace-Space-F1

One-handed mode on/off: Backspace-1-2-3

Bluetooth on/off: Backspace-3-4-5-6

Wireless LAN on/off: Backspace-1-4-5-6

Typing mode on/off: Backspace-Space-Enter-M (dots 1-3-4)

Change Braille Code: Backspace-Enter-b (dots 1-2)

Scroll voice on/off: F3-up scroll button

Eight dot mode on/off: F1-down scroll button

Select Braille cursor type: F4-up scroll button

Change TTS Voice: F3-V (dots-1-2-3-6)

Select display of control information: F2-down scroll button

Unmount USB or SD: Backspace-Enter-u (dots 1-3-6) or Backspace-Enter-3-6

17.2 Quick Launch Commands

Use the following commands to launch their associated applications from anywhere on the BrailleSense 6.

Use F1 in combination with each letter to launch the following applications:

Address Manager: A (dot 1)

Web Browser: B (dots 1-2)

DAISY Player: D (dots 1-4-5)

Email: E (dots 1-5)
File Manager: F (dots 1-2-4)
Help: H (dots 1-2-5)
Exchange E-mail: L (dots-1-2-3)
Media Player: M (dots 1-3-4)
Notepad: N (dots-1-4-5)
FM Radio: R (dots 1-2-3-5)
Schedule Manager: S (dots 2-3-4)
Google search: U (dots-1-3-6)
Word Processor: W (dots 2-4-5-6)
Excel Viewer: X (dots 1-3-4-6)
Online DAISY: Z (dots 1-3-5-6)

Use F3 in combination with each letter to launch the applications below:

Wake up alarm: A (dot 1)
Bluetooth Manager: B (dots 1-2)
Calculator: C (dots 1-4)
Format: F (dots 1-2-4)
Setup Internet: I (dots 2-4)
Backup/Restore Personalized Settings: K (dots 1-3)
BrailleSense Math: M (dots-1-3-4)
Password Protection: N (dots 1-3-4-5)
Terminal for Screen Reader: S (dots 2-3-4)

Set Time & Date: T (dots 2-3-4-5)

Upgrade BrailleSense: U (dots 1-3-6)

Stopwatch: W (dots 2-4-5-6)

Other Quick Commands:

Display time and date: Space-t (dots 2-3-4-5)

Display network status: Space-n (dots 1-3-4-5)

Display power status: Space-dots-1-6

Display compass heading: Enter-dots 5-6

Change device name: Backspace-Enter-e (dots 1-5)

Open sleep Timer: Space-j (dots 2-4-5)

Open Global Options: Space-o (dots 1-3-5)

Open Sense Dictionary: Backspace-Enter-d (dots 1-4-5)

Open Menu Manager: From Main Menu, Enter-H (dots-1-2-5)

17.3 File Manager

17.3.1 NAVIGATION KEYS FOR THE FILE AND MENU LIST

Move to previous item: Space-1 or up scroll button

Move to next item: Space-4 or down scroll button

Move to the beginning of the list: Space-1-2-3 or Space-1-3

Move to the end of the list: Space-4-5-6 or Space-4-6

17.3.2 NAVIGATION KEYS FOR THE FILE LIST

Jump to folder list or file list: Space-5-6

Move between file list and address window: Tab (Space-4-5 or F3) / shift-tab (Space-1-2 or Space-F3)

Enter selected folder or open selected file: Enter

Move up one folder level: Backspace

Jump to the top level of the disk: Space-1-2-5-6

Move up 32 items: Space-1-2-3 or F1-F2

Move down 32 items: Space-3-4-5 or F3-F4

17.3.3 SELECTION COMMANDS

Select individual item: Space

Continuous selection: Enter-b (dots 1-2)

Select all: Enter-a (dot 1)

17.3.4 FILE MENU COMMANDS

Open: Enter-o (dots 1-3-5)

Open with: Enter-e (dots 1-5)

Zip: Enter-z (dots 1-3-5-6)

Unzip: Enter-u (dots 1-3-6)

Send To: Enter-s (dots 2-3-4)

Rename: Enter-r (dots 1-2-3-5)

New document: Enter-n (dots 1-3-4-5)

New folder: Enter-f (dots 1-2-4)

File conversion: Enter-t (dots 2-3-4-5)

Find file: Space-f (dots 1-2-4)

Find Text: Backspace-f (dots 1-2-4)

Merge files: Enter-m (dots 1-3-4)

Split files: Enter-p (dots 1-2-3-4)

Information: Enter-i (dots 2-4)

17.3.5 EDIT MENU COMMANDS

Copy: Enter-c (dots 1-4)

Cut: Enter-x (dots 1-3-4-6)

Paste: Enter-v (dots 1-2-3-6)

Delete: Enter-d (dots 1-4-5)

Select all: Enter-a (dot 1)

17.3.6 VIEW MENU COMMANDS

Sort files by: Enter-g (dots 1-2-4-5)

Set file info: Enter-l (dots 1-2-3)

Display only files of type: Enter-w (dots 2-4-5-6)

Hide system folders: Enter-h (dots-1-2-5)

Document Open Settings: Enter-D (dots-1-4-5)

17.4 Word Processor

17.4.1 FILE MENU COMMANDS

New document: Enter-n (dots 1-3-4-5)

Open: Enter-o (dots 1-3-5)

Save: Enter-s (dots 2-3-4)

Save as: Space-s (dots 2-3-4)

Export to Braille: Backspace-Enter-X (dots-1-3-4-6)

Close current document: Space-q (dots 1-2-3-4-5)

Print: Space-p (dots 1-2-3-4)

Settings: Enter-e (dots 1-5)

Set File open method: Backspace-Enter-O (dots-1-3-5)

Dictionary: Backspace-Enter-d (dots 1-4-5)

Exit: Space-z (dots 1-3-5-6)

17.4.2 EDIT MENU COMMANDS

Start selection: Enter-b (dots 1-2)

Copy: Enter-c (dots 1-4)

Add to the clipboard: Enter-p (dots 1-2-3-4)

Clear clipboard: Enter-d (dots 1-4-5)

Delete: Space-d (dots 1-4-5)

Cut: Enter-x (dots 1-3-4-6)

Paste: Enter-v (dots 1-2-3-6)

Delete blank lines: Backspace-e (dots 1-5)

Select all: Enter-a (dot 1)

Check spelling: Enter-k (dots 1-3)

Check spelling of the current word: Backspace-k (dots 1-3)

17.4.3 INSERT MENU COMMANDS

Set Mark: Enter-m (dots 1-3-4)

Jump to mark: Enter-j (dots 2-4-5)

Delete Mark: Backspace-d (dots 1-4-5)

Insert from file: Enter-i (dots 2-4)

Insert date: Enter-w (dots 2-4-5-6)

Insert time: Backspace-w (dots 2-4-5-6)

17.4.4 GO TO COMMANDS

Find: Space-f (dots 1-2-4)

Find next: Enter-f (dots 1-2-4)

Find previous: Backspace-f (dots-1-2-4)

Find and Replace: Enter-r (dots 1-2-3-5)

Go to location: Enter-l (dots 1-2-3)

Go To Previous Sentence: Space-2-3-6

Go To Next Sentence: Space-3-5-6

Go to previous page: Space-1-2-6 or F1-F2

Go to next page: Space-3-4-5 or F3-F4

Go to previous document: Tab (Space-1-2 or Space-F3)

Go to next document: Shift-tab (Space-4-5 or F3)

17.4.5 READ COMMANDS

Read selected text: Backspace-b (dots 1-2)

Read beginning of selected text: Enter-u (dots 1-3-6)

Read from beginning to cursor: Backspace-g (dots 1-2-4-5)

Read from cursor to end: Enter-g (dots 1-2-4-5)

Auto scroll: up-down scroll buttons

Read current paragraph: Space-2-3-5-6

Read Current Sentence: Space-1-4-5-6

Read current line: Space-1-4

Read current word: Space-2-5

Read current character: Space-3-6

Read status: Space-3-4

Read current cursor position: Space-1-5-6)

Read character and space count: Enter-3-4

Read Font info: Backspace-Enter-N (dots-1-3-4-5)

17.4.6 LAYOUT COMMANDS

Set font: Enter-backspace-f (dots 1-2-4)

Print document layout: Backspace-t (dots 2-3-4-5)

Print paragraph layout: Backspace-2-3-4-6

17.4.7 MISCELLANEOUS HOT KEYS

Move forward by heading: Space-dots-2-5-6

Move backwards by heading: Space-2-3-5

Move forward by double Enter: Space-Enter-5-6

Move back by Double Enter: Backspace-Space-2-3

Delete current paragraph: Backspace-2-3-5-6

Delete current line: Backspace-1-4

Delete current word: Backspace-2-5

Delete current character: Space-d (dots 1-4-5)

Set view format characters: Space-1-4-6

Set reading unit: Space-2-4-6

Set read only: Space-3-4-6

Move back one slide in PPT: Backspace

Move forward one slide in PPT: Space

Go to Slide: Enter-L (dots-1-2-3)

Insert page break: Backspace-space-Enter-p (dots 1-2-3-4)

Insert tab: Backspace-Space-Enter-l (dots 2-4)

Center line: Backspace-Space-Enter-c (dots 1-4)

Left justify: Backspace-Space-Enter-J (dots-2-4-5)

Right justify: Backspace-Space-Enter-R (dots-1-2-3-5)

Bold On/off: Control-B (dots-1-2)

Italics On/off: Control-I (dots-2-4)

Underline On/Off: Control-U (dots-1-3-6)

Toggle Math Mode on/off: Backspace-m (dots 1-3-4)

17.5 Notepad

17.5.1 FILE MENU COMMANDS

New document: Enter-n (dots 1-3-4-5)

Open: Enter-o (dots 1-3-5)

Save: Enter-s (dots 2-3-4)

Save as: Space-s (dots 2-3-4)

Close current document: Space-q (dots 1-2-3-4-5)

Print to embosser: Space-p (dots 1-2-3-4)

Settings: Enter-e (dots 1-5)

Dictionary: Backspace-Enter-d (dots 1-4-5)

Exit: Space-z (dots 1-3-5-6)

17.5.2 EDIT MENU COMMANDS

Start selection: Enter-b (dots 1-2)

Copy: Enter-c (dots 1-4)

Add to the clipboard: Enter-p (dots 1-2-3-4)

Clear clipboard: Enter-d (dots 1-4-5)

Delete: Space-d (dots 1-4-5)

Cut: Enter-x (dots 1-3-4-6)

Paste: Enter-v (dots 1-2-3-6)

Delete blank lines: Backspace-e (dots 1-5)

Select all: Enter-a (dot 1)

Check spelling: Enter-k (dots 1-3)

Check spelling of the current word: Backspace-k (dots 1-3)

17.5.3 INSERT MENU COMMANDS

Set Mark: Enter-m (dots 1-3-4)

Jump to mark: Enter-j (dots 2-4-5)

Delete Mark: Backspace-d (dots 1-4-5)

Insert from file: Enter-i (dots 2-4)

Insert date: Enter-w (dots 2-4-5-6)

Insert time: Backspace-w (dots 2-4-5-6)

17.5.4 GO TO COMMANDS

Find: Space-f (dots 1-2-4)

Find next: Enter-f (dots 1-2-4)

Find previous: Backspace-f (dots-1-2-4)

Find and Replace: Enter-r (dots 1-2-3-5)

Go to location: Enter-l (dots 1-2-3)

Go To Previous Sentence: Space-2-3-6

Go To Next Sentence: Space-3-5-6

Go to previous page: Space-1-2-6 or F1-F2

Go to next page: Space-3-4-5 or F3-F4

Go to previous document: Tab (Space-1-2 or Space-F3)

Go to next document: Shift-tab (Space-4-5 or F3)

17.5.5 READ COMMANDS

Read selected text: Backspace-b (dots 1-2)

Read beginning of selected text: Enter-u (dots 1-3-6)

Read from beginning to cursor: Backspace-g (dots 1-2-4-5)

Read from cursor to end: Enter-g (dots 1-2-4-5)

Auto scroll: up-down scroll buttons

Read current paragraph: Space-2-3-5-6

Read Current Sentence: Space-1-4-5-6

Read current line: Space-1-4

Read current word: Space-2-5

Read current character: Space-3-6

Read status: Space-3-4

Read current cursor position: Space-1-5-6)

Read character and space count: Enter-3-4

17.5.6 LAYOUT COMMANDS

Braille document layout: Backspace-p (dots 1-2-3-4)

Braille paragraph layout: Backspace-1-2-4-6

17.5.7 MISCELLANEOUS HOT KEYS

Delete current paragraph: Backspace-2-3-5-6

Delete current line: Backspace-1-4

Delete current word: Backspace-2-5

Delete current character: Space-d (dots 1-4-5)

Set reading unit: Space-2-4-6

Set read only: Space-3-4-6

17.6 E-mail/Exchange E-mail

17.6.1 HOT KEYS FOR THE INBOX

Open "Accounts Manager": Enter-m (dots 1-3-4)

Switch from Offline to Online mode (IMAP): Enter-dots1-2-3

Check for new mail: Enter-n (dots 1-3-4-5)

Receive old mail (IMAP): Backspace-p (dots1-2-3-4)

Move to previous account: Backspace-dots 1-2-6

Move to next account: Backspace-dots 3-4-5

Check current account: Space-3-4

Move to previous mailbox: Enter-dots 1-2-6

Move to next mailbox: Enter-dots 3-4-5

Write new message: Enter-w (dots 2-4-5-6)

Reply: Enter-r (dots 1-2-3-5)

Reply all: Backspace-r (dots 1-2-3-5)

Forward: Enter-f (dots 1-2-4)

Forward as attachment: Backspace-w (dots 2-4-5-6)

Open Set Path: Enter-p (dots 1-2-3-4)

Open Set Options: Enter-o (dots 1-3-5)

Open Set Spam: Enter-e (dots 1-5)

Find: Space-f (dots 1-2-4)

Find again: Backspace-f (dots 1-2-4)

Move to unread message: Enter-u (dots 1-3-4)

Move to mailbox: Backspace-x (dots 1-3-4-6)

Copy to mailbox: Backspace-c (dots 1-4)

Import messages: Backspace-l (dots-2-4)

Add a sender to the address list: Enter-i (dots 2-4) (This is valid only at the "from" item of the "inbox")

Delete current message: Space-d (dots 1-4-5) (valid only from the Subject and Date fields of the Inbox.

Empty Trash: Enter-Backspace-q (dots 1-2-3-4-5)

Save current message as text file: Space-s (dots 2-3-4)

Print e-mail: Space-p (dots 1-2-3-4)

Cancel: Space-e (dots 1-5)

Cancel and exit e-mail program: Space-z (dots 1-3-5-6)

17.6.2 HOT KEYS FOR SENDING E-MAIL

Search address list: Enter-l (dots 1-2-3)

Attach file: Enter-f (dots 1-2-4)

Send e-mail: Enter-s (dots 2-3-4)

Save to Outbox: Space-s (dots 2-3-4)

Cancel: Space-e (dots 1-5)

Cancel and exit e-mail program: Space-z (dots 1-3-5-6)

17.6.3 HOT KEYS FOR READING/WRITING AN E-MAIL MESSAGE

Open link in message body: Enter-L (dots-1-2-3)

Start selection: Enter-b (dots 1-2)

Select all: Enter-a (dot 1)

Copy to the clipboard: Enter-c (dots 1-4)

Cut to the clipboard: Enter-x (dots 1-3-4-6) (Only available when writing a message)

Paste from the clipboard: Enter-v (dots 1-2-3-6) (only available when writing a message)

Delete selection/delete current character: Space-d (dots 1-4-5)

Cancel selection: Space-e (dots 1-5)

17.7 Media

17.7.1 MEDIA PLAYER

17.7.1.1 File Commands

Open file: Enter-o (dots 1-3-5)

Open folder: Enter-f (dots 1-2-4)

Add file: Backspace-o (dots 1-3-5)

Add folder: Backspace-f (dots 1-2-4)

Save playlist: Enter-i (dots 2-4)
Save as playlist: Space-s (dots 2-3-4)
Delete item: Space-d (dots 1-4-5)
Open URL: Enter-u (dots 1-3-6)
Get tag information: Enter-t (dots 2-3-4-5)

17.7.1.2 Playback Commands

Play: Enter, Play button
Pause/play: Space, Play button
Stop: Backspace, Stop button
Previous track: Dot 3, Previous button
Next track: Dot 6, Next button
Back 5 tracks: Dots 2-3
Forward 5 tracks: Dots 5-6
First track: Dots 1-3
Last track: Dots 4-6
Volume up: Space-Up Scroll button
Volume down: Space-Down Scroll button
Speed up: Dots 3-4-5
Slow down: Dots 1-2-6
Change time index: Dot 2 or Dot 5
Move to next position by time index: Dot 4
Move to previous position by time index: Dot 1

Delete playlist: Space-d (dots 1-4-5)
Set marked position: Enter-m (dots 1-3-4)
Delete marked position: Enter-d (dots 1-4-5)
Jump to marked position: Enter-j (dots 2-4-5)
Go to position: Enter-g (dots 1-2-4-5)
Set mark: Backspace-m (dots 1-3-4)
Move to mark: Backspace-j (dots 2-4-5)
Move to previous mark: Backspace-dot 3
Move to next mark: Backspace-dot 6
Delete mark: Backspace-d (dots 1-4-5)
Mark manager: Backspace-k (dots 1-3)
Configuration dialog: Backspace-c (dots 1-4)
Playback settings dialog: Enter-s (dots 2-3-4)
Record settings dialog: Backspace-s (dots 2-3-4)
Effect settings dialog: Enter-e (dots 1-5)

17.7.1.3 Recording Commands

Open the Record dialog: Enter-r (dots 1-2-3-5), Record button
Record: Record button
Pause/Continue recording: Space, Record button
Play: Play button

17.7.1.4 Hot Keys in the Play Information Tab

Move to the next item: Space-dot-4

Move to the previous item: Space-dot-1

Open menu: F2 or Space-m (dots 1-3-4)

Open help: Space-h (dots 1-2-5)

Move to Playlist tab: Space-4-5/Space-1-2

Exit Media Player: Space-z (dots 1-3-5-6)

17.7.1.5 Hot Keys in the Playlist Tab

Move to the next file: Space-4

Move to the previous file: Space-1

Move to the last file: Space-4-6

Move to the first file: Space-1-3

Play current file: Space

Start file selection: Enter-b (dots 1-2)

Start playing the selected files: Enter

Stop playing: Backspace

Delete the selected files from the playlist: Space-d (dots 1-4-5)

Move to "playback information" tab: Space-4-5/Space-1-2

17.7.2 FM RADIO

Open the Presets dialog: Enter-e (dots 1-5)

Open the menu: Space-m (dots 1-3-4) or F2

Internal speakers on/off: Space-x (dots 1-3-4-6)

Move to next frequency: Dot 6

Move to previous frequency: Dot 3

Autoscan backward: Dot 2, short press Previous button

Autoscan forward: Dot 5, short press Next button

Go to frequency: Space-f (dots 1-2-4)

Move to previous preset: Dot 1, long press Previous button

Move to next preset: Dot 4, long press Next button

Save preset: Enter-s (dots 2-3-4) or long press Record button

Delete preset: Space-d (dots 1-4-5) or long press Stop button

Open Record dialog: Enter-r (dots 1-2-3-5)

Record pause/resume: short press Record button

Stop recording: short press Stop button

Mute on: short press Stop button, Space

Mute off: short press Play button, Space

Volume up: Space-Up Scroll button

Volume down: Space-Down Scroll button

17.7.3 DAISY PLAYER

Read title: Space-dots 1-2-5-6

Open DAISY book: Enter-o (dots 1-3-5)

Voice settings: Enter-s (dots 2-3-4)

Check book-info: Enter-i (dots 2-4)

Exit: Space-z (dots 1-3-5-6)

Play/Pause: Space

Go to beginning: Space-1-3 or Space-1-2-3

Go to end: Space-4-6 or Space-4-5-6

Increase volume: Backspace-dot-4

Decrease Volume: Backspace-dot-1

Increase Speed: Backspace-dot-5

Decrease Speed: Backspace-dot-2

Increase Pitch: Backspace-dot-6

Decrease Pitch: Backspace-dot-3

Move to the next page: Space-3-4-5

Move to the previous page: Space-1-2-6

Go to page: Enter-g (dots 1-2-4-5)

Select previous text/time unit: Dot 1

Select next text/time unit: Dot 4

Move to previous text/time unit: Dot 2

Move to next text/time unit: Dot 5

Select previous movement unit: Dots 2-3

Select next movement unit: Dots 5-6

Move to the previous unit: Dot 3 or short press Previous button

Move to the next unit: Dot 6 or short press Next button

Move to the next heading: Space-5 or long press Next button

Move to the previous heading: Space-2 or long press Previous button
Move to the last heading: Space-4-6
Move to the first heading: Space-1-3
Check the current-reading level and heading: Enter-w (dots 2-4-5-6)
Scan the heading list from the present to the end: Enter-n (dots 1-3-4-5)
Find heading: Space-f (dots 1-2-4)
Headings list: Enter-h (dots 1-2-5)
Find again: Enter-f (dots 1-2-4)
Set mark at current position: Enter-m (dots 1-3-4)
Move to the next mark: Space-dot 6
Move to the previous mark: Space-dot 3
Delete mark: Space-d (dots 1-4-5)
Create memo: Backspace-m (dots-1-3-4)
Read memo: Backspace-r (dots-1-2-3-5)
Read previous memo: Backspace-p (dots-1-2-3-4)
Read Next Memo: Backspace-n (dots-1-3-4-5)
Delete Memo: Backspace-d (dots-1-4-5)
Exit and erase position without prompt: Enter-z (dots-1-3-5-6)
Exit and erase marks without prompt: Backspace-z (dots-1-3-5-6)

17.8 Organizer

17.8.1 ADDRESS MANAGER

17.8.1.1 Navigation When Adding an Address

Move to the previous field or menu item: Space-1 or Up Scroll button

Move to the next field or menu item: Space-4 or Down Scroll button

Move to the first field or menu item: Space-1-2-3

Move to the last field or menu item: Space-4-5-6

17.8.1.2 Navigation in Search Results List

Move to the previous record: Space-1 or Up Scroll button

Move to the next record: Space-4 or Down Scroll button

Move to the first record: Space-1-2-3

Move to the last record: Space-4-5-6

Move to Record information: F3

17.8.1.3 General Commands

Add address: Enter-i (dots 2-4)

Find address: Enter-s (dots 2-3-4)

Edit address: Enter-m (dots 1-3-4)

Delete address: Space-d (dots 1-4-5)

Select all: Enter-a (dot 1)

Save as a file: Enter-t (dots 2-3-4-5)

Print address: Space-p (dots 1-2-3-4)

Backup address list: Enter-u (dots 1-3-6)

Restore address list: Enter-r (dots 1-2-3-5)

Set backup options: Enter-e (dots 1-5)

Setting add/searching address fields: Enter-o (dots 1-3-5)

Import from CSV: Backspace-i (dots 2-4)

Export to CSV: Backspace-x (dots 1-3-4-6)

17.8.2 SCHEDULE MANAGER

17.8.2.1 General Commands

Move to the next field: Space-4-5 or F3

Move to the previous field: Space-1-2 or Space-F3

Add appointment: Enter-i (dots 2-4)

Search appointments: Enter-s (dots 2-3-4)

Modify appointment: Enter-m (dots 1-3-4)

Delete appointment: Space-d (dots 1-4-5)

Delete all appointments: Backspace-Enter-q (dots 1-2-3-4-5)

Backup appointments: Enter-u (dots 1-3-6)

Restore appointments: Enter-r (dots 1-2-3-5)

Set backup options: Enter-e (dots 1-5)

Alarm options: Enter-o (dots 1-3-5)

Save appointments as file: Enter-t (dots 2-3-4-5)

Print appointments: Space-p (dots 1-2-3-4)

Refresh sync with cloud: Backspace-R (dots 1-2-3-5)

17.8.2.2 Navigating Search Results

Move to the previous appointment: Space-2

Move to the next appointment: Space-5

Move to the first appointment: Space-2-3 or Space-1-2-3

Move to the last appointment: Space-5-6 or Space-4-5-6

Move to the previous appointment on the same day: Space-1 or Up Scroll button

Move to the next appointment on the same day: Space-4 or Down Scroll button

Move to the first appointment on the same day: Space-1-3

Move to the last appointment on the same day: Space-4-6

17.9 Web tools

17.9.1 WEB BROWSER

17.9.1.1 General Commands

Open URL: Enter-u (dots 1-3-6)

Open: Enter-o (dots 1-3-5)

Save as: Space-s (dots 2-3-4)

Information: Enter-i (dots 2-4)

File Download Dialog: Backspace-3-4

Exit: Space-z (dots 1-3-5-6)

Read from beginning to cursor: Backspace-g (dots 1-2-4-5)

Read from cursor to end: Enter-g (dots 1-2-4-5)

Auto scroll: Up Scroll button-Down Scroll button

Start selection: Enter-b (dots 1-2)
Copy: Enter-c (dots 1-4)
Add to clipboard: Enter-p (dots 1-2-3-4)
Copy URL: Backspace-d (dots 1-4-5)
Copy link: Backspace-l (dots 1-2-3)
Go to home page: Backspace-h (dots 1-2-5)
Go to previous page: Backspace-Space-3 or Backspace-p (dots 1-2-3-4)
Go to next page: Backspace-Space-6 or Backspace-n (dots 1-3-4-5)
Go to previous heading: Backspace-b (dots 1-2)
Go to next heading: Backspace-f (dots 1-2-4)
Go to previous text chunk: Space-2-4-6
Go to next text chunk: Space-1-2-4-5-6
Go to previous visited link: Space-2-3-6
Go to next visited link: Space-3-5-6
Refresh: Enter-r (dots 1-2-3-5)
Open the history list: Enter-h (dots 1-2-5)
Check the address of the selected title in the history list: Space-i (dots 2-4)
Find: Space-f (dots 1-2-4)
Find again: Enter-f (dots 1-2-4)
Links list: Backspace-i (dots 2-4)
RSS Feed List: Enter-j (dots 2-4-5)
Set current as your home page: Enter-s (dots 2-3-4)
Add to favorites: Backspace-a (dot 1)

Favorites list: Enter-l (dots 1-2-3)

Option settings: Backspace-o (dots 1-3-5)

17.9.1.2 Hot Keys When Playing Audio Files in Web Browser

Turn up the volume: Space-Up Scroll button

Turn down the volume: Space-Down Scroll button

Play: Play button on the front panel

Stop: Stop button on the front panel

17.9.1.3 Hot Keys for Reading Web Pages

Move to the previous control: Shift-tab (Space-1-2)

Move to the next control: Tab (Space-4-5)

Select the next item in a combo box (list box): Backspace-Space-4

Select the previous item in a combo box (list box): Backspace-Space-1

Move to previous form control: Space-1-2-3-4-6

Move to next form control: Space-1-3-4-5-6

Move to previous table: Space-3-4-5-6

Move to next table: Space-1-4-5-6

Move to previous cell: Space-2-3

Move to next cell: Space-5-6

Move to upper cell: Space-3-5

Move to lower cell: Space-2-6

Read current cell: Space-2-3-5-6

Move to previous cell of upper level table: Backspace-2-3

Move to next cell of upper level table: Backspace-5-6

Move to upper cell of upper level table: Backspace-3-5

Move to lower cell of upper level table: Backspace-2-6

Check current position: Space-1-5-6

Move to previous frame: Space-3-4-6

Move to next frame: Space-1-4-6

Read current line: Space-r (dots 1-2-3-5)

17.10 Extras

17.10.1 EXCEL VIEWER

17.10.1.1 Navigation Commands

Move to next row: Space-Dot 4

Move to previous row: Space-Dot 1

Move to next column: Space-Dot 6

Move to previous column: Space-Dot 3

Move to beginning of row: Space-Dots 1-3

Move to end of row: Space-Dots 4-6

Move to top of column: Space-Dots 2-3

Move to bottom of column: Space-Dots 5-6

Move to top of worksheet: Space-1-2-3

Move to bottom of worksheet: Space-4-5-6

Move to next worksheet: Space-3-4-5

Move to previous worksheet: Space-1-2-6

Move to next group: Backspace-Dots 5-6

Move to previous group: Backspace-Dots 2-3

Move to next group in row: Space-Dot 5

Move to previous group in row: Space-Dot 2

Move to next group in column: Space-Dots 5-6

Move to previous group in column: Space-Dots 2-3

17.10.1.2 File Commands

Open: Enter-o (dots 1-3-5)

Open Hyperlink: Enter-h (dots 1-2-5)

Set User table: Enter-t (dots 2-3-4-5)

Cell properties: Enter-i (dots 2-4)

Quick Setup: Enter-x (dots 1-3-4-6)

Save as: Space-s (dots 2-3-4)

17.10.1.3 Go To Commands

Find: Space-f (dots 1-2-4)

Find Next: Enter-f (dots 1-2-4)

Find Previous: Backspace-f (dots 1-2-4)

Move to cell: Enter-j (dots 2-4-5)

Next sheet: Space-3-4-5

Previous sheet: Space-1-2-6

Sheet list: Enter-l (dots 1-2-3)

Set Mark: Backspace-m (dots 1-3-4)

Go to Mark: Backspace-j (dots 2-4-5)

Delete Mark: Backspace-d (dots 1-4-5)

17.10.1.4 Read Commands

Set Options: Backspace-o (dots 1-3-5)

Sheet title: Space-1-5-6

Read from top of column to cursor: Backspace-Dot 1

Read from cursor to bottom of column: Backspace-Dot 4

Read from beginning of row to cursor: Backspace-Dot 3

Read from cursor to end of row: Backspace-Dot 6

Read current row: Space-c (dots 1-4)

Read current column: Space-y (dots 1-3-4-5-6)

Read header cell of row: Backspace-c (dots 1-4)

Read header cell of column: Backspace-y (dots 1-3-4-5-6)

Read from beginning to cursor: Backspace-g (dots 1-2-4-5)

Read from cursor to end: Enter-g (dots 1-2-4-5)

Read One character (When in Cell Activation): Space-Dots 3-6

Read Current Cell: Space-Dots 3-6

Read File Name: Space-Dots 3-4

Read Selected Text: Backspace-b (dots 1-2)

17.10.1.5 View Commands

View Hide Sheet: Enter-s (dots 2-3-4)

View Hide Row: Enter-r (dots 1-2-3-5)

View Hide Column: Enter-y (dots 1-3-4-5-6)

Cell Activation: Enter

Activation Cancel: Backspace

Set Time and Date: Enter-d (dots 1-4-5)

17.10.1.6 Edit Commands

Start Selection: Enter-b (dots 1-2)

Select All: Enter-a (dot 1)

Copy Selected Text: Enter-c (dots 1-4)

17.10.2 ONLINE DAISY

Move between category and booklist: F3 and Space-F3

Move among list items: Space-1 and Space-4

Download book: Enter

Return book: Enter-r (dots 1-2-3-5)

17.11 Utilities

17.11.1 CALCULATOR

Clear calculator: Backspace-c (dots 1-4)

Delete numbers or operators: Backspace

Calculation history: Enter-h (dots 1-2-5)

General functions: Enter-g (dots 1-2-4-5)

PI: Enter-p (dots 1-2-3-4)

Exp: Enter-e (dots 1-5)

Trigonometric Functions: Enter-s (dots 2-3-4)

Sine: Enter-s (dots 2-3-4), s (dots 2-3-4)

Arc Sine: Enter-s (dots 2-3-4), a (dot 1)

Hyperbolic Sine: Enter-s (dots 2-3-4), h (dots 1-2-5)

Cosine: Enter-i (dots 2-4), i (dots 2-4)

Arc Cosine: Enter-c (dots 1-4), a (dot 1)

Hyperbolic Cosine: Enter-c (dots 1-4), h (dots 1-2-5)

Tangent: Enter-t (dots 2-3-4-5), t (dots 2-3-4-5)

Arc Tangent: Enter-t (dots 2-3-4-5), a (dot 1)

Hyperbolic Tangent: Enter-t (dots 2-3-4-5), h (dots 1-2-5)

Logarithm Function: Enter-l (dots 1-2-3)

Natural Logarithm: Enter-l (dots 1-2-3), e (dots 1-5)

Common Logarithm: Enter-l (dots 1-2-3), n (dots 1-3-4-5)

Recall Items: Backspace-r (dots 1-2-3-5)

Delete all saved items: Enter-d (dots 1-4-5)

Save Items: Backspace-s (dots 2-3-4)

Convert Unit: Enter-u (dots 1-3-6)

Copy to Clipboard: Enter-c (dots 1-4)

Option Settings: Enter-o (dots 1-3-5)

Fraction options: Space-dots 3-4

Return formula: Enter-r (dots 1-2-3-5)

Change Braille code: Space-g (dots 1-2-4-5)

17.11.2 BRAILLESENSE MATH

17.11.2.1 File Menu Commands

Export: Enter-E (dots-1-5)

Import: "Enter-I (dots-2-4)

Save to History: "Space-S (dots-2-3-4)

History: "Enter-H (dots-1-2-5)

Save Graph As Braille: "Enter-F (dots-1-2-4)

Exit: "Space-Z (dots-1-3-5-6)

17.11.2.2 The Edit Menu

Copy: "Enter-C (dots-1-4)

Cut: "Enter-X" (dots-1-3-4-6)

Paste: "Enter-V (dots-1-2-3-6)

Delete: "Space-D (dots-1-4-5)

Select All: "Enter-A (dot-1)

Start Selection: "Enter-B (dots-1-2)

Copy Graph: "Enter-G (dots-1-2-4-5)

Symbol picker: "Backspace-S (dots-2-3-4)

Set Math Code: "Backspace-G (dots-1-2-4-5)

17.11.2.3 The Graph Menu

Axis: "Backspace-X (dots-1-3-4-6)

Graph label: "Backspace-L (dots-1-2-3)

Graph Style: Backspace-C (dots-1-4)

Show/Hide graph: "Backspace-H (dots-1-2-5)

17.11.2.4 The Output Menu

Display Tactile Graph: Enter-T (dots-2-3-4-5)

Send to Embosser: Enter-P (dots-1-2-3-4)

17.11.3 DISPLAY COMPASS HEADING

Open Display Compass Heading: Space-F3-F4

Move to the next control: Space-4-5 or F3

Move to the previous control: Space-1-2 or Space-F3

Change compass display type: Backspace or Space

Exit: Space-z (dots 1-3-5-6) or Space-e (dots 1-5)

17.11.4 ALARM

Add Alarm: Enter-A (dot-1)

Modify Alarm: Enter-M (dots-1-3-4)

Delete Alarm: Space-D (dots-1-4-5)

17.11.5 STOPWATCH

Stopwatch start, pause, restart: Enter

Check the elapsed time: Backspace

Stop and restart Braille display: Space-F2

Switch between stopwatch and countdown timer: Space-4-5 or Space-1-2

Count down timer pause or restart: Enter

Count down timer Initialize: Backspace

Exit stopwatch or countdown timer: Space-z (dots 1-3-5-6)

17.11.6 TERMINAL FOR SCREEN READER

Enter terminal Clipboard: Space-Enter-i (dots 2-4)

Send Terminal Clipboard content: Enter-s (dots 2-3-4)

Exit terminal for Screen reader: Space-Backspace-z (dots 1-3-5-6)

17.11.7 MACRO COMMANDS

Start/stop recording Macro: F2-R (dots-1-2-3-5)

Insert Delay Time: F2-D (dots-1-4-5)

Execute Macro: F2-e (dots-1-5)

Macro Manager: F2-L (dots-1-2-3)

17.11.8 OTHER UTILITY COMMANDS

Display network status: Space-n (dots 1-3-4-5)

Display power status: Space-1-6

Format: Space-1-2-3

Set Sleep Timer: Space-j (dots 2-4-5)

Check upgrade download progress: Space-i (dots 2-4)

Check date and time: Space-t (dots 2-3-4-5)

17.12 Settings

17.12.1 SETTING TIME AND DATE

Move to the next day: Space-6

Move to the previous day: Space-3

Move to the next week: Space-5

Move to the previous week: Space-2

Move to the next month: Space-4

Move to the previous month: Space-1

Move to the next year: Space-5-6

Move to the previous year: Space-2-3

Move to 1 hour after: Space-4

Move to 1 hour before: Space-1

Move to 10 minutes after: Space-5

Move to 10 minutes before: Space-2

Move to 1 minute after: Space-6

Move to 1 minute before: Space-3

Switch between A.M. and P.M.: Space-x (dots 1-3-4-6)

17.12.2 BLUETOOTH MANAGER

Rescan for Bluetooth devices: Enter-r (dots 1-2-3-5)

Connect device: enter on the device name

Connect service: enter on the service name

Disconnect the service in Bluetooth service list: Enter-d (dots 1-4-5)

Move to Bluetooth device list in service list: Backspace

Remove device settings in Bluetooth device list: Space-d (dots 1-4-5)

17.12.3 BACKUP/RESTORE PERSONALIZED SETTINGS

Move to previous item: Space-1 or up scroll button

Move to next item: Space-4 or down scroll button

Move to previous control: Shift-tab (Space-1-2 or Space-F3)

Move to next control: Tab (Space-4-5 or F3)

Select/unselect check box: Space

Backup: Enter-b (dots 1-2)

Restore: Enter-r (dots 1-2-3-5)

17.12.4 PASSWORD PROTECTION.

Set password: Enter-i (dots 2-4)

Modify password: Enter-m (dots 1-3-4)

Password configuration: Enter-c (dots 1-4)

Reset to factory defaults: i (dots 2-4)

17.13 Screen Reader Commands

17.13.1 MOVING FOCUS

Move to next object: Space-4-5 or F3

Move to previous object: Space-1-2 or Space-F3

Move to top of screen: Space-1-2-3

Move to bottom of screen: Space-4-5-6

Scroll down one screen: Space-3-4-5

Scroll up one screen: Space-1-2-6

Execute current item: Enter or Cursor routing key

Back button: Space-e, F4 or Space-Z

Move to specific object: press first letter of object name

Go to Home screen: F1

Activate App Actions Menu: Enter-I on focused app

Open Notification Shade: Space-1-2-3-4-5-6

Activate Notification: Enter

Delete notification: Space-D (dots-1-4-5)

Clear all notifications: Space-Enter-D (dots-1-4-5)

Activate Edit Box: Enter

Increase Slider Value: Backspace-Space-4

Decrease Slider Value: Backspace-Space-1

Double Tap: Space-Enter

Double Tap and Hold: Backspace-Space-Enter

Start/Stop Hold: Backspace-Space-Dot-2

Reset BrailleSense Launcher: F2-F3-1-2-3-4-5-6-7-8

Change object navigation method: Backspace-Enter-H (dots-1-2-5)

17.13.2 WHEN EDITING TEXT

Go to next character: Space-6

Go to previous character: Space-3

Go to next word: Space-5

Go to previous word: Space-2

Go to next line: Space-4

Go to previous line: Space-1

Go to next paragraph: Space-5-6

Go to previous paragraph: Space-2-3

Go to beginning of paragraph: Space-1-3

Go to end of paragraph: Space-4-6

Go to top of document: Space-1-2-3

Go to bottom of document: Space-4-5-6

Delete current character: Space-1-4-5

Delete previous character: Backspace

Delete current word: Backspace-2-5

Delete current line: Backspace-1-4

Delete current paragraph: Backspace-2-3-5-6

Move to next display length: Down Scroll

Move to previous display length: Up Scroll

17.13.3 MEDIA KEY FUNCTIONS WHEN IN "APP MODE"

Open Main menu: Stop button

Recent Apps: Play button

Back: Record button

Move to previous object: Previous button

Move to next object: Next button

Android Media play/Pause: Hold Play button

Android previous track: Hold Previous button

Android Next track: hold Next button

Mobile Screen Reader on-off: Stop and Next buttons

Call Google Voice Assistant: Record and Play buttons

FCC regulatory conformance:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference.
- (2) This device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a **Class B** digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

RF Exposure Information and Statement

The SAR limit of USA (FCC) is 1.6 W/kg averaged over one gram of tissue. Device types Tablet PC (FCC ID: 2AL4DH632B) has also been tested against this SAR limit. The highest SAR value reported under this standard during product certification for use on the body is 2.4GHz:0.639W/kg, 5GHz:0.973W/kg. This device was tested for typical body-worn operations with the back of the handset kept 0mm from the body. To maintain compliance with FCC RF exposure requirements, use accessories that maintain a 0mm separation distance between the user's body and the back of the handset. The use of belt clips, holsters and similar accessories should not contain metallic components in its assembly. The use of accessories that do not satisfy these requirements may not comply with FCC RF exposure requirements, and should be avoided.

5G WIFI indoor used only