ZEPP CLARITY ONE

Follow the safety information to avoid damage or injury.

physician, preferably an ear specialist, for diagnosis of potential associated medical conditions. Furthermore, children should
receive a formal hearing evaluation and rehabilitation since
hearing loss may cause problems in language development and

- Unilateral hearing loss of sudden or recent onset within the
previous 90 days.

- Audiometric air-bone gap equal to or greater than 15 decibels


anything related may arise, please contact a physician.

• The wireless capabilities featured in your hearing aid are approved to operate at a radio frequency specific to your country or region


hearing aid to prevent any damage. To prevent damage and/or malfunction, do not modify the hearing aid, charging case, or accessories.
Do not repair or replace the lithium-ion rechargeable battery yourself. Contact the manufacturer if you believe a battery The sound output should not be uncomfortable or painful.
 You should turn down the volume or remove the hearing aid if
 the sound output is uncomfortably loud or painful.
 The hearing aid will not restore normal hearing and will not
 prevent or improve a hearing impairment resulting from organic conditions.


hearing aid. The hearing aid evaluation will enable the audiologist or dispenser to select and fit a hearing aid to your individual


• Fatigue due to greater listening effort.

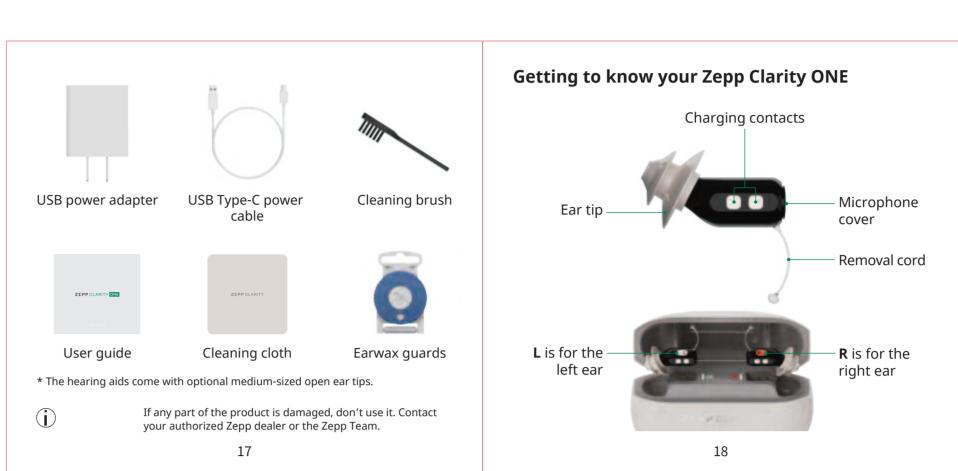



Lid: Protects the Zepp Clarity ONE during charging and storage. case can also be used to personalize your listening experience in each ear. Your charging case is the safest place for your Zepp Clarity ONE when not being worn. To charge the charging case, connect the small end of the USB-C status of the Zepp Clarity ONE (page XX). cable to the USB-C port on the bottom of the charging case. You can connect the larger end of the power cord to the included USB power adapter. inside will also be charged. The Zepp Clarity ONE automatically

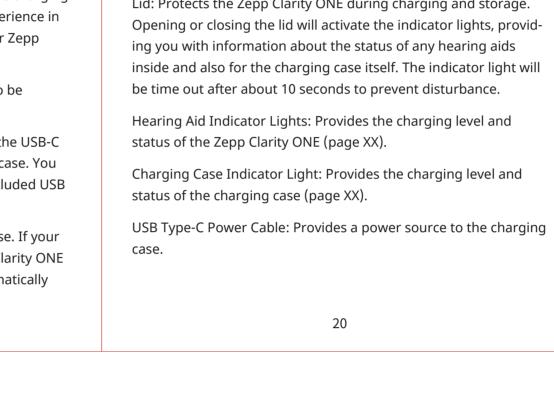


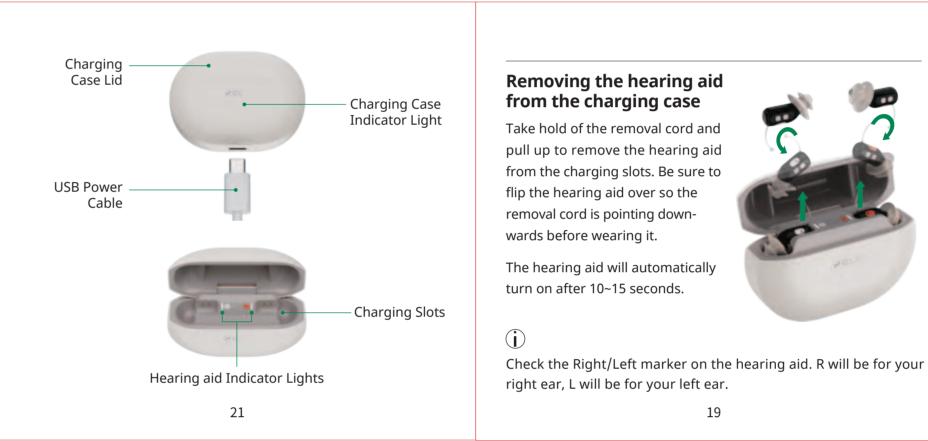


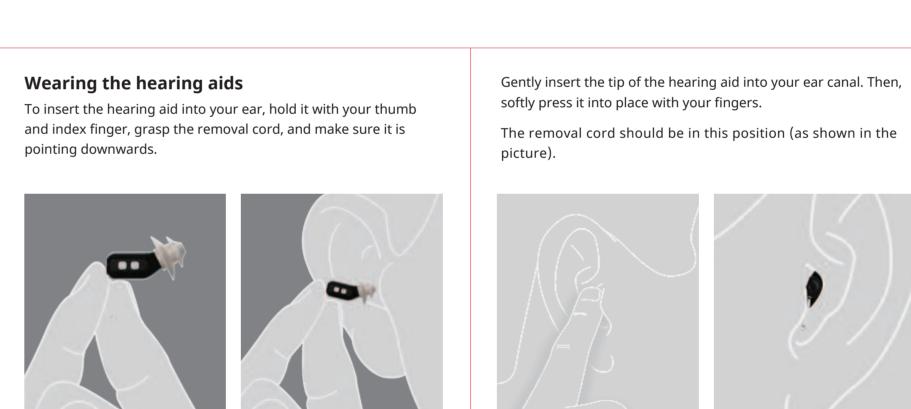

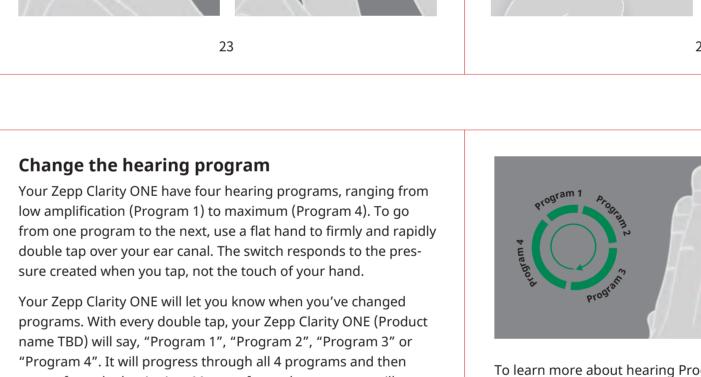

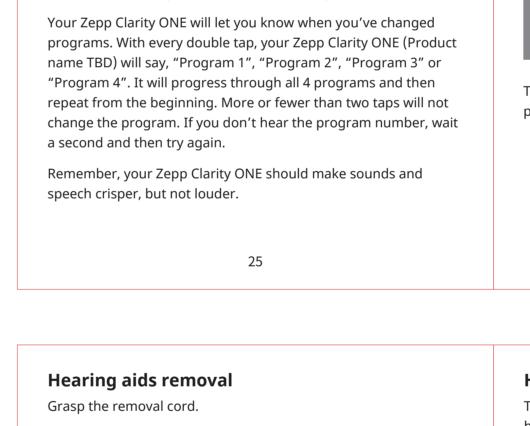

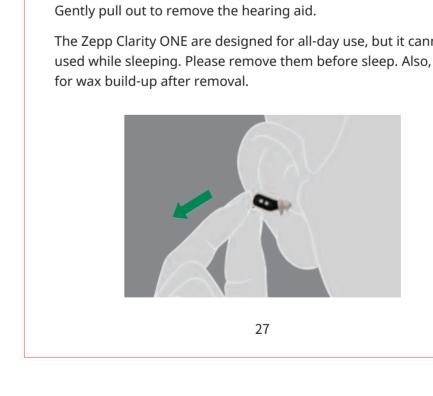

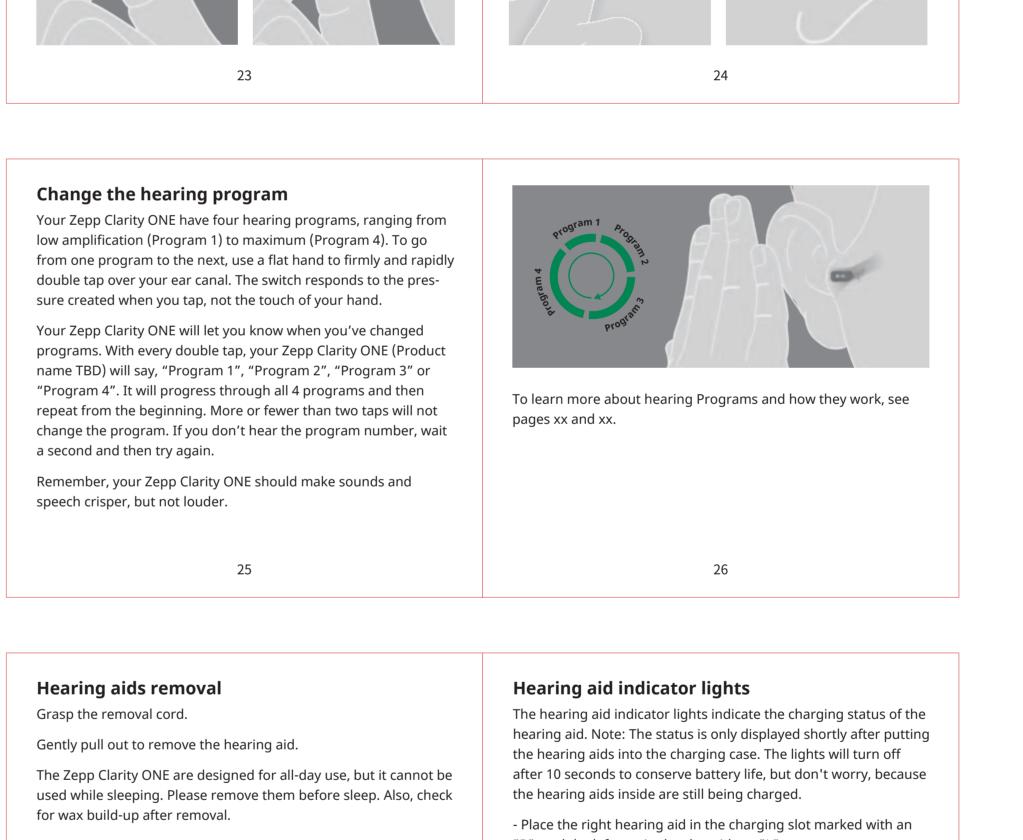



Place the right hearing aid in the charging slot marked with an "R", and the left one in the slot with an "L".
The hearing aids automatically turn off and start charging.


If the hearing aid indicator lights do not light up when you insert the hearing aids, check the following:


- The hearing aids are placed incorrectly. Push them gently into the slots.


- The battery of the charging case is too low for mobile charging. Connect the charging case to a power outlet.

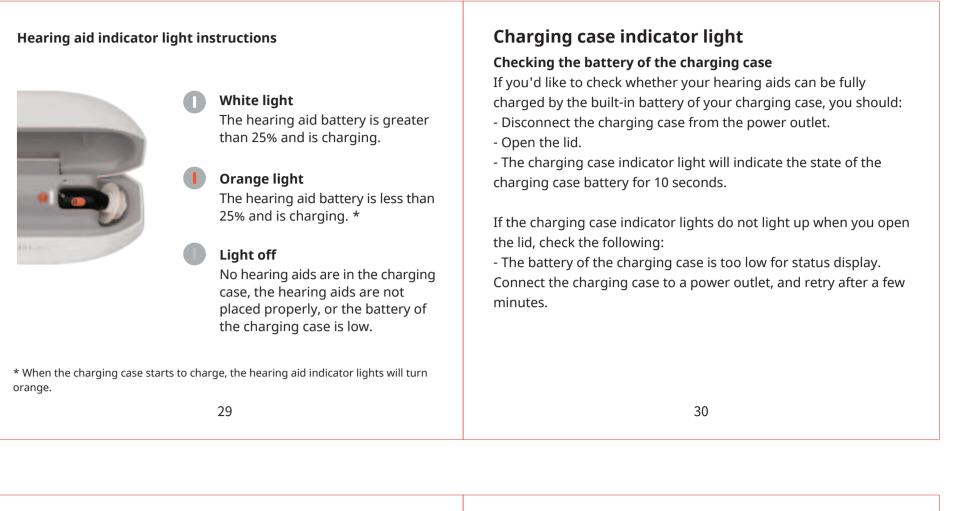


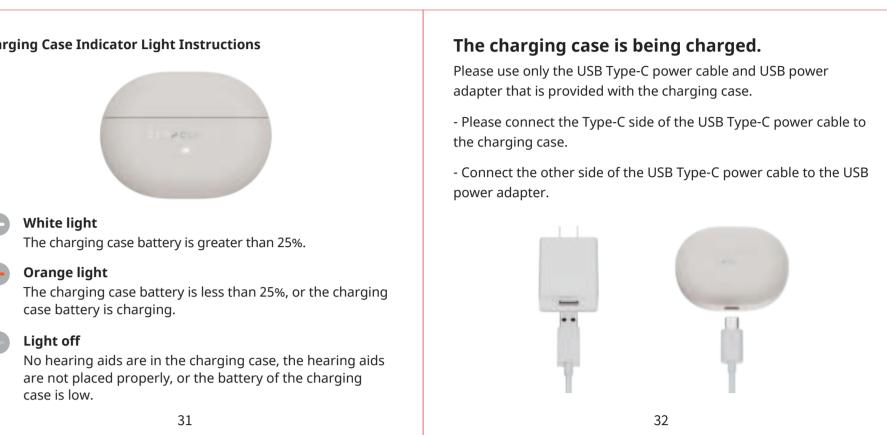


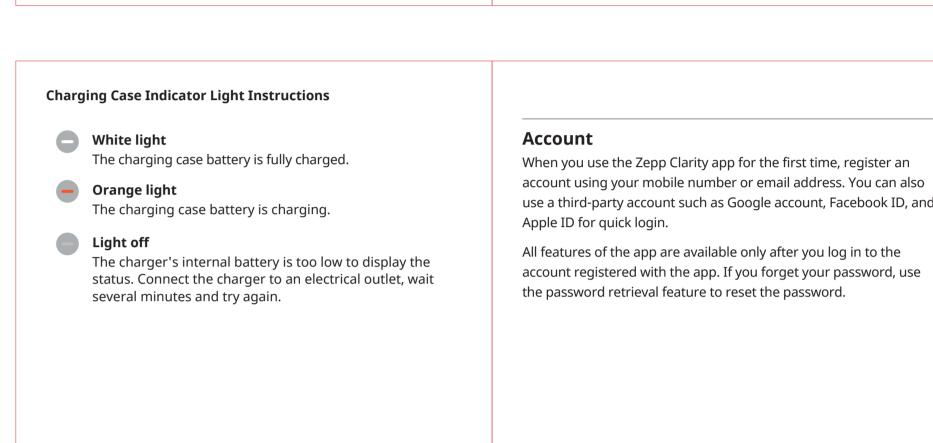


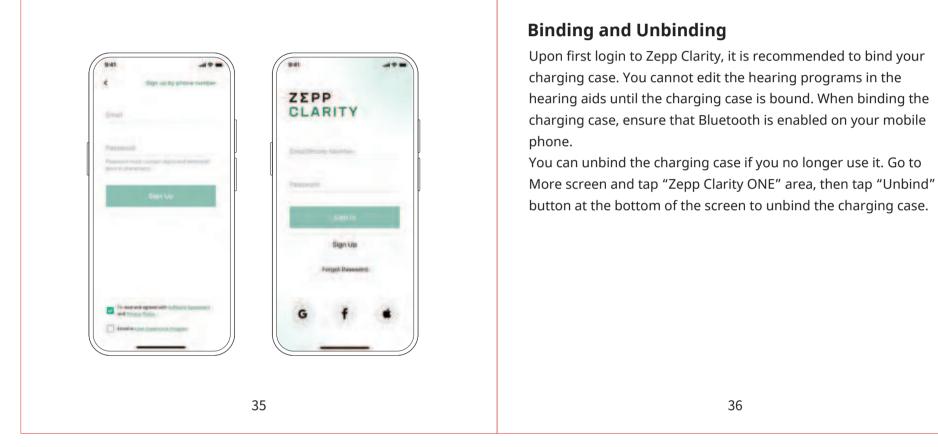


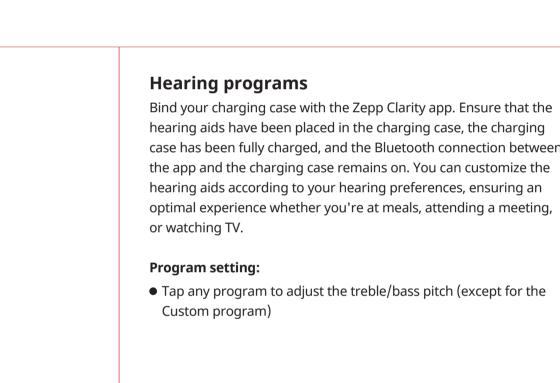


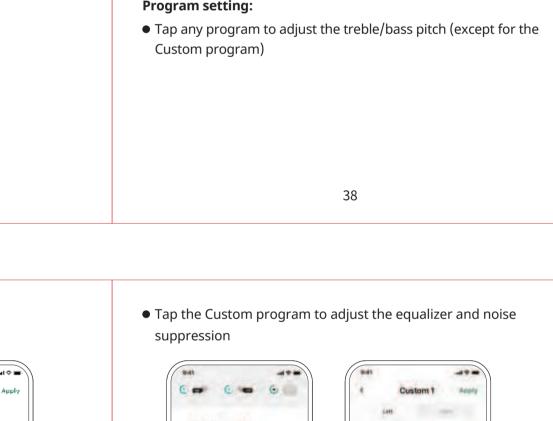


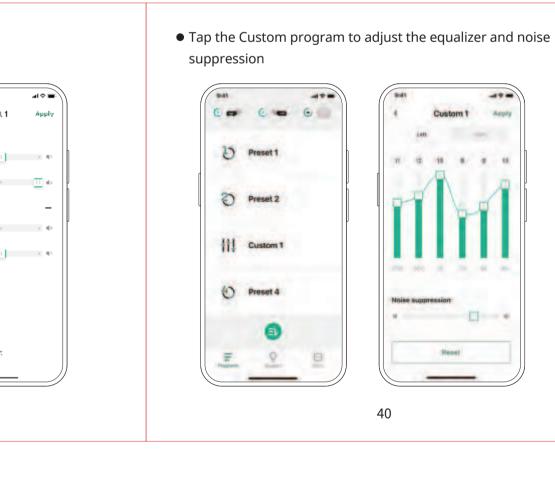

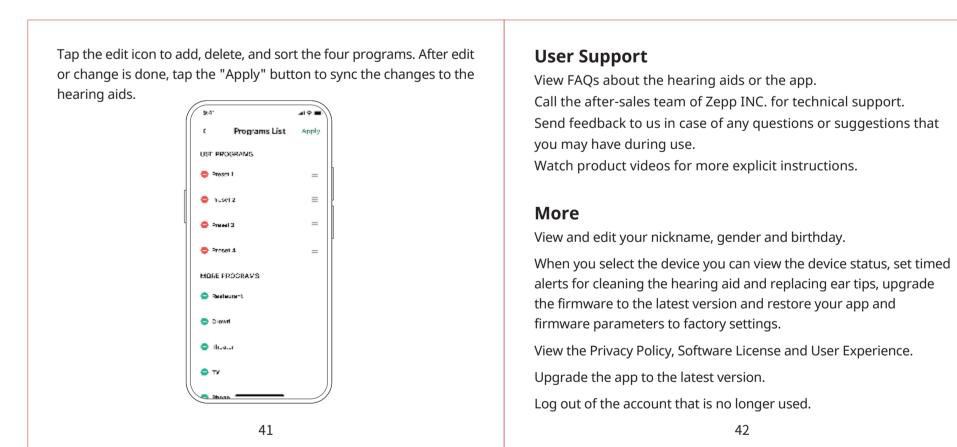


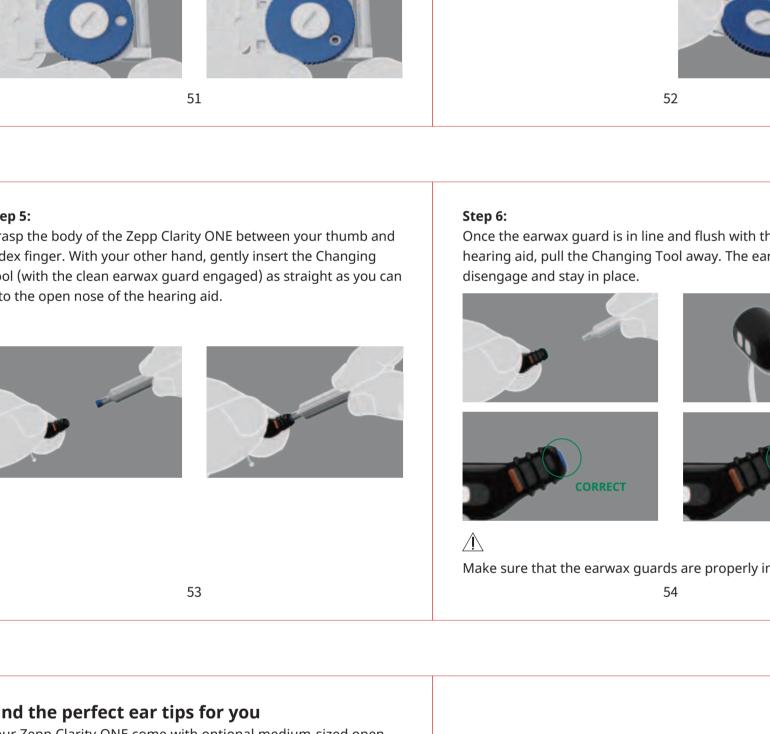


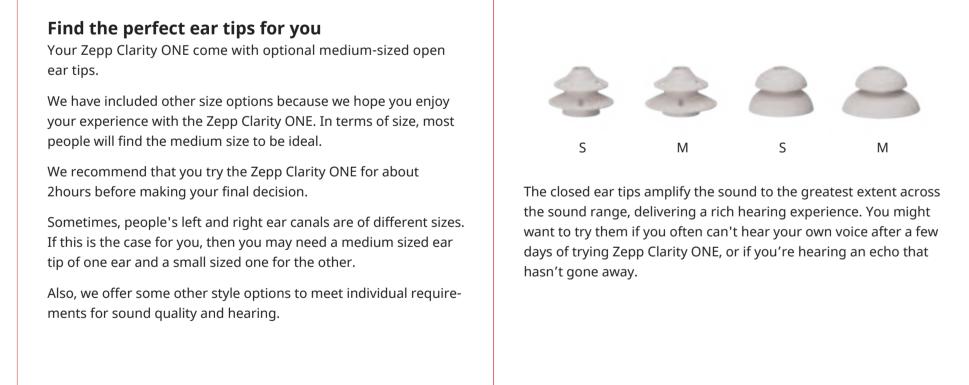


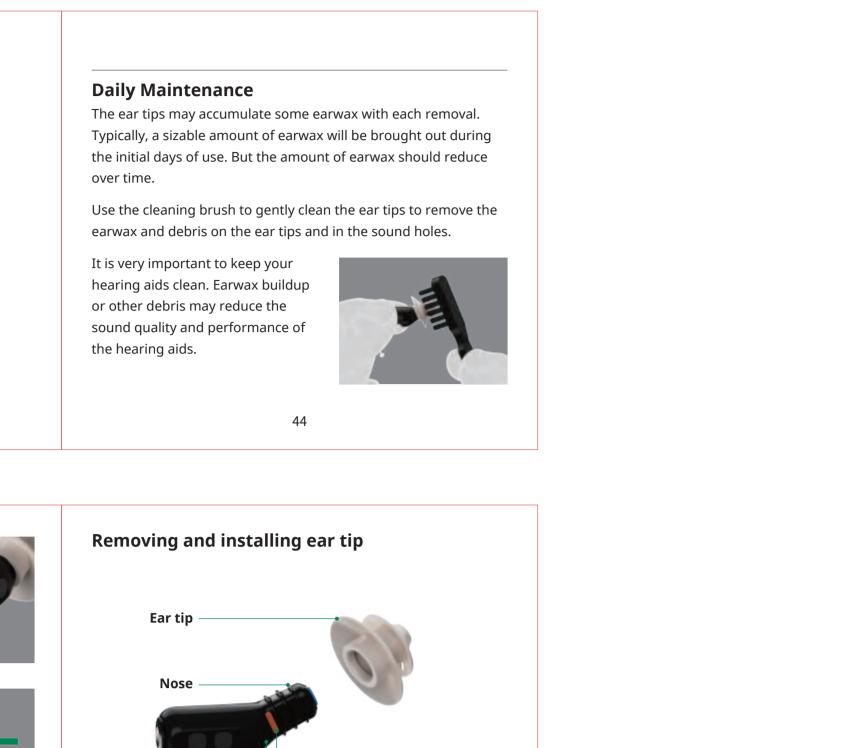




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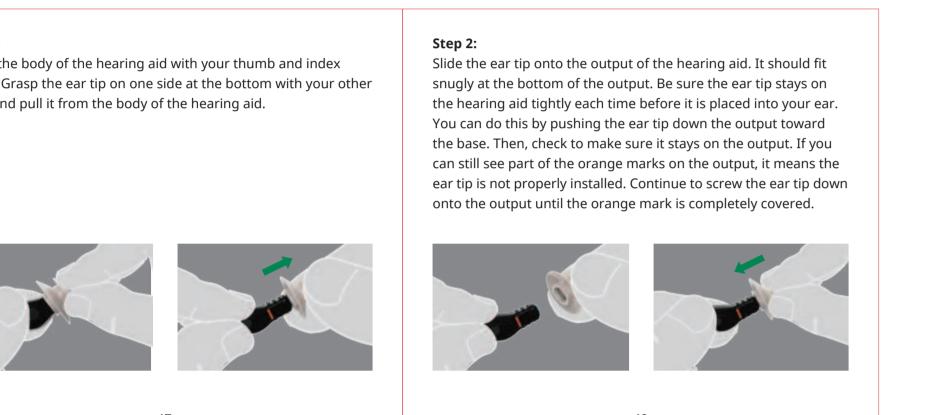


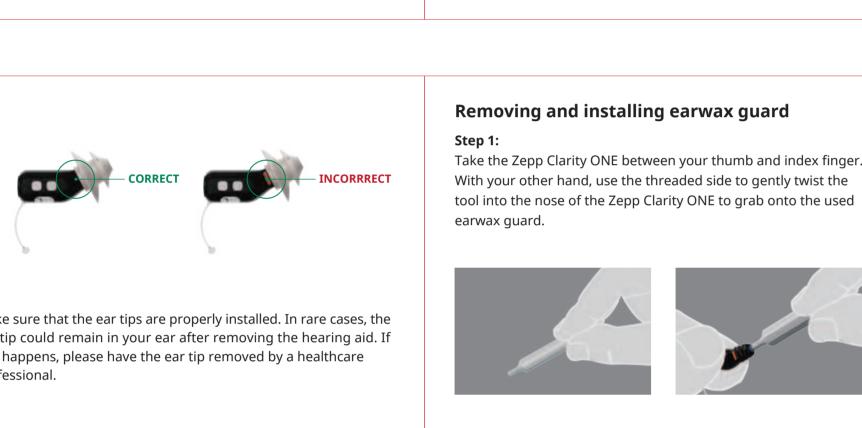


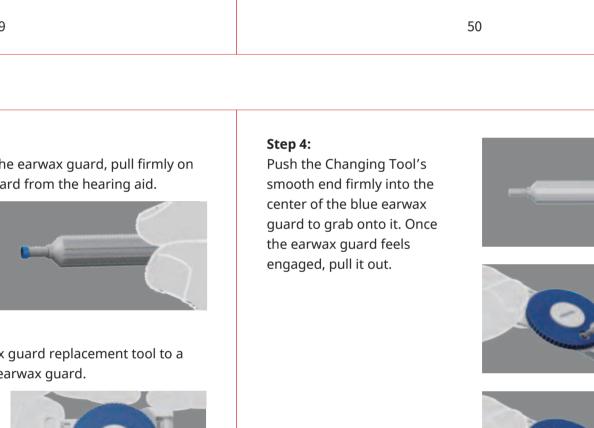

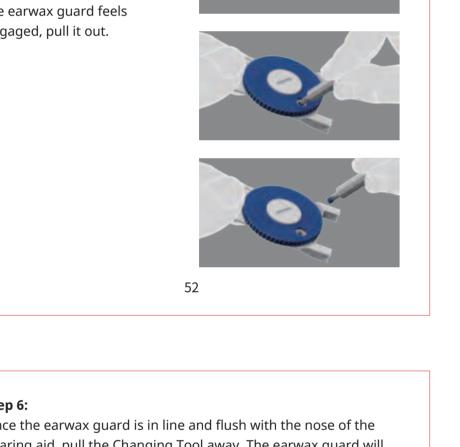





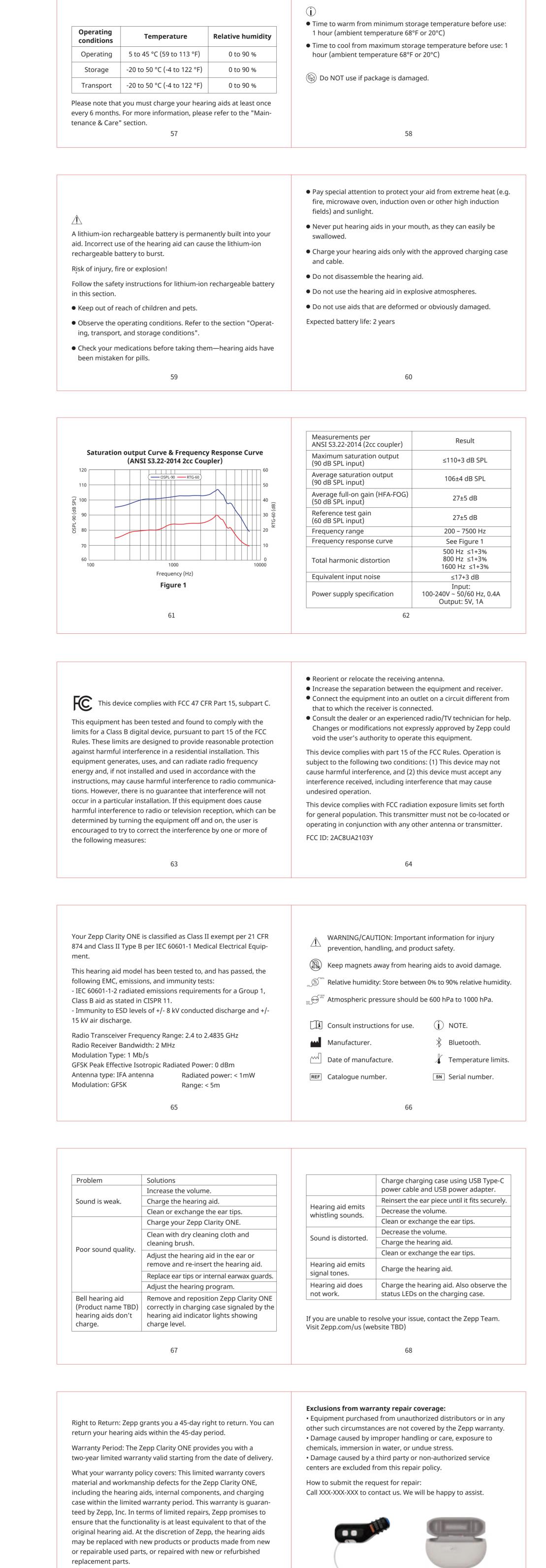





印刷一连线哑油一模切一折页

模切走位≤±0.5mm

颜色及专色: PANTONG Black C CMYK

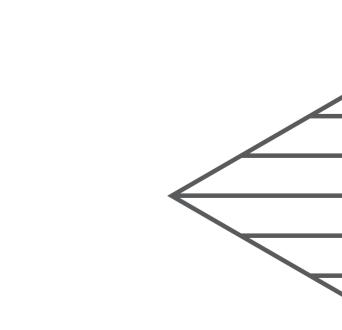

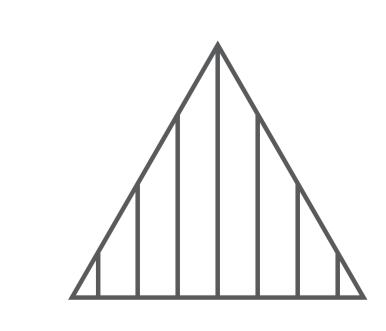

比例 1:1 单位 mm

