# RestOn

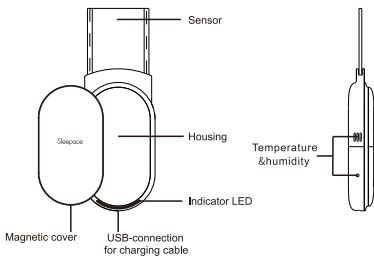
Sleep Tracker

Achieve your best sleep

Z400TWP

Sleepace


## 1 Device and controls

Items supplied and packaging:

- 1 \*Sleeppace Sleep Tracker      2 \*Power Adaptor      3 \*Instruction manual

## 2 Activating and positioning the Sleep Tracker

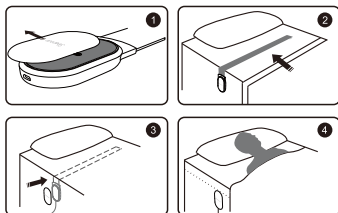- 1 Remove the magnetic cover.
- 2 Put the piezo strap between mattress and sheet as shown approximately at the height of your chest. The housing will be fixed on the side of the bed (on the sheet).
- 3 Put back the magnetic cover, connect power supply to start the device.
- 4 Connect the app and start sleep monitoring.

### 3 Downloading the app

Download and install the app on your smartphone. You can download the app free of charge from Google Play Store or Apple App Store. Search for " Sleepware".

iOS 8.0 or later, Android 4.3 or later.

### 4 Connecting to the app

- 1.Create an account
- 2.Remove the magnetic cover and put it back to restart the device
- 3.Start the connecting process as the app prompts
- 4.Connecting process completed, please follow the app's instructions to monitor your sleep and check the sleep report

### 5 Indicator LED Instructions

Power On	Indicator LED lights up in green for 1 seconds
Magnetic cover's position is in active mode	Indicator LED lights up in green, then goes off when exit the AP mode

### 6 Technical specifications

Housing	104mm*54mm*18.6mm
Sensor	830mm*45mm*1.5mm
Weight	220g
Power Supply	Power adaptor 5.0V / 1A
Operating Conditions	0°C -40°C (32°F -104°F)
Transmitting Method	Wi-Fi
Compatibility	iOS 8.0 or later, Android 4.3 or later

### 7 Safety notice

Only use the unit for the specific purpose described in the instruction manual. Any misuse will void the warranty.

- The unit is only designed for indoor use.
- The appliance is only suitable for use on flat and even bed surfaces. Do not use the Sleepace Sleep Monitor on mobile beds (such as inflatable beds or hammocks), as this could damage the appliance.
- This device is not a medical device and cannot be used to prevent sudden death.
- The use of this device does not replace the advice of a physician.
- People with health problems should principally consult a physician before using this device.
- This item must not be used on patients with a cardiac pacemaker.
- Do not excessively bend this product. The built-in sensor may be damaged.

- Do not let the device come in contact with liquids. It is not waterproof.
- The device must not be used close to any source of vibration (e.g. washing machines, loud music etc.) as the vibrations could affect the functions of the device.
- Only use the device within the ambient conditions mentioned in the chapter "technical specifications"
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.

## 8 Warranty

Please contact your dealers for warranty service.

For more information on the product warranty or service questions, please go to [www.sleepace.com](http://www.sleepace.com).

## 9 Certification

changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

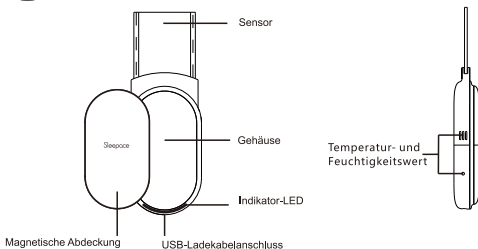
- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with a minimum distance of 20cm between the radiator & your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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