

				Regulatory Information Enclosed
--	--	--	--	---

XXXXXX-XXX

⚠ WARNING:
When to see a doctor.

If you have any of the problems listed below, please see a doctor, preferably an ear-nose-throat doctor (an ENT).

- Your ear has a birth defect or an unusual shape. Your ear was injured or deformed in an accident.
- You saw blood, pus, or fluid coming out of your ear in the past 6 months.
- Your ear feels painful or uncomfortable.
- You have a lot of ear wax, or you think something could be in your ear.

- You get really dizzy or have a feeling of spinning or swaying (called vertigo).
- Your hearing changed suddenly in the past 6 months.
- Your hearing changes: it gets worse then gets better again.
- You have worse hearing in one ear.
- You hear ringing or buzzing in only one ear

⚠ WARNING:
Medical evaluation for people younger than 18.
The use of a hearing aid in people

younger than 18 years old without a medical evaluation may worsen impairment or disability. A prospective hearing aid user who is younger than 18 should have a recent medical evaluation from a doctor, preferably an ear-nose-throat doctor (an ENT). Prior to purchase, a doctor should determine that the person is a candidate for the use of a hearing aid.

NOTE: Ask about trial-rental or purchase-option programs.

If you're unsure about your ability to get used to using a hearing aid, you should ask about a trial-rental or purchase-option program. Many hearing

instrument specialists offer programs that allow you to wear a hearing aid for a short time, at a nominal fee, before you decide to buy the hearing aid.

FCC ID:

EOA-24GENMICRO (mRIC R)
EOA-24GENMACRO (RIC RT)
EOA-24GENITELCH
(ITC R/HS R/ITE R left); EOA-24GENI-
TERCH (ITC R/HS R/ITE R right)
EOA-24LIVIORCHG (RIC R)
EOA-24LIVIOBCHG (BTE R)
EOA-24LIVIOITEL
(ITC R/HS R/ITE R left); EOA-24LIV-
IOITER (ITC R/HS R/ITE R right)

These devices comply with part 15 of the FCC rules. Operation is subject to the following two conditions:

(1) The device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation of the device.

Qty 1 pair of hearing aids with zinc-air batteries. See user manual for battery size of your hearing aid.



Starkey Laboratories
(Germany) GmbH
Weg beim Jäger 218-222
22335 Hamburg
Germany