


Keep Track of Your Injections

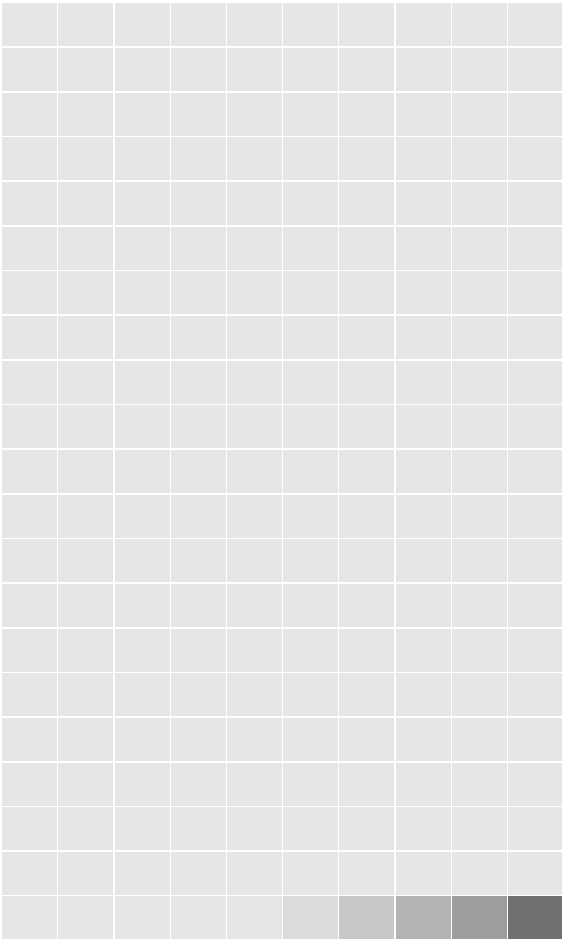

Device expiration

02 2024-08-27 8020100530_04

This is your
Instructions for Use

- For quick overview of steps, read your Quick Reference Guide
- For training video, go to www.skytrofa.com



Instructions
for Use



Quick
Reference Guide



www.skytrofa.com

Training
Video

Ascendis Pharma Customer Support



1-844-44ASCENDIS
(1-844-442-7236)



Distributed by:

Ascendis Pharma Endocrinology Inc.
Princeton, NJ 08540
USA



Phillips-Medisize A/S
Gimsinglundvej 20
DK-7600 Struer
Denmark

This Instructions for Use has been approved by
[Placeholder]
Approved: Month, Year

Instructions
for Use



Single-patient use



Getting Started

How to Use



Troubleshooting and Care

Product Information

Status Icons



Green Check Mark
Injection successfully completed



Green Eye
Mixed medicine is ready for inspection in window




Green Mixing
Medicine mixing is in progress

➤ Read more about status icons on pages 26-45 (and pages 52-63 for troubleshooting)



Constant Green
➔ Battery is fully charged
➔ The auto-injector is ready to use

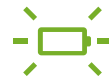

Flashing Green
➔ At least 1 injection remaining, but charging recommended after use



Flashing Orange
➔ Battery needs charging

➤ Read more about battery status icons on page 25

Parts Overview

Green Top

Starts injection when pressed against skin

Inspection Window

Allows for inspection of mixed medicine

Progress Bar (with six light elements)

Shows progress of mixing and injection

➤ Read more on pages 30-35 and 42-43 (and pages 52-53 for troubleshooting)

Status Icons

Green Button

To turn on the SKYTROFA Auto-Injector or to reject mixed medicine

Needle

To be screwed onto SKYTROFA Cartridge before inserting the cartridge in green top

SKYTROFA Cartridge

Contains medicine and water for injection. If refrigerated, leave it at room temperature for 15 minutes before use

Battery Icon

