

# The Dash<sup>PRO</sup>

tailored by Starkey<sup>®</sup>

## Quick Start Guide

v1.0, 2017-01, V3C QSG English

**BT:** XXXX-XXXXMHz **NFMI:** XX.XXXMHz  
**FCC ID:** XXXXXXXXXXXXXL **FCC ID:** XXXXXXXXXXXXXR  
**IC:** XXXX-XXXXXXXXXX



**Starkey Hearing Technologies**, Minneapolis, MN, USA & **Bragi GmbH**, Munich, Germany

Starkey is a trademark of Starkey Laboratories, Inc., registered in the U.S. and other countries.

All rights reserved. Patents pending. BRAGI, The Dash, "The World's First Hearable", "Listen. Track. Communicate.", and the B logo are trademarks of Bragi GmbH in the U.S. or one or more other countries. The absence of a product or service name or logo from this list does not constitute a waiver of trademark or other intellectual property rights concerning that name or logo.

**Thank you.**





BRAGI




## Charge & Activate.



Charge your Dash  
before first use.


# Charge.

Before using The Dash, charge it for at least 1 hour. This will also activate your Dash.

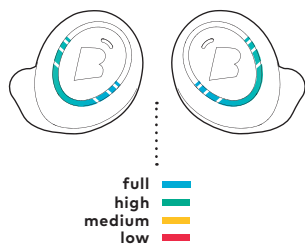

## Shake.


See the current battery status by shaking your Dash.

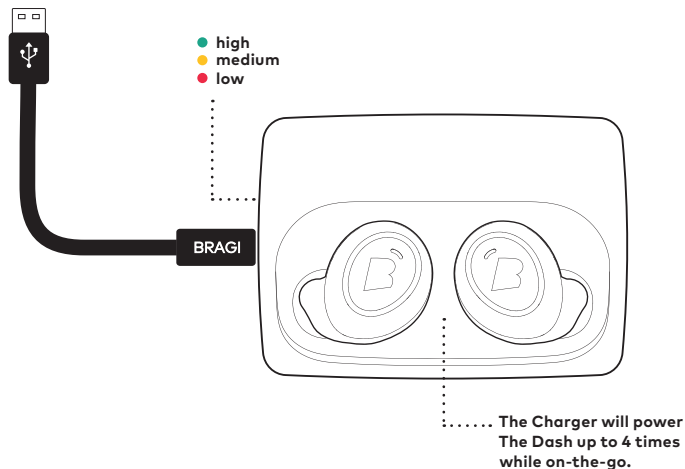

With The Dash in The Charger, connect to your computer or a power supply via USB.

# Update.

Your Dash learns new features when updated to the newest **Bragi OS**.  
Download from **[update.bragi.com](https://update.bragi.com)**


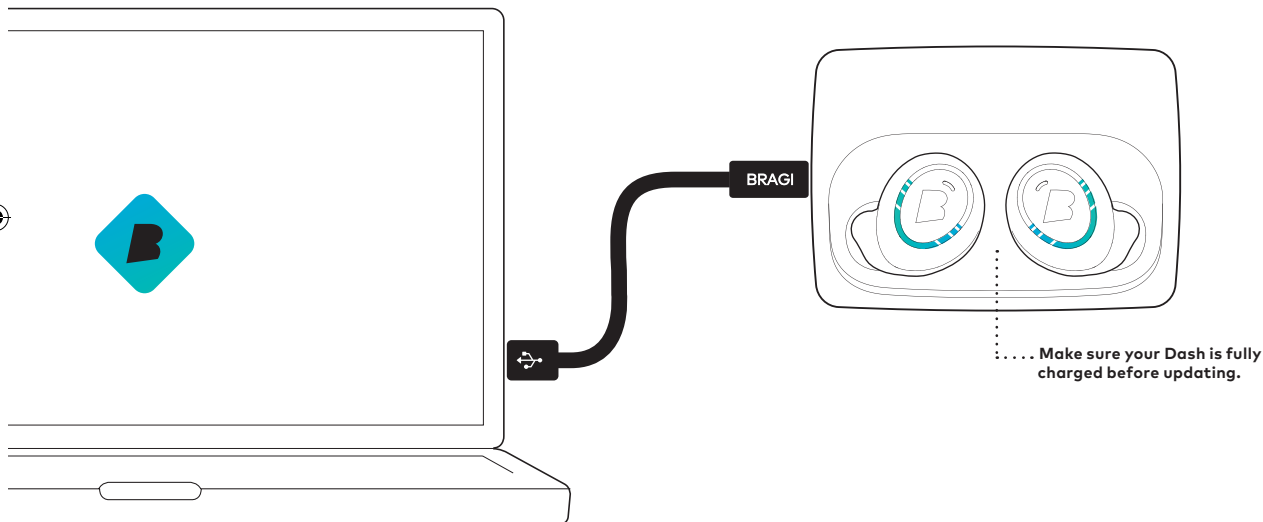


▶ Video tutorials on **[bragi.com/support](https://bragi.com/support)**


**[update.bragi.com](https://update.bragi.com)**

# Insert.

1. Tilt The Dash slightly forward.
2. Insert the tip firmly in your ear canal.
3. Adjust The Dash by rotating it backwards.

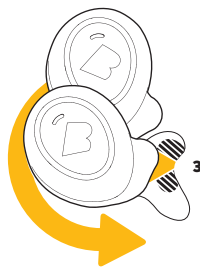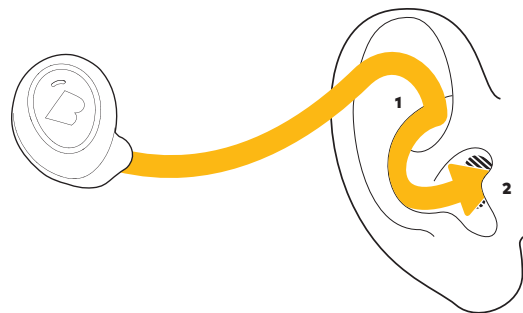



The PerfectFit

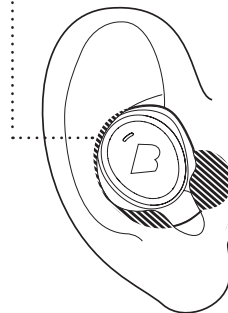

# HearClears.


## Replace your wax guard...

1. Place the blank side of the applicator in your HearClear® and twist it clockwise.
2. Pull it out.
3. Insert the new HearClear® from the other end of the applicator.

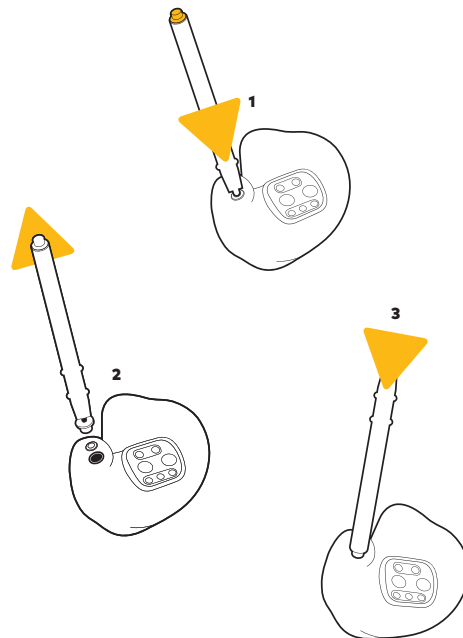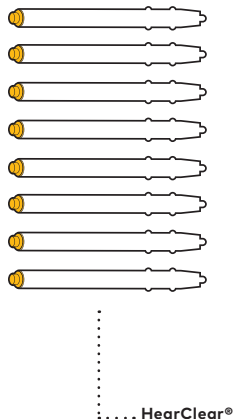

! Replace the HearClears of your Dash, if the sound is muffled.



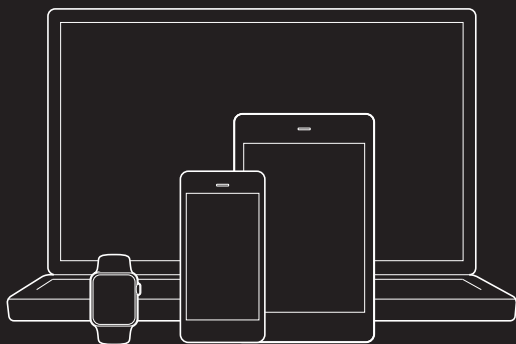

**Connect.**





**Control.**

## Pairing.

Insert The Dash into your ear and search for **The Dash** in the Bluetooth® menu of your device.

! If your Dash does not show up in the list, do a long hold (5 sec) on the Right Dash.

## Connect Audio.

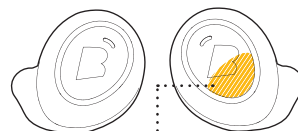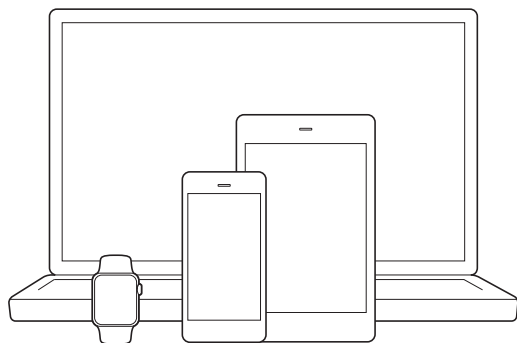


Long Hold on the  
Right Dash to initiate  
Bluetooth® pairing

## Pairing.

To send activity data from the Dash sensors to your device, open the Bragi App and follow the instructions on how to connect.


## Connect Sensors.

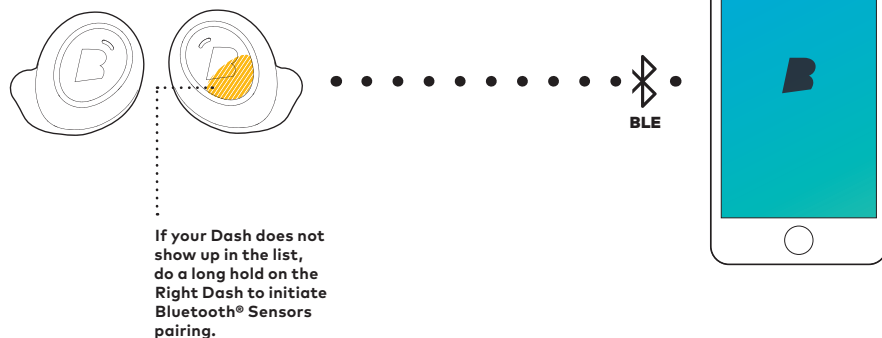

# Customize.

## Bragi App.

Download the Bragi app to set up and personalize your Dash. You can also use your smartphone to control the features of The Dash.

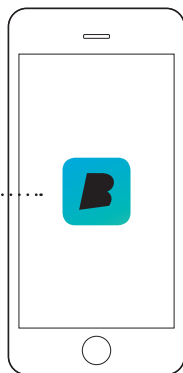


Use the Bragi App  
to set up:  
**Activities**  
**MyControls**  
**Sound**  
**Settings**  
**Profile**



## Free download!


# EarTouch.

Control your Dash with tap, hold and swipe gestures using the EarTouch Interface.

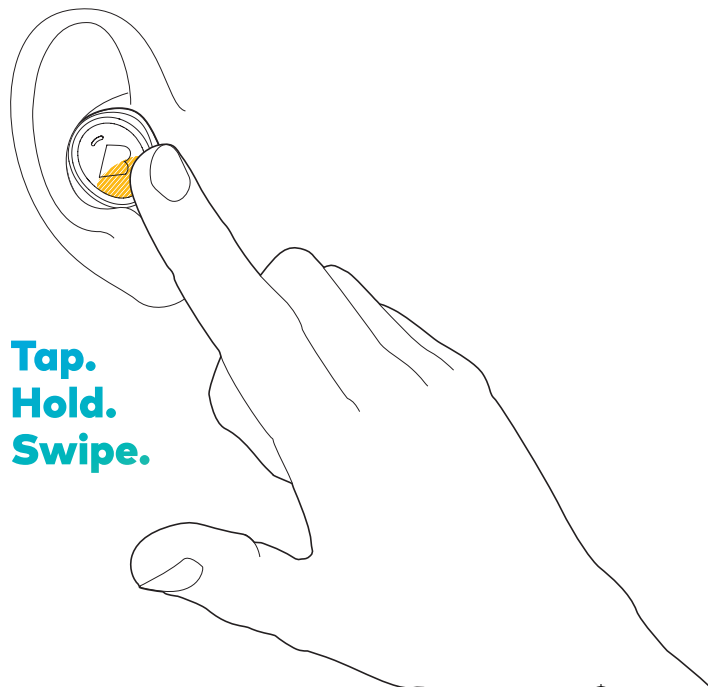

Tap.  
Hold.  
Swipe.

The EarTouch Interface  
is located on the lower  
half of each Dash.

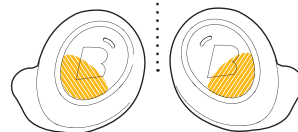

Hold on both Dash simultaneously  
to turn **TouchLock** on



# Basic Controls.



## Right Dash

..... Listen & Communicate

Tap	Play/Pause	●
2xTap	Next song	●●
3xTap	Previous song	●●●
Swipe	Volume Up/Down	◀▶
Hold 1sec.	Main menu	▬
Hold 5sec.	Bluetooth® settings	▬




## Left Dash

..... Control Applications

Swipe	Audio Transparency	◀▶
Hold 1sec.	App menu	▬

## Menu navigation

Tap	Select	●
Swipe	Next / Previous menu item	◀▶
Hold 1sec.	Close menu	▬

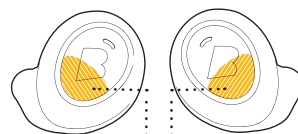

Hold 1sec. to  
open app menu


Hold 1sec. to  
open main menu



See more features on  
[bragi.com/support](https://bragi.com/support)

# Languages.

Use the Bragi Updater to change the language of your Dash. **Download from [update.bragi.com](https://update.bragi.com)**

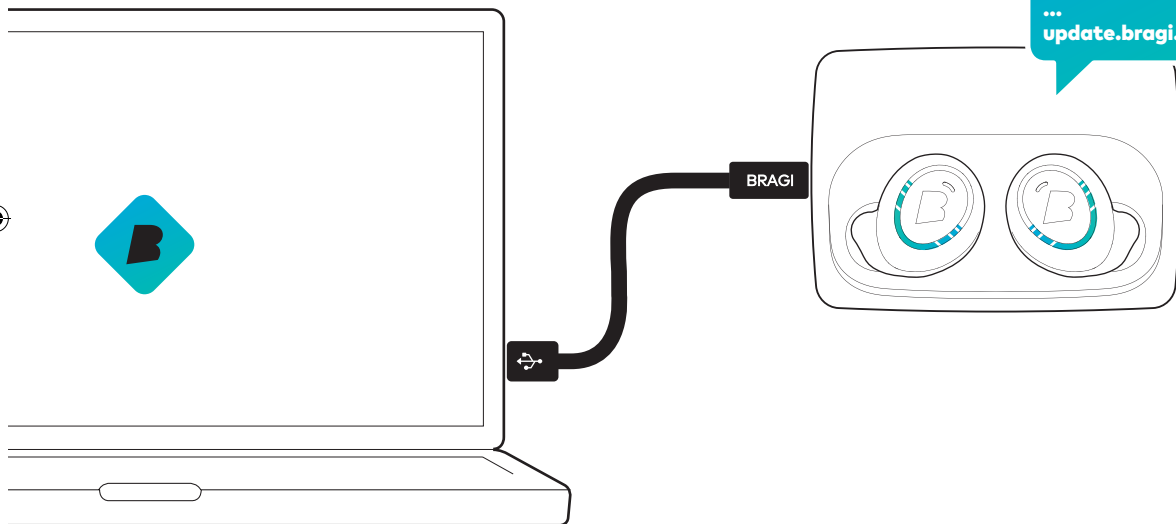

▶ Video tutorials on [bragi.com/support](https://bragi.com/support)

[update.bragi.com](https://update.bragi.com)

# Questions?

We are here to help.

## **Starkey Limited Customization Refit**

If you are not satisfied with the fit of The Dash - tailored by Starkey Hearing Technologies, you shall have 30 days from receipt of The Dash - tailored by Starkey Hearing Technologies purchase to notify the practice where purchased of User's desire to seek a replacement custom fit at no cost.

**[support.bragi.com](https://support.bragi.com)**